

<p>Meerkat</p>																					
<p>User information appropriate for use by people with visual, reading or cognitive disability</p>	<p>The latest version of all instructions are always available on the etac website etac.com and can be printed in larger sizes</p>																				
<p>Information available in pictogram</p>	<p>The instructions are made as pictograms</p>																				
<p>Intended use and intended environment including standard options</p>	<p>Intended purpose The device is a standing frame for children and adults with disabilities.</p> <p>Indications The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age. The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product.</p> <p>Contra indications There are no known contra indications.</p> <p>Precautions Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:</p> <ul style="list-style-type: none"> • Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories. • Movement function (e.g. experience involuntary movements). • Ability to maintain a standing position, use the product with accessories (e.g. chest or back support). • Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head. <p>Application areas The product is for indoor use</p>																				
<p>Maintenance instructions</p>	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1" data-bbox="574 1523 1356 2016"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>Before every use</td> <td>- All nuts and bolt on this product must be checked and tightened to avoid failures</td> </tr> <tr> <td>Daily</td> <td>- Use a dry cloth to clean the product</td> </tr> <tr> <td></td> <td>- Visual inspection. Repair or replace any damaged or worn-out parts</td> </tr> <tr> <td></td> <td>- Ensure that all fasteners are in place and fitted correctly</td> </tr> <tr> <td></td> <td>- Check all fittings and buckles for signs of worn out parts</td> </tr> <tr> <td></td> <td>- Be aware of any MAX mark indications before adjustment</td> </tr> <tr> <td></td> <td>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</td> </tr> <tr> <td>Weekly</td> <td>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</td> </tr> <tr> <td></td> <td>- Disinfect the product. please check the disinfection section in this User Guide for more information</td> </tr> </tbody> </table>	Frequency	Description	Before every use	- All nuts and bolt on this product must be checked and tightened to avoid failures	Daily	- Use a dry cloth to clean the product		- Visual inspection. Repair or replace any damaged or worn-out parts		- Ensure that all fasteners are in place and fitted correctly		- Check all fittings and buckles for signs of worn out parts		- Be aware of any MAX mark indications before adjustment		- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use	Weekly	- Check that all castors are moving freely and that all wheel locks/brakes are easily activated		- Disinfect the product. please check the disinfection section in this User Guide for more information
Frequency	Description																				
Before every use	- All nuts and bolt on this product must be checked and tightened to avoid failures																				
Daily	- Use a dry cloth to clean the product																				
	- Visual inspection. Repair or replace any damaged or worn-out parts																				
	- Ensure that all fasteners are in place and fitted correctly																				
	- Check all fittings and buckles for signs of worn out parts																				
	- Be aware of any MAX mark indications before adjustment																				
	- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use																				
Weekly	- Check that all castors are moving freely and that all wheel locks/brakes are easily activated																				
	- Disinfect the product. please check the disinfection section in this User Guide for more information																				

Meerkat	
	Every 6 months - Check the condition of the upholstery and replace it if necessary Annually - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty - The product has to be checked and tightened by a qualified technician at least every 12 months to avoid failures.
Cleaning description	Washing: The cushioning is removable and the cover can be washed at max. 60° using a mild detergent in a washing machine. Warewashing machine This product can be washed at 60° using a mild detergent for 15-20 minutes in a warewashing machine designed for medical devices. Use the machine's drying function to dry the product. Remove all accessories before washing the main product. Clean the accessories separately. Please allow the product to dry before reuse.
Disinfection description	The product can be disinfected with a 70% IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth wrung in warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	http://www.etac.com - Select product
Mass	http://www.etac.com - Select product
Maximum slope	http://www.etac.com - Select product
Type(s) of tyres	Castors
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. The Meerkat might loose its flame resistant characteristics if using other than original R82 cushioning.
List of accessories, detachable parts and materials	http://www.etac.com - Select product
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	See pictograms in the User Guide
Used in motor vehicles	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A