

Wombat Living													
User information appropriate for use by people with visual, reading or cognitive disability	The latest version of all instructions are always available on the R82 website <a href="http://support.R82.org">support.R82.org</a> and can be printed in larger sizes												
Information available in pictogram	The instructions are made as pictograms												
Intended use and intended environment	<p>The Wombat Living is suitable for users needing a practical indoor activity chair e.g. children with GMFCS level 2-4, children who need extra help while sitting.</p> <p>The wide range of height adjustments makes it possible for the child to participate in activities at the floor level or at the table level.</p> <p>The tilt adjustments of both the seat and backrest together with a large modularized accessory program provide the possibility of placing a child in the posture which is most beneficial for the individual child.</p> <p>At configuration, it is possible to choose between two ways to operate tilt-in-space and three ways to operate the height adjustment. All versions are delivered with a multi-adjustable seat, back and footrest.</p> <p>The product is available with gas-spring or an electric motor.</p> <p>The Wombat Living comes in 3 sizes and is for indoor use only</p>												
Maintenance instructions	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1<sup>st</sup> Day</td> <td> <ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store it in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul> </td> </tr> <tr> <td>Daily</td> <td> <ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fit-ted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any MAX mark indications before adjustment</li> <li>- Check that all cables connecting the power system are in good shape</li> </ul> </td> </tr> <tr> <td>Weekly</td> <td> <ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul> </td> </tr> <tr> <td>Monthly</td> <td> <ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> </ul> </td> </tr> <tr> <td>Annually</td> <td> <ul style="list-style-type: none"> <li>- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty</li> </ul> </td> </tr> </tbody> </table>	Frequency	Description	1 <sup>st</sup> Day	<ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store it in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul>	Daily	<ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fit-ted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any MAX mark indications before adjustment</li> <li>- Check that all cables connecting the power system are in good shape</li> </ul>	Weekly	<ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul>	Monthly	<ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> </ul>	Annually	<ul style="list-style-type: none"> <li>- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty</li> </ul>
Frequency	Description												
1 <sup>st</sup> Day	<ul style="list-style-type: none"> <li>- Read the User Guide carefully before use and store it in a safe place for future reference</li> <li>- Store the enclosed tools on/at the product</li> </ul>												
Daily	<ul style="list-style-type: none"> <li>- Use a dry cloth to clean the product</li> <li>- Visual inspection. Repair or replace any damaged or worn-out parts</li> <li>- Ensure that all fasteners are in place and fit-ted correctly</li> <li>- Check all fittings and buckles for signs of worn out parts</li> <li>- Be aware of any MAX mark indications before adjustment</li> <li>- Check that all cables connecting the power system are in good shape</li> </ul>												
Weekly	<ul style="list-style-type: none"> <li>- Check that all castors are moving freely and that all wheel locks/brakes are easily activated</li> <li>- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use</li> </ul>												
Monthly	<ul style="list-style-type: none"> <li>- All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures</li> <li>- Oil the swing-away parts. We recommend you use a professional lubricant system</li> </ul>												
Annually	<ul style="list-style-type: none"> <li>- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty</li> </ul>												

Wombat Living	
	<ul style="list-style-type: none"> <li>- The product has to be checked and tightened by a qualified technician at least every 12 months to avoid failures.</li> </ul>
Cleaning description	<p>The cushion is removable and the cover can be washed at max. 60°C using a mild detergent in a washing machine. The cushion covers can be washed at 60°C using a mild detergent in a washing machine. It is not recommended to wash the foam.</p> <p>This product can be washed at 60° using a mild detergent in 15-20 minutes in a washing machine de-signed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.</p> <p>Before washing:</p> <ol style="list-style-type: none"> <li>1. Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.</li> <li>2. Remove all accessories and remove the power column</li> </ol> <p>After washing:</p> <ol style="list-style-type: none"> <li>3. Replace the power column going backwards</li> <li>4. Check that the handset functionalities work correct</li> </ol>
Disinfection description	The product without accessories and cushion can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Mass	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	N/A
Transport instructions	-10°C to +50°C (power version)
Sound power level	Noise level: ≤ 50 db (power version)