---

## **Monteringsvejledning (DK)**

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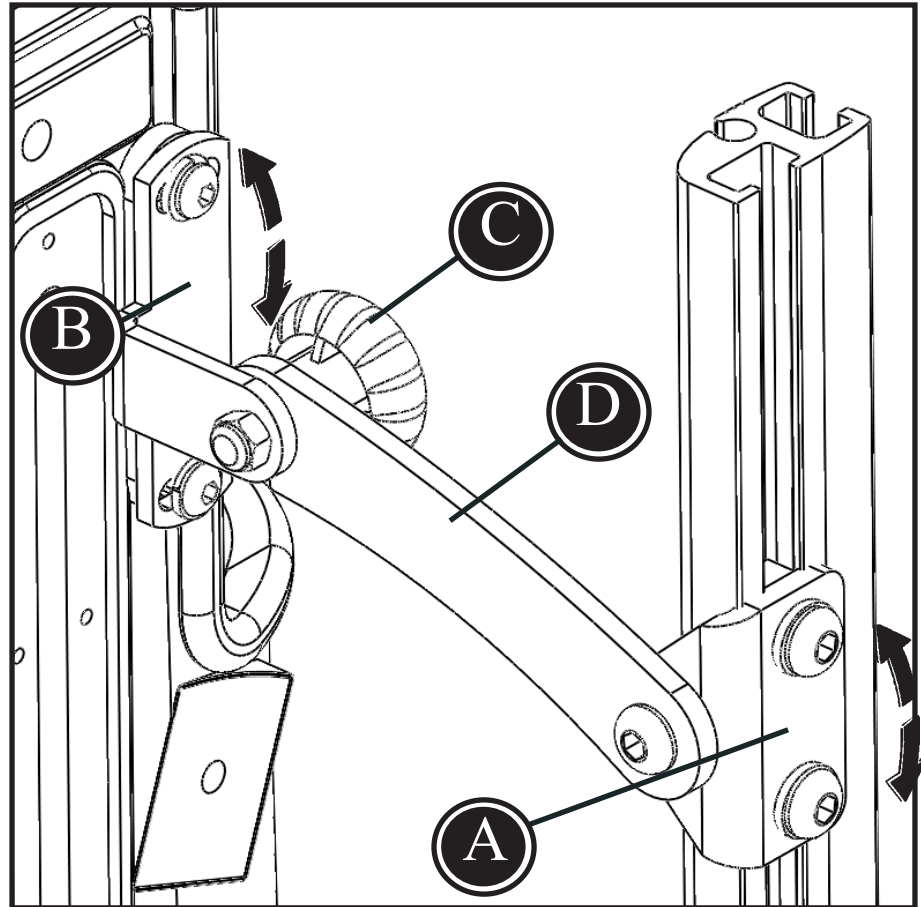
# Mounting instruction

R82<sup>®</sup>

## Mounting transport fitting on Combi Frame with Panda Futura seat

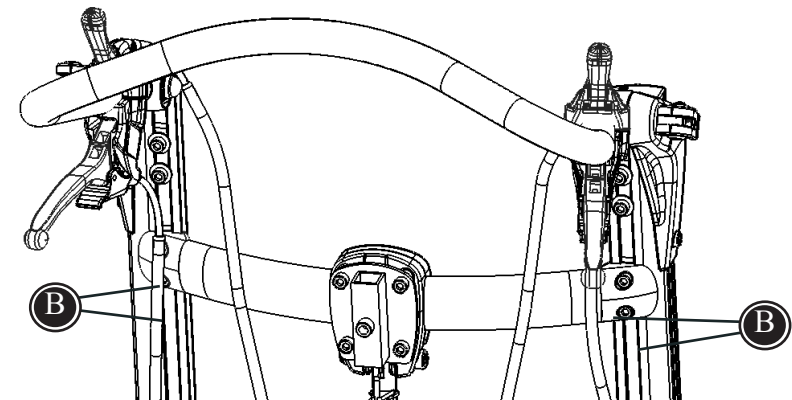
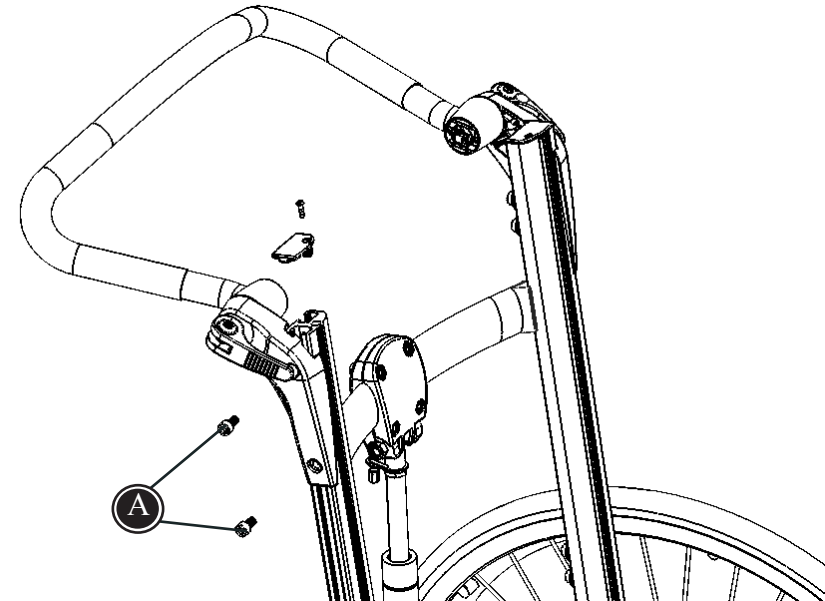
 This mounting instruction applies only to transport fitting for the Panda Futura seat. The Combi Frame also has to be secured separately depending on the foot support mounted - see separate mounting instructions.

 The fitting **MUST** be mounted for transportation.

 On Combi Frame size 3 with Panda Futura size 4, the fitting is placed above the back brace. Skip step 1 and 2, and go to step 3 on next page.

 Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

1. Remove the push brace by loosening the 2 bolts (A) on each side.
  - \* Pull up the push brace.
2. Remove the top back brace by loosening the 4 screws (B).
  - \* Tilt back the vertical bars.
  - \* Slide up the back brace.

# Mounting instruction

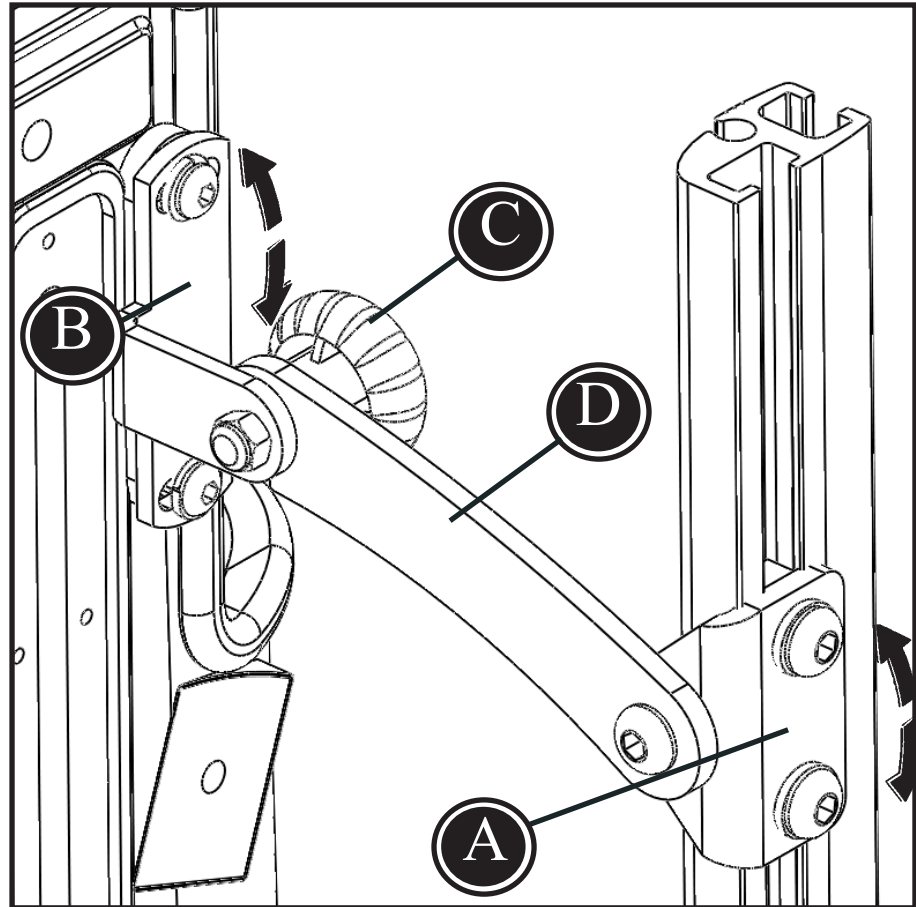
R82<sup>®</sup>

## Mounting transport fitting on Combi Frame with Panda Futura seat

3. Make sure that the seat back and the back of the frame runs parallel in a vertical position.
- \* Mount fitting (A) on the Combi Frame profile. Please note that the fitting is 180° reversible depending on the combination of the seat and frame size.
  - \* Mount fitting (B) on the Panda Futura back, as shown on the drawing. Please note that the fitting is 180° reversible depending on the combination of the seat and frame size.
  - \* The fingerscrew (C) is screwed into the fitting (B). Fasten the nut at the end of the fingerscrew.
  - \* The brace (D) mounted on the fitting (A) is brought over the fingerscrew (C).
  - \* Tighten the fingerscrew.
  - \* The fitting (A) can be loosened for final adjustments.

The seat can easily be removed by loosening the fingerscrew (C) and lifting the brace (D) mounted on the fitting (A).

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