Monteringsvejledning (DK)

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Mounting instruction

R82[®]

Mounting transport fitting on the Panther

Follow the below instruction to mount the transport fitting on the Panther and prepare the wheelchair to transportation in vehicles:

Mount the enclosed two fittings (A) on the back of the wheelchair.








Mount the enclosed two fittings (B) on the front of the wheelchair.

The transport fittings are equipped with a secure symbol (C).

Secure the wheelchair to the vehicle, using the 4-point strap-type tiedown system, approved according to ISO 10542-2.

Fasten the 3-point belt. The belt restraints should be adjusted as tightly as possible, consistent with user comfort. Further the belt webbing should not be twisted when in use.

Please find the Panther manual on www.r82.com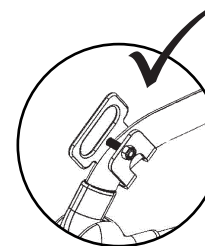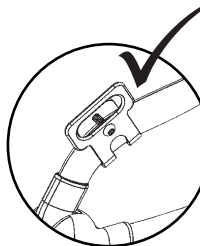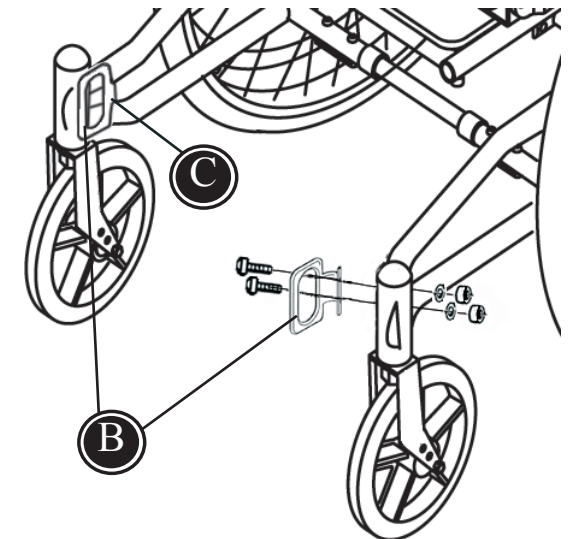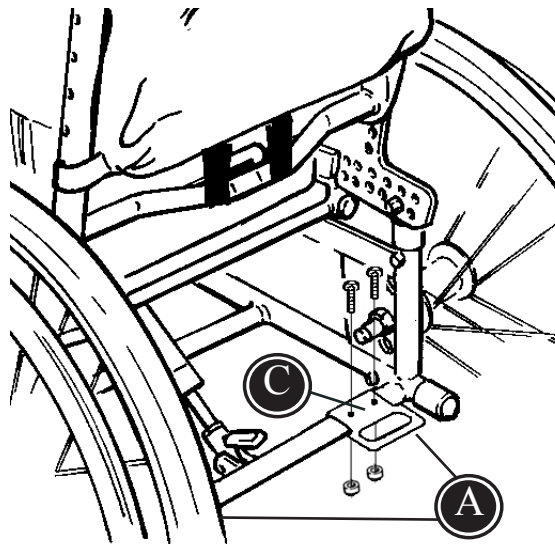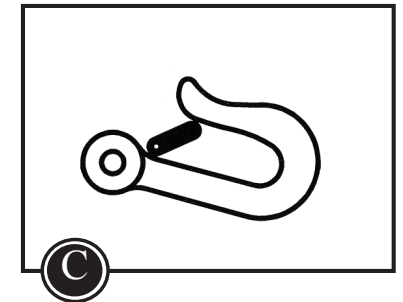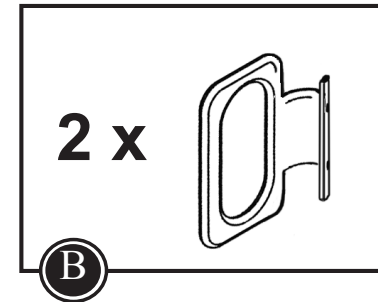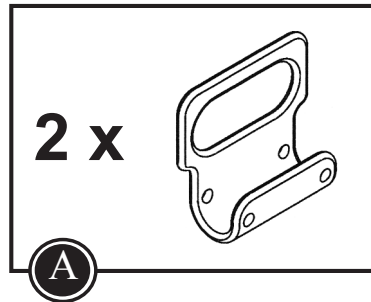

The Panther wheelchair is approved for transporting children in motor vehicles, when the wheelchair is placed in forward-facing position. Follow the below to ensure the safety:

 The child should transfer to the vehicle seat and use the vehicle-installed restraint system whenever it is feasible.

 The wheelchair has been dynamically tested according to ANSI/RESNA WC/Vol. 1-1998, section 6.4.1, litra c.


 The wheelchair must be placed in forward-facing position, when used as a seat in a motor vehicle. Use an approved 4-point strap-type tiedown system.

 Use an approved 3-point belt in the wheelchair. Restraints should not be held away from the body by wheelchair components such as armrests or wheels.

 All accessories have to be removed from the wheelchair and secured separately in the vehicle. Accessories which can not be removed, have to be secured to the wheelchair but positioned away from the child with energy-absorbing padding placed between the accessories and the child.

 Alterations or substitutions should not be made to the wheelchair without consulting the dealer.

 The wheelchair should be inspected by the dealer before reuse following involvement in any type of vehicle impact.

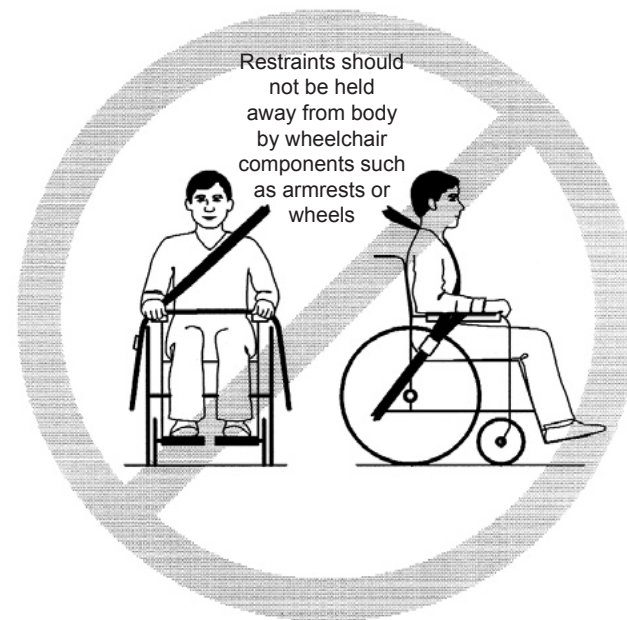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