Monteringsvejledning (DK)

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Mounting instruction

Mounting transport fittings on the Serval

R82[®] 

Follow the instructions below to mount the transport fittings on the Serval and prepare the pushchair to transportation in vehicles:






Secure the child in the pushchair:

Dismount the 4 fixlocks (C) on the back of the Serval seat using a 3 mm Allen key.

Mount the fittings (A) where the fixlocks were mounted. Fixlocks can be mounted again on the outside of the transport fittings. Mount with enclosed nuts, bolts and a 3 mm Allen key.

Mount a hip-belt, approved according to ISO 10542-1 or SAE J2249 and secure the child in the pushchair.

The back of the seat must be in vertical position.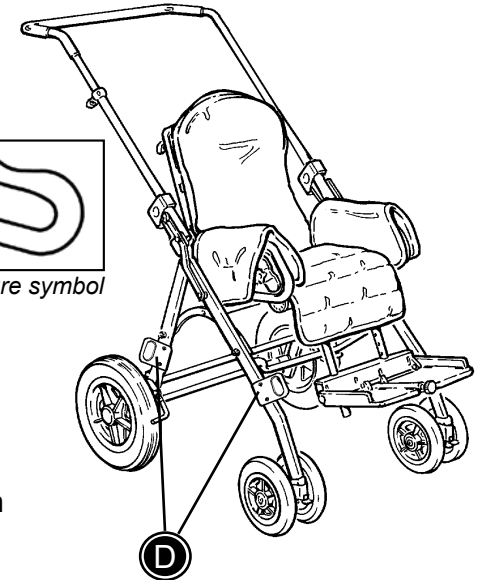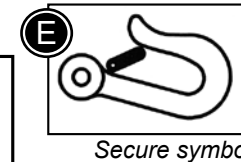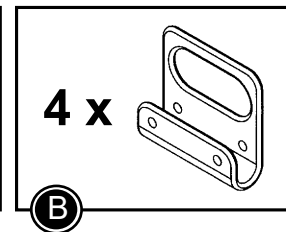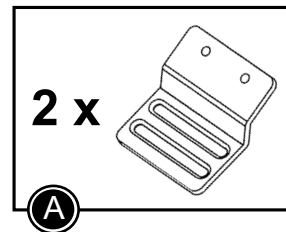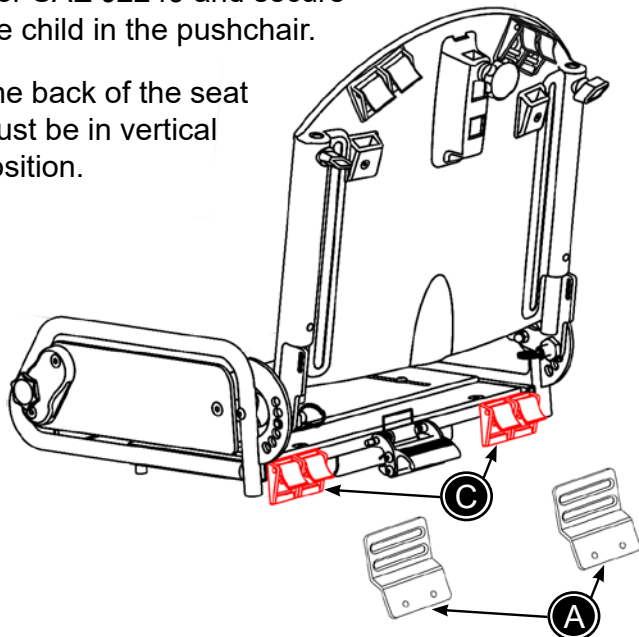

Secure the Serval in the vehicle:


You have to mount 4 transport fittings (B): 2 on each side of the Serval

Mount the transport fittings in the predrilled holes (D) Remove the plastic plugs from the frame and fasten the transport fittings with nuts, bolts and a 4 mm Allen key.

Stick the enclosed stickers (E) on the four transport fittings (B).

Secure the pushchair to the vehicle, using a 4-point strap type tiedown system, approved according to ISO 10542-2 and SAE J2249.

Use the 3-point belt to secure the user. The belt restraints should be adjusted as tightly as possible, consistent with user comfort. Further the belt webbing should not be twisted when in use.

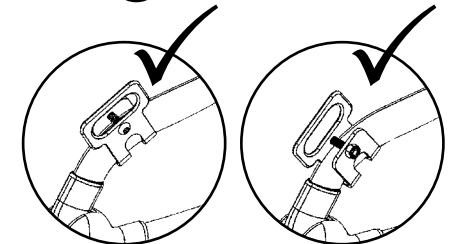Use an approved 3-point belt to secure the user. It is not sufficient to use only a hip belt!

Please find the Serval manual on www.R82.com

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