Monteringsvejledning (DK)

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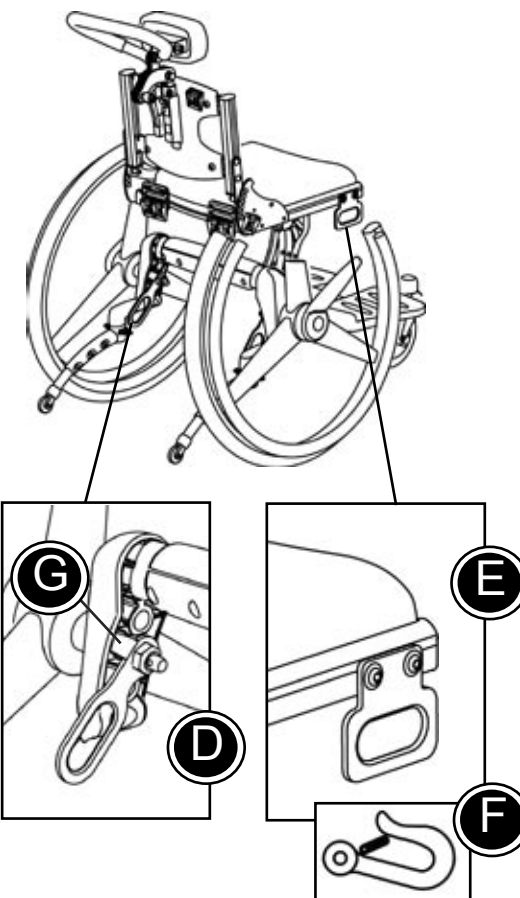
Mounting instruction

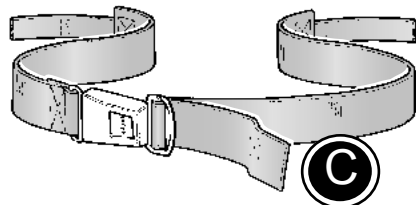
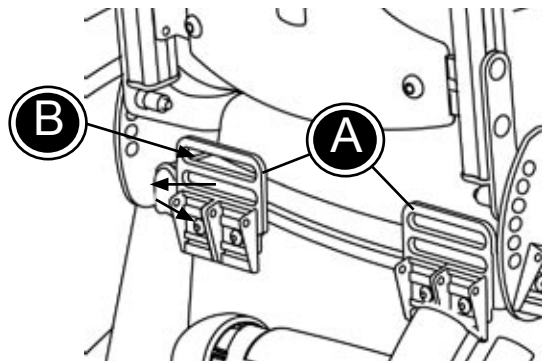
Mounting transport fitting on the Cheetah

Follow the below instruction to mount the transport fitting on the Cheetah and prepare the pushchair to transportation in vehicles:

Seat belt attachment:

- * Mount the two belt fittings (A), for the seat belt, in the slots on the back side of the seat.
- * Mount the ends of the seat belt in the fittings and fasten in the snap locks (B).
- * Secure the user in the wheelchair and fasten the seat belt using the buckle.
- * Pull the strap (C) until the seat belt is secured in the correct position.

Secure the Cheetah in the vehicle:

- * Mount the 4-point strap-type tiedown system in the vehicle. (Follow the manufacturers instructions).
- * Mount the fittings (D) with the symbol (F) on the back of the chair.

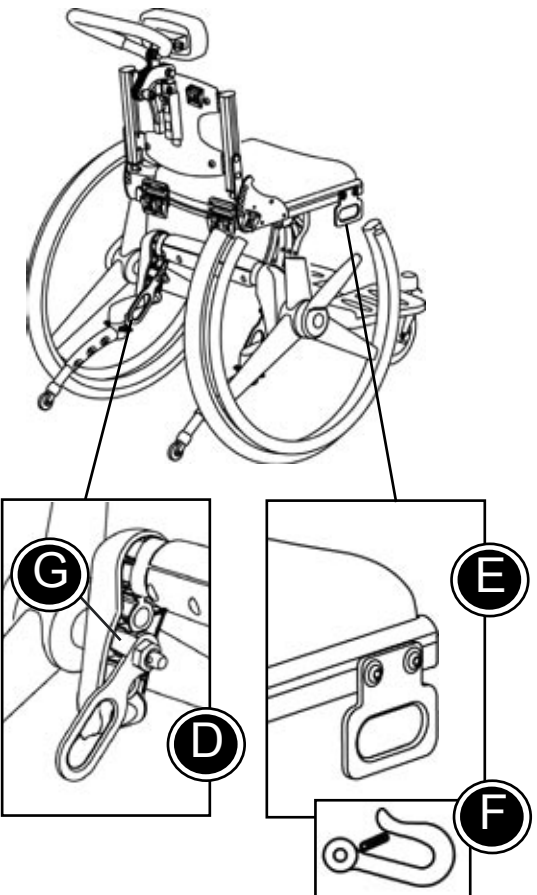
 Please be aware of the left/right transport fittings. The small side wing (G) must be fixated in the wheel fitting.

- * Mount the fittings (E) with the symbol (F) on the front of the chair. Click the fittings into the holes in rails and fasten the screws. The fittings are now mounted correctly.
- * Secure the wheelchair to the vehicle, using the 4-point strap-type tiedown system. Use either a hook or a strap in the fittings (D) and (E).

 Use a 4-point strap-type tiedown system approved according to ISO 10542-2.

Use the 3-point belt to secure the user. The belt restraints should be adjusted as tightly as possible, consistent with user comfort. Further the belt webbing should not be twisted when in use.

Please find the Cheetah manual on www.r82.com

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