Monteringsvejledning (DK)

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

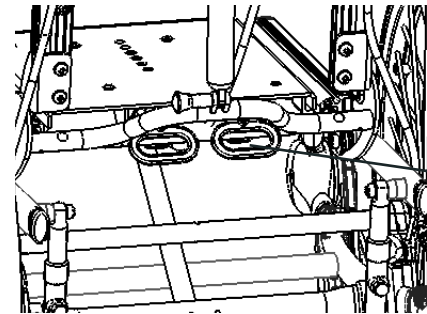
Koko 3+4

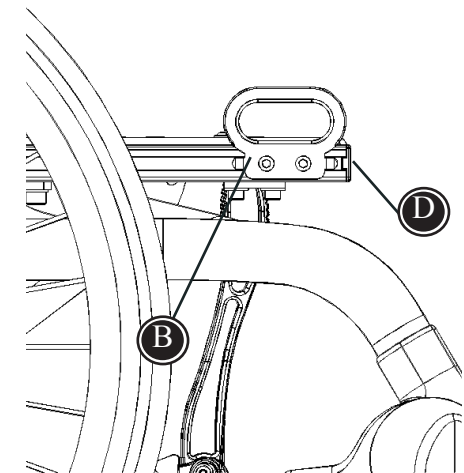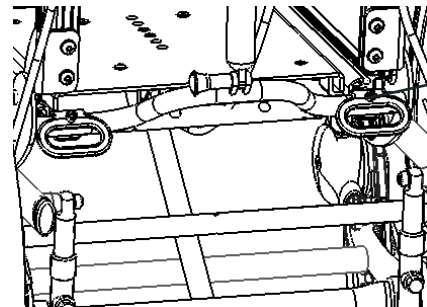Mounting instruction

Mounting transport fitting on Combi Frame with centre mounted foot support

Placing the Combi Frame in the vehicle:

1. By use of a 4-point strap system, the frame is secured in the vehicle. (Follow instructions from the manufacturer of the vehicle).

 A 4-point strap-system, approved according to ISO 10542-2, must be used.

 Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

On the rear of the frame, fitting A:

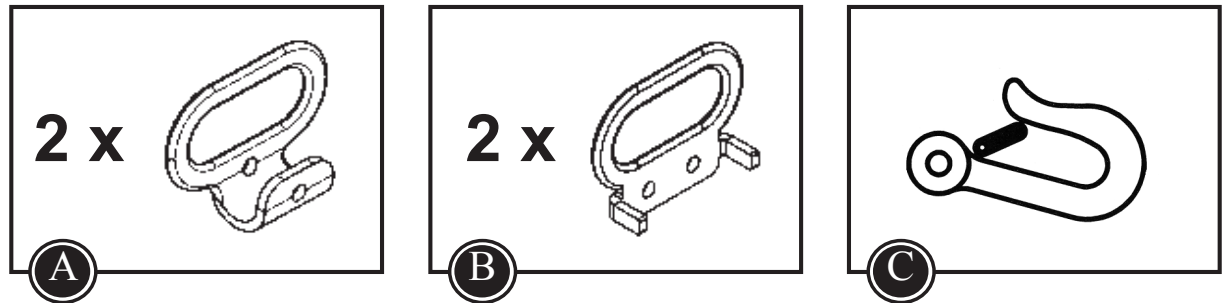
2. * Remove the plugs.
* Place the fittings (A) on the rear bar.
* Mount with the enclosed 6 mm long bolts and nuts.
(Note: the different positions for the size 1 & 2 and the size 3 & 4 - see drawings).

On the front of the frame, fitting B:


3. * Remove the caps (D) in the side rails.
* Slide the thread block into the side rails.
* Mount the caps (D) again.
* Place the fitting (B) over the thread block.
* Tighten with the enclosed 6 mm. bolts and washer through the holes in the thread block. It is important that the 'legs' on the fitting (B) is placed in the holes in the side rails. The fitting (B) can be turned upwards (as on the drawing) or downwards if you wish.

4. The wheelchair is fastened in the vehicle by use of a 4-point strap-system. Either a hook or a strap can be used for fastening the fittings (A) and (B).



Stick the enclosed stickers (C) on the four fittings (A) and (B).

Size 1+2

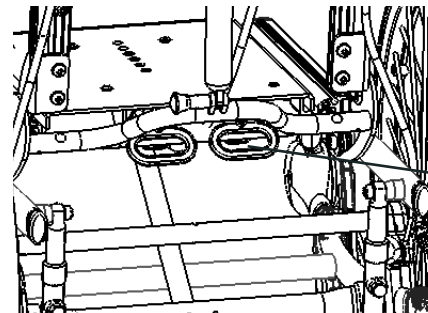Size 3+4

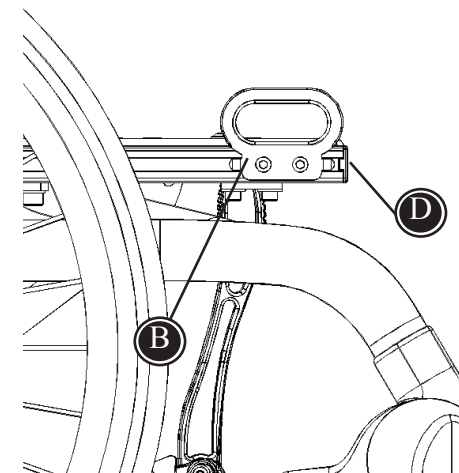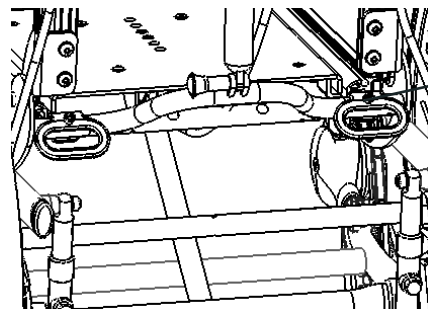Please find the Combi Frame manual on www.R82.com

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