Monteringsvejledning (DK)

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






Mounting instruction


Mounting power-tilt on the Buffalo

This is to show how you mount Power Tilt on a standard Buffalo. If your Buffalo was ordered with Power Tilt from the beginning it is already installed. Power Tilt is only available for Buffalo size 2 & 3.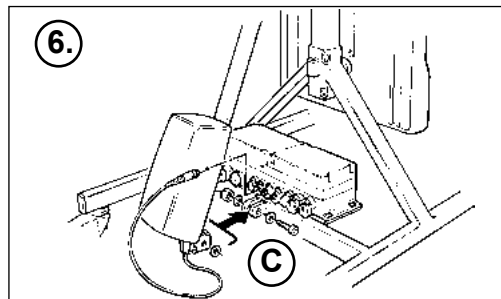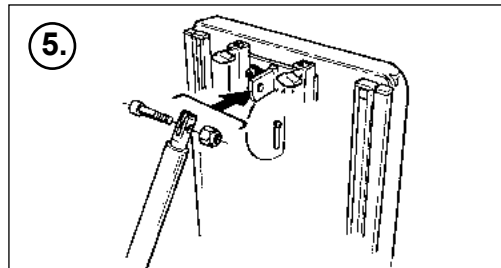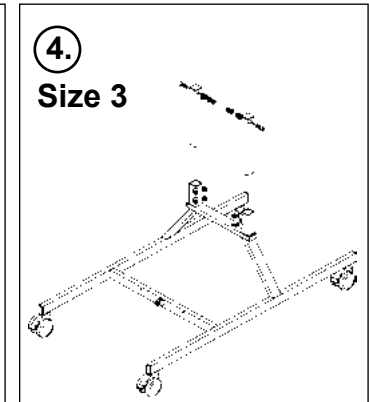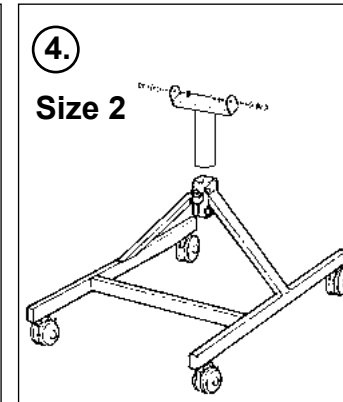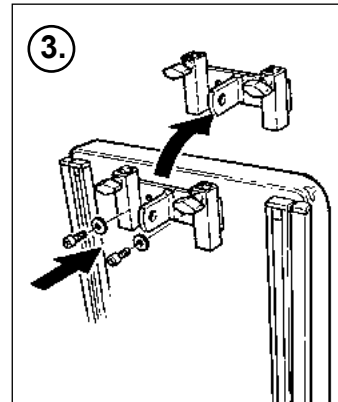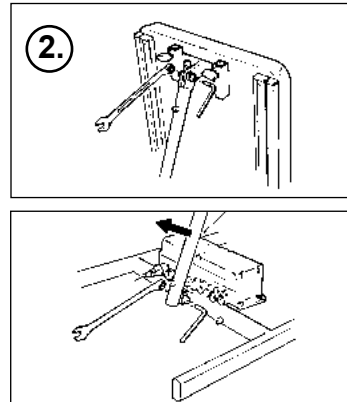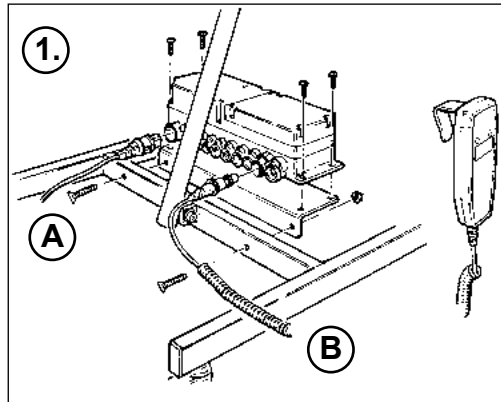

1. In order to attach the control box, fasten the plate in the holes first. Then attach the control box to the plate. A = power cable (220 V) B = remote control cable
2. Dismount the bar for the manual tilt. Use Allen key and 13 mm fork wrench.
3. Replace the fitting.
4. On the Buffalo size 2: Replace the center bar.
On the Buffalo size 3: Replace the two bars.
5. Place the bar for the power tilt in the new fitting. Use Allen key and 17 mm fork wrench.
6. Place the bar for the power tilt in the fitting below. Then connect cables from the electric motor and the remote control to the control box. C = control box cable

Charging the batterybox: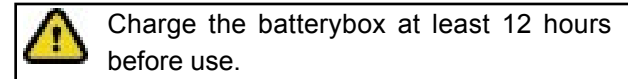

Charge the batterybox at least 12 hours before use.

- * When the battery box is plugged it starts charging.

It is not possible to charge the batteries too much, but do not let them discharge completely. Charge the Buffalo every 24 hours. The power angling of Buffalo is still possible during the charging.

Change the batteries within a period of 4 years.