---

## Monteringsvejledning (DK)

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# Mounting instruction

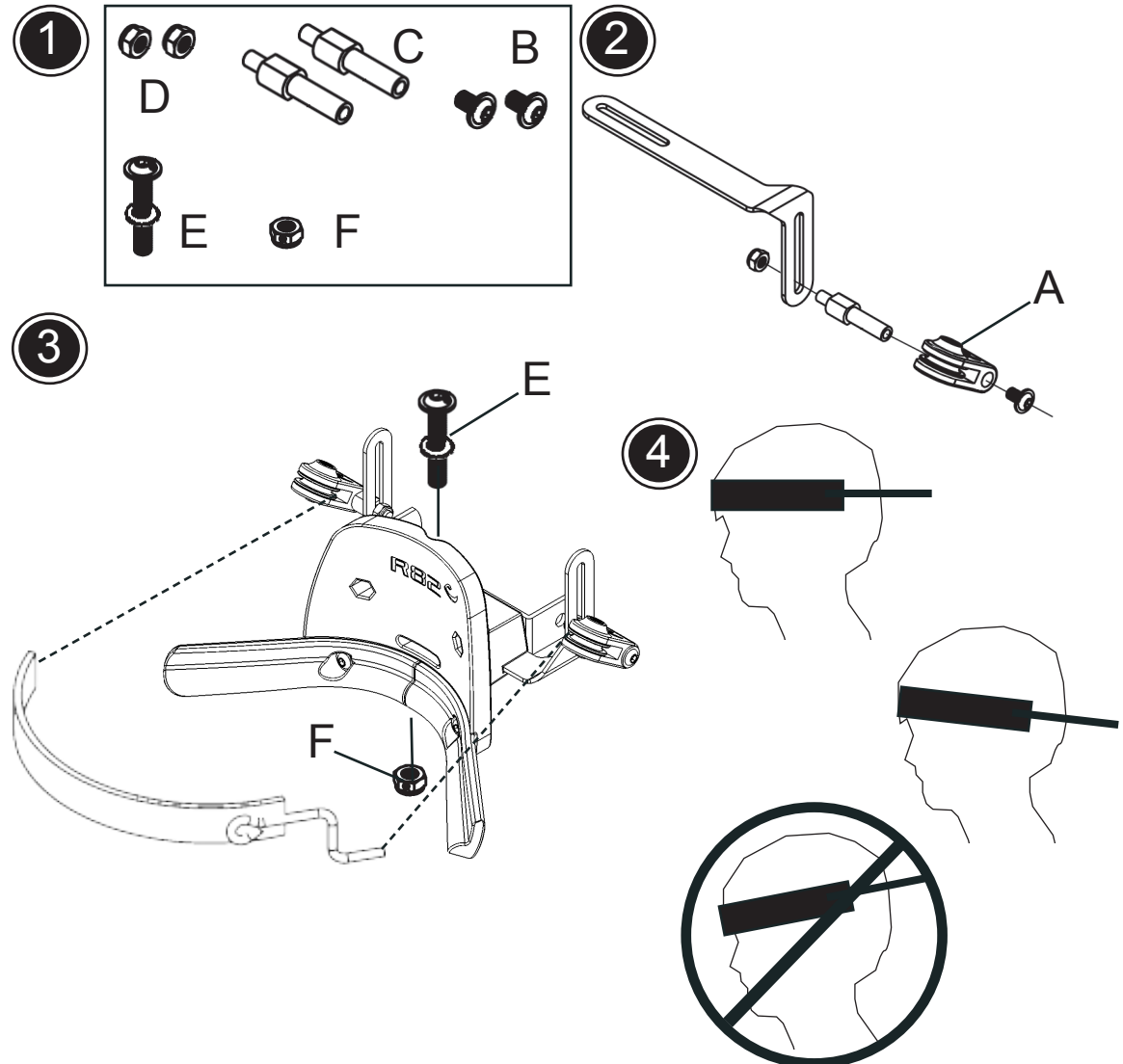
## How to mount head band on the anatomic head support

1. The package includes screws and nuts for mounting of the head band.
2. The holder for the head band is mounted on the fitting (A) by using the enclosed screw (B), the distance piece (C) and the nut (D). Use 4 mm Allen key and a 10 mm wrench to do the mounting.
3. Mount the head band fitting on the head support. Mount the two fittings from each side of the head support and adjust the width. Use the enclosed screw (E) and the nut (F) to secure the fitting to the head support. Make sure the screw is mounted through the slots in both head band fittings.
4. It is important that the head band is tightened against the forehead of the user with sufficient tension to prevent slippage off the forehead. The tension is adjusted at the rear of the headband by holding the sliding mechanism with one hand and adjusting the cord with the other.

The angle of pull on the headband should be horizontal or below the forehead as illustrated. The sub-occipital support on the headrest should be positioned below the occiput to provide support to the head when the headband is in use.

Note, that the head band fitting (2) can be turned in different ways to fulfill this need.

 Only to be used as head band and under supervision. Incorrect use may cause injury or suffocation.

 Warning - cannot be used in place in transportation.

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