UB200626B

etac®

Monteringssats Bygel

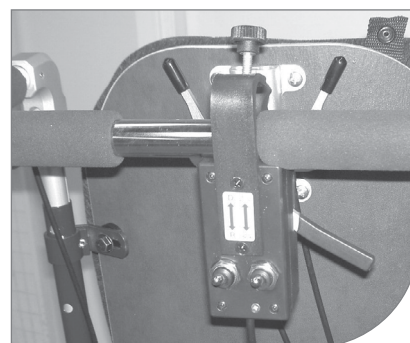
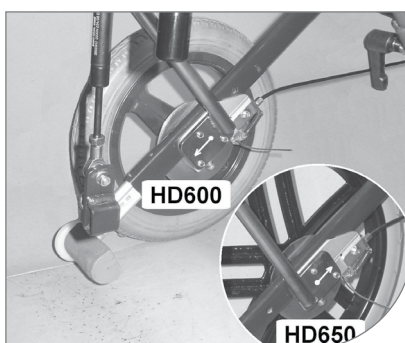
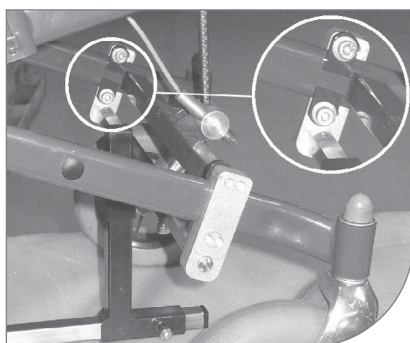
Monteringssett Bøjle

Assembly kit Bow

75018.3 2022-08-15

Rullstol, Tillverkare:
HD600-650, 38 - 45cm, Hantverksdesign

Monteringsanvisning

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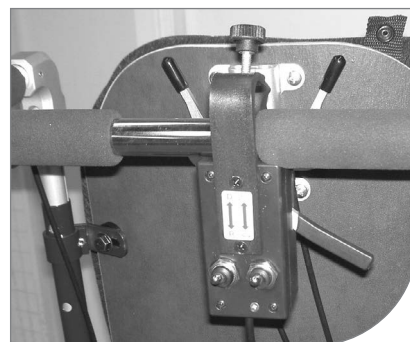
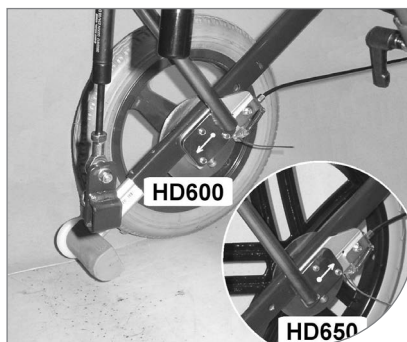
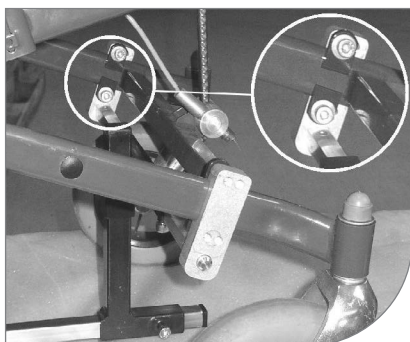
Mounting Instructions

Lille Viking 2000 is mounted on to the wheel chair after being adjusted to the user.

Front

The front brackets are placed as close behind the cross bar as possible. The bracket is fitted on the outside of the frame, with the hole pointing downwards and is then fastened with the bolts in the rear holes. The cross bar shall be placed with the centered tube down, facing backwards. Remember to check if the cross bar in the front centre section can move freely without being in contact with other parts.

Rear

The rear mounting arms is placed inside of the chair's wheel bracket. The newer HD-chair models have a cross bar which has to be removed before mounting. First remove the wheels so the chair will stand at the anti tip. Then mount the arms, those at the HD600 is turning with the plate backwards, and those at the HD650 with the plate forwards. The mounting angles are pushed into the pressure part, the bow's stoppers are put on, and the whole arrangement is placed in the mounting arms so that

the pressure part is in a height of about 49 cm above the ground. Lille Viking supporting motor is installed underneath the chair and the pulling bar is adjusted so that the driving wheels are on parallel lines with the wheelchair's rear wheels. The pressure is adjusted so that Lille Viking 2000 is lifted free off the ground during disengaging and the gas spring is pushed about halfway together when the handle is in the lowest position.

Check that the space between the chairs wheel brackets still are the same as before removing the cross bar. Remember to check that all functions are working properly after having installed Lille Viking 2000 onto the chair.

NB

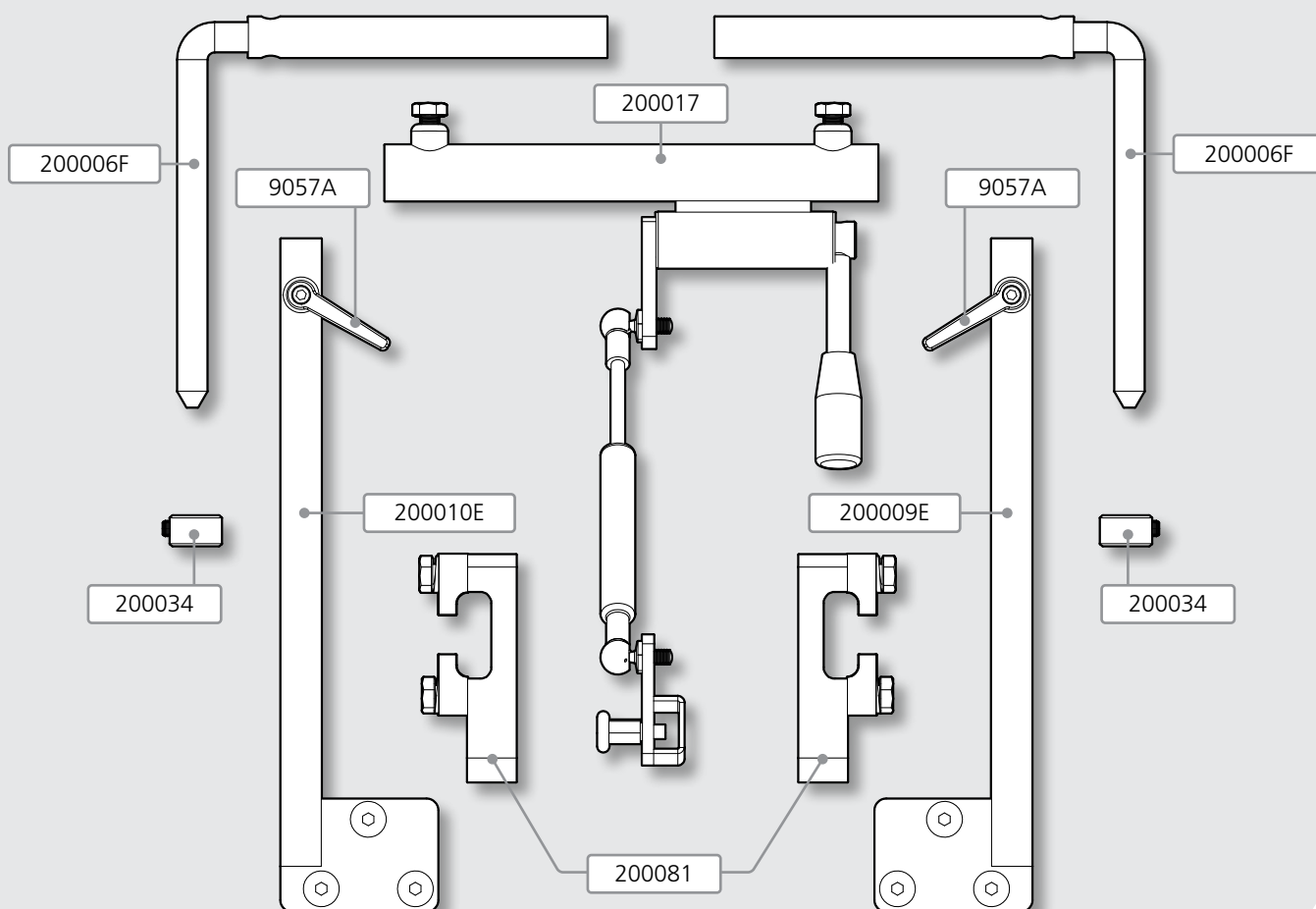
The wheelchair must always be equipped with tilt protection when operating using the auxiliary motor and this must be folded down. This is especially important for the HD650-chair, which tilts easier than the HD600.

SPEEDERBOX

We recommend that the wheelchair is fitted with a driving bow and separate brake handles.

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