UB200618A

Monteringssats Bygel

Monteringssett Bøjle

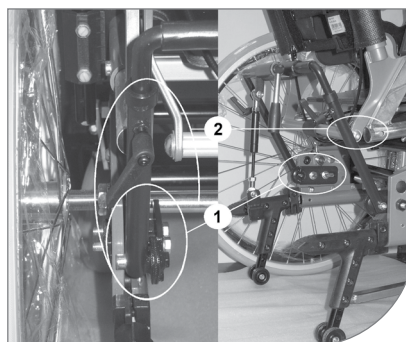
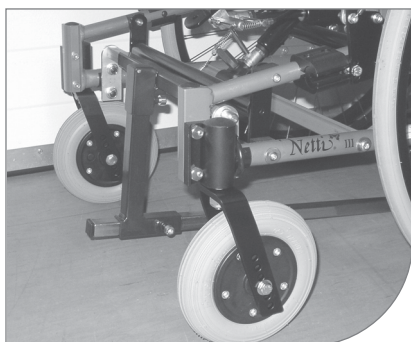
Assembly kit Bow

75071.2 2022-08-15

Rullstol, Tillverkare:

Netti, 38 - 45 cm, Alu Rehab

Netti Svipp, 38-45 cm. Alu Rehab

SV

Monteringsinstruktioner

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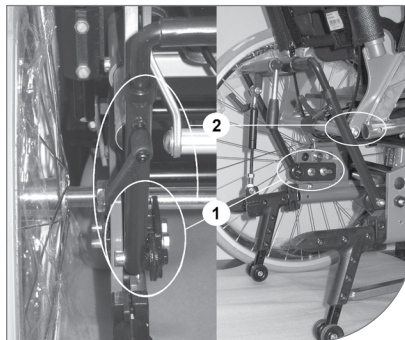
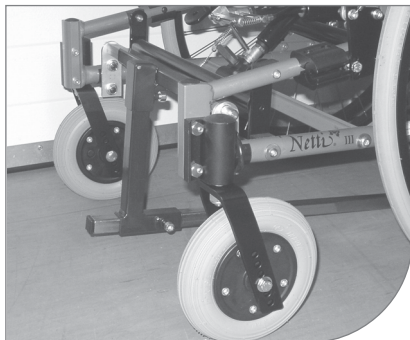
Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Netti, 38 - 45 cm, Alu Rehab
Netti Svipp, 38-45 cm. Alu Rehab

ENG

Mounting Instructions

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user.

The wheelchair has to have these frontwheels:

7x 1 $\frac{3}{4}$ - front wheels, Alu Rehab part no. 22193

Front

The front brackets are fitted right behind the front wheels, at the lowest horizontal tube, so that the holes for the telescopic mounting bars are pointing upwards. At small chairs the brackets should be fitted outside the frame.

The cross bar centre section must be turned so the pull bar bracket is mounted in front, and the finger screw is pointing downwards (see photo for).

Rear

The steel brackets are mounted in the two hindmost holes in a row, and are turned with the cog washers outwards. See details 1 at the photo.

The tubes are to be mounted at the cog washers, so they pass closely behind the seat's length adjustment – details 2. The bow system is adjusted in place, not hindering the anti tilt- and seat functions.

NB. The anti tilt might be adjusted backwards to keep the functionality.

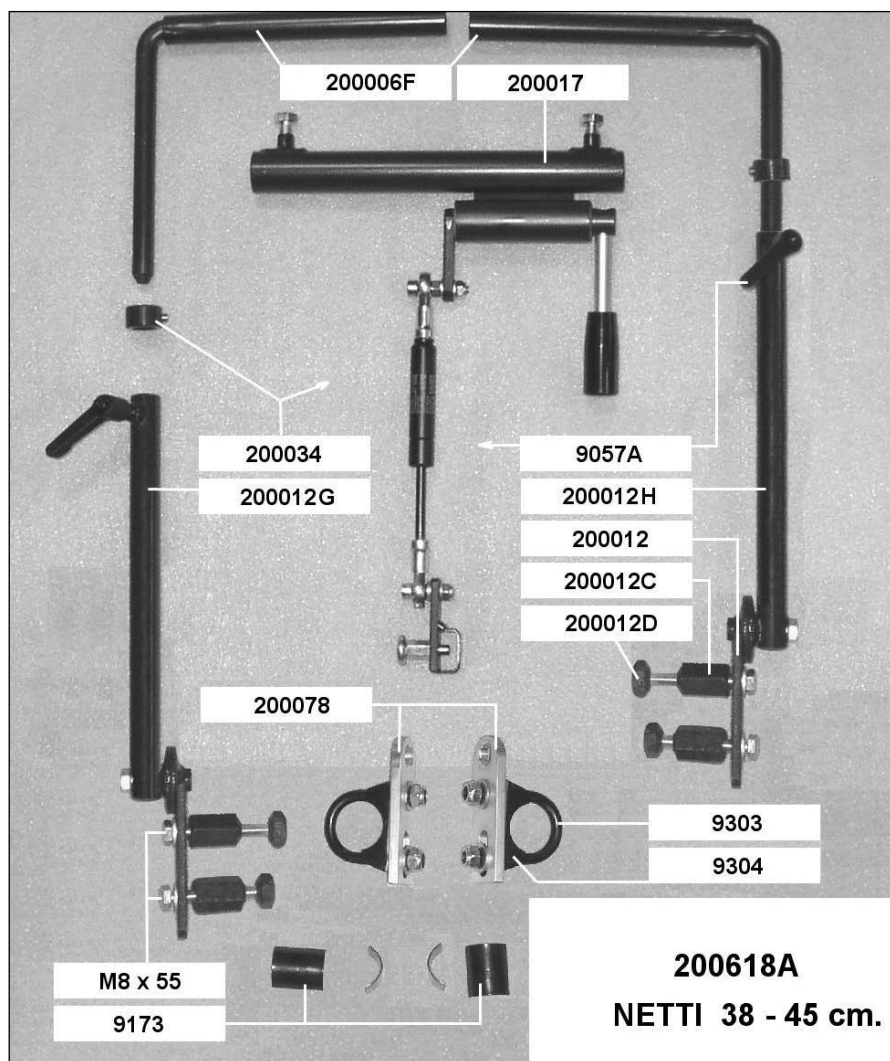
OBS

Remember to check that all functions are working properly after installing Lille Viking 2000 on to the chair.

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ENG

Art. no.	Description	Qty
UB9057A	Latch lever M6 x 12	2
UB9173	Tube protector mounting bracket	4
UB9303	Mounting clamp bow 28 mm	2
UB9304	Mounting clamp basin 28 mm	2
UB200006F	Mounting angle long	2
UB200012	Steel bracket with cog washer	2
UB200012C	Spacer	4
UB200012D	Nut	4
UB200012G	Mounting arm, 200 mm right	1
UB200012H	Mounting arm, 200 mm left	1
UB200017	Pressure part complete	1
UB200034	Bolt for bow	2
UB200078	Fittings bracket front round tubes 22-32 mm	2