

UB200624M

Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75072.2 2022-08-15

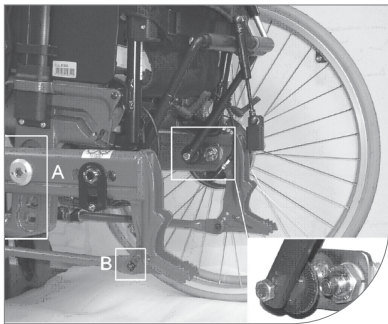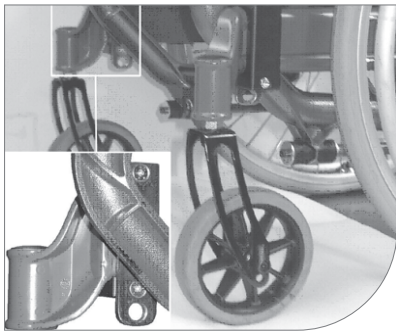Rullstol, Tillverkare:
Next Comfort 50 – 55 cm, Etac AB, Sverige

max
150 kg

SV

Monteringsinstruktioner

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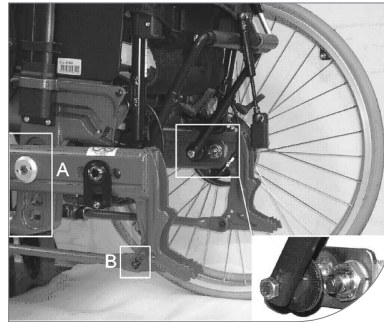
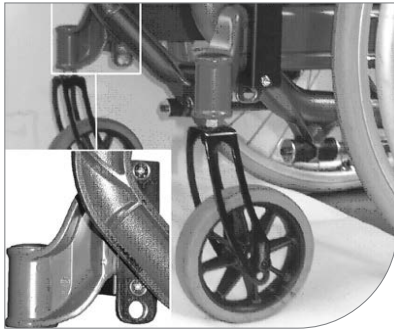
ENG

Mounting instructions

It is not allowed to fit Lille Viking 2000 to the wheelchair until it has been altered to suit the user. For space reasons, the seat cannot be placed in the lowest position - see position A in the picture.

Front

The front brackets are mounted as shown, so that they are fitted right behind the front wheels of the wheelchair frame. If the seat frame gets in the way of assembly, it can be tilted by loosening the screws at the rear of the longitudinal arms - see point B in the picture.

Rear

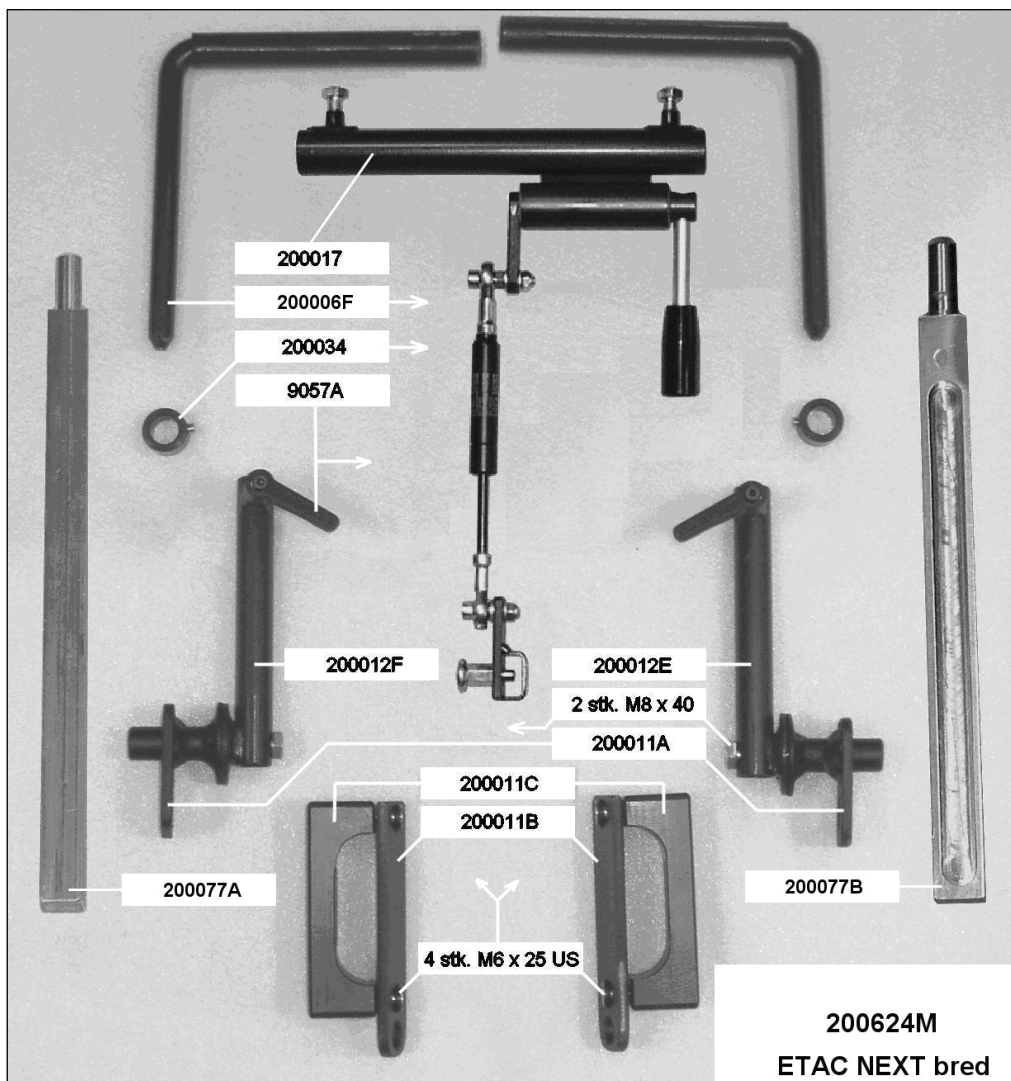
The rear brackets are assembled with the wheel hubs on the inside of the frame. The brackets are fitted so that the cog washers are facing each other, and so that the pin on the bracket is inserted in the hole BEHIND the wheel hub. The bracket may only be turned around so that the pin ends up in the front hole only when the hub is fitted in the hole at the very back of the frame, as shown in the picture.

The assembly arms are screwed to the brackets at an angle so that the pressure unit finishes up as close to the seat/back support as possible. The rings are attached to angled brackets which are then pressed into the pressure unit, and then the entire device can be attached to the assembly arms. The height is now adjusted so that the pressure unit finishes up about 49 cm above the floor level and the pressure unit is then adjusted so that it is horizontal with the gas spring in the middle. The connecting unit must still be attached to the Lille Viking bracket when the pressure unit has to be adjusted to a position where the gas spring is about half compressed when the handle is set to the lowest position. At the same time, ensure that the supporting motor is 2 to 3 cm above floor level during disengaging, and the handle is stable in both of these positions. When the final adjustment is made, you can fit the rings and retighten all screws.

Note! Remember to check that all functions are working properly after Lille Viking has been adjusted to fit the chair.

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ENG

Art. no.	Description	Qty
UB9057A	Latch lever M6 x 12	2
UB200006F	Mounting angle long	2
UB200011A	Mounting bracket rear Next	2
UB200011B	Mounting bracket inner Next	2
UB200011C	Mounting bracket outer Next	2
UB200012E	Mounting arm 140 mm right	1
UB200012F	Mounting arm 140 mm left	1
UB200017	Pressure part compl.	1
UB200034	Bolt for bow	2
UB200077A	Telescopic mounting bar 250 mm with rounded tip	1
UB200077B	Telescopic mounting bar 250 mm with precut track	1

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Etac Supply Center AB
Next Comfort 50 – 60 cm, Etac AB, Sverige