UB200624N

etac®

Monteringssats Bygel

Monteringssett Bøjle

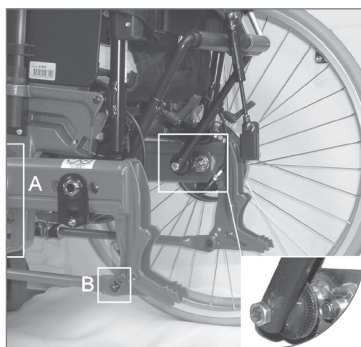
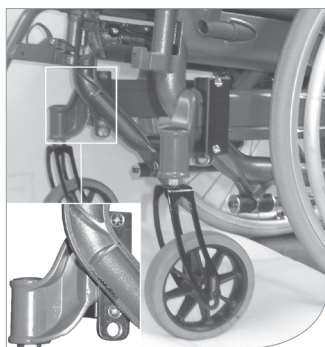
Assembly kit Bow

75073.2 2022-08-16

Rullstol, Tillverkare:

Etac Next 37,5 - 47,5 cm, Etac

Etac Next Comfort, 37,5 - 47,5 cm, Etac

max
150 kg

SV

Monteringsanvisning

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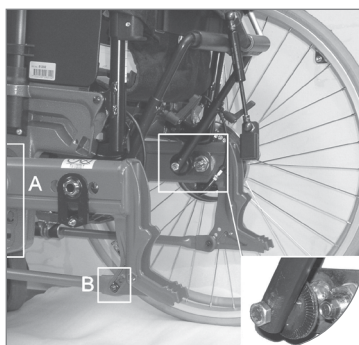
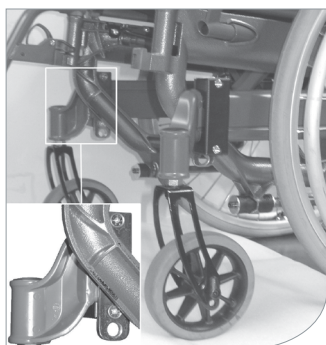
ENG

Mounting Instructions

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user. Considered the space, the seat should not be placed at the lowest position – see A at the photo below.

Front

The front brackets are fitted as illustrated, right behind the wheelchair's front wheels on the chair's basic frame. If the seat frame is in the way for the assembly, it can be tilted, when the screws at the back of the parallel arms are removed – see B at the photo below.

Rear

The rear brackets are mounted together with the wheel hubs internally on the frame. The plates are placed, thus the cog disks turn against each other and the peg on the plate is at the hole behind the wheel boss. Only if the wheel hub is placed at the frame's rear hole, the plate is turned, thus the peg is at the hole in front – see photo. The assembly arms are angle mounted on the plates so the pressure part fits as close to the seat back as possible. The bolts on the bow are placed onto the angled mounting bars which are pushed into the pressure part, and the whole assembly is fitted in the mounting bars. The height is adjusted so that the pressure part is about 49 cm above ground and the pressure part is adjusted – partly so that it is horizontal and partly so that the gas spring is in the centre.

While the connecting part is fitted on to Lille Viking 2000's brackets, the pressure part is readjusted so that the gas spring is pushed about halfway together when the handle is in the lower position.

At the same time, you will control that Lille Viking 2000 is lifted about 2-3 cm off the floor during disengaging, and that the handle stays stable in both positions. When the final adjustment has been made, the bolts on the bow are adjusted and all screws are fastened properly.

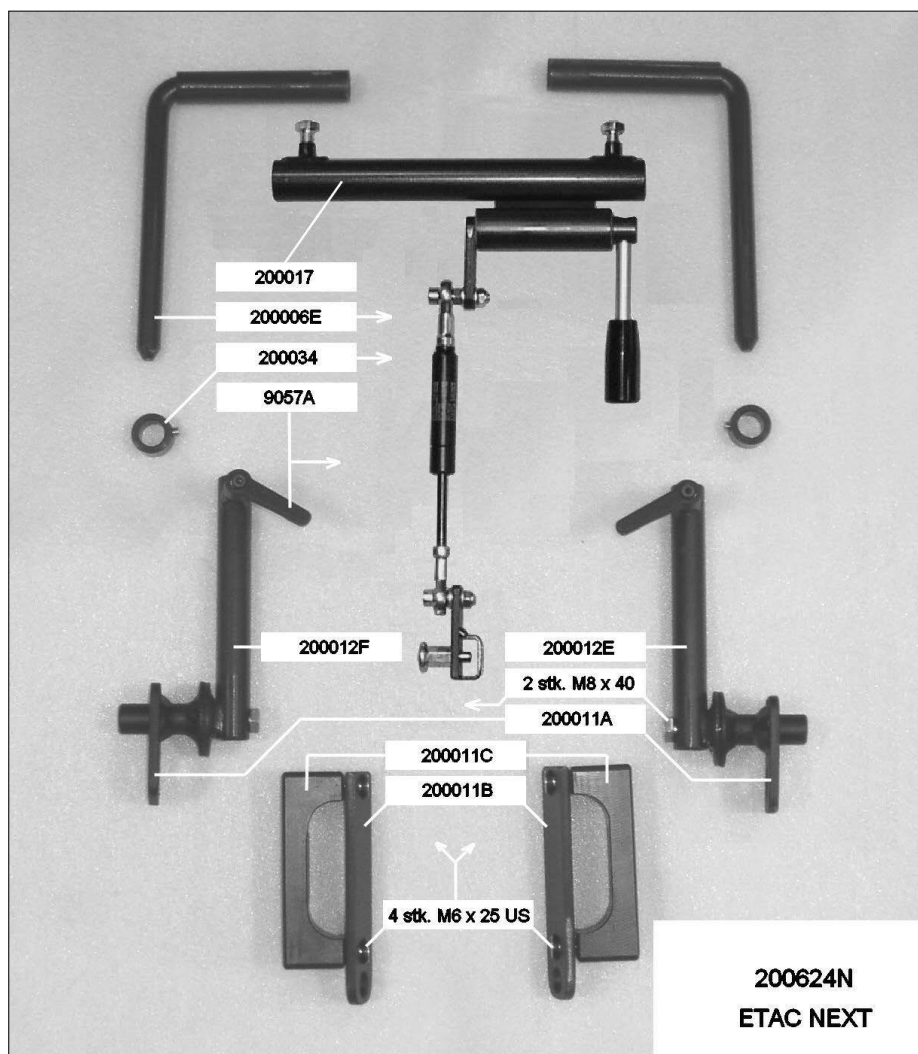
OBS.

Remember to check that all functions are working properly after installing Lille Viking 2000 on to the chair.

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ENG

Artnr.	Description	Qty
UB9057A	Latch lever M6x12	2
UB200006E	Mounting angle short	2
UB200011A	Mounting bracket rear Next	2
UB200011B	Mounting bracket inner Next	2
UB200011C	Mounting bracket outer Next	2
UB200012E	Mounting arm 140 mm right	1
UB200012F	Mounting arm 140 mm left	1
UB200017	Pressure part compl.	1
UB200034	Bolt for bow	2