
UB200624S

Monteringssats Rem
Monteringssett Rem
Assembly kit Strap

75074.2 2022-08-16

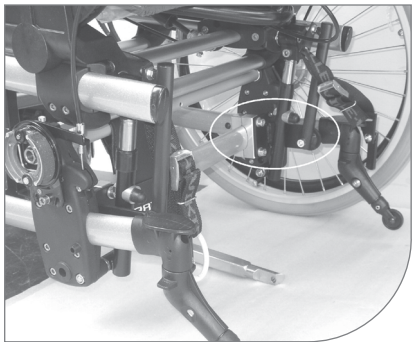Rullstol, Tillverkare:
Cougar, 35 - 50 cm, R82
Combi Frame, 35 - 50 cm, R82

SV

Monteringsinstruktioner

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Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Cougar, 35 - 50 cm, R82
Combi Frame, 35 - 50 cm, R82

ENG

Mounting instructions

Lille Viking 2000 is mounted on to the wheel chair after being adjusted to the user.

Front

The front fittings are mounted on to the inside of the lower tubes, right behind the plastic strengthening so the hole of the telescopic mounting bars are facing downwards. Make sure the inclined side of the aluminium fittings are turned so the black telescopic mounting brackets are on parallel lines. The cross bar in the front centre section is turned in order to mount the angled bracket in front with the screw facing up.

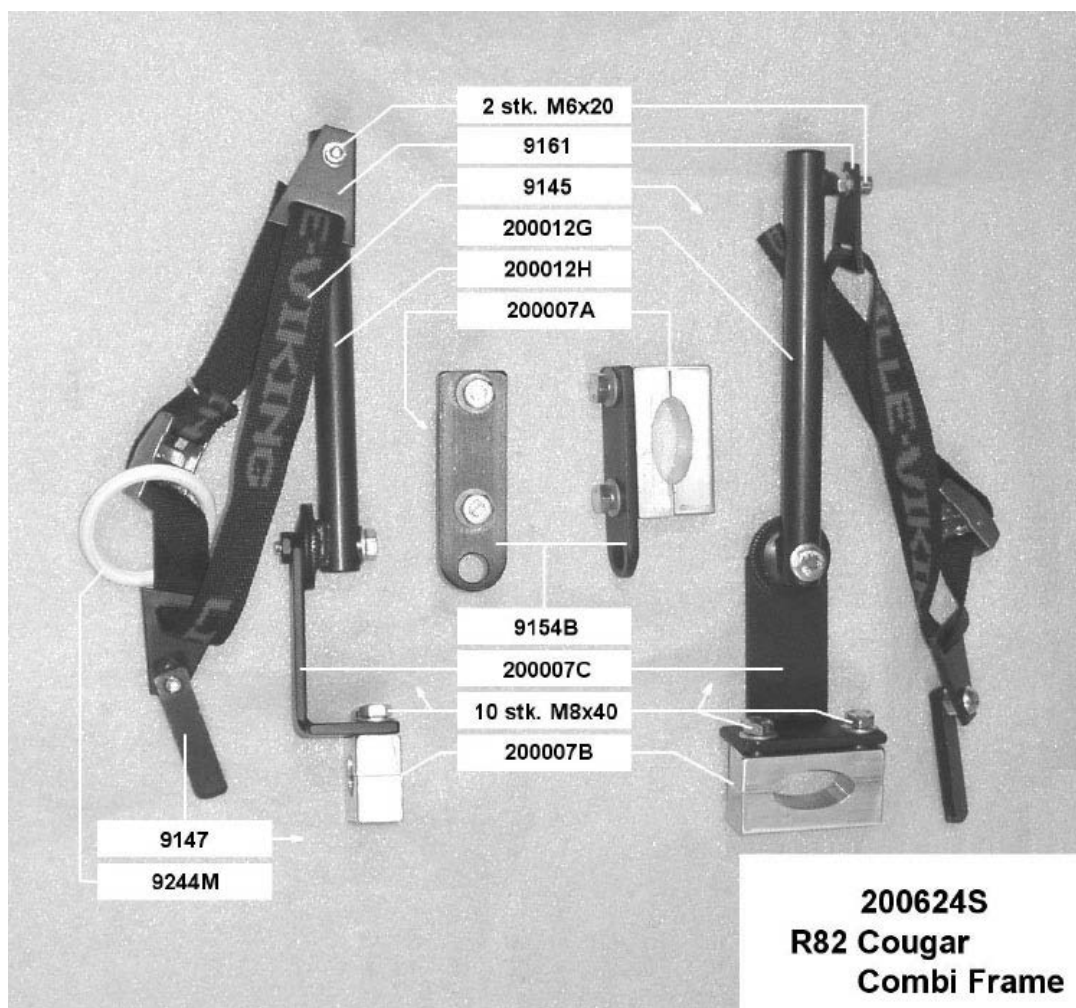
Rear

The fittings are mounted on to the horizontal tube which is straight across underneath the chair. The brackets are mounted on to the horizontal tube which is straight across beneath the chair (see details on photo). On narrow chairs, the brackets are mounted far out to the side against the plastic strengthening and the mounting brackets are turned so they are outside of the tubes to adjust the seat height. On the wider chairs, the mounting brackets can be placed further in on the tube, and the brackets are placed inside of the height adjustment. The tubes are mounted on to the cog washers pointing almost straight up, without being in contact with the chair. Lille Viking 2000 is then placed in a way that makes the wheel stand directly behind the crossing bar. Remember to leave enough space for the wheels not to be against the bar when the supporting motor is being disengaged. When the final position is found, the straps are adjusted to make the supporting motor lift about 3 cm above ground during disengaging.

Remember to check all functions after installing Lille Viking 2000 on to the chair.

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