
UB200626

Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75075.2 2022-08-16

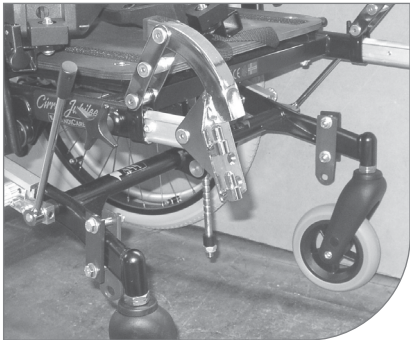Rullstol, Tillverkare:
Cirrus HandiCare, Cirrus Jubilee HandiCare, Cirrus 4 HandiCare

SV

Monteringsanvisning

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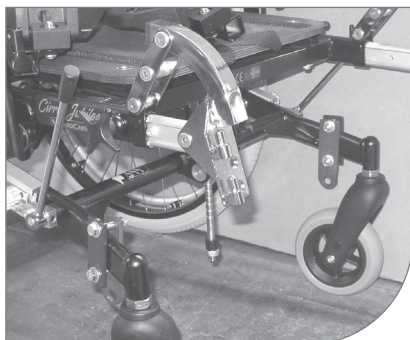
Cirrus HandiCare, Cirrus Jubilee HandiCare, Cirrus 4 HandiCare

Mounting instruction

Lille Viking 2000 is mounted on to the wheel chair after being adjusted to the user.

Front

The front fittings are put as close as possible behind the front wheels, inside of the frame and with the hole facing downwards. NB: The fittings are marked with "R" – Right and "L" – left and should be placed in parallel positions. The cross bar in the front centre section is placed with the centre tube downwards, facing forward, so that it is not in contact with the edge of the seat. Make sure the front wheels turn freely.

Rear

The rear fittings are placed on the top of the sideways tubes, right beneath the cross bar, so far up that it will not be a problem for the anti-tilt. The fittings are put as follows: - on broad chairs they should face each other – on narrow chairs they should face away from each other. The mounting arms are screwed tightly on to the cog washers, so they are in a backward inclination, with the screws on the back. The mounting angles are pushed into the pressure part, the clamp bolts are fitted and the whole arrangement is placed into the mounting brackets so that the pressure part is about 49 cm above ground. The supporting motor is placed beneath the chair and the pulling bar is adjusted, to make the driving wheels on parallel lines with the wheel chairs' rear wheels. The pressure part is set so that Lille Viking 2000 can be lifted freely from the floor during disengaging and the gas spring is pushed about halfway together when the handle is in lower position.

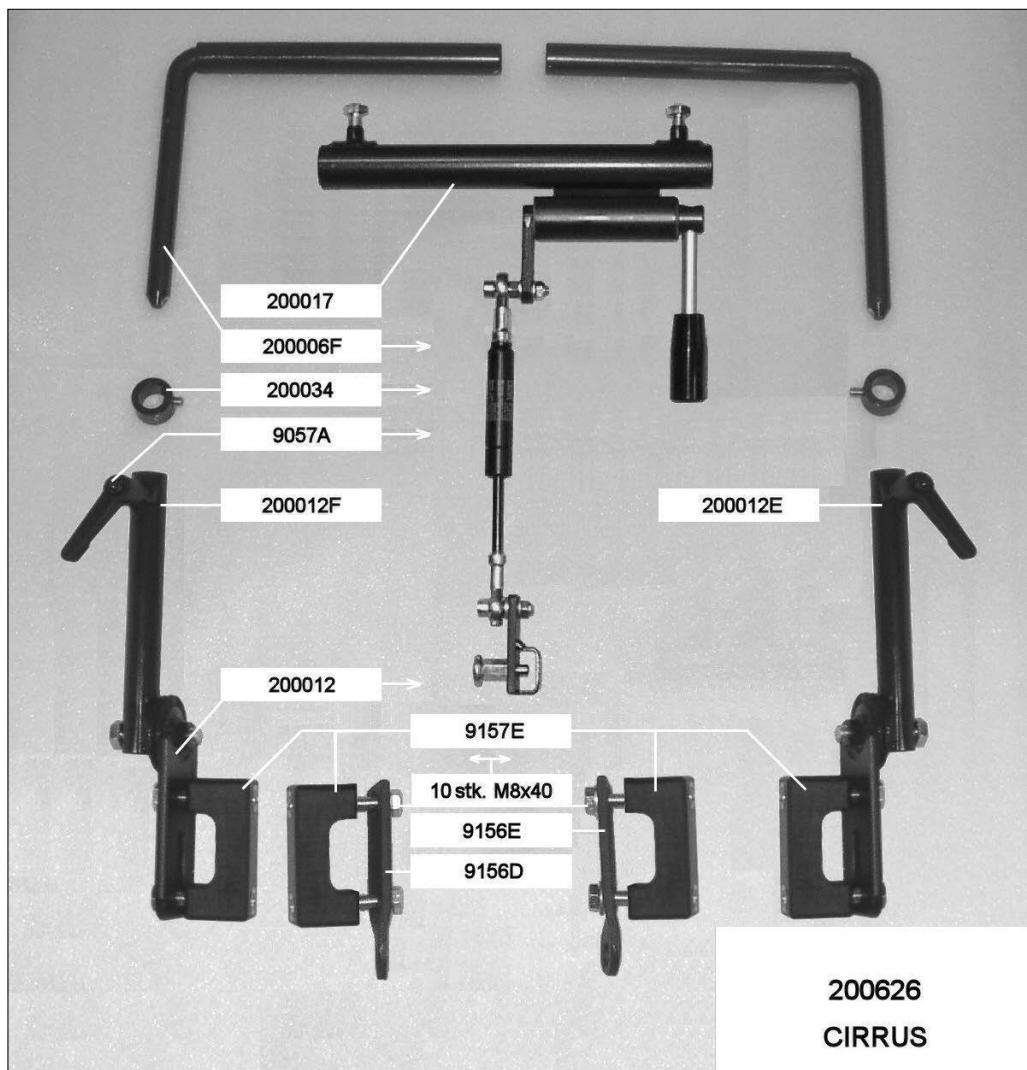
OBS!

Remember to check all functions after fitting Lille Viking 2000 on to the wheelchair.

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Etac Supply Center AB