UB200626D

Monteringssats Bygel

Monteringssett Bøjle

Assembly kit Bow



75077.2 2022-08-16

Rullstol, Tillverkare:

Impera 39-51cm, HandiCare

Legend 39-51cm, HandiCare

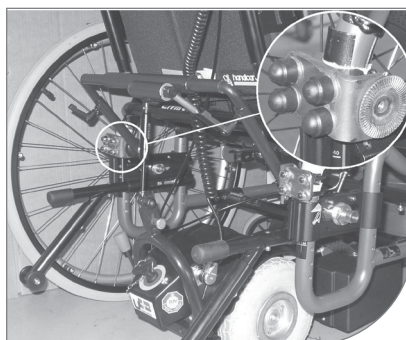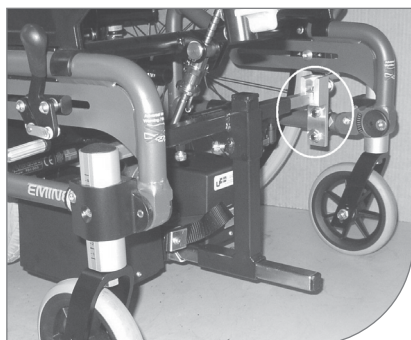Emineo 39-51cm, HandiCare

max
150 kg

SV

Monteringsinstruktioner

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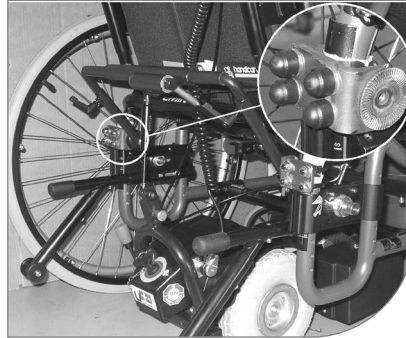
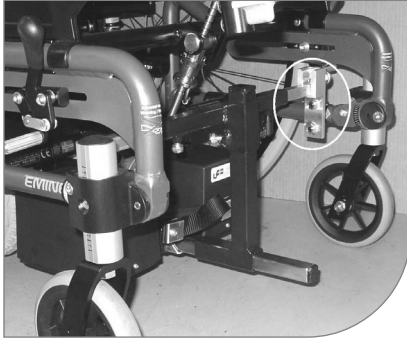
Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Impera 39-51cm, HandiCare
Legend 39-51cm, HandiCare
Emineo 39-51cm, HandiCare

ENG

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Lille Viking 2000 is mounted onto the wheelchair after being adjusted to the user.

Front

The front brackets are fitted right behind the front wheels, on the wheelchair's horizontal tube, preferably so that the holes on the telescopic mounting brackets face downwards. The cross bar in the front centre section is turned so that the angled bar is mounted in front and the finger screw is turned upwards.
NB. On 39 cm wide chairs, the brackets can be fitted on the outside of the frame, or the telescopic mounting bar which does not have a precut track, can be shortened 5-6 cm.

Rear

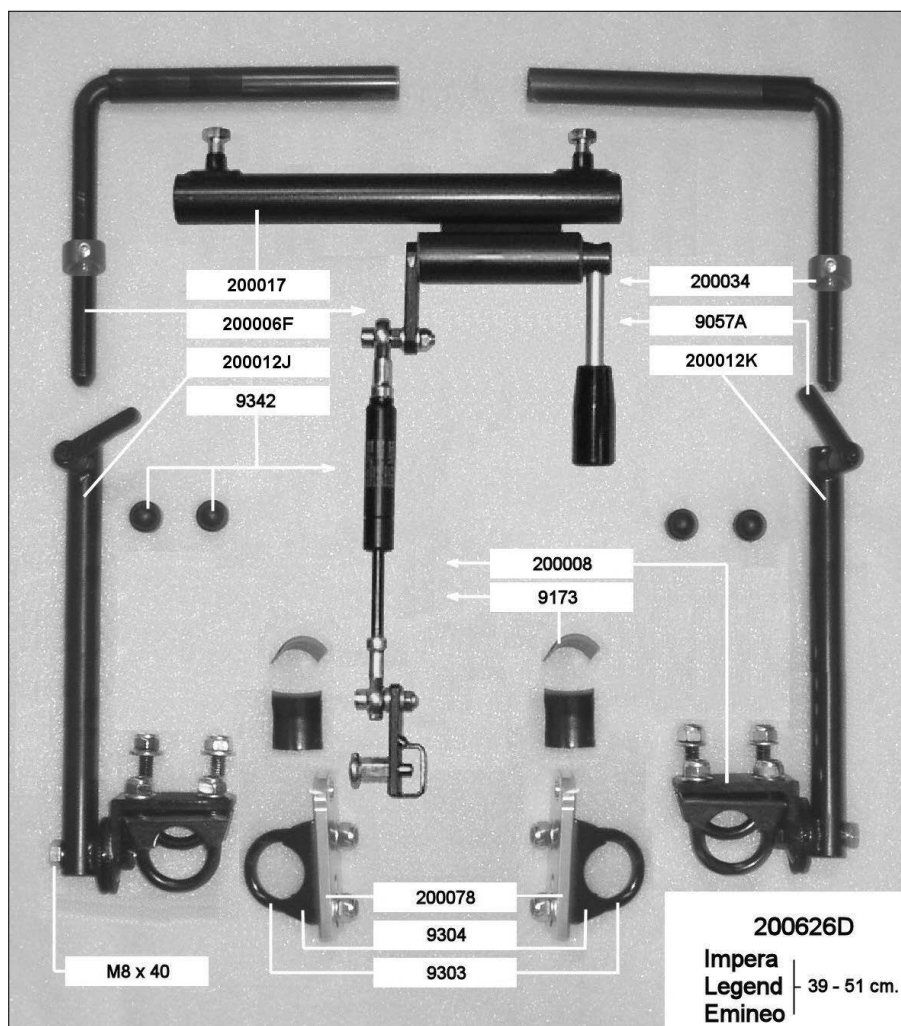
The steel brackets are mounted onto the horizontal tube beneath the wheel fittings, so that the cog washers are turned against each other. This makes it possible to use the anti-tilts without problems. If the wheelchair has been adjusted to a low seat height – with the wheel bosses placed high up on the frame – the steel brackets should be turned in order to make the cog washers face upwards (as illustrated).

The mounting bars are fitted in an upwards inclination on the cog washers, close to the back edge of the seat. The angled mounting bars are pushed into the pressure part, the bolts for the bow is put on and the whole assembly is fitted into the mounting bars so that the pressure part is 49 cm above ground. The supporting motor is installed underneath the chair and the pulling bar is adjusted. The driving wheels should now be on parallel lines with the wheelchair's rear wheels. The pressure part is adjusted so that Lille Viking 2000 is lifted free off the floor during disengaging and the gas spring is pushed about halfway together when the handle is in lower position.

Remember to check that all functions are working properly after installing Lille Viking 2000 onto the chair.

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ENG

Art.no.	Description	Qty
UB9057A	Latch lever M6x12	2
UB9173	Tube protector mounting bracket	12
UB9303	Mounting clamp bow 28 mm	6
UB9304	Mounting clamp basin 28 mm	6
UB9342	Nut protection M8	8
UB200006F	Mounting angle long	2
UB200008	Steel bracket rear Handicare	2
UB200012J	Mounting arm 300 mm right	1
UB200012K	Mounting arm 300 mm left	1
UB200017	Pressure part complete	1
UB200034	Bolt for bow	2
UB200078	Fittings bracket front round tubes 22-32 mm	2