UB200626G

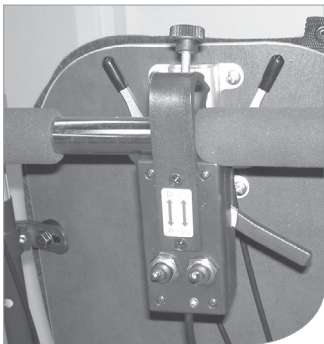
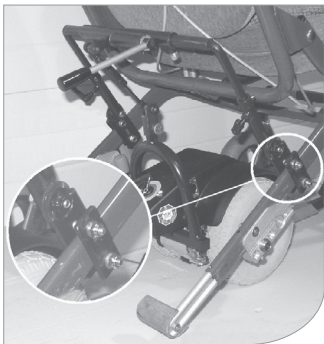
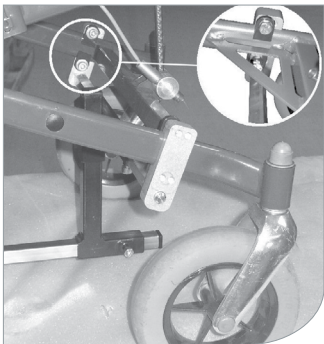
Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75078.3 2022-08-16

Rullstol, Tillverkare:
HD500 Humlan, 52 cm, Hantverksdesign AB

Monteringsanvisning

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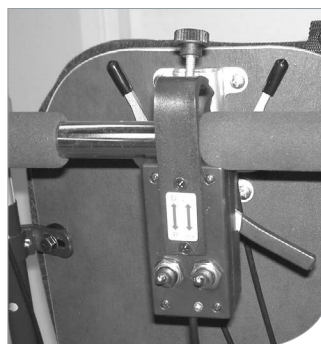
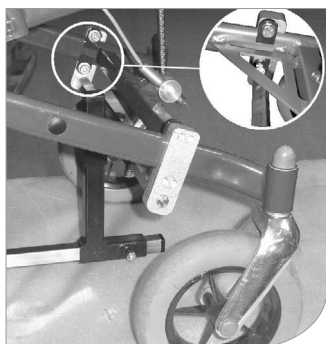
Mounting instructions

Lille Viking 2000 is mounted onto the wheelchair after being adjusted to the user.

Front

The front brackets are fitted as close behind the crossing bar as possible. The aluminium bracket is mounted on the outside of the frame, with the hole pointing downwards. The grey steel bracket is fitted as illustrated, and is fastened with the bolts in the rear holes. The cross bar in the front centre section is fitted with the centre tube downwards and somewhat backwards. Remember to control that the cross bar in the front centre section can move freely without being in contact with other parts.

Rear

The rear brackets are fitted at the top of the inclined tubes, right below the curve, but so far up that it will not be a problem to the anti-tilt. The mounting bars are screwed onto the cog washers, making them point out about 45 degrees backwards, with the fingerscrews on the underside. The angled mounting bars are pushed into the pressure part, the bolts for the bow are fitted and the whole assembly is fitted into the mounting bars.

The supporting motor is installed underneath the chair and the pulling bar is adjusted so that the driving wheels are on parallel lines with the wheelchair's rear wheels. The pressure part is adjusted so that Lille Viking 2000 is lifted free off the floor during disengaging and the gas spring is pushed about halfway together when the handle is in lower position. Remember to control that all functions are working properly after having installed Lille Viking 2000 onto the chair.

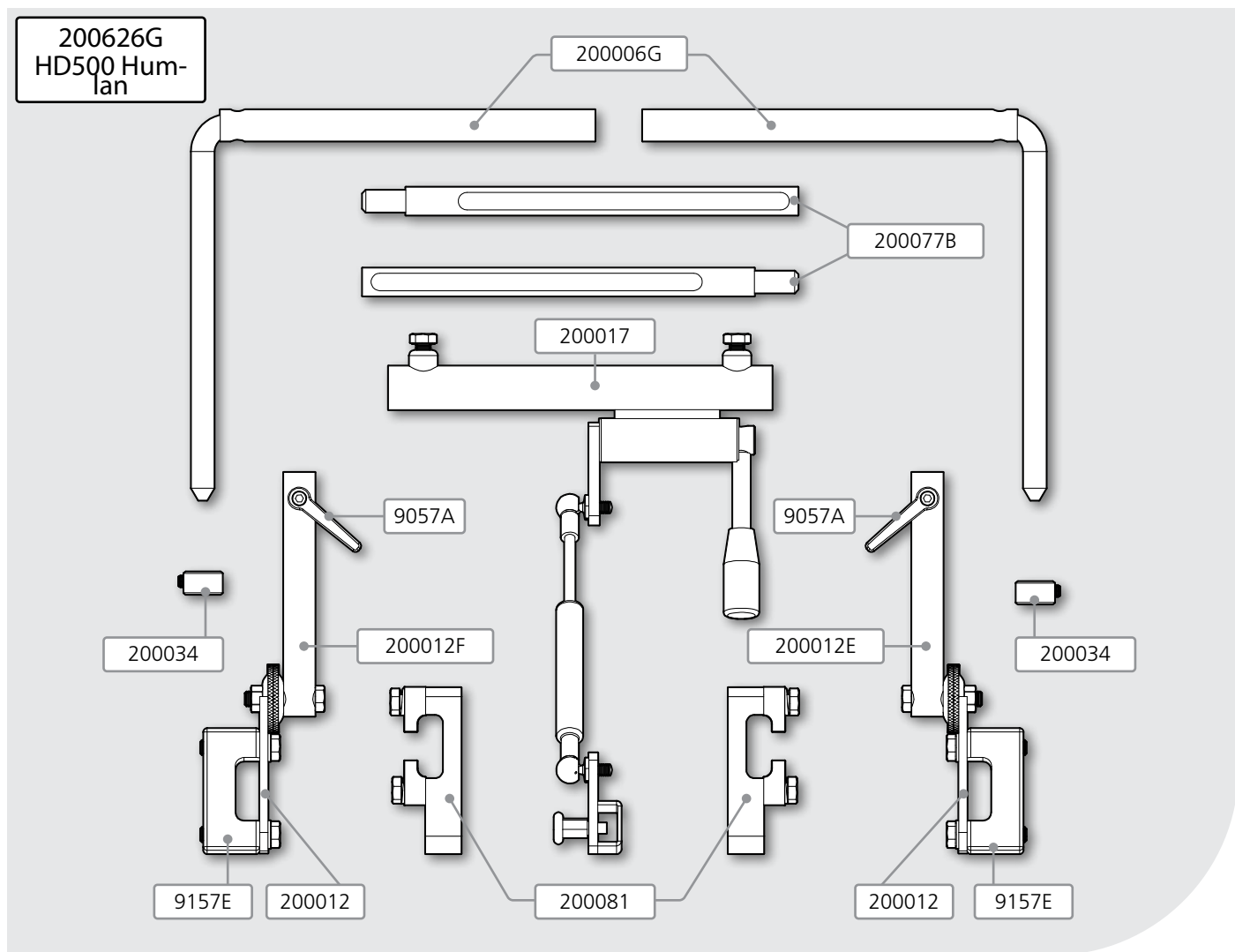
NB

The wheelchair must always be equipped with tilt protection when operating using the auxiliary motor and this must be folded down.

Speederbox

We recommend that the wheelchair is fitted with a driving bow and separate brake handles.

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EN

Item. no.	Description	Qty
UB9057A	Latch lever M6x12	2
UB9157E	Bracket rear HD/Cirrus	2
UB200006G	Mounting angle extra long	2
UB200012	Steel bracket with cog washer	2
UB200012E	Mounting arm 140 mm right	1
UB200012F	Mounting arm 140 mm left	1
UB200017	Pressure part compl.	1
UB200034	Bolt for bow	2
UB200077B	Telescopic mounting bar 250 mm with precut track	2
UB200081	Bracket front HD	2