UB200626H

etac®

Monteringssats Bygel

Monteringssett Bøjle

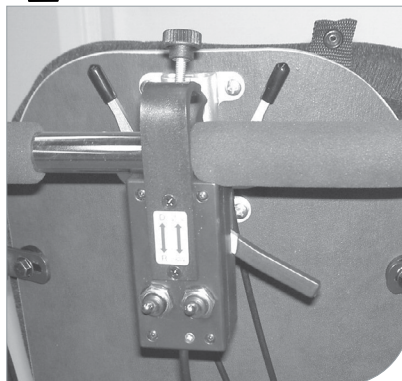
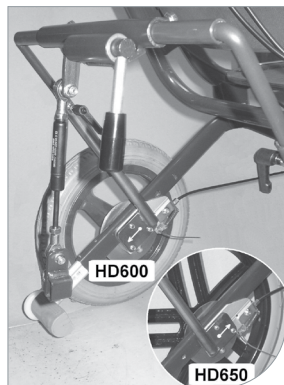
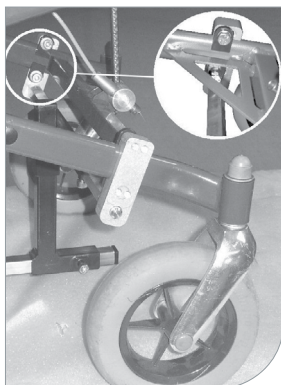
Assembly kit Bow

75079.3 2022-08-16

Rullstol, Tillverkare:
HD650 Humlan, 52cm, Hantverksdesign

MAX
150 kg
330 lbs

Monteringsanvisning

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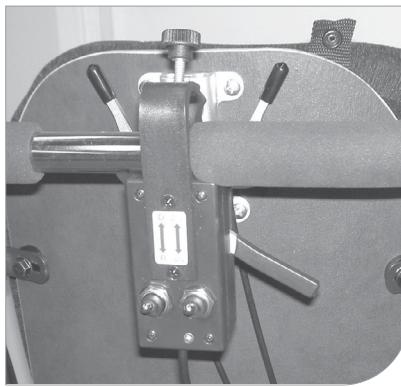
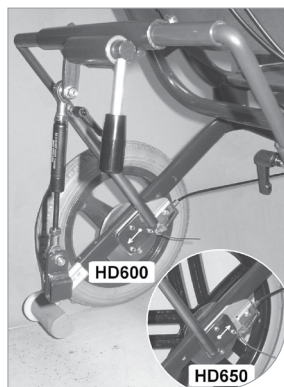
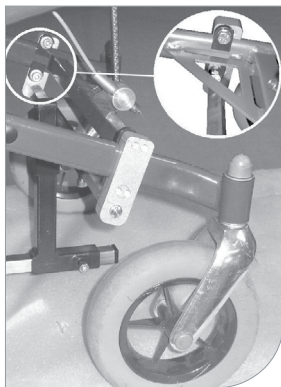
Mounting Instructions

Lille Viking 2000 is mounted onto the wheelchair after being adjusted to the user.

Front

The front brackets are fitted as close behind the crossing bar as possible. The aluminium bracket is mounted on the outside of the frame, with the hole pointing downwards. The grey steel bracket is fitted as illustrated, and is fastened with the bolts in the rear holes. The cross bar in the front centre section is fitted with the centre tube downwards and somewhat backwards. Remember to check that the cross bar in the front centre section can move freely without being in contact with other parts.

Rear

Remove the wheels, so the chair will stand at the anti tip. The Humlan-chair has a cross bar between the wheel brackets which has to be removed before mounting. Fasten the arms on the inside of the wheel brackets, so that the plate is pointing forward, like the HD650. The angled mounting bars are pushed into the pressure part, mount the bow stoppers, and fit the whole assembly into the mounting bars so that the pressure part is in a height of about 49 cm above ground.

Lille Viking 2000 support motor is placed beneath the chair, and the pull bar is adjusted, so that the driving wheels are on parallel lines with the wheelchairs' rear wheel. The pressure part is adjusted so that Lille Viking 2000 is free off the ground during disengaging, and the gas spring is pushed about halfway together when the handle is in the lowest position. Check that the space between the chair's wheel bracket is the same as before removing the cross bar. Remember to check that all functions are working properly after having installed Lille Viking 2000 onto the chair.

NB.

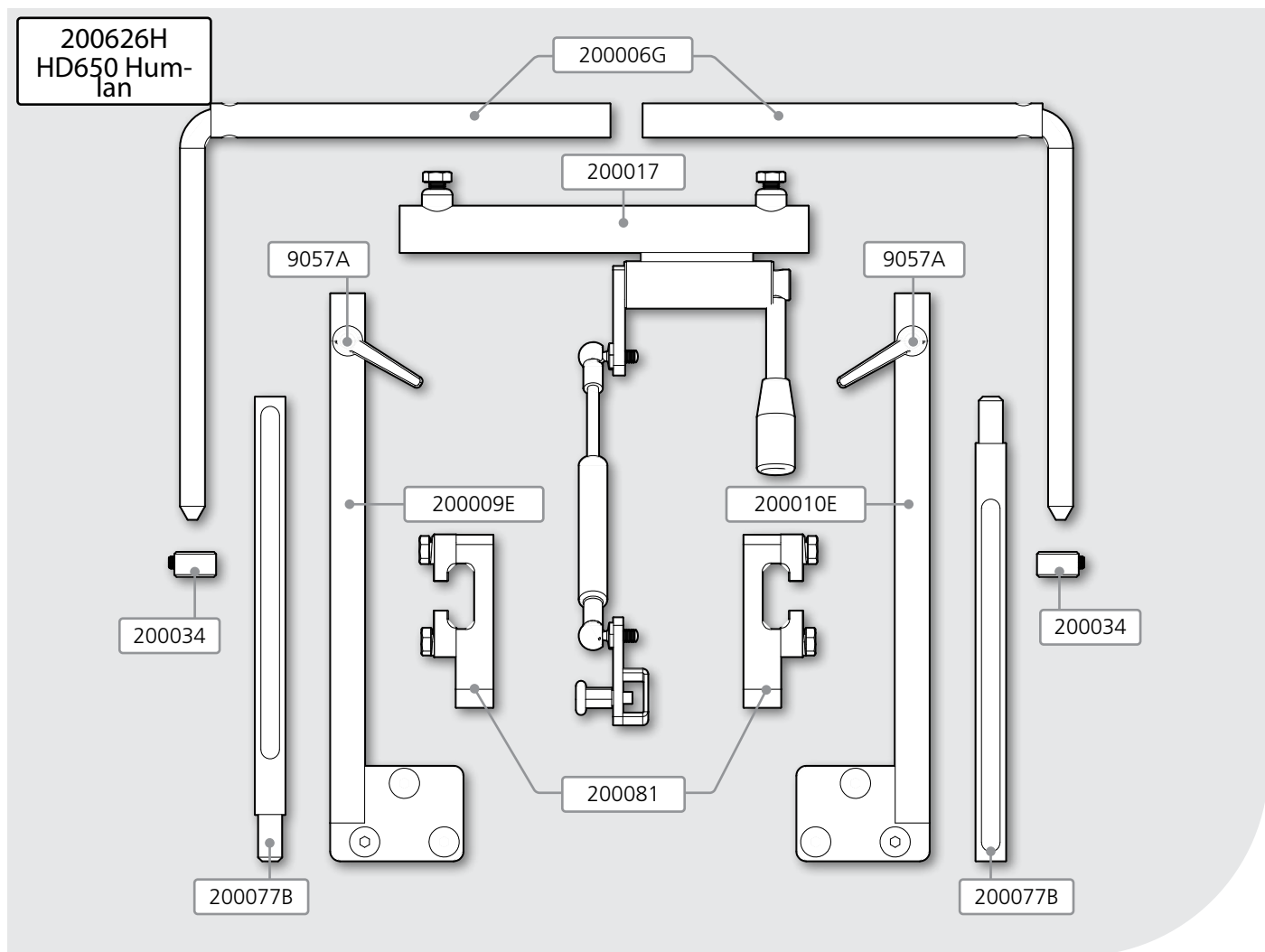
The wheelchair must always be equipped with tilt protection when operating using the auxiliary motor and this must be folded down.

Speederbox

We recommend that the wheelchair is fitted with a driving bow and separate brake handles.

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