UB200628A

Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75080.2 2022-08-16

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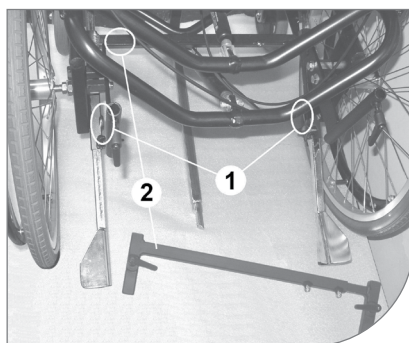
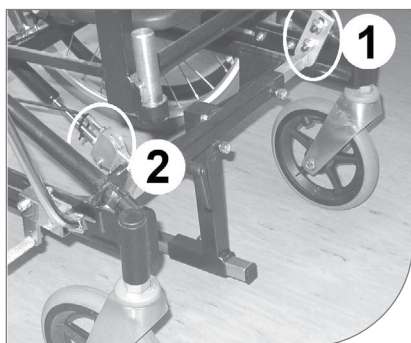
ENG

Mounting instructions

Lille Viking 2000 is mounted onto the wheelchair after being adjusted to the user.

Front

The front brackets are mounted on the inside of the frame's top tubes, right behind the front wheels and with the holes on the telescopic mounting brackets pointing downwards.

If the bracket of the seat adjustment is in the way of the pulling bar, it can be turned so that the cableconnection now is upwards.

Cross bar (old chairs)

On chairs that are produced before 2005 it is necessary to remove the bar which is straight across between the anti-tilts, before Lille Viking 2000 is installed. The wheelchair's serial number which is placed on this bar, is moved to a different place on the frame.

The cross bar, which replaces the bar you have recently removed, is mounted onto the ends of the anti-tilts, in front of the wheel locks and is to be turned in the direction as illustrated. When the bar is fitted, the finger screws are fastened first, and then the two bolts on the cross bar.

OBS. The cross bar must stay on the wheelchair when the supporting motor has not been installed!

Rear

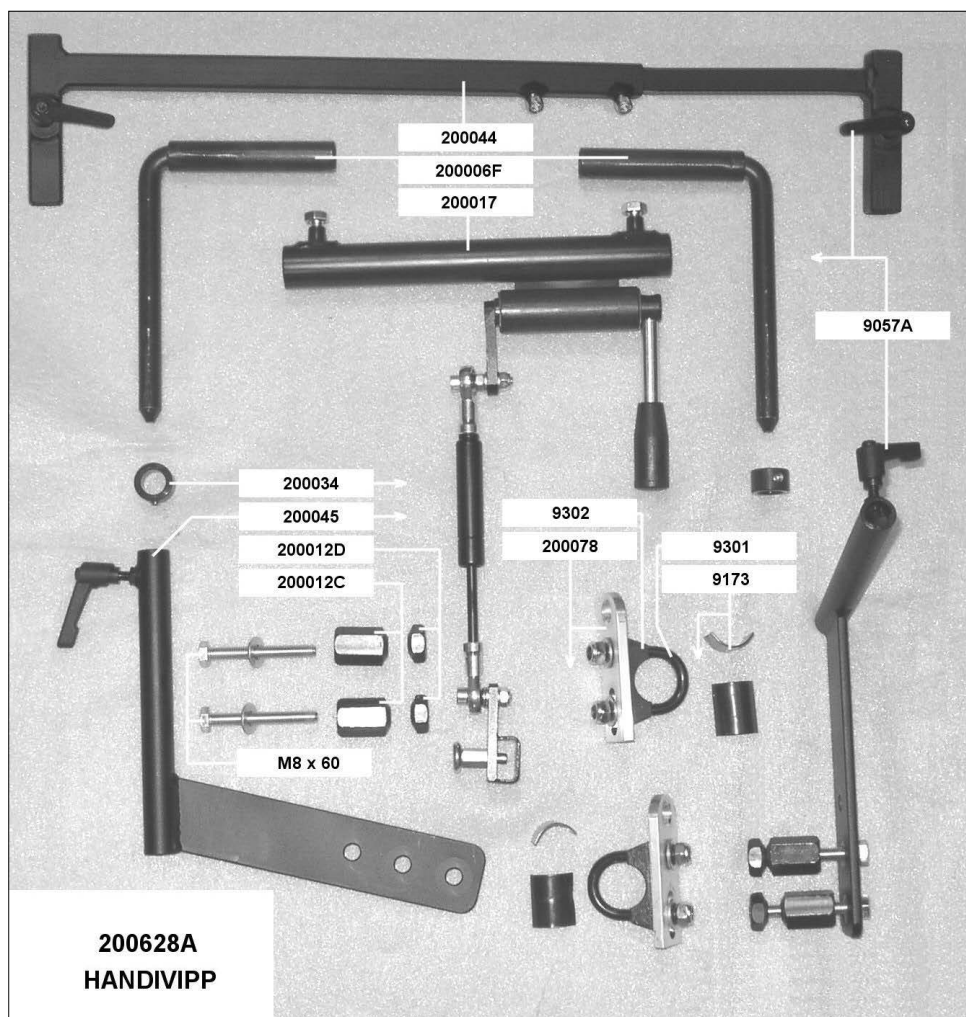
The steel brackets are screwed on the inside of the perforated plates in the two front holes in the flat iron, so it points out backwards, horizontally, and the tubes are turned upwards. Lille Viking 2000 is installed underneath the chair and is adjusted in the height and lengthwise so that the gas spring of the seat tilt almost contacts the motor. That way, the supporting motor will be about 12-15 cm behind the wheelchair's rear wheels.

The companion is to be instructed in possible limitations of the seat functions.

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