UB200629C

Monteringssats Rem

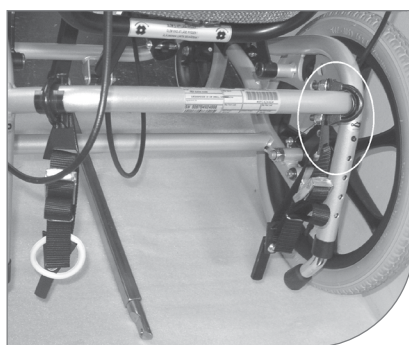
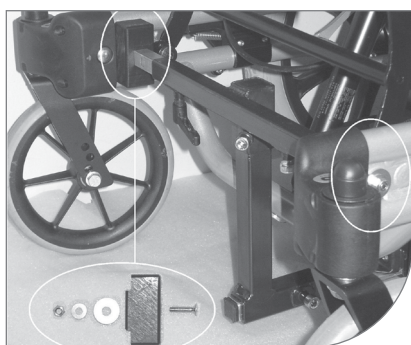
Monteringssett Rem

Assembly kit Strap

75081.2 2022-08-16

Rullstol, Tillverkare:

Azalea, Invacare

SV

Monteringsinstruktioner

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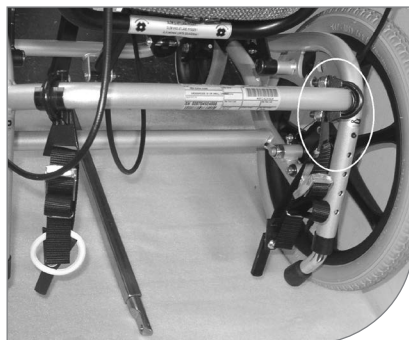
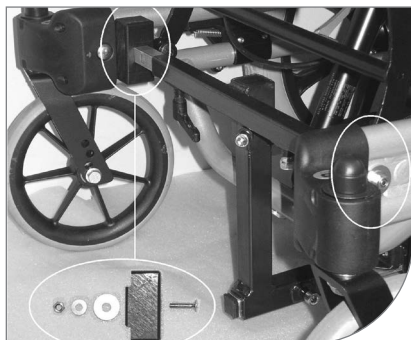
Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Azalea, Invacare

ENG

Mounting Instructions

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user.

Front

The black plastic brackets are mounted into the holes at the frame, right behind the front wheels. It might be necessary to unfasten the fixing device a little, to get the bracket in place.

The plastic bracket is tightened firmly with the countersunk screws through the holes in the telescopic bars, with washers and nuts on the outer side of the frame. The cross bar centre section is turned so the square tube for the angle bracket is turning backwards and downwards.

Rear

The brackets shall be put on the cross bar behind the seat, the strap holders should be on the front side of the tube. (see photo). The distance between the straps should be at least 32 cm.

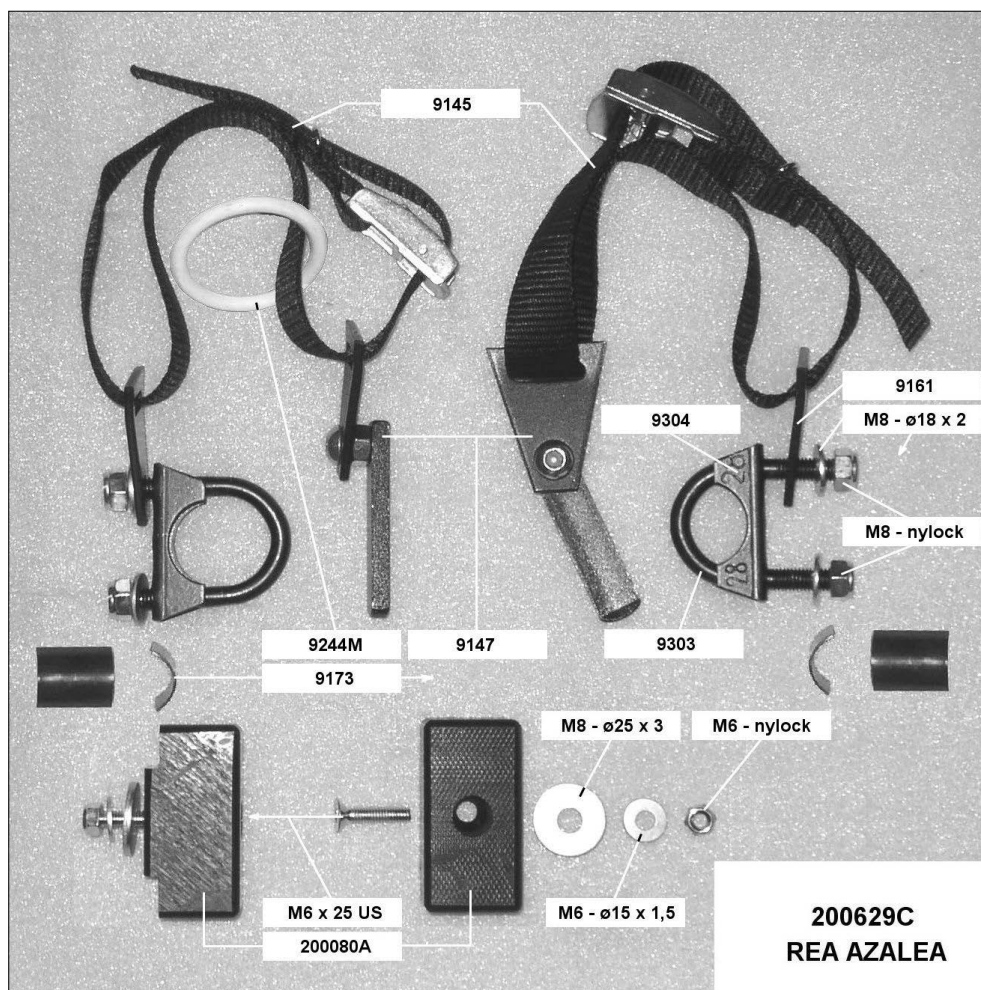
OBS

It is important that the chair's rear wheels have maximum pressure! The supporting motor is placed under the chair, and shall be adjusted lengthwise so that the tube between the chair's rear wheels is passing between the battery and the wheels. The height during disengaging is precisely adjusted by the straps, so that the motor will go all the way up against the transverse tube behind the seat.

The companion must be given directions in adjustments of the strap length, and also in possible restrictions for the seat functions.

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ENG

Art.no.	Description	Qty
UB9145	Strap Lille Viking	2
UB9147	Strap fastening with holder	2
UB9161	Belt holder	2
UB9173	Tube protector mounting bracket	4
UB9244M	Rubber ring white	1
UB9303	Mounting clamp bow 28 mm	2
UB9304	Mounting clamp basin 28 mm	2
UB200080A	Mounting bracket front Bellis	2