UB200629D

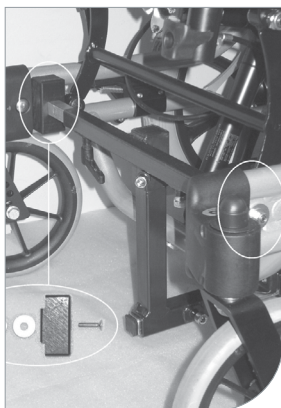
Monteringssats Bygel

Monteringssett Bøjle

Assembly kit Bow

75082.3 2022-08-16

Rullstol, Tillverkare:
Azalea, Invacare

SV

Monteringsanvisning

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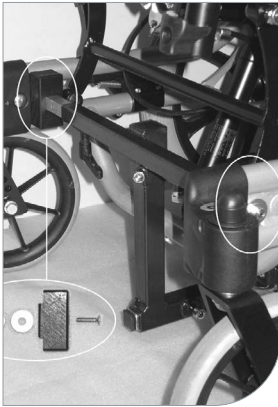
Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Azalea, Invacare

Mounting Instructions

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user.

Front

Mounting bracket UB200080A are mounted into the holes at the frame, right behind the front wheels. It might be necessary to unfasten the fixing device a little, to get the bracket in place.

The plastic bracket is tightened firmly with the countersunk screws through the holes in the telescopic bars, with washers and nuts on the outer side of the frame. The cross bar centre section is turned so the square tube for the angle bracket is turning backwards and downwards.

Rear

The brackets are mounted as far back as possible at the lowest horizontal tube, so that the cog washers are turned upwards and are facing each other.

The tubes are to be mounted on the cog washers, so that they pass by close behind the horizontal tube. Then the bow system is mounted and adjusted preferably so the seat functions not becomes limited.

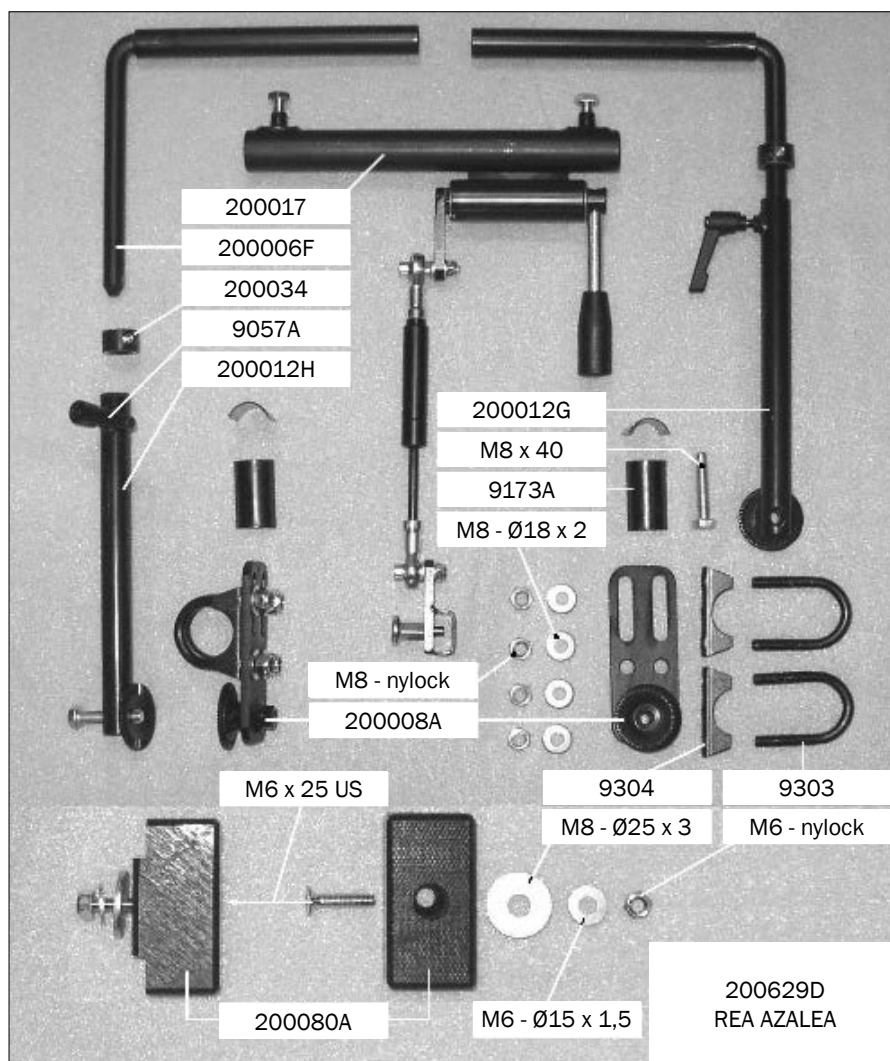
NB

It is important that the chair's rear wheels have maximum pressure! The supporting motor is placed under the chair, and adjusted lengthwise, so the tube between the chair's rear wheels is passing between the battery and the wheels. The lifting height when disengaged is adjusted precisely, so that the motor will go all the way up against the transverse tube behind the seat.

The companion must be given directions of possible restrictions for the seat functions.

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ENG

Art.no.	Description	Qty
UB9057A	Latch lever M6 x 12	2
UB9173A	Half basin long	4
UB9303	Mounting clamp bow 28 mm	4
UB9304	Mounting clamp basin 28 mm	4
UB200006F	Mounting angle long	2
UB200008A	Bracket with cog washer	2
UB200012H	Mounting arm 200 mm left	1
UB200012G	Mounting arm 200 mm right	1
UB200017	Pressure part complete	1
UB200034	Bolt for bow	2
UB200080A	Mounting bracket front Bellis	2