

UB200633F

Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75085.2 2022-08-16

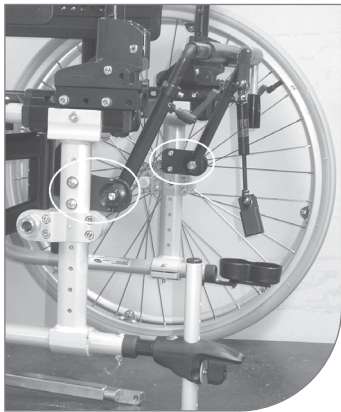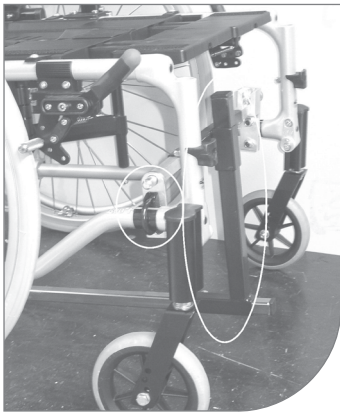Rullstol, Tillverkare:
Rea Focus 38-53, Invacare

max
150 kg

SV

Monteringsanvisning

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Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Rea Focus 38-53, Invacare

Mounting Instructions

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user.

Front

The front brackets are placed on the frame tubes right behind the front wheels, with the hole for the telescopic bars pointing upwards. The cross bar 's centre section must be turned so the angled bracket is mounted in front, and the screw is pointing downwards (see photo).

NB. It is necessary to use a high model of the pull bar bracket.

Rear

The rear fittings are placed on the upper half of the aluminium section, where the wheel hubs are mounted. For narrow chairs the fittings should be turned with the cog washers pointing outwards, and for wide chairs the cog washers should face each other. The mounting bars are fitted aslant upwards, on the cog washers, close to the rear edge of the seat.

The angled mounting bars are pushed into the pressure part, the bow stoppers are put on and the whole assembly is fitted into the mounting bars so that the pressure part is ca. 49 cm above ground.

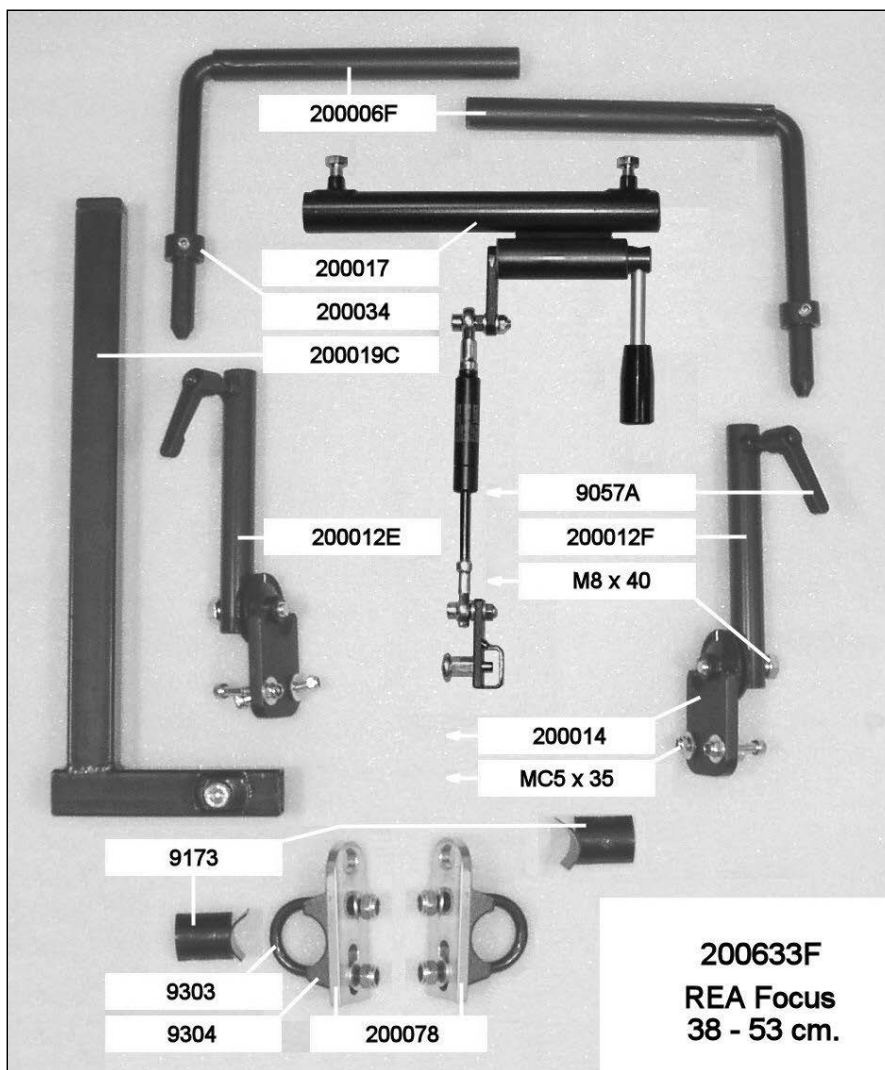
The supporting motor is installed underneath the chair and the pulling bar is adjusted. The driving wheels should now be on parallel lines with the wheelchair's rear wheels.

The pressure part is adjusted so that Lille Viking 2000 is lifted free off the ground during disengaging and the gas spring is pushed about halfway together when the handle is in the lowest position.

NB! Remember to check all functions after fitting Lille Viking 2000 on to the wheelchair.

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ENG

Art.no.	Description	Qty
UB9057A	Latch lever M6x12	2
UB9173	Tube protector mounting bracket	4
UB9303	Mounting clamp bow 28 mm	2
UB9304	Mounting clamp basin 28 mm	2
UB200006F	Mounting angle long	2
UB200012E	Mounting arm 140 mm right	1
UB200012F	Mounting arm 140 mm left	1
UB200014	Bracket rear Roxx	2
UB200017	Pressure part compl.	1
UB200019C	Pull bar bracket	1
UB200034	Bolt for bow	2
UB200078	Fitting bracket front round tubes 22-32 mm	2