UB200633K

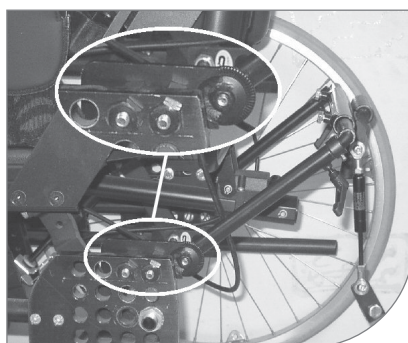
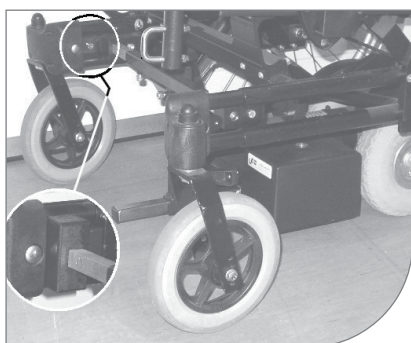
Monteringssats Bygel

Monteringssett Bøjle

Assembly kit Bow

75086.2 2022-08-16

Rullstol, Tillverkare:
Bellis, Invacare

max
150 kg

SV

Monteringsanvisning

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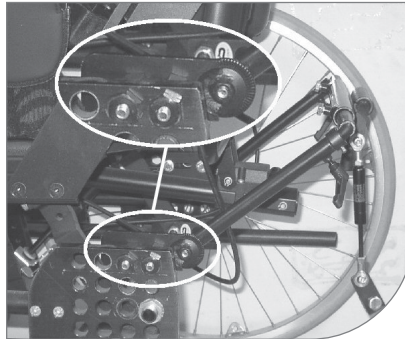
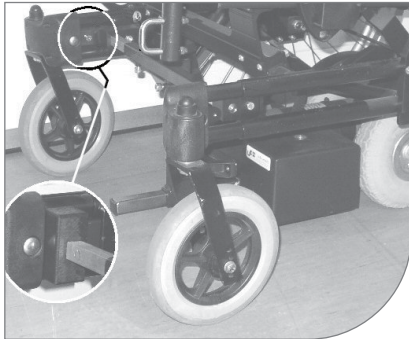
Brug af tipsikring anbefales.

Wheelchair, manufacturer:
Bellis, Invacare

Mounting Instructions

Lille Viking 2000 is mounted on to the wheel chair after being adjusted to the user. Because of the free height beneath the chair, it is necessary however, that the rear wheels are mounted in one of the 2 lower ranges in the perforated plates.

Front

Remove the two 6 mm screws that are located inside of the frame just behind the front wheels. Then, mount the two large black plastic covers and tighten them with the lower died screws through the hole to the releaser devices. The cross bar in the front centre section is turned so that the tube in the centre of the angled bracket faces downwards.

Rear

Because of the lack of space underneath the chair, caused by a cross beam, it can be necessary to move the wheels down to one of the 2 lower mounting positions (see photo). It can also be necessary to adjust the wheelbase so the locking mechanism on the shaft will lock safely, and the wheels can be turned freely. The tubes shall be mounted on the toothwashers, so that they are pointing in an angle backwards, and the bow system has to be adjusted so that the pressure parts is fixed in a position 49-50 cm above the floor.

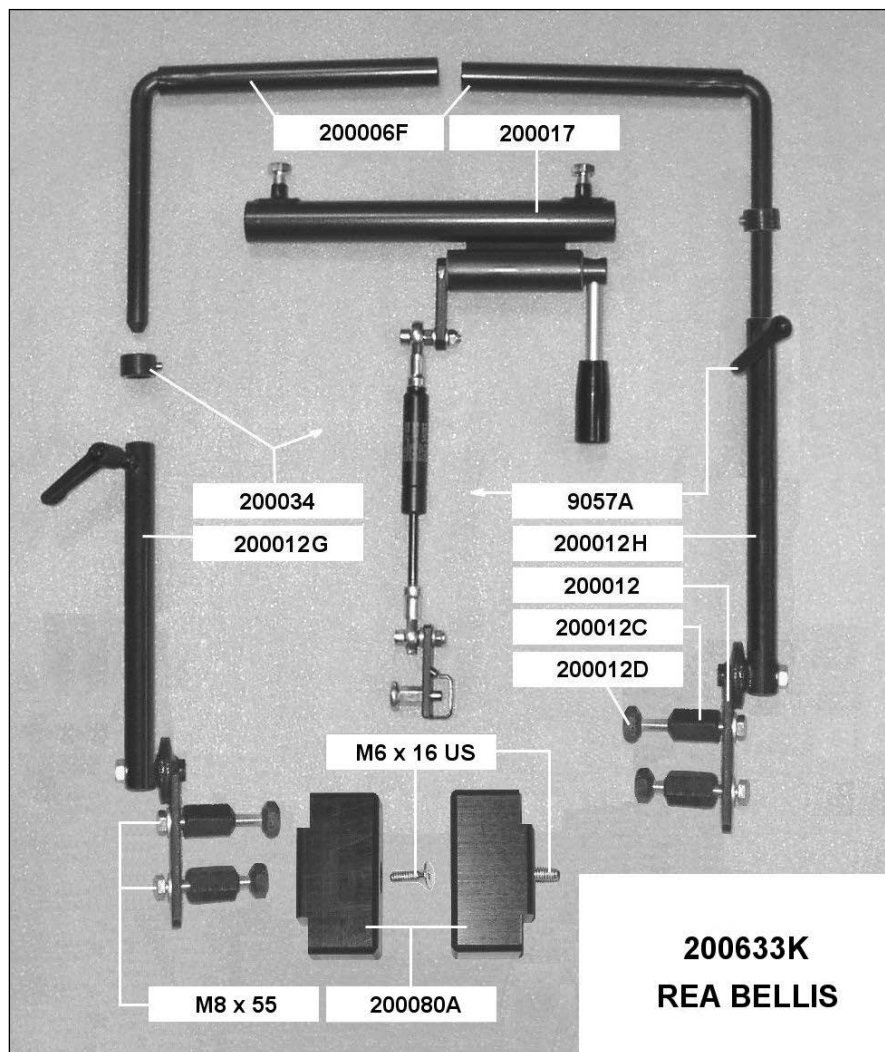
The support motor is placed beneath the chair and is adjusted lengthwise so that the wheels touch ground right in the middle under the chair's rear wheel. It can be an advantage however, to adjust it somewhat further back, to make more space for the seat to tilt.

The companion should be instructed in any possible limitations of the seat functions.

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