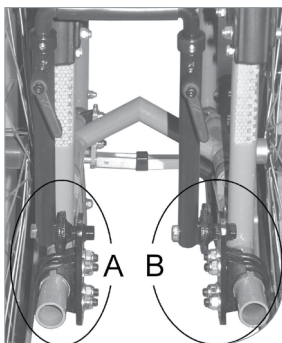
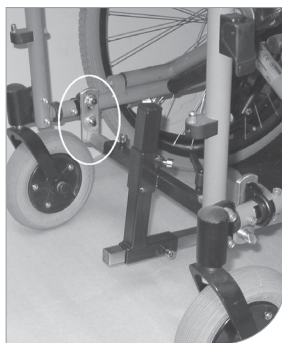
UB200635A

Monteringssats Bygel
Monteringssett Bøjle
Assembly kit Bow

75087.2 2022-08-16

Rullstol, Tillverkare:
Runda rör, 22 - 25 mm, diverse/ generell
Rea 702, Invacare, 702 TV, Invacare, 703LX, Invacare, 150, Invacare

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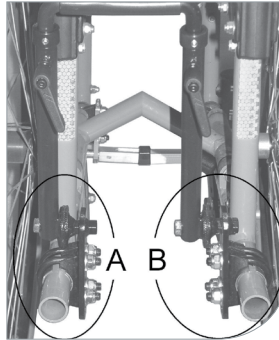
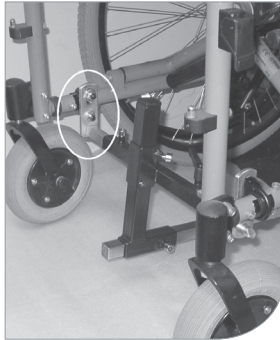
Rea 702, Invacare, 702 TV, Invacare, 703LX, Invacare, 150, Invacare

**Mounting Instructions**

Lille Viking 2000 is mounted on to the wheelchair after being adjusted to the user.

Front

The front brackets are fitted to the chairs lowest horizontal tube, right behind the front wheels and as far in front as possible. The front brackets are fitted on the frame's inside. For small chairs it might be necessary to fasten the brackets on the frame's outer side. The cross bar and the angle bracket is mounted so that the pull bar right below the chair is horizontal, or if possible points somewhat upwards. Finally it shall be adjusted lengthwise, so that the driving wheels are on parallel line with the wheelchairs' rear wheels.

Rear

The brackets are mounted at the rear part of the frame. When they are mounted at the back of the vertical tube, or upwards at the horizontal tube (see photo), it is possible to turn them in two ways -

A: Narrow chairs B: Wide chairs

The mounting arms are placed on the brackets, and are locked up in a position where they end out about 10 cm behind the seat. The angled mounting bars are pushed into the pressure part, the screws for the bow are put on loosely and the whole assembly is fitted into the arms.

The pressure part is adjusted, so that the coupling part is horizontal, before it's strapped firmly on the angles, and the height is adjusted to 48-50 cm above the ground. The Lille Viking 2000 supporting motor is lifted about 2-3 cm off the ground during disengaging, and the gas spring is pressed about half way together in its lowest position.

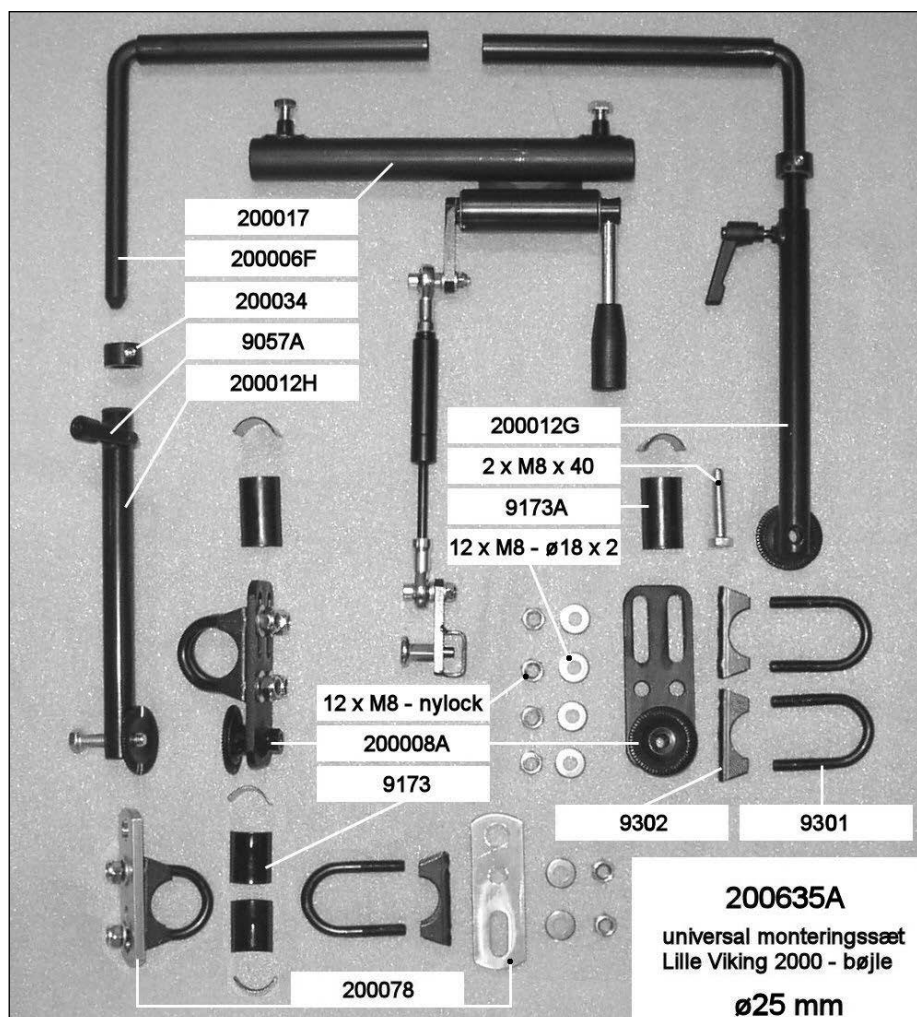
REMEMBER to place the bow stoppers correctly and tighten all screws.

When Lille Viking 2000 is mounted and adjusted, all functions has to be checked before the chair is delivered to the user.

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ENG

Art.no.	Description	Qty
UB9057A	Latch lever M6x12	2
UB9173	Tube protector mounting bracket	4
UB9173A	Half basin long	4
UB9301	Mounting clamp bow 25 mm	6
UB9302	Mounting clamp basin 25 mm	6
UB200006F	Mounting angle long	2
UB200008A	Bracket with cog washer	2
UB200012H	Mounting arm 200 mm left	1
UB200012G	Mounting arm 200 mm right	1
UB200017	Pressure part complete	1
UB200034	Bolt for bow	2
UB200078	Fittings bracket front round tubes 22-32 mm	2