## **Monteringsvejledning (DK)**

Montering af R82 transportsele

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Montering av R82 transportsele

## **Mounting instruction (GB)**

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
Producer  
R82 A/S  
Parallevej 3  
DK-8751 Gedved


# Mounting instruction
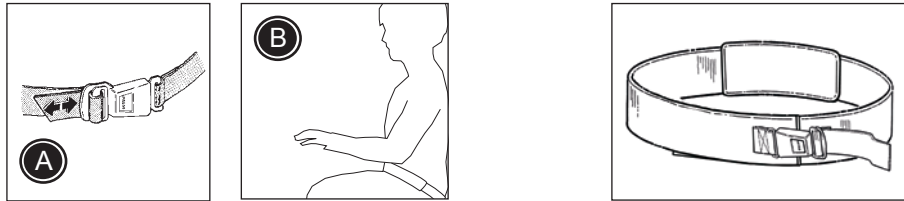
## How to mount the R82 Transportation belt

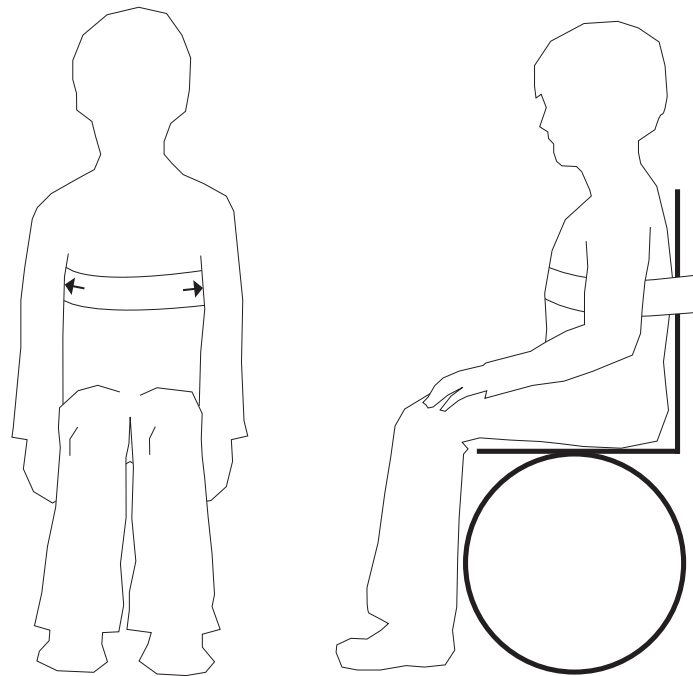The belt goes around the persons chest and around the back of the chair. Use the velcro for closing the belt and lock with the metal lock. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The transportation belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (B) together with the transportation belt.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.





 Carefully read the care label on the harness to maintain and wash the transportation belt correctly.

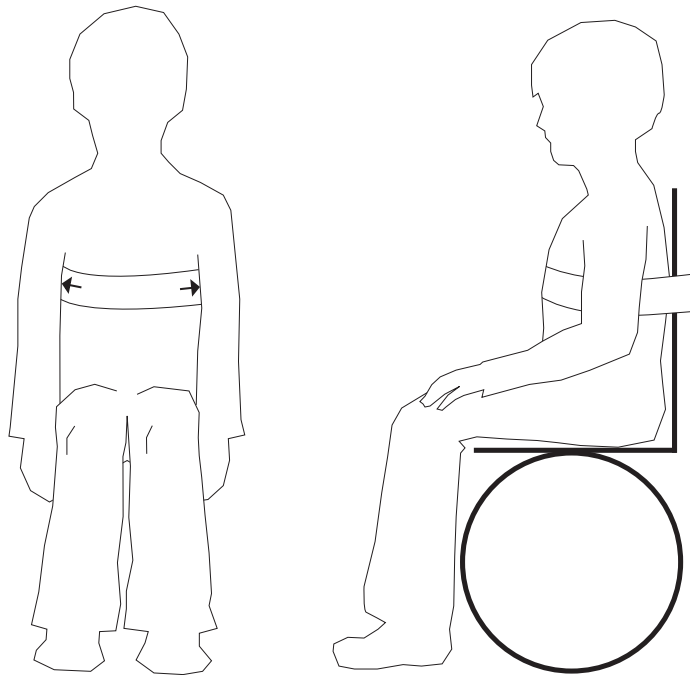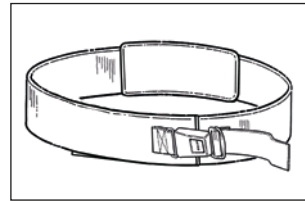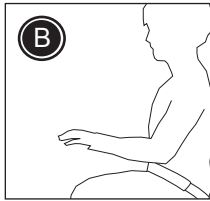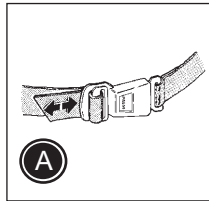 The latest version of the mounting instruction is always available online: [R82.org](http://R82.org)

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