Monteringsvejledning (DK)

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Mounting instruction (GB)

How to mount the R82 Thermo bag

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Montage des R82 Schlupfsack

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Monteren van de R82 winterbekleding

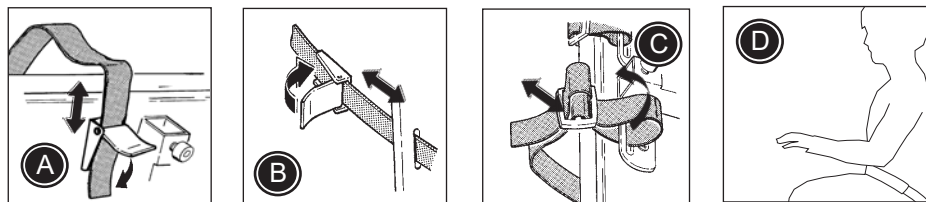
Instruction de montage (F)

Comment monter le sac thermique R82

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R82®

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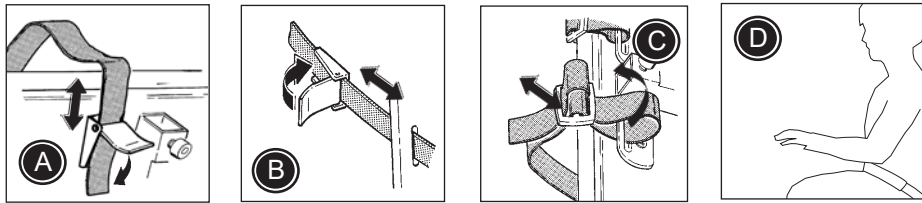
R82®

How to mount the R82 Thermo bag

Use the fittings (A) and (B) on the chair to mount the Thermo bag. If no fittings are mounted, please use the enclosed fix locks and belt fittings to mount the thermo bag as shown on picture (C). Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

* Always mount the shoulder straps (1) a little higher than the shoulder height of the user.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The straps is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (D) together with the thermo bag.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

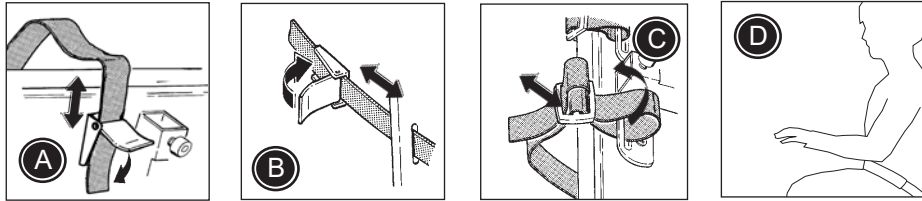
 Carefully read the care label on the harness to maintain and wash the thermo bag correctly.

 The latest version of the mounting instruction is always available online: R82.org

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