## **Monteringsvejledning (DK)**

Montering af R82 sternumsele

## **Monteringsanvisning (SV)**

Montera R82 sternumsele

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Montering av R82 sternumsele

## **Mounting instruction (GB)**

How to mount the R82 Sternum belt

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Montage eines R82 Sternumgurt

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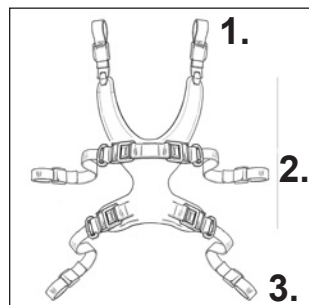
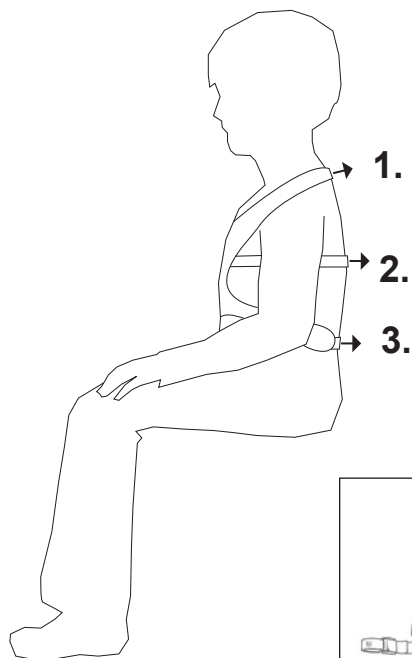
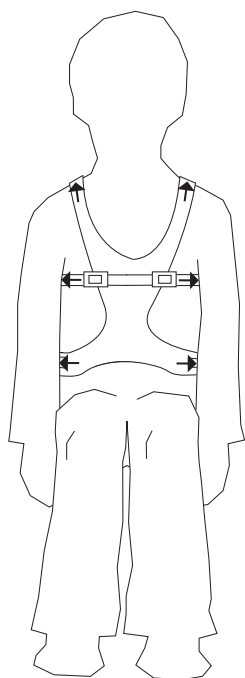
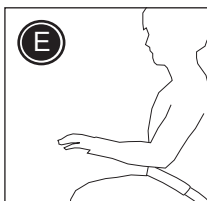
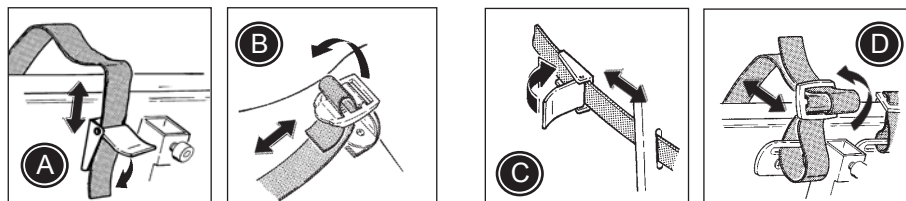
## **Instruction de montage (F)**

Comment monter le harnais thoracique R82

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R82®

## Montering af R82 sternum sele

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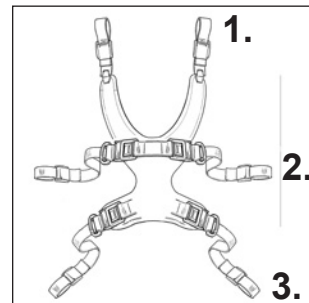
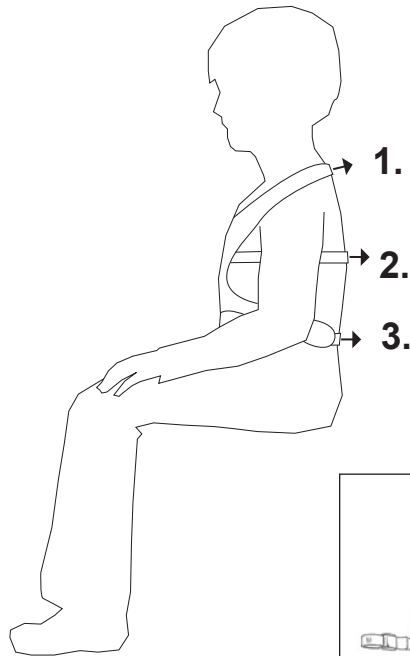
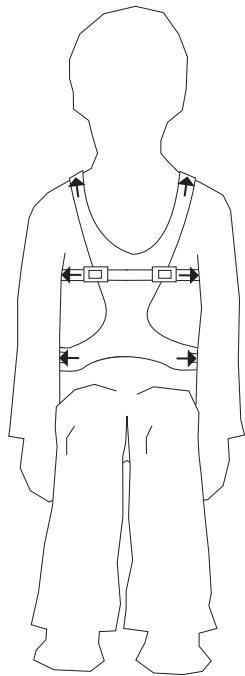
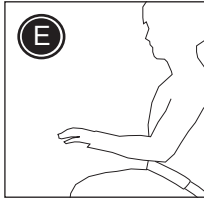
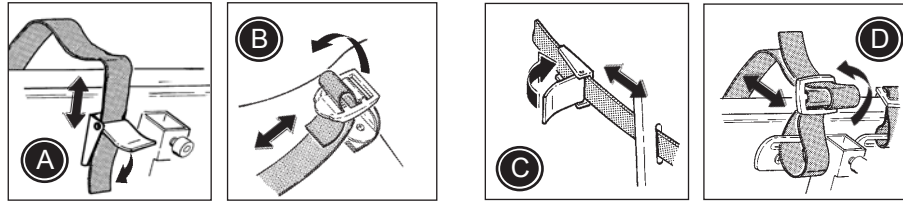
# Mounting instruction

## How to mount the R82 Sternum belt

Use the fittings (A) and (B) on the chair to mount the sternum belt. If no fittings are mounted, please use the enclosed fix locks and belt fittings to mount the sternum belt as shown on picture (C) and (D). Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

- \* Always mount the shoulder straps (1) a little higher than the shoulder height of the user.
- \* Mount the middle straps (2) beneath the user's arms.
- \* Mount the lower straps (3) at waist height.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The sternum belt is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (E) together with the sternum belt.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

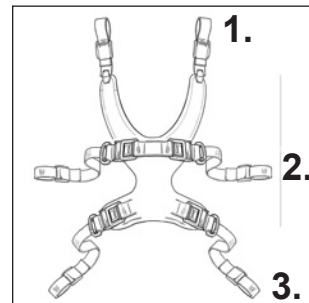
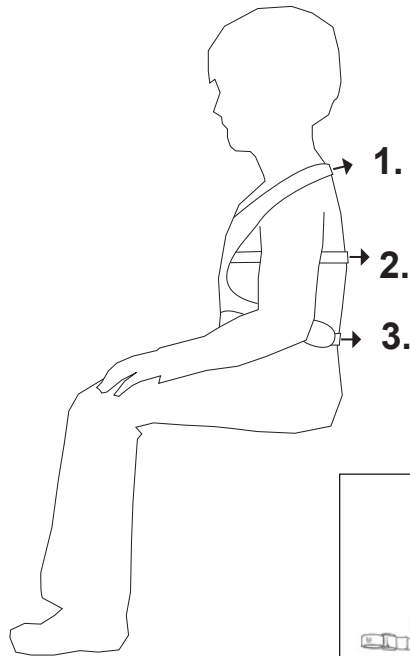
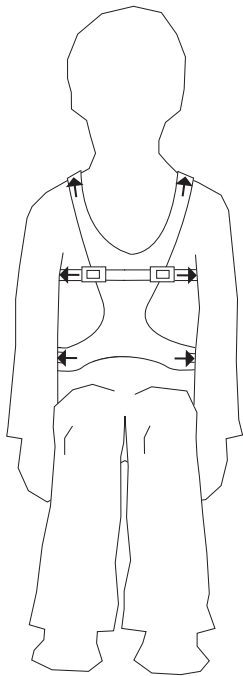
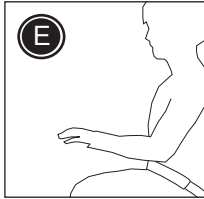
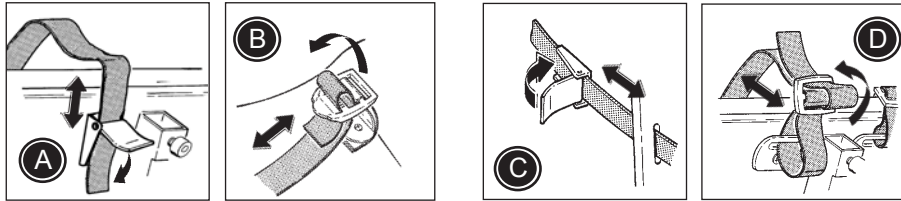
 Carefully read the care label on the harness to maintain and wash the sternum belt correctly.

 The latest version of the mounting instruction is always available online: [R82.org](http://R82.org)

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