Monteringsvejledning (DK)

Montering af R82 håndledsmanchet

Monteringsanvisning (SV)

Montera R82 handledsmanschett

Monteringsveiledning (N)

Montering av R82 håndleddsmansjett

Mounting instruction (GB)

How to mount the R82 Hand cuff

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Montage der R82 Manschette

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Comment monter support de poignet R82

Instruction de montage (F)

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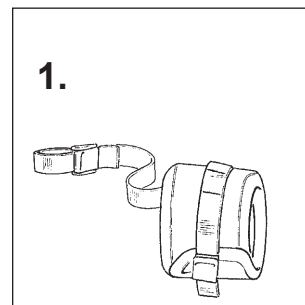
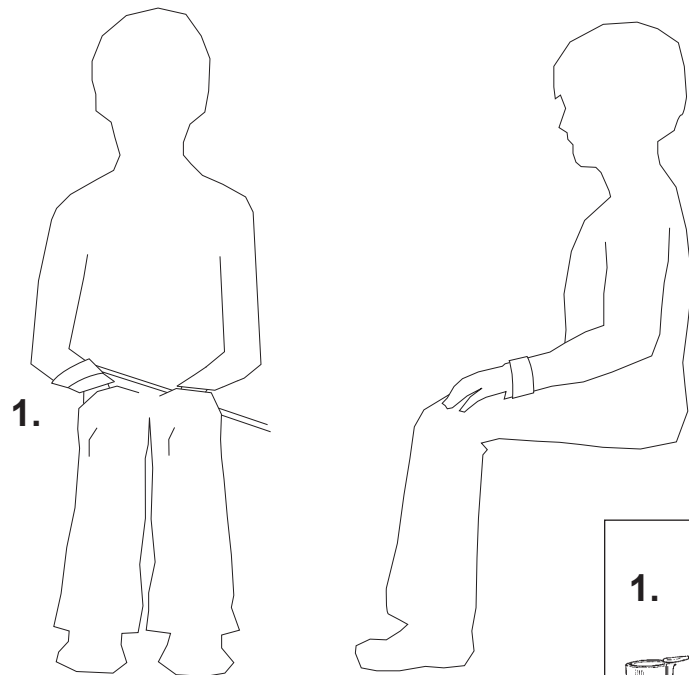
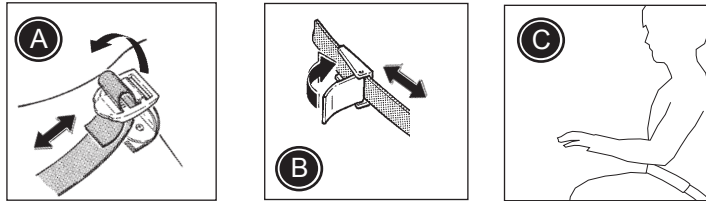
Mounting instruction

How to mount the R82 Hand cuff

Use the fittings (A) on the chair to mount the hand cuff. If no fittings are mounted, please use the enclosed fix locks and belt fittings (B) to mount the hand cuff as shown on picture. Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

* Mount the straps (1) on the opposite side, beneath the user's arms.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The hand cuff is fitted correctly when two fingers can be put between the cuff and the child's handwrists. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (C) together with the hand cuff.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 All buckles must be checked and cleaned on a regular basis. To obtain optimal safety, always keep the buckles cleaned.

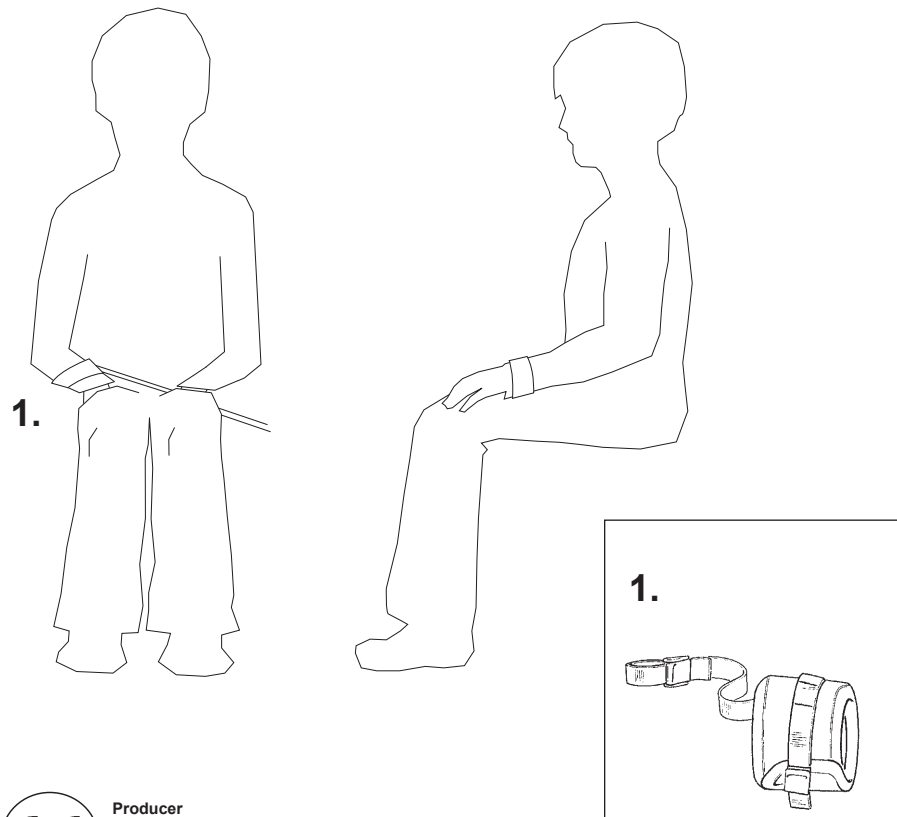
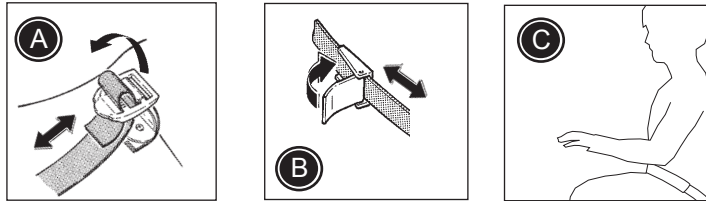
 Carefully read the care label on the harness to maintain and wash the hand cuffs correctly.

 The latest version of the mounting instruction is always available online: R82.org

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