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Manual (SE)

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Brukerveiledning (NO)

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User guide (GB)

Adjustable head supports

Bedienungsanleitung (D)

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Adjustable head supports

Head support with 4 laterals

Adjusting the upper laterals: Loosen the screws (A) with a 5 mm Allen key, place the laterals in desired position and fasten the screws (A) again.

Adjusting the lower laterals: Loosen the screws (B) with a 5 mm Allen key, place the laterals in desired position and fasten the screws (B) again.

If the head support is only mounted with the lower laterals (C), you still adjust the laterals by the screws (B).

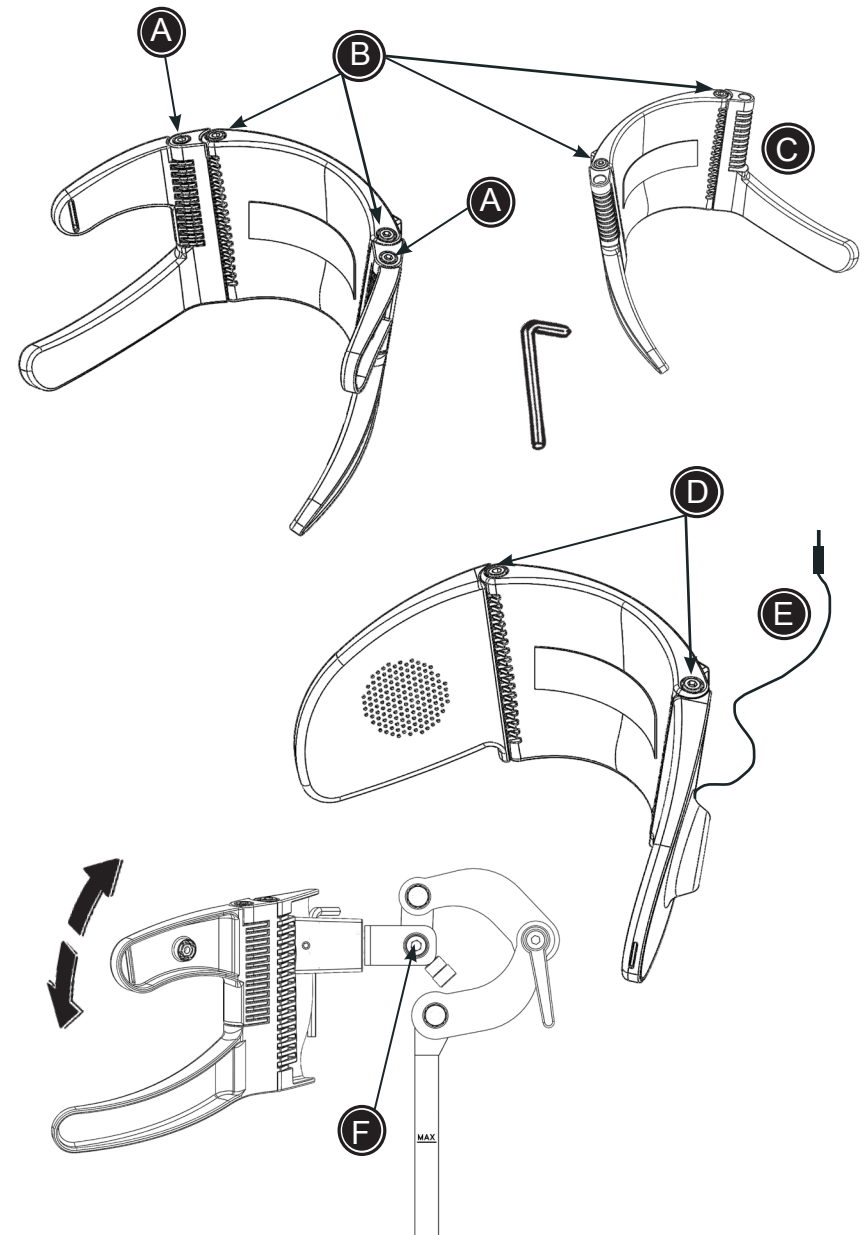
Head support with 2 large laterals

Adjusting the laterals: Loosen the screws (D) with a 5 mm Allen key, place the laterals in desired position and fasten the screws (D) again.

Connecting a music unit to the speakers: Plug the cable (E) in to your music unit.

Adjusting the angle (for both head supports)

Loosen the screws (F) in both sides with a 6 mm Allen key, place the head support in desired position and fasten the screws again.

Always use a suitable cover for the head supports

Be careful not to fasten the screws too much when adjusting the laterals

Do not pull the wire for the speakers

Beware of hearing damages! To protect your hearing, keep the volume below 80 decibels and do not use for an extended period of time (max 8 hours).

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