

# Mover 205,300 Partner 255


## oetac<sup>®</sup>


art:1420021- Ambulating arm stopper

BM1201 2019-06-18 Rev A

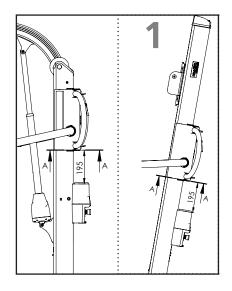

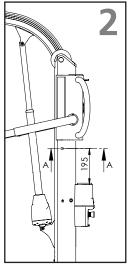

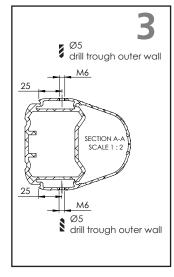

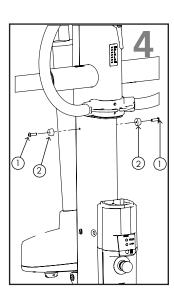

#### FN

#### How to mount stopper

- 1
- Remove battery
- Adjust ambulating arm to 195 mm above top of battery case.
- Set a mark on both side of column at A-A
- 2
- Set the ambulating arm to upper posistion
- 3
- Make a mark 25 mm from the column edge at the previos marked height. Do this on both sides of columb
- Drill a Ø5 hole trough the outer wall of columb. Do this on both sides of columb
- Make M6 threds in both holes.

### 4

- Insert bolt (1) into rubber stop (2). Screw bolt into columb and tighten. Do this on both sides of columb
- Innsert battery, then lower ambulating arms until it stops against the rubber stop. Make sure it is possible to remove the battery.