



Assembly instruction - EN


Molift Raiser Pro

Non-slip footboard kit

molift®
by Etac

BM43101 - 2025-07-21 - Rev. 1.0

01. Take the Non-slip footboard kit and Cleaning wipe out of the bag.
02. Use the Cleaning wipe to thoroughly clean the surface of the footboard.
03. Remove the protective backing from one of Non-slip footboard kit.
04. Apply the Non-slip footboard kit, adhesive side down, aligning it with one of the two marked foot shapes on the footboard. Press down evenly to ensure full adhesion.
05. Repeat the procedure with the second Non-slip footboard kit aligning it with the other marked foot shape.

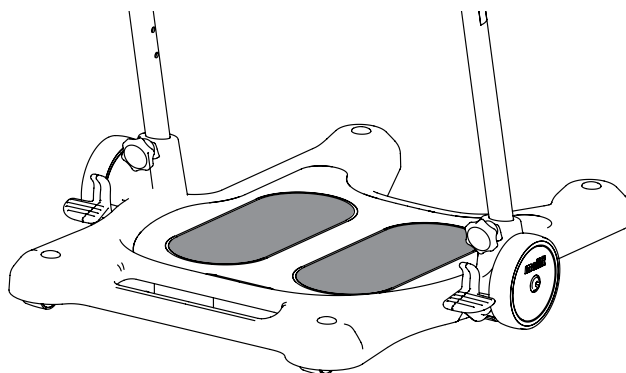

www.etac.com





Etac Ltd.

Unit 60, Hartlebury Trading Estate,
Hartlebury, Kidderminster,
Worcestershire, DY10 4JB
+44 121 561 2222



Sodimed SA

Ch. Praz-Devant 12
CH-1032 Romanel-sur-Lausanne
+41 58 911 06 06



Etac A/S

Parallevej 3
DK-8751 Gedved



+45 796 858 33



patienthandling@etac.com



www.etac.com

molift[®]
by Etac