

# Twist/Twist R

## Etac Xact wheelchair


### Combination guide

1a21260.1 2025-05-20

## Xact

### Rear position

To fit Twist R onto Xact you will need to set the chair up in the seat height 42.5 cm/45 cm (rear/front) or above, and with the rear axle plate a minimum of 60 mm from the rear frame bar (see below). Please check for clearance through the full rotation of the Twist device and that the resulting balance position safely meets the user’s needs.

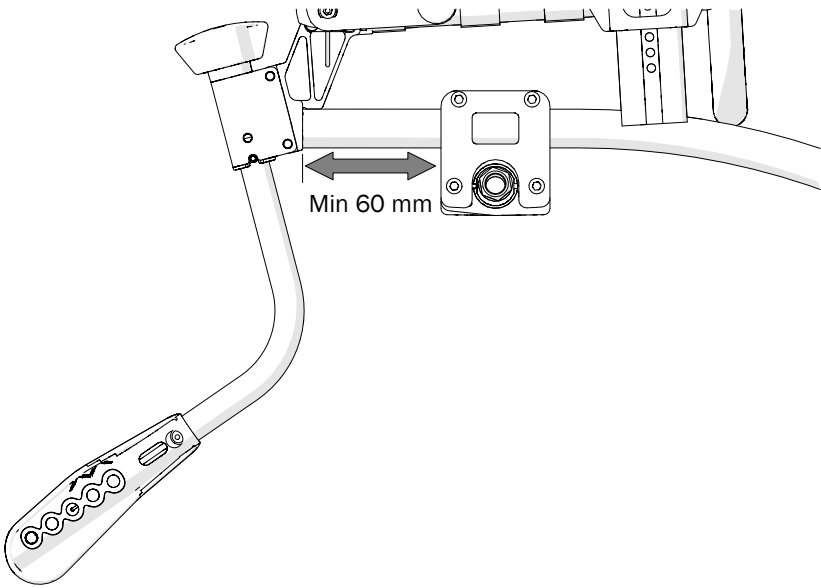

NC	NC
NC	NC
NC	NC
NC	NC
425	455
438	468
450	480
463	493
475	505
NC = Non-conformance	

### Front position

It is not currently possible to fit Xact to the front position.