

Gazelle PS

<p>User information appropriate for use by people with visual, reading or cognitive disability</p>	<p>The latest version of all instructions are always available on the etac website etac.com and can be printed in larger sizes</p>								
<p>Information available in pictogram</p>	<p>The instructions are made as pictograms</p>								
<p>Intended use and intended environment</p>	<p>Intended purpose The device is a standing frame for children and adults with disabilities. Indications The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age. The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product. Contra indications There are no known contra indications. Precautions Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:</p> <ul style="list-style-type: none"> • Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories. • Movement function (e.g. experience involuntary movements). • Ability to maintain a standing position, use the product with accessories (e.g. chest or back support). • Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head. <p>Application areas The product is for indoor use</p>								
<p>Maintenance instructions</p>	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1" data-bbox="590 1523 1348 1982"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1st Day</td> <td> <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product </td> </tr> <tr> <td>Daily</td> <td> <ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment </td> </tr> <tr> <td>Weekly</td> <td> <ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated </td> </tr> </tbody> </table>	Frequency	Description	1 st Day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 	Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment 	Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated
Frequency	Description								
1 st Day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 								
Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment 								
Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated 								

Gazelle PS

- | | |
|----------|--|
| | <ul style="list-style-type: none"> - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Disinfect the upholstery |
| Monthly | <ul style="list-style-type: none"> - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system |
| Annually | <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty - The product has to be checked and tightened by a qualified technician at least every 12 months to avoid failures |

Cleaning description

This product can be washed at 60° using a mild detergent in 15-20 minutes in a washing machine de-signed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Disinfection description

The product can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Dimensions

<http://www.etac.com> - Select product or click [here](#)

Mass

<http://www.etac.com> - Select product or click [here](#)

Used in combination with other products

N/A

Warning about dangerous combinations

The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.

List of accessories, detachable parts and materials

<http://www.etac.com> - Select product or click [here](#)

Programmable controller information

N/A

Operator control adjustments

N/A

Folding and dismantling

N/A

Transport instructions

N/A

Sound power level

N/A