

<b>Penguin</b>													
User information appropriate for use by people with visual, reading or cognitive disability	The latest version of all instructions is always available on the R82 website <a href="http://support.R82.org">support.R82.org</a> and can be printed in larger sizes												
Information available in pictogram	The instructions are made as pictograms												
Intended use and intended	The Penguin makes bathing comfortable for the child. It provides support for head and trunk together with great freedom of movement for arms and legs so that the child can stay longer in the bath. With its adjustable abduction block it easily fits the individual child.												
Maintenance instructions	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1<sup>st</sup> day</td> <td>- Read the User Guide carefully before use and store it in a safe place for future reference.</td> </tr> <tr> <td>Before every use</td> <td>- Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts</td> </tr> <tr> <td>After every use</td> <td>- Disinfect the product. please check the disinfection section in this User Guide for more information.</td> </tr> <tr> <td>Monthly</td> <td>- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures</td> </tr> <tr> <td>Annually</td> <td>- Inspect the product for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty</td> </tr> </tbody> </table>	Frequency	Description	1 <sup>st</sup> day	- Read the User Guide carefully before use and store it in a safe place for future reference.	Before every use	- Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts	After every use	- Disinfect the product. please check the disinfection section in this User Guide for more information.	Monthly	- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures	Annually	- Inspect the product for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Frequency	Description												
1 <sup>st</sup> day	- Read the User Guide carefully before use and store it in a safe place for future reference.												
Before every use	- Visual inspection. Repair or replace any damaged or worn-out parts - Check all fittings and buckles for signs of worn out parts												
After every use	- Disinfect the product. please check the disinfection section in this User Guide for more information.												
Monthly	- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures												
Annually	- Inspect the product for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty												
Cleaning description	<p>This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.</p> <p>This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.</p>												
Disinfection description	The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.												
Dimensions (Incl. max load)	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>												
Mass	N/A												

Penguin	
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	<a href="http://www.R82.org">http://www.R82.org</a> - Select product or click <a href="#">here</a>
Programmable controller information	N/A
Operator control adjustments	Manual
Folding and dismantling	N/A
Transport instructions (unoccupied)	Land and/or air
Sound power level	N/A
Turning diameter	NA
Forces on the wall	N/A
Electromagnetic environment warning	N/A
Electromagnetic influence warning	N/A
Life-time	The lifetime of the product in normal use is 5 years