

Rabbit Up

User information appropriate for use by people with visual, reading or cognitive disability

The latest version of all instructions is always available on the etac website etac.com and can be printed in larger sizes

Information available in pictogram

The instructions are made as pictograms

Intended use and intended environment

Intended purpose

The device is a standing frame for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

Rabbit Up mobile standing frame can also be used by children and adults with disabilities, who experience limitations in their ability to move around within the room or between rooms / outdoors.

The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product.

Contra indications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories.
- Movement function (e.g. experience involuntary movements).
- Ability to maintain a standing position, use the product with accessories (e.g. chest or back support).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

Rabbit Up (standing frame): For indoor use.

Rabbit Up (Mobile standing frame): For indoor and outdoor use (on even level and hard ground).

Maintenance instructions

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
-----------	-------------

Rabbit Up

- | | |
|---------------------|---|
| 1 st Day | - Read the User Guide carefully before use and store it in a safe place for future reference |
| Daily | <ul style="list-style-type: none"> - Store the enclosed tools on/at the product - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fitted correctly - Check all fittings and buckles for signs of worn out parts - Be aware of any MAX mark indications before adjustment |
| Weekly | <ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Check that the angle mechanism functions correctly and that the safety lock is working correctly |
| Monthly | <ul style="list-style-type: none"> - Disinfect the product. - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system - The product must be checked and tightened by a qualified technician at least every 12 months to avoid failures. |
| Annually | - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty |

Cleaning description This product can be washed at:
 60°C in 10 minutes
 85°C in 3 minutes
 ...using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Disinfection description The product can be disinfected with a 70% disinfectant solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Dimensions <http://www.etac.com> - Select product or click [here](#)

Mass <http://www.etac.com> - Select product or click [here](#)

Used in combination with other products N/A

Rabbit Up	
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	http://www.etac.com - Select product or click here
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	N/A
Transport instructions	N/A
Sound power level	N/A