

Wombat Solo

<p>User information appropriate for use by people with visual, reading or cognitive disability</p>	<p>The latest version of all instructions is always available on the R82 website support.R82.org and can be printed in larger sizes</p>												
<p>Information available in pictogram</p>	<p>The instructions are made as pictograms</p>												
<p>Intended use and intended environment</p>	<p>Intended purpose The Wombat Solo is a chair for children with disabilities. Contra indications There are no known contraindications. Caution! When used for: Children with limitation in their ability to maintain the position of their head, use the product with a headrest. Children with limitation in their ability to maintain a sitting position, use the product with accessories (e.g. side supports or vest). Children who experience severe or complete impairments in involuntary movement functions of the trunk, consider a modular seating system with a dynamic back. Application areas The product is for indoor use</p>												
<p>Maintenance instructions</p>	<p>Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.</p> <table border="1" data-bbox="593 1086 1348 1836"> <thead> <tr> <th>Frequency</th> <th>Description</th> </tr> </thead> <tbody> <tr> <td>1st Day</td> <td> <ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product </td> </tr> <tr> <td>Daily</td> <td> <ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts </td> </tr> <tr> <td>Weekly</td> <td> <ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Disinfect the product. </td> </tr> <tr> <td>Monthly</td> <td> <ul style="list-style-type: none"> - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system </td> </tr> <tr> <td>Annually</td> <td> <ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty </td> </tr> </tbody> </table>	Frequency	Description	1 st Day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 	Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts 	Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Disinfect the product. 	Monthly	<ul style="list-style-type: none"> - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system 	Annually	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
Frequency	Description												
1 st Day	<ul style="list-style-type: none"> - Read the User Guide carefully before use and store it in a safe place for future reference - Store the enclosed tools on/at the product 												
Daily	<ul style="list-style-type: none"> - Use a dry cloth to clean the product - Visual inspection. Repair or replace any damaged or worn-out parts - Ensure that all fasteners are in place and fit-ted correctly - Check all fittings and buckles for signs of worn out parts 												
Weekly	<ul style="list-style-type: none"> - Check that all castors are moving freely and that all wheel locks/brakes are easily activated - Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use - Disinfect the product. 												
Monthly	<ul style="list-style-type: none"> - All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures - Oil the swing-away parts. We recommend you use a professional lubricant system 												
Annually	<ul style="list-style-type: none"> - Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty 												
<p>Cleaning description</p>	<p>The cushions are removable and the cover can be washed at max. 60°C using a mild detergent in a washing machine. The cushion covers can be washed at 60°C using a mild detergent in a washing machine. It is not recommended to wash the foam.</p>												

Wombat Solo	
	<p>Warewashing machine This product can be washed at 60° using a mild detergent in 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories seperately. Before washing Avoid degreasing of the piston rod, lower the seat to minimum height</p>
Disinfection description	The product without accessories and cushions can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.
Dimensions	http://www.R82.org - Select product or click here
Mass	http://www.R82.org - Select product or click here
Used in combination with other products	N/A
Warning about dangerous combinations	The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.
List of accessories, detachable parts and materials	http://www.R82.org - Select product or click here
Programmable controller information	N/A
Operator control adjustments	N/A
Folding and dismantling	N/A
Transport instructions	N/A
Sound power level	N/A