



etac®

Etac Alto

Mobile shower commode

Elevating the caregiving experience

Alto is a compact, powered, mobile shower commode with adjustable height and tilt, which can be used for showering, toileting, and transportation. It is designed with both caregivers' and users' needs in mind, making it highly suitable for various settings including homecare, hospitals, and long-term care facilities.



Paving the way for dignity and independence

Limited bathroom space presents challenges, often leading to aid choices based on limitations rather than individual needs, resulting in compromises. Caregivers face physically demanding tasks, spending significant time in poor postures, which can cause injuries.

Crafted for small spaces, Alto provides easy access, which simplifies care tasks and eliminates costly renovations. Caregivers benefit from reduced stress and injury risk, translating into better, more dignified care.

Discover the features

A chair with excellent maneuverability and 360 degree access, specifically designed to make personal care tasks easier for the carer while ensuring comfort for the user.



Alto

Alto is a chair with powered, adjustable height and tilt function. It offers exceptional maneuverability and ergonomic 360-degree access, reducing the risk of injury for caregivers.

Ergonomic caregiver access

Assist in the entire hygiene routine without performing physically demanding tasks, overextending, or assuming awkward postures that can lead to work-related injuries.

Easy to manoeuvre and operate

Enables single-handed transportation of the user to, from, and within the bathroom. This makes the caregiver's job easier, safer, and more enjoyable

Compact and purposeful design

Its purposeful design allows it to fit over most toilets and effortlessly pass through doorways, allowing users to remain in their homes with little or no remodelling required.

Enhanced comfort and convenience

With its adjustable height, tilt, and numerous accessories, user comfort and caregiver convenience are enhanced, thus improving the overall quality of care, dignity, and well-being.



Alto comes with a 5-year warranty and an 8-year service life (1-year warranty for the battery).

Alto

	Item no.
Alto, EU with pan holder	80229600
Alto, UK with pan holder	80229601
Alto, CH with pan holder	80229602
Alto, US with pan holder	80229603
Alto, AU with pan holder	80229604
Alto, EU wipeable back support incl. pan holder	80229605
Alto, UK wipeable back support incl. pan holder	80229606
Alto, EU upright back, upright head support incl. pan holder	80229608
Alto, EU	80229609
Alto, EU upright back, head support, pan holder, pan w lid, leg support minus	80229610
Alto, EU upright back, upright head support, wipeable back, pan holder, pan w lid, angled leg support minus	80229611
Alto, UK upright back, upright head support with pan holder	80229612
Alto, EU wipeable back, pan holder, flip up foot supports	80229613
Alto, EU upright back, upright head support, wipeable back, pan holder, flip-up foot supports	80229614

Materials

Chassis: powder-painted steel
 Seat/Arm supports: polypropylene
 Soft back support: polyester/velcro
 Foot support: polypropylene
 Castors: polyamide/TPE, lockable, ball bearing
 Wipeable back support: polypropylene



max
160
kg

max
353
lbs



Manual cleaning:

Clean the product with standard non-abrasive cleaners (pH between 5-9). Rinse and dry
 Disinfect the product with 70% rubbing alcohol. Allow to air dry.

Machine cleaning:

Clean the product in a washing machine or tunnel. Cleaning by washing machine or tunnel should be done in accordance with EN/ISO 60601-2-52

Max user weight: 160 kg / 353 lbs.

Solving the challenges of residential bathrooms

Working in residential bathrooms can be challenging due to limited space and narrow doorways. With a variety of toilet models available, it can be difficult to find wall-mounted options that work well with tilt-in-space, mobile shower commodes

Alto is purposefully designed for home care environments, offering a compact footprint that is ideal for smaller spaces. It includes a version with an upright back and upright head support, a unique combination that allows the chair to be placed closer to the wall, and ensures a seamless fit over wall-mounted toilets and many smart toilets.

The upright head support allows easy sideways adjustments without the need for tools, as well as height and depth adjustments to ensure a comfortable experience for for all users.



Alto with upright back and upright head support over wall-mounted toilet.

Hygienic and easy to clean

In addition to the space limitations, there are other considerations to bear in mind. For institutions and hospitals, infection control and hygiene are important factors. A mobile shower commode must be easy to clean, in order to accommodate multiple daily users.

Alto's design ensures easy cleaning and maintenance, for a safe and sanitary environment. It offers a version with hygienic back support, easy-to-clean surfaces, and few visible screw heads where dirt can accumulate. This makes Alto suitable for both home care and institutions.

The wipeable back support is designed to support the spine's natural curve, promoting good posture and comfort. It is slightly flexible in the lower part to adapt to users' different shapes. It is easy to assemble and requires no tools. Additionally, the wipeable back support's height can easily be adjusted by sliding the support either up or down the chassis.



Alto with wipeable back support, makes cleaning a quick and easy task.

Accessories

Alto is compatible with a wide range of Swift Mobil-2 accessories, allowing for individual customisation.



Seat closed rear

84001337

A seat that is closed at the rear and features a hygiene opening at the front for extra support around the seating.

Material: Polypropylene



Soft cushion with oval opening

80209067

Foam cushion with a cover made from water-repellent fabric. Height 4 cm.

Material: Polyether/coated plastic



Soft comfort seat, 4 cm rear opening

80209519

Ergonomically shaped with substantial softness providing comfortable relief and support. Designed with a recess at the front.

Material: Polyurethane



Head support

80209521

Multi-adjustable head support with shaped pillow. Can be adjusted in height and depth, sideways and at an angle. Complete with bracket. Material: Pillow: Polyurethane

Brace: Stainless steel



Soft comfort seat, 2 cm rear opening

80209518

Designed with substantial softness and round shapes for maximum comfort. Designed with a recess at the front.

Material: Polyurethane



Head support upright

80209533

When combined with the Upright back, the chair can be placed closer to the wall. Multi-adjustable with shaped pillow. Complete with bracket. Material: Pillow: Polyurethane

Brace: Stainless steel



Soft comfort seat, 4 cm

Small opening 15 cm (6"): 80209507

Large opening 18 cm (7"): 80209506

Ergonomically shaped, soft comfort seats with recesses at the front. Height is 4 cm at the lowest point.

Material: Polyurethane



Comfort cover

80209455

Soft back with integrated arm support cushions. The comfort padding provides a soft support, while the gripping surface on the armrest is kept free.

Material: Polyethylene



Soft comfort seat, 2 cm

Opening 18 cm: 80209306

Shaped, soft comfort seat with recess at the front. Height is 2 cm at the lowest point.

Material: Polyurethane



Comfort kit

80209457

Soft back with integrated armpads and soft seat.

Material: Polyethylene



Seat cover

80209434

A comfortable, closed seat when you want to cover the hygiene opening. Can be used in the shower. Height 2 cm.

Material: Polyurethane



Side support

80209509

Offers a soft side support and facilitates positioning in the chair. Adjustable in width.

Material: Polyester fabric, Velcro, Polyethylene



Soft cushion with standard opening

80209065

Foam cushion with a cover, made using water-repellent fabric. Height 2 cm.

Material: Polyether/coated plastic



Crossbar

60.5 cm 80303034

66.5 cm 80209452

A safe support to hold on to. Can also be used as a strain support. Attaches to the arm supports.

Material: Aluminium, polyethylene

Accessories



Soft back pad

80209448

Provides the user with warmth and softness. Used inside the arm supports, which makes it possible to fold up the arm supports and create accessibility for side transfer.

Material: Polyethylene



Arm support pad

80209508

Used to create softness around the arm supports. Provides a warm, soft surface to support the arms, while keeping the gripping surface free, making it safe to stand up. Sold in pairs. Material: Polyethylene



Arm support Hemiplegia

80209035

Provides support for the forearm with its contoured shape. Equipped with a heat-reflecting material on the outside. Fits both the left and right armrest.

Material: Polyethylene



Arm support lock

80209445

The locking pin can be used when you want to lock the arm supports in either the folded-down or folded-up position.

Material: Stainless steel



Splash guard

80209430

A soft protection against urine splashes at the front. Attaches to the front recess of the seat and can easily be removed.

Material: Polyurethane



Hip/Chest belt, upholstered

80209524

2-part belt for positioning that can be used either as a hip or chest belt. Adjustable with 3 buckles between 65-130 cm length. The material is comfortable and quick-drying, and provides firm support when positioning.



Hip belt

80209440

Easy to adjust with a plastic buckle. Attaches around the back tubes in the rear with velcro straps.

Material: Polyester



Chest belt

80209442

Easy to adjust and attach around the straps of the back support upholstery. We recommend using a hip belt in conjunction with the chest belt. Material: Polyester



Pan with lid and handle

80209255

Unique interior shape that breaks the stream and prevents splashes. The lid has a handle that locks onto the pan and ensures that the content does not spill.

Material: Polypropylene



Pan

Pan without lid 80209254

Lid with handle 80209256

Unique interior shape that breaks the stream and prevents splashes.

Material: Polypropylene



Pan holder

80209257

Suitable for both the Etac pan and traditional hospital pans. Used together with the attachment holders, which are ordered separately.

Material: Polypropylene



Pan locker

80209049

Holds the pan in place when the shower chair is tilted. Compatible with Swift Mobil Tilt-2 variants.

Material: Polypropylene



Attachment holder

80209435

To be mounted firmly under the seat, enabling the use of pan holders. Fitted using a screwdriver. Complete set.

Material: Polyamide



Low mounting of pan/bucket

80209259

Used when you want the pan/bucket placed lower under the seat. Mounted with the pan or the bucket holders. Lowers the bed pan/bucket holders by 6 cm.

Material: Polypropylene

Accessories



Bucket holder

80209258

Used as a holder for pans/buckets or bed pans with higher edges. Used together with the attachment holders, which are ordered separately.

Material: Polypropylene



Bucket with handle and lid

80209265

Traditional bucket that is used with a bucket holder.

Material: Polypropylene



Short foot support

80209520

A short foot support that swings sideways. Can be adjusted by 12.5 cm, and provides a foot support length of 24-36.5 cm. The foot plate has a rounded shape. Sold as a pair.

Material: stainless steel and polypropylene



Leg support with flip-up footplates

80209446

Height adjustable and swing-away leg support with foldable footplates. For those who need a more open foot angle. Heel straps are included. Sold as a pair.

Material: Stainless steel and polypropylene



Leg and calf support

80209437

Leg support with an adjustable angle that can be used for both left and right legs. Comes with 4 fixed-angle positions. Sold per piece.

Material: Stainless steel and ABS-plastic



Cover leg and calf support

80209535

Enhanced comfort with a soft surface that easily fits by sliding over the support. Quick and easy to clean with a simple wipe.

Material: Polyester, Polyurethane, Nylon, Spandex



Foot support for leg and calf support

80209443

Kit with separate foot plate, to be mounted on the leg and calf support.

Material: ABS plastic



Angled leg support plus

80209531

Brings the foot support further away from the frame. Sold as a pair.

Material: Stainless steel and polyurethane



Angled leg support minus

80209532

Brings the foot support closer to the frame. Sold as a pair.

Material: Stainless steel and polyurethane



Amputee leg support

80209510

Adjustable in height, length, and angle. Can be used on both left and right sides. If the user is an amputee, we recommend using anti-tippers.

Material: Stainless steel and polyurethane



Wipeable back support

80209530

Supports the spine's natural curve, promoting good posture and comfort. Adjustable in height and slightly flexible in the lower part. Easy to clean, maintain, and mount.

Material: Polypropylene



Castor direction lock

80209534

Provides the option of locking the wheel direction for easier manoeuvring in corridors.

Brake colour: black

Material: Polyamide, Polypropylene, TPE, Fe

See washing instructions on page 10.

Washing instructions for accessories

Soft Comfort seats and Seat cover, Head support, Head support upright, Arm support lock, Splash guard, Pan, Pan holder, Attachment holder, Low mounting attachment, Bucket holder, Bucket, Foot support, Leg and calf support, Amputee leg support, Castor direction lock, Seat closed rear, and Wipeable back support



Back support upholstery, Hip/chest belt upholstered, Hip belt, Chest belt, and Heel strap



Soft cushions, Comfort cover, Soft back pad, Arm support pad, Arm support hemiplegia, Cross bar, and Cover leg- and calf support



Washing symbols explained



Machine wash – max. temperature 60 °C / 140 °F



Manual cleaning: Clean the product with a solvent-free cleaning agent with a pH level between 6–9, or with a 70% disinfectant solution.




Machine cleaning: The product can be cleaned in a cabinet washer-disinfector at max. 85 °C / 185 °F for 3 minutes.




Measurements



Model	Length	Width	Seat height	Back height	Seat depth
Alto	972 mm 38 ^{1/4} "	600 mm 23 ^{1/2} "	500 / 930 mm 19 ^{3/4} " / 36 ^{1/2} "	540 mm (590 mm*) 21 ^{3/4} " (23 ^{1/4} "*)	465 mm 18 ^{1/4} "

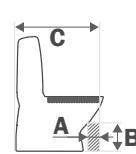

Model	Foot support height	Seat width	Arm support height	Arm support to arm support	Dimensions over toilet
Alto	340 / 465 mm 13 ^{1/2} " / 18 ^{1/4} "	500 mm 19 ^{3/4} "	220/250/280 mm 8 ^{3/4} "/9 ^{3/4} "/11"	420/480/540 mm 16 ^{1/2} "/19"/21 ^{1/4} "	A= Min 105 mm B = 190 mm C = Min 600 mm (510 mm)* 23 ^{1/2} " (20"*)

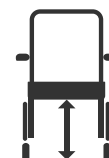

Model	Height	Castors	Clearance over toilet	Clearance over toilet	Tilt angle
Alto	1040-1470 mm (1085-1515mm*) 41"-58" (42 ^{3/4} "/59 ^{3/4} "*)	125 mm 5"	430 mm (406 mm)** 17" (16")**	445 mm 17 ^{1/2} "	-5° - 30°



Model	Turning radius	Max user weight	Weight		
Alto	ø 1005 mm ø 39 ^{1/2} "	160 kg 352 lbs	40 kg 88 lbs		

*Applies to 8022 9607, 8022 9608 **If the device shall be centered



Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit www.etac.com

Etac



+46 371 587 330



info@etac.se



www.etac.com

etac®

Creating Possibilities