



etac®

Etac My-Loo Fixed

Raised toilet seat with fixed mounting

Next level – A new generation of toilet aids that creates possibilities



My-Loo Fixed raised toilet seat with fixed mounting belongs to a whole new generation of smart products from Etac. By setting the highest requirements on everything from safety and comfort to adaptability and ease of installation, we can present a range that offers innovative, unique solutions.

With this new product series, we quite simply take toilet aids to a new level. ‘Design drives development’ has always been our guiding principle, and we are convinced that My-Loo Fixed will help users and prescribers to create new possibilities.



Patented arm support solution

My-Loo Fixed's innovative arm support feature has an entirely unique height-adjustment mechanism. In addition, the arm supports can be conveniently folded up and down with no pinching risk for the user. This patented solution provides optimal individualised support around the toilet.



The height of the arm supports is set with a simple manual adjustment. Choose a setting by sliding the orange control into one of the three positions. The steps are clearly marked with 1, 2, 3 on the armrest.

The arm supports are mounted from above – perfect for confined spaces. The arm supports are marked with L and R - intended for the user's left and right side respectively.

For dismantling, the arm support is folded up and pulled straight upwards to remove it from its bracket.

The arm supports can be locked into the desired position using a screw (included).

When only one arm support is used, the opening of the empty bracket is shielded by a cover (included).

My-Loo Fixed

With a unique, patented arm support solution, a well-designed seat and quick, simple installation, My-Loo Fixed with fixed mounting offers safe support, a high level of comfort and ease of handling. The raised toilet seat is suitable for most toilets and sits securely in place.

Developed to give independence with dignity

My-Loo Fixed enables users with reduced mobility or balance problems to manage toilet visits independently and safely.

The raised seat and stable arm supports make it easier to sit down and stand up from the toilet. An extra-large seat opening and generous hygiene recesses make personal hygiene easier.

Available in five different versions

My-Loo Fixed with fixed mounting is available in different designs, with or without arm supports and in seat heights of 2, 6 and 10 cm. If necessary, arm supports can be added later to a model without arm supports.

My-Loo Fixed comes with a 5-year warranty and a 5-year service life.



My-Loo Fixed raised toilet seat

My-Loo Fixed with arm supports, 20 mm	80301510
My-Loo Fixed with arm supports, 60 mm	80301511
My-Loo Fixed with arm supports, 100 mm	80301512
My-Loo Fixed, 60 mm	80301513
My-Loo Fixed, 100 mm	80301514

Weight

My-Loo Fixed with arm supports, 20 mm: 3.2 kg My-Loo Fixed, 60 mm: 2.2 kg
My-Loo Fixed with arm supports, 60 mm: 3.4 kg My-Loo Fixed, 100 mm: 2.4 kg
My-Loo Fixed with arm supports, 100 mm: 3.7 kg

Maximum user weight

150 kg

Together with accessory Supporting legs, 80301603: 200 kg

Material

Arm support: Polypropylene
Seat/Lid: Polypropylene
Attachment: Stainless steel, Polypropylene, TPE

Design

Myra Industriell Design AB

Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

myra



Features



Adjustable arm supports

Using a unique, patented solution, the arm supports can quickly and easily be adjusted in height. They are easily folded up and down with no pinching risk for the user. If needed My-Loo Fixed can be used with only one arm support.



Simple, convenient installation

My-Loo Fixed is installed quickly and easily using an innovative new solution. Practical, quick nuts make it possible to attach the raised toilet seat from above. The seat and arm supports can then be clicked smoothly into their brackets.



Spacious and practical

Generous recesses and an extra-large seat opening provide excellent accessibility. The raised toilet seat's design, with free space around the seat, provides plenty of room for all body shapes.



Stable

My-Loo Fixed sits securely in place and has a max user weight of 150 kg. For additional support and increased weight capacity, the accessory supporting legs can be added (max user weight 200 kg).



Safe and comfortable

With stable, easy-grip arm supports and a well-designed seat, My-Loo Fixed makes toilet visits safe and comfortable for the user. For extra comfort, a soft seat can be added.



Easy to clean

The clean, smooth surfaces are easy to wipe with a soft cloth. Should separate cleaning be required, the seat and arm supports are easy to remove from their brackets and equally simple to put back in place.



My-Loo Fixed with arm supports
2 cm



My-Loo Fixed with arm supports
6 cm



My-Loo Fixed with arm supports
10 cm




My-Loo Fixed
6 cm



My-Loo Fixed
10 cm

Dimensions

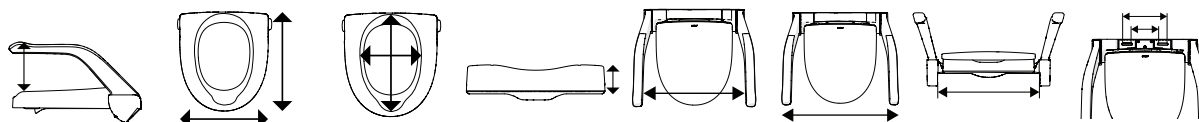

Model	Height arm supports	Width/Length	Inner dimensions	Height	Width between arm supports	Total width	Attachment width (max toilet width)	Dimensions between holes
80301510	215/245/285 mm	360/400 mm	215/285 mm	20 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301511	175/205/245 mm	360/400 mm	230/360 mm	60 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301512	135/165/205 mm	360/400 mm	230/360 mm	100 mm	480 mm	570 mm	400 mm	c/c 123-203 mm
80301513	-	360/400 mm	230/360 mm	60 mm	-	-	400 mm	c/c 123-203 mm
80301514	-	360/400 mm	230/360 mm	100 mm	-	-	400 mm	c/c 123-203 mm

My-Loo Fixed 2 cm – functional toilet arm support

My-Loo Fixed 2 cm is the perfect arm support solution for around the toilet if you want to avoid drilling into the walls or floor. The product consists of arm supports and a toilet seat mounted directly onto the toilet. As with the other versions with a raised seat, the arm supports can be set to three different heights to suit the needs of different users. With its modern, discreet design, My-Loo Fixed blends in beautifully with most bathroom environments.



My-Loo Fixed is delivered in a smart flatpack carrycase for easy handling

Accessories



Soft seat

80301604

Ergonomically shaped, soft and comfortable. Provides excellent pressure relief and comfortable support.

Material: Polyurethane



Supporting legs

80301603

For additional stability. Increases max. user weight to 200 kg. Foldable.

Material: Aluminium, polyamide, polypropylene, TPE, stainless steel



Storage basket

80301602

Attaches to the arm support.

Material: Polyamide, polyester/mesh, stainless steel



Arm supports

80301606

Sold in pairs, right, left version.


Material: Polypropylene



Top-mounted screw set

80301626

For toilets without through mounting holes (blind fitting). Should only be used with the included supporting legs. Suitable for mounting holes Ø20 mm. Material: Aluminium, polyamide, stainless steel, EPDM

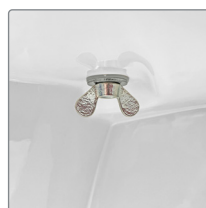

Attachment kit

80301053

An alternative attachment kit including shorter screws, wingnuts, metal nuts and accompanying washers

Material: Polypropylene, stainless steel



Toilet paper holder

80301601

Attaches to the arm support.

Material: Polyamide, stainless steel



Cross bar

80303034

A secure support to hold on to. Attaches to the arm support. 60.5 cm

Material: Aluminium, polyethylene





Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit www.etac.com

Etac

+46 371 587 330

info@etac.se

www.etac.com

 **etac**[®]
Creating Possibilities