



**etac®**

# Etac My-Loo

Toilet seat raiser with brackets

# My-Loo

## Toilet seat raiser, with brackets

Together with experienced industrial designers, we have developed the raised toilet seat with brackets – My-Loo. A product that gives concepts such as hygienic, safe and appealing a new meaning. With its gently sweeping lines and excellent ergonomics, it offers extra-comfort and a design ideally suited to the modern bathroom.

## Designed for function and comfort

My-Loo has been developed to blend into the bathroom environment and has a design that gives optimal comfort and accessibility. The gentle lines and the large opening make My-Loo suitable for most users. The seat raiser is offered in two heights, with or without a lid.

## Easy to keep clean and fresh

My-Loo toilet seat raiser consists of a small number of parts made of durable materials. It is easy to wipe off with a soft cloth. Should separate cleaning be required, it is easy to remove the seat and equally simple to put it back in place.

## Safe and quick installation

It takes only a few seconds to install My-Loo. You fix it in place using two specially designed brackets which are easily adapted to different toilet shapes. The extra-large friction surfaces and flexible material of the brackets provide a firm attachment to the porcelain.

Put the raised toilet seat on the toilet and push the brackets into place. Click, and the installation is finished!



### My-Loo

### Item no.

6 cm	80301520
10 cm	80301521
6 cm with lid	80301522
10 cm with lid	80301523

### Weight

6 cm: 0,8 kg	10 cm: 1,1 kg
6 cm with lid: 1,2 kg	10 cm med lock: 1,4 kg

### Maximum userweight

190 kg

### Material

Polypropen och TPE

### Design

Myra Industriell Design AB

myra

### Underhåll



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3 minutes.



# Features



## Easy to keep clean and fresh

The clean, smooth surfaces are easy to clean with a soft cloth. Should separate cleaning be required, the seat is easy to remove and equally simple to put back in place.



## Designed for function and comfort

The soft shape of the seat gives safe support and high comfort. An extra large opening and generous recesses facilitate accessibility during personal hygiene.



## Safe, quick installation

The easily attached brackets with wide adjustability and large friction surfaces make My-Loo at all times safely fixed to, and easily mounted on, any toilet porcelain. Entirely without screws.



## Variants






My-Loo raised toilet seat, with brackets, is available in two different heights, 6 and 10 cm, with or without lid.



My-Loo comes with a 5-year warranty and a 10-year service life.

## Dimensions

Outer width x depth	36 cm x 40 cm
Opening width x depth	23 cm x 36 cm
Seat height	6 cm, 10 cm
Distance between brackets	29 cm - 38,5 cm

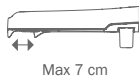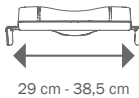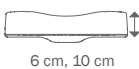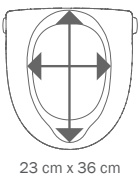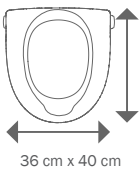



Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit [www.etac.com](http://www.etac.com)

Etac

+46 371 587 330

info@etac.se

[www.etac.com](http://www.etac.com)

**etac**<sup>®</sup>  
Creating Possibilities