



immedia

Immedia LeanOnMe

Relieving and supporting positioning wedges, triangles and cushions

Immedia LeanOnMe



LeanOnMe is a family of soft, moldable cushions, along with firm and stable wedges and triangles. LeanOnMe can be used individually or in combination. They provide the individual body support in either sitting or lying positions, and can also ease caregivers' tasks during care routines.

Two different fillings

Foam & beans



Cushion

The LeanOnMe cushions are filled with a well-balanced mix of granulated viscoelastic polyurethane foam residuals from mattress production and EPP beans (Expanded Polypropylene). The EPP beans are firm, quiet, water resistant and 100% recyclable.

The filling allows the limbs to sink into the soft material and allows molding to individual needs. Its elasticity and repeatability contributes to resilient pressure distribution and load-bearing, even under heavy loads. Both materials return to their original shape, ideal for soft positioning.

All materials are designed to keep noise to a minimum, making the positioning cushions optimal for users with sensitive hearing.

Foam



Wedge, Triangle

The PU foam in the LeanOnMe Wedges and Triangles is a flexible, open-cell material designed for maximum comfort, support, and durability.

Its excellent airflow helps keeping users cool and comfortable, while its ability to distribute weight effectively helps reduce pressure. This makes it an ideal choice for both support and comfort-focused applications.

Three different surfaces

Soft-Touch



Cushion, Wedge, Triangle

All LeanOnMe devices come with a unique soft and skin- friendly surface made from sustainable and natural organic materials. The Soft-Touch textile is a quilted fabric which makes the surface very pleasant and soft allowing the limbs to sink into it.

The Soft-Touch cover on the LeanOnMe Wedges and LeanOnMe Triangles has a non-slip polyurethane material on the bottom that helps to keep the them in place independent of surface material.


Hygienic



Cushion

The LeanOnMe cushions have a uniquely soft and skin-friendly surface that is waterproof and can be wiped off. It is very pleasant and soft, allowing the limbs to sink into it.

Non-Slip

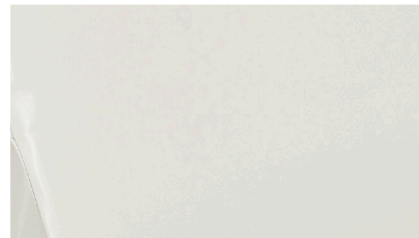

Wedge, Triangle

The Non-Slip surface is unique antislip eco material with a skin frindly surface and cover helps to keep the LeanOnMe Wedges and Triangles in place, independent of surface material.

Additional benefits include:

- Air circulation
- Temperature regulation
- Barrier against dust mites, bacteria, fungi and bed bugs
- Silent materials that provide a minimum of auditive stimuli

Additional benefits include:

- Antibacterial

Additional benefits include:

- STANDARD 100 by OEKO-TEX® certified
- Waterproof and breathable
- Soft and bi-elastic, even after many washes

The LeanOnMe products are produced environmental friendly with consideration for its footprint and a focus on reducing waste and its effects on the environment. This is achieved by using eco-friendly materials that are made from organic ingredients and residual materials from preceding production, as well as both recycled and recyclable packaging.



Material

LeanOnMe Cushions

Inner cushion: 95% polyester, 5% elastane

Filling: granulated residual polyurethane foam, EPP (Expanded Polypropylene) beans

Soft-touch/hygienic cover: 75% Lyocell (Tencel™ + Smartcel™ sensitive), 25% Modal (Tencel™). 100% polyester filling and polyurethane membrane

LeanOnMe Wedges and Triangles

Filling: PU comfort foam

Non-Slip material: 100% polyester with 100% polyurethane

Soft-touch cover: 75% Lyocell (Tencel™ + Smartcel™ sensitive), 25% Modal (Tencel™). 100% polyester filling and polyurethane membrane

Washing instructions



Machine wash - max. temperature 80°C. Tumble drying - low temperature; max. 60°C. Do not use solvents or bleach. Do not iron. Remove inner cushion and foam before washing.

Immedia LeanOnMe Wedge

The wedges provide comfortable support for resting and personal hygiene situations. They offer a firm and stable support and come with two different covers: one with high friction to keep them in place and another with a soft, skin-friendly surface made from sustainable materials, making LeanOnMe Wedges gentle for users with sensitive skin.

The wedges are ideal for individuals who need positioning support. Positioning with LeanOnMe Wedges contributes to spare vulnerable body areas, such as those at risk of pressure injuries. For users with mobility limitations or ageing skin, the wedges can enhance comfort and reduce pressure, providing essential stability during positioning or hygiene routines.

LeanOnMe Wedges can help create a safe, comforting environment, enhancing comfort during rest and care routines.



Positioning options



LeanOnMe Wedges can be used in combination with two or more wedges to support where needed or as an aid for raising up from the bed to standing position.



LeanOnMe Wedges offer many possibilities for supporting the user in different resting positions. In addition to supporting the back, the wedges can also be used to support the legs in supine position.



LeanOnMe Wedges have two angles and can support a stable position when the patients back lies against the wedge or during personal hygiene tasks, or in combination with a LeanOnMe Triangle.

Product name	Size	Item no.
LeanOnMe Wedge XS + Soft-Touch Cover	W300xL265xH200 mm	IM53XSST
LeanOnMe Wedge XS + Non-Slip Cover	W300xL265xH200 mm	IM53XSNS
LeanOnMe Wedge S + Soft-Touch Cover	W300xL530xH200 mm	IM53SST
LeanOnMe Wedge S + Non-Slip Cover	W300xL530xH200 mm	M53SNS
LeanOnMe Wedge L + Soft-Touch Cover	W300xL800xH200 mm	IM5380LST
LeanOnMe Wedge L + Non-Slip Cover	W300xL800xH200 mm	IM5380LNS
Non-Slip Cover for IM5380LNS	W300xL800xH200 mm	IM530080LNS
Non-Slip Cover for IM53SNS	W300xL530xH200 mm	IM5300SNS
Non-Slip Cover for IM53XSNS	W300xL265xH200 mm	IM5300XSNS
Soft-Touch Cover for IM5380LST	W300xL800xH200 mm	IM530080LST
Soft-Touch Cover for IM53SST	W300xL530xH200 mm	IM5300SST
Soft-Touch Cover for IM53XSST	W300xL265xH200 mm	IM5300XSST

Immedia LeanOnMe Triangle

The triangles are versatile positioning aids designed to fill gaps between the body and mattress, providing targeted support that promotes relaxation and stability. The triangles are ideal for addressing gaps under the stomach, hips, or beneath a pillow for additional head support. In this way, they help enhance alignment and support, that can increase comfort and calmness throughout the body.

The triangles help create a soothing, supportive environment that can promote restful sleep, relieves pressure on vulnerable areas, and fosters comfort and calmness during rest and care activities. This makes LeanOnMe Triangles an essential aid for both users and caregivers.

The LeanOnMe Triangle XS is a professional alternative to traditional supports and can fill various gaps. The LeanOnMe Triangle S is perfect for side-lying support, making personal care tasks more accessible and comfortable. It can also provide stable support under the knees or feet.



Positioning options



A multi-purpose support LeanOnMe Triangle XS which is placed where smaller gaps occur between body and mattress when a person is positioned; typically, at the stomach, hips or under the pillow.



LeanOnMe Triangles are ideal for providing a comfortable and stable side-lying position, making tasks such as care routines, dressing in bed, and wound care around the lumbar or pelvic areas easier for the carer.



Positioning the user well supported can contribute to relaxation and sleep..

Product name	Size	Item no.
LeanOnMe Triangle XS + Soft-Touch Cover	W200xL250xH100 mm	IM54XSST
LeanOnMe Triangle XS + Non-Slip Cover	W200xL250xH100 mm	IM54XSNS
LeanOnMe Triangle S + Soft-Touch Cover	W250xL400xH150 mm	IM54SST
LeanOnMe Triangle S + Non-Slip Cover	W250xL400xH150 mm	IM54SNS
Soft-Touch Cover for IM54XSST	W200xL250xH100 mm	IM5400XSST
Non-Slip Cover for IM54XSNS	W200xL250xH100 mm	IM5400XSNS
Soft-Touch Cover for IM54SST	W250xL400xH150 mm	IM5400SST
Non-Slip Cover for IM54SNS	W250xL400xH150 mm	IM5400SNS

Immedia LeanOnMe Roll

The roll can be bent, shaped and shaken so that it becomes fuller in some areas and thinner in others. This makes it adjustable to the user's specific needs.

When the user is lying in lateral side position, the LeanOnMe Roll can advantageously be shaped to support both the head, back, and legs. It can be well suited for users who benefits from tactile stimulation.



Positioning options



When used as a nest, the LeanOnMe Roll can have a pressure- and pain relieving effect on the shoulders, and will elevate the arms and hands. This can contribute to relieved breathing.



When hugged in the lateral position, the LeanOnMe Roll can provide a calming effect, and supports the user in a comfortable position.



When the LeanOnMe Roll is held into the stomach, it can contribute to a calming and stabilizing effect and relieves arms and shoulders.

Product name	Size	Item no.
LeanOnMe Roll S + Soft-Touch Cover	W330xL1000 mm	IM15112
LeanOnMe Roll L + Soft-Touch Cover	W330xL2300 mm	IM15113
LeanOnMe Roll S + Hygienic Cover	W330xL1000 mm	IM16112
LeanOnMe Roll L + Hygienic Cover	W330xL2300 mm	IM16113
Soft-Touch Cover for IM15112	W330xL1000 mm	IM10512
Soft-Touch Cover for IM15113	W330xL2300 mm	IM10513
Hygienic Cover for IM16112	W330xL1000 mm	IM10612
Hygienic Cover for IM16113	W330xL2300 mm	IM10613

Immedia LeanOnMe Channel

The channel-sewn cushion is a flexible cushion that provides support and relief. The cushion can be rolled and shaped depending on the needs.

The LeanOnMe Channel cushion can be placed under the leg(s), between the legs, under the calves to help minimising pressure on the heels, or under an arm to relieve shoulder blades and/or elbows. In a lateral position, the cushion can be placed under the user's back or between the user's legs to help distribute pressure and relieve exposed areas. When sitting, the cushion can be placed in the back as support.



Positioning options



When a LeanOnMe Channel is placed under both legs, it can relief edama. It also can provide pain- and pressure relieving effect.



The LeanOnMe Channel can be used for elevation of legs or for pressure relief on the ankle and/or heel.



In a lateral position a LeanOnMe Channel cushion can be placed between the legs to help exposed areas, such as inner knees and ankles, from pressure.

Product name	Size	Item no.
LeanOnMe Channel + Soft-Touch Cover	W750xL750 mm	IM15108
LeanOnMe Channel + Hygienic Cover	W750xL750 mm	IM16108
Soft-Touch Cover for IM15108	W750xL750 mm	IM10508
Hygienic Cover for IM16108	W750xL750 mm	IM10608

Immedia LeanOnMe Wing

The backrest cushion with a wing is used in a lateral position. The user lies on the wing so that the user's weight holds the cushion in place. Then the user's back is comfortably supported in a stable position

The LeanOnMe Wing cushion can also be used for supine positioning the pillow can be positioned to help minimise pressure on exposed body parts from the bed rails.



Positioning options



When placed in the back in a lateral position with the wing under the user, the cushion stays in place, and ensures the user does not tilt backwards.



The LeanOnMe Wing cushion can protect exposed body parts from pressure from bed rails.



Placed under an arm in supine position, the LeanOnMe Wing can contribute to pressure-, elevating- and pain-relieving effect.

Product name

LeanOnMe Wing + Soft-Touch Cover
LeanOnMe Wing + Hygienic Cover
Soft-Touch Cover for IM15107
Hygienic Cover for IM16107

Size

W450xL800 mm + wing 200 mm
W450xL800 mm + wing 200 mm
W450xL800 mm + wing 200 mm
W450xL800 mm + wing 200 mm

Item no.

IM15107
IM16107
IM10507
IM10607

Immedia LeanOnMe Mini

A small, multi-purpose support cushion which is placed where smaller gaps occur between body and mattress when a person is positioned; e.g. under lower back, at the ankle and knee.

Applying a LeanOnMe Mini can have a relaxing effect on the body part. It can also be used for opening a hand affected by spasticity.



Positioning options



A LeanOnMe Mini placed under the lower back to relieve pressure on the pelvis.



Use a LeanOnMe Mini for elevation and for opening the hand.



The LeanOnMe Mini can be placed where smaller gaps occur.

Product name	Size	Item no.
LeanOnMe Mini + Soft-Touch Cover	W150xL300 mm	IM15100
LeanOnMe Mini + Hygienic Cover	W150xL300 mm	IM16100
Soft-Touch Cover for IM15100	W150xL300 mm	IM10500
Hygienic Cover for IM16100	W150xL300 mm	IM10600

Immedia LeanOnMe Basic

Traditional positioning cushions in six shapes and sizes, The cushions can be used in positioning to support the back, arms, legs, knees, and ankles.

The LeanOnMe Basic positioning cushions can also be used to reduce pressure on exposed body parts from bed rails.



Positioning options



LeanOnMe Basic cushions can be used in supine position for supporting the body or extremities.



A LeanOnMe Basic cushion can be held onto the stomach for a pain relieving and/or calming effect.



Elevation of arm and hand for minimising the risk of contractures, edema and pressure.

Product name	Size	Item no.
LeanOnMe Basic XS +Soft-Touch Cover	W300xL400 mm	IM15101
LeanOnMe Basic S + Soft-Touch Cover	W400xL600 mm	IM15102
LeanOnMe Basic M + Soft-Touch Cover	W500xL700 mm	IM15103
LeanOnMe Basic L + Soft-Touch Cover	W250xL800 mm	IM15104
LeanOnMe Basic LW + Soft-Touch Cover	W450xL800 mm	IM15105
LeanOnMe Basic XL + Soft-Touch Cover	W550xL900 mm	IM15106
LeanOnMe Basic XS + Hygienic Cover	W300xL400 mm	IM16101
LeanOnMe Basic S + Hygienic Cover	W400xL600 mm	IM16102
LeanOnMe Basic M + Hygienic Cover	W500xL700 mm	IM16103
LeanOnMe Basic L + Hygienic Cover	W250xL800 mm	IM16104
LeanOnMe Basic LW + Hygienic Cover	W450xL800 mm	IM16105
LeanOnMe Basic XL + Hygienic Cover	W550xL900 mm	IM16106
Soft-Touch Cover for IM15101	W300xL400 mm	IM10501
Soft-Touch Cover for IM15102	W400xL600 mm	IM10502
Soft-Touch Cover for IM15103	W500xL700 mm	IM10503
Soft-Touch Cover for IM15104	W250xL800 mm	IM10504
Soft-Touch Cover for IM15105	W450xL800 mm	IM10505
Soft-Touch Cover for IM15106	W550xL900 mm	IM10506
Hygienic Cover for IM16101	W300xL400 mm	IM10601
Hygienic Cover for IM16102	W400xL600 mm	IM10602
Hygienic Cover for IM16103	W500xL700 mm	IM10603
Hygienic Cover for IM16104	W250xL800 mm	IM10604
Hygienic Cover for IM16105	W450xL800 mm	IM10605
Hygienic Cover for IM16106	W550xL900 mm	IM10606

Immedia LeanOnMe Ring

The ring cushion is used to provide stable and relieving support when a normal pillow is not considered appropriate.

It is well suited for users who are in risk of developing occipital pressure ulcers. It can also be used in a half-seated bed or armchair to provide good headrest for users with poor head control.

The LeanOnMe Ring can easily be taken on trips to relieve sore neck muscles and vertebrae.



Positioning options



For relieving pressure on ears, and/or pain relief of neck muscles when lying in lateral position.



For pressure relief on the back of the head, and/or for head control in lying position.



Use the LeanOnMe Ring to support the head in a sitting position.

Product name

Size

Item no.

LeanOnMe Ring S + Soft-Touch Cover

Ø400 mm

IM15110

LeanOnMe Ring M + Soft-Touch Cover

Ø500 mm

IM15111

LeanOnMe Ring S+ Hygienic Cover

Ø400 mm

IM16110

LeanOnMe Ring M + Hygienic Cover

Ø500 mm

IM16111

Soft-Touch Cover for IM15110

Ø400 mm

IM10510

Soft-Touch Cover for IM15111

Ø500 mm

IM10511

Hygienic Cover for IM16110

Ø400 mm

IM10610

Hygienic Cover for IM16111

Ø500 mm

IM10611



Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit www.etac.com

Etac



+46 371 587 330



info@etac.se



www.etac.com

etac®

Creating Possibilities