



etac®

Etac Swift Mobil-2

Mobile shower commode chairs

Introducing the Swift Mobil-2 family

Swift Mobil-2 shower commode chairs are easy to manoeuvre and simplify patient transfers as well as transportation to and from the bathroom. They are designed to make life easier for the people who use them. We understand what's important to carers and users, because we ask. By listening to our customers, we understand the importance of things like ease of access, adaptability, dignity and comfort.



The Swift Mobil-2 family comes with a 5-year warranty and a 10-year service life.

Facilitates access

A majority of carers express that lack of access is a challenge during personal care routines and often force strained work positions. To facilitate access, the Swift Mobil-2 seat has a rear opening and free space on both sides. This seemingly minor adjustment will help make personal hygiene routines swifter and more dignified for users as well as their caregivers.

Product flexibility and comfort

All Swift Mobil-2 models have a generous maximum user weight of 160 kg and can easily be adapted to smaller or larger users with various conditions and needs. The chairs are adjustable in many ways and have a wide range of accessories to enhance comfort and stability. Making a few small adjustments can make a big difference to the safety and well-being of the user and the working conditions for the carer.

Discover the benefits

A range of functional and flexible mobile shower commode chairs, designed to facilitate personal care activities for both carer and user. Swift Mobil-2 is offered in three different models; a standard shower commode chair, a tilt-in-space model and a 24" self-propelled model.

Multi-adjustable head support

Designed to provide maximum comfort and flexibility and can be adjusted to suit a range of user needs.

Comfortable back support

Adjustable and quick-drying. Stable fit with double fasteners around the pushbar.

Convenient access

Seat with rear opening, front recess and free space on both sides.

Height adjustable

Easy to adjust to different fixed seat heights, without using tools.



Swift Mobil-2



Swift Mobil 24"-2

Flexible arm support

Adjustable in height or width.

"Etac Easy-Clean"

A unique surface especially developed for Etac with a perfect balance between friction and cleanliness, non-slip and hygienic!

Ergonomic foot support

The gently curved footplates provide instep and arch support, ensuring comfort, stability and relaxation.



Swift Mobil Tilt-2

Swift Mobil-2

The Swift Mobil-2 is a flexible attendant-propelled shower commode chair that gives carers exceptional access for personal care tasks. It is easy to manoeuvre and can be customised to many different user needs.

Unique seat

The seat has a rear opening, giving the carer easy access when assisting the user with personal care tasks. Additionally, the “Etac Easy Clean” friction pattern on the seat surface is unique for Etac and provides the perfect balance between friction and cleanliness.

Comfortable relief

For sensitive users, extra-soft seat cushions are available, providing both support and comfortable relief. The soft and quick-drying back supports and the curved foot supports increase user comfort further.

Highly Adaptable

The Swift Mobil-2 has a generous maximum user weight of 160 kg as standard and is height adjustable between 45-60 cm. The arm supports are height and width adjustable, 6 cm in each direction. The swing away foot supports are height adjustable and are also offered in a shorter version, e.g. when higher soft seats are used.

Configuration options

Configurations with a wider back frame is available for larger users, providing an extra 6 cm between the arm supports. These models also have an increased width of 12 cm between the leg supports, thanks to widening adapters. The XL-back frame and the leg support adapters are available as accessories for other Swift Mobil variants.

The Swift Mobil-2 also offers an upright back configuration that, combined with the upright head support, allows the chair to be positioned 7 cm closer to the wall, ensuring a better fit over wall-mounted toilets.



Swift Mobil-2

| | |
|--|----------|
| Swift Mobil-2, partly assembled | 80229400 |
| Swift Mobil-2, assembled | 80229401 |
| Swift Mobil-2, with pan holder | 80229406 |
| Swift Mobil-2 XL, partly assembled with XL back | 80229467 |
| Swift Mobil-2 XL, partly assembled with pan holder and pan | 80229483 |
| Swift Mobil-2, partly assembled with pan holder and upright back | 80229490 |

Standard: back support, seat, arm support, foot support and 4 lockable castors

Weight

16,6 kg

Dimensions

Total width: 58/64 cm. Width between arm supports: 42/48/54 cm.
Total width XL: 61/64/70 cm. Width between arm supports: 48/54/60 cm.

Materials

Chassis: powder painted steel
Seat/Arm support: polypropylene
Back support: polyester/Velcro
Foot support: polypropylene
Castors: polyamid/TPE, lockable, ball bearing



max
160
kg

max
353
lbs



Manual cleaning: Clean the product with a solvent-free cleaning agent with a pH level between 6–9, or with a 70% disinfectant solution.

Machine cleaning: The product can be cleaned in a cabinet washer-disinfector at max. 85 °C / 185 °F for 3 minutes.

Max user weight: 160 kg / 353 lbs

Features




Seat with rear opening

Providing ease of access for the carer and comfort for the user. The unique surface, especially developed for Etac, has the perfect balance between friction and cleanliness, non-slip and hygienic!



Adjustable back support

Easily adjusted for posture, body shape and maximum comfort. Fitted with two fasteners around the pushbars. Soft and quick-drying material.

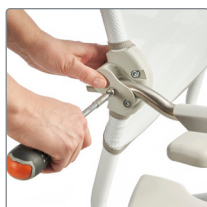

Adjustable arm support as standard

Easily adapted to various user needs. Adjustable both in height and width, +/- 6 cm in each direction.



Ergonomic foot support

Curved footplates provide instep and arch support, ensuring comfort, stability and relaxation.



Swift Mobil Tilt-2

The Swift Mobil Tilt-2 is designed with both carer and user needs in mind. It handles the multitude of challenges faced by caregivers in their daily work and provides the comfort and safety users need to feel secure.

Unique tilt-in-space function

The Swift Mobil Tilt-2, has a unique tilt feature that gives increased security for the user and an ergonomic working position for the carer. Its -5° to 35° tilt function can be operated from either side of the chair. This allows the carer to reassure the user when tilting.

Ease of access and comfort

With ease of access around the seat area, personal care routines are made easier for the carer and can be performed with dignity for the user. For sensitive users, extra-soft, seat cushions are available. The soft and quick-drying back support and curved foot support of the Swift Mobil-2, add to user comfort.

Fits many users

The shower commode chair is adaptable to fit many user needs and sizes. It has a generous maximum user weight of 160 kg as standard, and seat-to-floor height adjustability (50-65 cm). The arm support are adjustable in height or width, and the height adjustable, swing away foot support are also offered in a shorter version.

Swift Mobil Tilt-2 XL

Models for larger users are available with an additional 12 cm between the leg supports, and a wider back frame providing an extra 6 cm between the arm support.



Swift Mobil Tilt-2

Item no.

| | |
|--|----------|
| Swift Mobil Tilt-2, assembled | 80229410 |
| Swift Mobil Tilt-2, with bucket holder | 80229411 |
| Swift Mobil Tilt-2, with pan holder | 80229412 |
| Swift Mobil Tilt-2, with pan holder and pan | 80229415 |
| Swift Mobil Tilt-2 XL back | 80229480 |
| Swift Mobil Tilt-2 XL back, assembled with pan holder | 80229481 |
| Swift Mobil Tilt-2 XL back, assembled with bucket holder | 80229485 |

Standard: head support, back support, seat, arm support, foot support and 4 lockable castors

Weight

20,4 kg

Dimensions

Total width: 58/64 cm. Width between arm supports: 42/48/54 cm
Total width XL: 61/64/70 cm. Width between arm supports: 48/54/60 cm.

Materials

Chassis: powder painted steel
Seat/Arm support : polypropylene
Back support: polyester/Velcro
Foot support: polypropylene
Castors: polyamid/TPE, lockable, ball bearing



max
160 kg
max
353 lbs



Manual cleaning: Clean the product with a solvent-free cleaning agent with a pH level between 6-9, or with a 70% disinfectant solution.

Machine cleaning: The product can be cleaned in a cabinet washer-disinfector at max. 85 °C / 185 °F for 3 minutes.

Max user weight: 160 kg / 353 lbs

Features



Seat with rear opening

Providing ease of access for the carer and comfort for the user. The unique surface is especially developed for Etac and has the perfect balance between friction and cleanliness, non-slip and hygienic!



Unique and safe tilting function

Can be operated from either side of the chair enabling a good working position for the carer and reassurance for the user during tilting.



Adjustable arm support as standard

Easily adapted to various user needs. Adjustable both in height and in width, +/- 6 cm in each direction.



Multi-adjustable head support

The head support is designed to provide maximum comfort and flexibility and can be adjusted to suit a range of user needs.



Swift Mobil 24"-2

Swift Mobil 24"-2 is a self-propelled model, designed for the independent user.

Easy to manoeuvre

The shower commode chair is simple to use and easy to manoeuvre. The rear wheels are positioned for maximum control and are easy to reach. Flip up arm supports and generous space in front of the wheels, allow for easy side transfers. The relatively even shape of the seating area also simplifies side transfers.

Comfortable seat

To facilitate access, the seat has a rear opening and a hygiene recess in front. The seat is designed to provide comfort, regardless of the users posture or pelvis width. For additional comfort, ergonomic soft seats are available.

Fits many user needs

Swift Mobil 24"-2 is adaptable to fit many user needs and sizes. It has a generous maximum user weight of 160 kg as standard and adjustable seat-to-floor height (45-60 cm). The arm supports are adjustable both in height and width, and the soft back support can be adjusted for positioning and comfort.

For larger users, an extra wide back frame is offered, providing an additional 6 cm between the arm supports. There is also a widening kit for the leg supports available, which increases the legroom with 12 cm.



Swift Mobil 24"-2

Item no.

| | |
|---------------------------------------|----------|
| Swift Mobil 24"-2 | 80229402 |
| Swift Mobil 24"-2, with bucket holder | 80229407 |
| Swift Mobil 24"-2, with pan holder | 80229408 |

Standard: back support, seat, arm support, foot support and 4 lockable castors

Materials

Chassis: powder painted steel
Seat/Arm supports: polypropylene
Back supports: polyester/Velcro
Foot supports: polypropylene
Castors: polyamid/TPE, lockable, ball bearing
Rear wheels: solid tyres, polyamid, ball bearing
Brakes: polyamid, stainless steel

Weight

25,1 kg

Dimensions

Total width: 74 cm
Width between arm supports: 42/48/54 cm



max
160 kg
max
353 lbs



Manual cleaning: Clean the product with a solvent-free cleaning agent with a pH level between 6-9, or with a 70% disinfectant solution.
Machine cleaning: The product can be cleaned in a cabinet washer-disinfector at max. 85 °C / 185 °F for 3 minutes.
Max user weight: 160 kg / 353 lbs

Features



Easy to manoeuvre

The position of the rear wheels makes it easy to propel the chair forward, as well as simplifying side transfers.



Adjustable back

Easily adjusted for posture, body shape and maximum comfort. Fitted with two fasteners around the pushbars. Soft and quick-drying material.



Adjustable seat height

Swift Mobil 24"-2 can be easily set at various fixed heights with toolfree adjustment ensuring a safe and comfortable height.



Ergonomic foot support

Curved footplates provide instep and arch support, ensuring comfort, stability and relaxation.



Accessories



Seat closed rear

84001337

A seat that is closed at the rear and features a hygiene opening at the front.
Material: Polypropylene



Soft cushion with oval opening

80209067

Foam cushion with a cover made of a water-repellent fabric. Height 4 cm.
Material: Polyether/coated plastic



Soft comfort seat, 4 cm rear opening

80209519

Ergonomically shaped with substantial softness providing comfortable relief and support. Designed with a recess at the front.
Material: Polyurethane



Head support

80209521

Multi-adjustable head support with shaped pillow. Adjustable in height, depth, sideways and in angle. Complete with bracket.
Material: Pillow: Polyurethane
Brace: Stainless steel, Polypropylene



Soft comfort seat, 2 cm rear opening

80209518

Designed with substantial softness and round shapes for maximum comfort. Designed with a recess at the front.
Material: Polyurethane



Head support upright

80209533

Closer placement to the wall in combination with Upright back. Sharing features with Head support, ensuring functionality and comfort.
Material: Pillow: Polyurethane
Brace: Stainless steel



Soft comfort seat, 4 cm

Small opening 15 cm (6"): 80209507
Large opening 18 cm (7"): 80209506
Ergonomically shaped soft comfort seats with recesses at the front. Height is 4 cm at the lowest point.
Material: Polyurethane



Comfort cover

80209455

Soft back with integrated arm support cushions. The comfort padding provides a soft support while the gripping surface on the arm support is kept free.
Material: Polyethylene



Soft comfort seat, 2 cm

Opening 18 cm: 80209306
Shaped, soft comfort seat with recess at the front. Height is 2 cm at the lowest point.
Material: Polyurethane



Side support

80209509

Offers a soft side support and facilitates positioning in the chair. Adjustable in width.
Material: Polyester fabric, Velcro, Polyethylene



Seat cover

80209434

A comfortable, closed seat when you want to cover the hygiene opening. Can be used in the shower. Height 2 cm.
Material: Polyurethane



Standard back

80209538

Fits most users, includes soft mesh back support and arm support. Compatible with Wipeable back. Adjustable arm supports by 6 cm each way. Material: Polyester, Powder coated steel, stainless steel, polypropylene



Soft cushion with standard opening

80209065

Foam cushion with a cover made of a water-repellent fabric. Height 2 cm.
Material: Polyether/coated plastic



XL-back

80209454

For larger users, offering more room for back and shoulders. Increases the width between the arm support to a total width of 48/54/60.
Material: Polyester, Powder coated steel, stainless steel, polypropylene

Accessories



Upright back

80209529

For an optimal fit over wall-mounted toilets. Allows the chair to be placed 70 mm closer to the wall compared to a standard back frame. Material: As XL back



Wipeable back support

80209530

Comfortable with an ergonomic shape and flexible lower section. Height adjustable for optimal support. Quick-wipe material and suitable for washing machine or tunnel. Easy to mount. Material: Polypropylene



Cross bar

60.5 cm 80303034

66.5 cm 80209452

A safe support to hold on to or it can be used as a strain support. Attaches to the armrests. Material: Aluminium, polyethylene



Soft back pad

80209448

Provides the user with warmth and softness. Used inside the arm supports, which makes it possible to fold up the arm support and create accessibility for side transfer. Material: Polyethylene



Arm support pad

80209508

Used to create softness around the arm supports. Provides a warm, soft surface to support the arms while keeping the gripping surface free, making it safe to stand up. Sold in pairs. Material: Polyethylene



Arm support Hemiplegia

80209035


Provides support for the forearm with its contoured shape. Equipped with a heat-reflecting material on the outside. Fits the left and right arm support. Material: Polyethylene



Arm support lock

80209445


A locking pin to be used when you want to lock the armrest in the folded-down or folded-up position. Material: Stainless steel

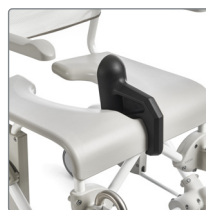

Splash guard

80209430

A soft protection against urine splashes at the front. Attaches to the front recess of the seat and can easily be removed. Material: Polyurethane

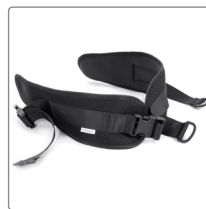

Hip/chestbelt upholstered

80209524

2-part belt for positioning that can be used either as hip- or chest belt. Adjustable with 3 buckles with up to 65-130 cm length. The material is comfortable and quick-drying and provides firm support when positioning.



Hip belt

80209440

Easy to adjust with a plastic buckle. Attaches around the back tubes in the rear with velcro straps. Material: Polyester



Chest belt

80209442

Easy to adjust and attaches around the straps of the back support upholstery. Use with a hip belt is recommended. Material: Polyester



Pan with lid and handle

80209255

Unique interior shape that breaks the stream and prevents splashes. The lid has a handle that locks onto the pan and ensures that the content does not spill. Material: Polypropylene



Pan

Pan without lid 80209254

Lid with handle 80209256

Unique interior shape that breaks the stream and prevents splashes. Material: Polypropylene



Pan holder

80209257

Suitable for both the Etac pan and traditional hospital pans. Used together with the attachment holders, which are ordered separately. Material: Polypropylene

Accessories



Pan locker

80209049

Holds the pan in place when the shower chair is tilted. Compatible with Swift Mobil Tilt-2 variants and Alto.

Material: Polypropylene



Attachment holder

80209435

To be mounted firmly under the seat enabling the use of pan holders. Fitted with a screwdriver. Complete set.

Material: Polyamide



Low mounting of pan/bucket

80209259

Used when you want the pan/bucket placed lower under the seat. Mounted with the pan or the bucket holders. Lowers the bed pan/bucket holders by 6 cm.

Material: Polypropylene



Bucket holder

80209258

Used as a holder for pans/buckets or bed pans with higher edges. Used together with the attachment holders, which are ordered separately.

Material: Polypropylene



Bucket with handle and lid

80209265

Traditional bucket that is used with a bucket holder.

Material: Polypropylene



Anti-tip back Swift Mobil Tilt-2

80209459

Provides protection against the chair tipping backwards. Increases safety when the centre of gravity falls far to the back.

Material: Polypropylene and Aluminium



Anti-tip back Swift Mobil-2

Anti-tip Including attachment: 80209439

Anti-tip excluding attachment: 80209438

Provides protection against the chair tipping backwards. Sold as a pair.

Material: Polypropylene and Aluminium



Anti-tip front Swift Mobil-2

80209526

Provides protection against the chair tipping forward. Includes attachment. Sold as a pair. Fits all Swift Mobil models.

Material: Stainless steel and polypropylene



Tilter

80209449

Facilitates transfer over e.g. thresholds or ramps. Sold complete with bracket.

Material: Polypropylene and Aluminium



Short foot support

80209520

A short foot support that swings sideways. Can be adjusted 12.5 cm and provides a footrest length of 24 - 36.5 cm. The foot plate has a rounded shape. Sold as a pair.

Material: stainless steel and polypropylene



Leg support with flip-up footplates

80209446

Height adjustable and swing-away leg supports with foldable footplates. For those who need a more open foot angle. Heel straps are included. Sold as a pair.

Material: Stainless steel and polypropylene



Widening kit leg support

80209525

Adapters for extra legroom. Increase the width between leg supports with 12 cm. Included in all XL/wide-back configurations. Sold as a pair.

Material: Polyamide



Leg and calf support

80209437

Leg support with an adjustable angle that can be used for both left and right legs. Comes with 4 fixed-angle positions. Sold per piece.

Material: Stainless steel and ABS-plastic



Cover leg and calf support

80209535

The soft surface enhances the comfort of the calf and leg support. Easy to fit by sliding it over the support. Clean quickly with a simple wipe. Material: Polyester, Polyurethane, Nylon, Spandex

Accessories



Foot support for Leg and calf support

80209443

Kit with separate foot plate to be mounted together with leg and calf support.

Material: ABS plastic



Amputee leg support

80209510

Adjustable in height, length and angle. Can be used on both left and right sides. If the user is an amputee we recommend using anti-tippers.

Material: Stainless steel and polyurethane



Angled leg support plus

80209531

Brings the foot support 5 cm further away from the frame. Sold as a pair.

Material: Stainless steel and polyurethane



Heel straps

80209432

Mounted on foot plates. Prevents the feet from sliding backwards. Adjustable in length. Sold as a pair.

Material: Polyester



Angled leg support minus

80209532

Brings the foot support 4 cm closer to the frame. Sold as a pair.

Material: Stainless steel and polyurethane



Rear wheel kit 24"

80209436

24" rear wheels with solid tyres and built-in tipping guards. Easy to mount. Includes attachment and brake. Fits Swift Mobil, not Swift Mobil Tilt. Material: Polyamide, stainless steel, polyurethane aluminium

See washing instructions on page 10.



Washing instructions

Seat closed rear, Soft Comfort seats and Seat cover, Head support, Head support upright, Arm support lock, Splash guard, Pan, Pan holder, Attachment holder, Low mounting attachment, Bucket holder, Bucket, Anti-tip, Tilter, Foot supports, Leg and calf support, Amputee leg support, Angles leg supports, Rear wheel 24" kit, Wipeable back support



Back support upholstery, Hip/chestbelt upholstered, Hip belt, Chest belt, Heel strap



Soft cushions, Comfort cover, Soft back pad, Arm support pad, Arm support hemiplegia, Cross bar, Cover leg and calf support



Washing symbols explained



Machine wash – max. temperature 60 °C / 140 °F



Manual cleaning: Clean the product with a solvent-free cleaning agent with a pH level between 6–9, or with a 70% disinfectant solution.



Machine cleaning: The product can be cleaned in a cabinet washer-disinfector at max. 85 °C / 185 °F for 3 minutes.


Measurements



| Model | Length | Width | Seat height | Back height | Seat depth |
|-----------------------|--------|-------------|----------------|-------------|------------|
| Swift Mobil-2 | 93 cm | 58/64 cm | 44 / 59 cm | 50 cm | 46,5 cm |
| Swift Mobil-2 XL | 93 cm | 61/64/70 cm | 44 / 59 cm | 50 cm | 46,5 cm |
| Swift Mobil 24"-2 | 106 cm | 74 cm | 45 / 60 cm | 50 cm | 46,5 cm |
| Swift Mobil Tilt-2 | 102 cm | 58/64 cm | 47,5 / 62,5 cm | 55 cm | 46,5 cm |
| Swift Mobil Tilt-2 XL | 102 cm | 61/64/70 cm | 47,5 / 62,5 cm | 55 cm | 46,5 cm |



| Model | Foot support height | Seat width | Arm support height | Arm support to arm support | Depth against wall |
|-----------------------|---------------------|------------|--------------------|----------------------------|--------------------|
| Swift Mobil-2 | 34 / 46,5 cm | 50 cm | 22/25/28 cm | 42/48/54 cm | 54 cm (47 cm*) |
| Swift Mobil-2 XL | 34 / 46,5 cm | 50 cm | 22/25/28 cm | 48/54/60 cm | 54 cm |
| Swift Mobil 24"-2 | 34 / 46,5 cm | 50 cm | 22/25/28 cm | 42/48/54 cm | 67 cm |
| Swift Mobil Tilt-2 | 34 / 46,5 cm | 50 cm | 22/25/28 cm | 42/48/54 cm | 60 cm |
| Swift Mobil Tilt-2 XL | 34 / 46,5 cm | 50 cm | 22/25/28 cm | 48/54/60 cm | 60 cm |

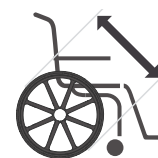

| Model | Height | Castors | Clearance over toilet | Clearance over toilet | Rear wheel |
|-----------------------|------------|---------|-----------------------|-----------------------|------------|
| Swift Mobil-2 | 95-110 cm | 12,5 cm | 40 cm | 40-55 cm | - |
| Swift Mobil-2 XL | 95-110 cm | 12,5 cm | 40 cm | 40-50 cm | - |
| Swift Mobil 24"-2 | 95-110 cm | 12,5 cm | 40 cm | 41-56 cm | 61 cm |
| Swift Mobil Tilt-2 | 100-115 cm | 12,5 cm | 40 cm | 43,5-58,5 cm | - |
| Swift Mobil Tilt-2 XL | 100-115 cm | 12,5 cm | 40 cm | 43,5-58,5 cm | - |



| Model | Turning radius | Max user weight | Weight | | |
|-----------------------|----------------|-----------------|---------|--|--|
| Swift Mobil-2 | ø 99 cm | 160 kg | 16,6 kg | | |
| Swift Mobil-2 XL | ø 99 cm | 160 kg | 17,3 kg | | |
| Swift Mobil 24"-2 | ø 121 cm | 160 kg | 25,1 kg | | |
| Swift Mobil Tilt-2 | ø 104 cm | 160 kg | 20,4 kg | | |
| Swift Mobil Tilt-2 XL | ø 104 cm | 160 kg | 21 kg | | |

* 8022 9489, 8022 9490, 8022 9491



Etac is a world-leading developer of innovative assistive devices and patient handling equipment. Since 1973, we have been committed to improving quality of life for the individual, family members and caregivers.

For the intended purpose of the products and latest information, please visit www.etac.com

Etac

+46 371 587 330

info@etac.se

www.etac.com

etac[®]
Creating Possibilities