


QG62001 - 2024-05-28 - Rev. 1.0


Molift Assist Strap

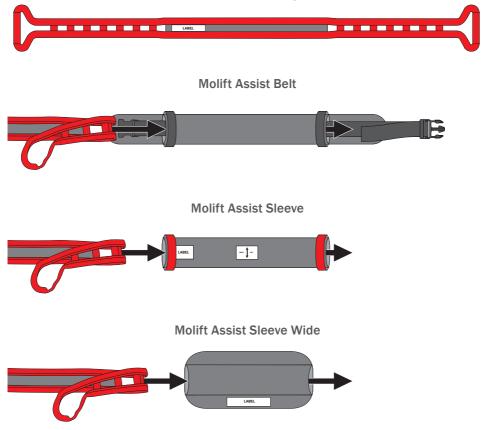











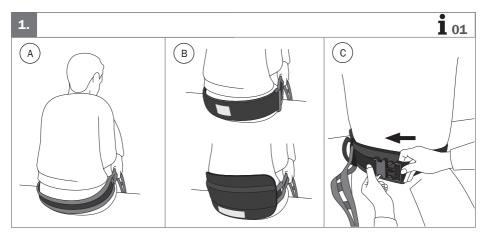

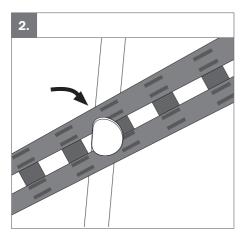


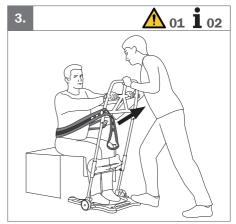

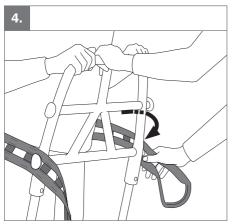


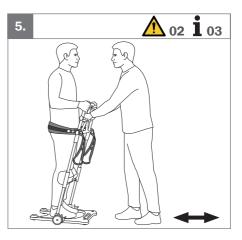




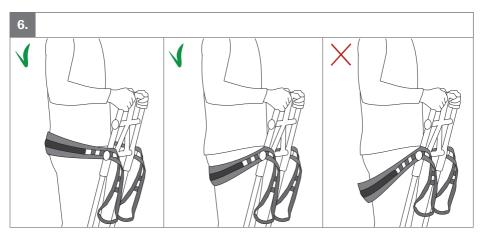




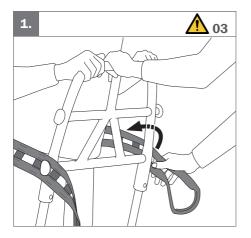


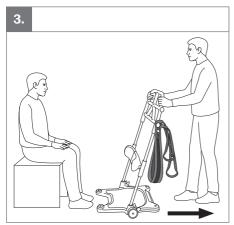







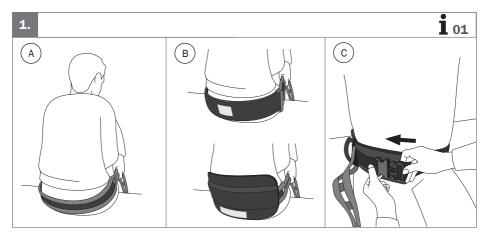

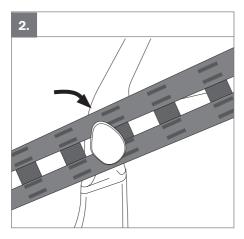




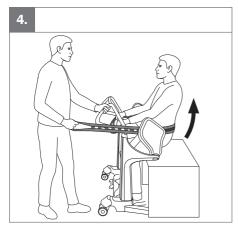


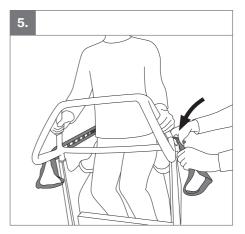





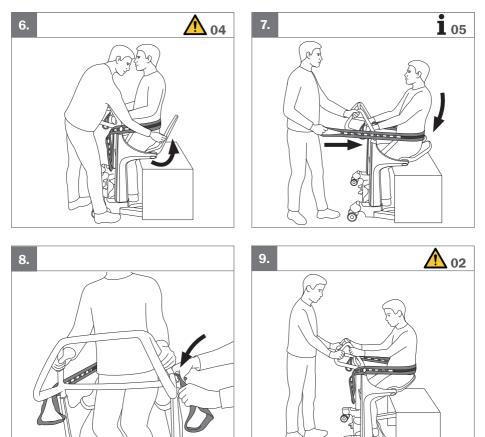




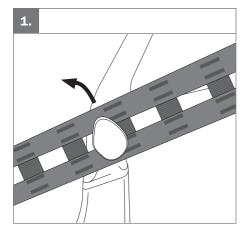

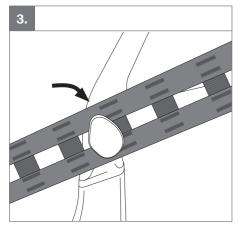

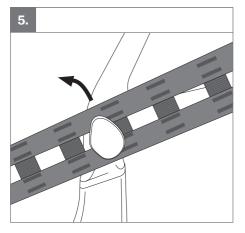


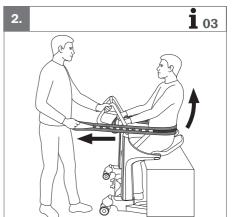







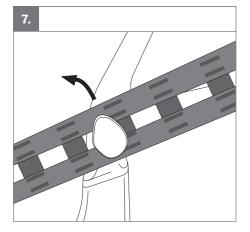



Handling



- 01. If using Sleeve, make sure to grab the handle on Assist Strap, not the handle of Sleeve.
- 02. The user should always hold on to the handles during transfer.
- 03. Make sure the user is weight-bearing when releasing Assist Strap. Only release in one side when lowering the user back to a sitting position.
- 04. The user must remain standing until both seat pads are rotated down.

Information

- 01. Choose applicable application using either method A (Strap), B (Sleeve/Sleeve Wide) or C (Belt).
- 02. Lock the wheels, and place one foot on the edge of the base plate. The carer must always counterbalance the user's weight when the user stands up or sits down.
- 03. Before performing the transfer make sure that the strap is tense and placed either underneath the buttocks or a hip height.
- 04. Make sure the straps are placed on the inside of the seat pads.
- 05. Adjust the strap attachment to loosen the strap while the user is seated on the seat pads.

Cleaning







 $igate{A} \otimes igate{A} \otimes igate{A}$

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.



This is a Quick Guide. The latest version of all user manuals can be downloaded as a "pdf-file" from the www.etac.com



Etac A/S

Parallelvej 3 DK-8751 Gedved



+45 796 858 33

patienthandling@etac.com



www.etac.com