









Quick Guide – EN






Molift Transfer Pro




molift[®]
by Etac

QG70101 - 2024-02-20 - Rev. 1.0

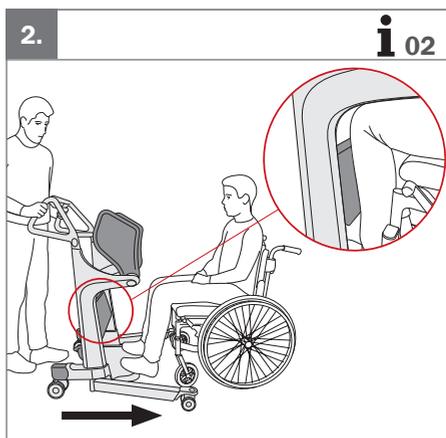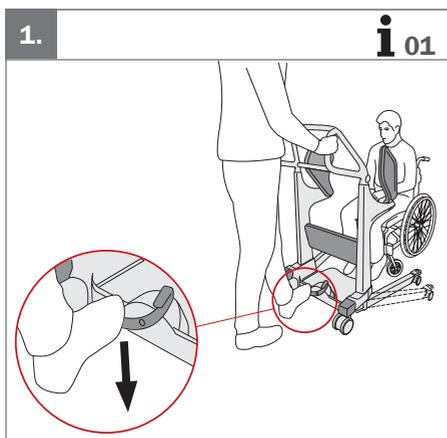

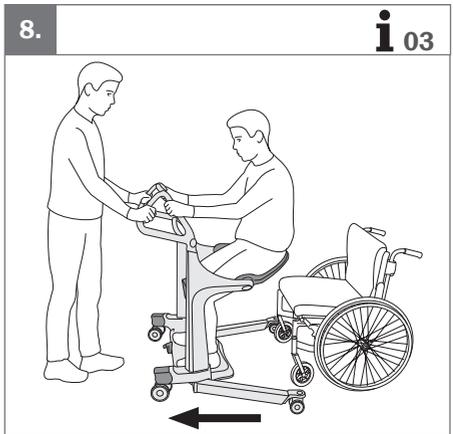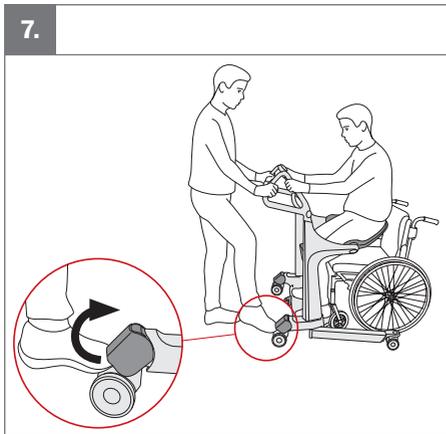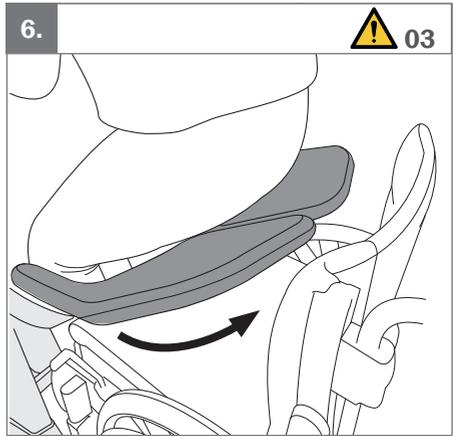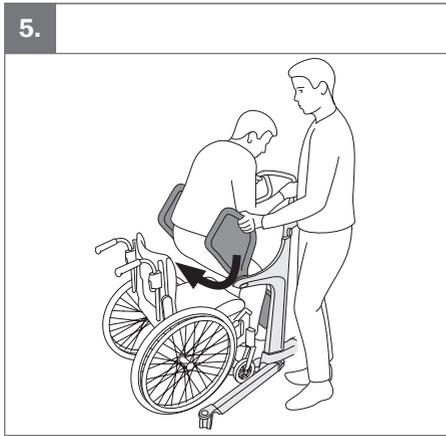

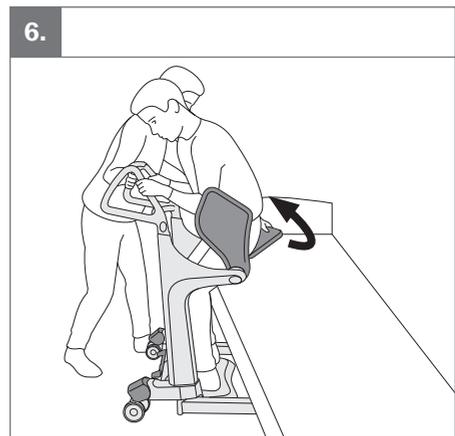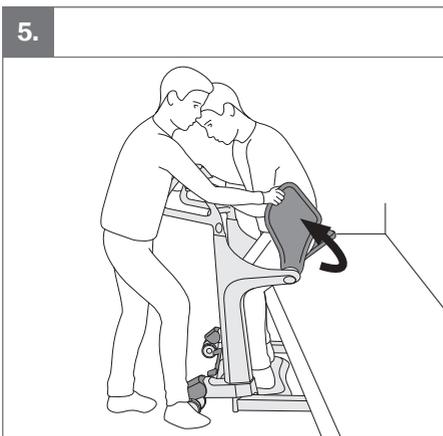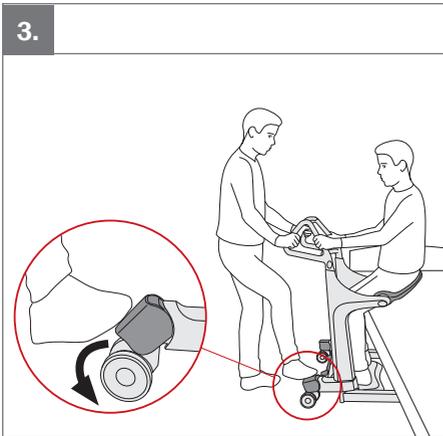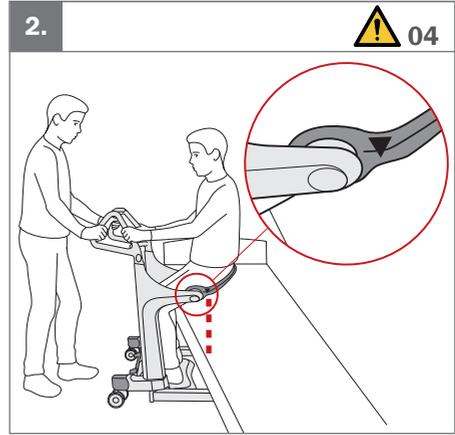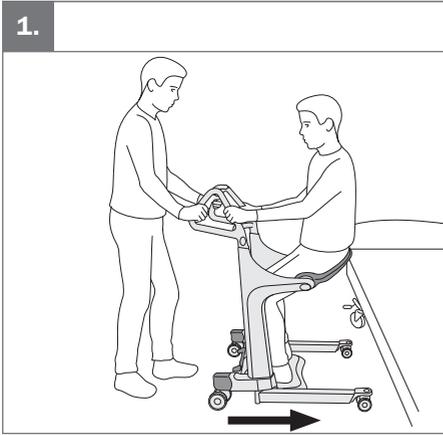

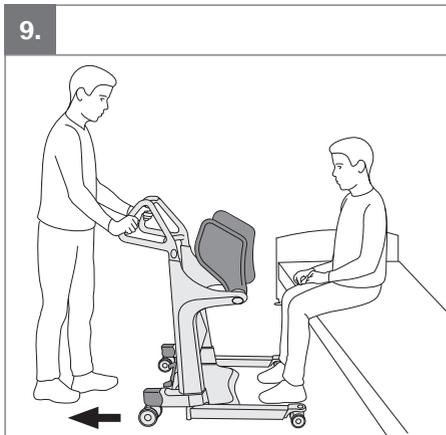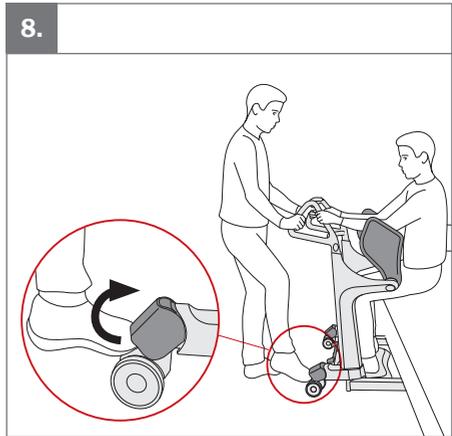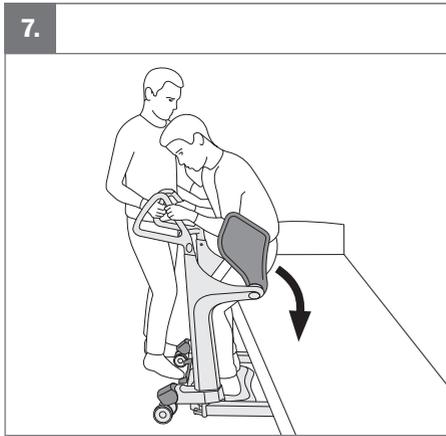

Handling



Warnings

01. The wheels must always be locked when the user moves between sitting and standing position.
02. The user should always hold on to the handles during transfer.
03. The user must remain standing until both seat pads are rotated down.
04. Ensure that the marking on the seat pad minimum aligns with the seat's edge before the user sits down.
05. Always rotate both seat pads up before the user sits down.



Information

01. Adjust the legs adjustment using the pedal. Left: wider. Right: narrower.
02. Place Transfer Pro so leg support reaches the user's shins.
03. It is optional to move the legs in during transfer.

Technical data



28,2kg
62lbs



max 200 kg
max 440 lbs



max 200cm
max 149cm

Cleaning



This is a Quick Guide. The latest version of all user manuals can be downloaded as a "pdf-file" from the www.etac.com



Etac A/S

Parallevej 3

DK-8751 Gedved



+45 796 858 33



patienthandling@etac.com



www.etac.com

molift®
by Etac