sv

Svenska

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English

Yearly maintenance

A technical aid service technician or a person with similar technical expertise should carry out maintenance once a year in accordance with the reconditioning/reuse checklist.

Cleaning

Clean the Lille Viking by wiping down the product using a damp cloth with a non-abrasive cleaning agent with a pH level between 5 and 9, or with a 70% disinfectant solution. Do not rinse the electrical parts of the product.

Reconditioning / Reuse:

To ensure reusers of the Lille Viking auxiliary motor receive an aid that is as secure and in as good condition as possible, it is important to carry out a technical inspection on the aid and clean it thoroughly before sending it to the new user.

Aid number:

Unique ID number:

Model Lille Viking 2015

☐

Serial number/manufacturing number:

Maintenance/reuse checklist:

Battery voltage check (24 volt)

☐

Battery pole connection check

☐

Charging contact connection check

☐

Charging voltage check

☐

Ignition key and main power switch check

☐

Battery fuse check

☐

Control unit function and connection check

☐

Throttle function, from 0 to max. 6 km/h

☐

Full set of assembly hardware check

☐

Cleaning the product

☐

Manual release handle check - operating mode

☐

Manual release handle check - released mode

☐

Motive force check

☐

Brake effect check

☐

Battery holder replacement if necessary

☐

Work carried out by:

Date:

English

Troubleshooting:

LED code		Explanation
LED not lit		no power or control circuit fault
The LED is lit all the time		the control circuit works, no fault
1,1	0 0	shut down due to temperature fault
1,2	0 00	throttle fault
1,3	0 000	fault in potentiometer for speed limitation
1,4	0 0000	voltage too low
1,5	0 00000	voltage too high
2,1	00 0	main contactor not released
2,2	00 00	main contactor has welded
2,3	00 000	DNC main contactor fault
2,4	00 0000	main contactor does not connect
3,1*	000 0	fault in processor/wiring (HPD fault >10s)
3,2	000 00	brake does not engage
3,3	000 000	charging fault
3,4	000 0000	brake does not release
3,5	000 00000	HPD (High Pedal Disable) fault
4,1*	0000 0	current sensor fault
4,2*	0000 00	motor voltage error (safety lock)
4,3*#	0000 000	EEPROM fault
4,4*	0000 0000	power circuit fault

*= acknowledged by turning the key-operated switch off and on

#= acknowledged via programming: select Program Menu, change value of any parameter, turn the key-operated switch off and on.

N.B. Only one fault at a time is indicated; the faults are queued.

English

Troubleshooting:

LED code	Shown in display	Explanation	Possible cause
1,1	THERMAL CUTBACK	shutdown due to too high/ low temperature	1. Temperature >92° or <-25°. 2. Too much weight on the chair. 3. The motor has to operate in extreme conditions. 4. The magnetic brake does not release correctly.
1,2	THROTTLE FAULT 1	voltage to potentiometers is outside the interval	1. The throttle wiring is broken or short-circuited. 2. The throttle potentiometer is defective. 3. Incorrect throttle type selected.
1,3	SPD LIMIT POT FAULT	fault in potentiometer for speed limitation	1. Wiring to speed limitation potentiometer is broken or short-circuited. 2. The speed limitation potentiometer is defective.
1,4	LOW BATTERY VOLTAGE	the battery voltage is too low	1. Battery voltage <17 V. 2. Bad connection to battery or control circuit.
1,5	OVERVOLTAGE	the battery voltage is too high	1. Battery voltage >36 V. 2. The chair is operated with the charger connected. 3. Intermittent disruptions in battery connection.
2,1	MAIN OFF FAULT	fault in main contactor for OFF	1. the main contactor does not release.
2,2	MAIN CONT WELDED	main contactor not released	1. The main contactors have welded. 2. Fault in main contactor. 3. Resistance in brake coil is too high.
2,3	MAIN CONT DNC	main contactor does not connect	1. The main contactor is stuck in the open position. 2. Fault in main contactor. 3. Resistance in brake coil is too high.
2,4	MAIN ON FAULT	fault in main contactor for ON	1. The main contactor does not connect.
3,1	PROC/WIRING FAULT	HPD fault 10s	1. Incorrectly adjusted throttle. 2. Faulty throttle or potentiometer.
3,2	BRAKE ON FAULT	brake does not engage	1. Short-circuit in magnetic brake. 2. Disruption in magnetic brake coil.
3,3	PRECHARGE FAULT	Fault during pre-charging	1. Fault in control circuit. 2. Low battery voltage.
3,4	BRAKE OFF FAULT	brake does not release	1. Disruption in magnetic brake. 2. Short-circuit in magnetic brake.
3,5	HPD	HPD fault	1. Incorrect impulse sequence from throttle, key-operated switch, cancel procedure 2. Incorrectly set throttle potentiometer.
4,1	CURRENT SENSE FAULT	current fault	1. Short-circuit in motor/winding 2. Fault in control circuit.
4,2	HW FAILSAFE	motor voltage fault	1. The motor voltage does not correspond to the position of the throttle. 2. Short-circuit in motor/winding 3. Fault in control circuit.
4,3	EEPROM FAULT	EEPROM fault	1. EEPROM defective
4,4	POWER SECTION FAULT	power circuit fault	1. EEPROM defective 2. Short-circuit in motor/winding 3. Fault in control circuit.

English

Parameter setup diagram

Specially adapted parameter settings can be reset to Etac's approved parameters. To reset the parameters to the factory setting, please refer to the setup diagram below.

Lille Viking	2015	Lille Viking	2015
MAIN C/L	40A	REV MIN SPD	25
MOTOR R	210	GEAR SOFTEN	0
IR COMP COEFF	40	SOFT START	40
KEY OFF DECEL	1,4	RAMP SHAPE	50
TREMOR COMP	1	AMPERE HOURS	33
ACCEL MAX SPD	1,6	BDI RESET	27
ACCEL MIN SPD	1,6	SLEEP DLY	5
DECEL MAX SPD	0,8	BRAKE DLY	0
DECEL MIN SPD	0,8	CREEP SPD	6
E STOP	0,6	THRTL TYPE	0
REV ACCEL MAX	0,8	THRTL DEADBAND	12
REV ACCEL MIN	0,8	THRTL GAIN	1
REV DECEL MAX	0,8	THRTL AUTOCAL	OFF
REV DECEL MIN	0,8	SPD SCALER	24,2
M1 MAX SPD	100	HIGH PEDAL DIS	ON
M2 MAX SPD	100	FAULT BEEP	OFF
M1 MIN SPD	40	SEAT LIFT	OFF
M2 MIN SPD	40	BRAKE FLTS	OFF
M1 REV MAX SPD	50	SL BRAKE FLTS	OFF
M2 REV MAX SPD	50	PUSH SPD	50

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Etac A/S
R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB