Monteringsvejledning (DK)

Montering af bord til x:panda str. 1 S-M-L+ 2 S, Panda Futura str. 1+2, Panther str. 1+2 og Wombat str. 1+2

Monteringsanvisning (SV)

Montering av bord till x:panda stl 1 S-M-L + 2 S, Panda Futura stl 1 + 2, Panther stl 1+2, Wombat stl 1+2

Monteringsveiledning (N)

Montering av bord til x:panda str. 1 S-M-L+ 2 S, Panda Futura str. 1+2, Panther str. 1+2 og Wombat str. 1+2

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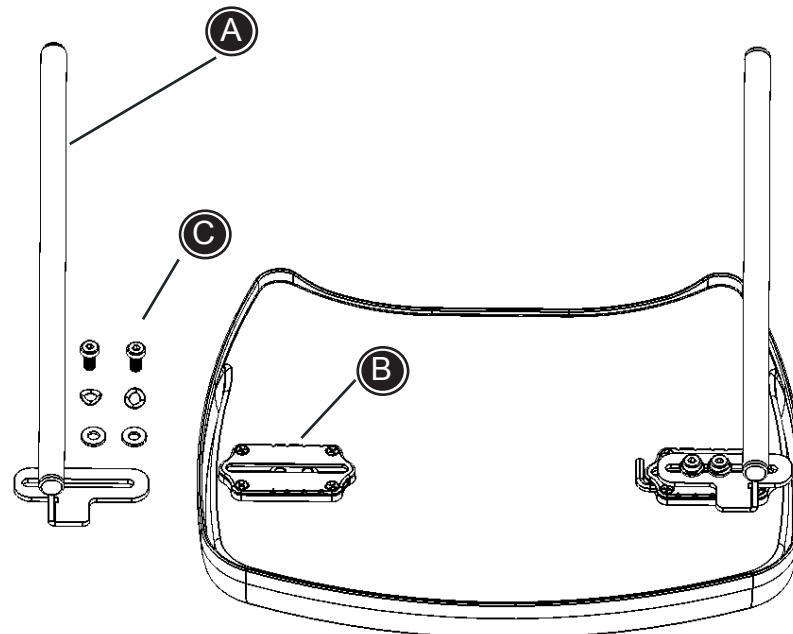
R82[®]

Mounting the tray

Follow the instructions below to assemble and mount the tray (the enclosed 5 mm Allen key can be used on all bolts):

1. The tray rods (A) are mounted in the fittings (B) with the enclosed bolts and washers (C). Please note that rods (A) can be placed on the left or the right depending on the width required to fit the seat. Do not tighten the bolts fully at this point. Leaving them loose will allow for width adjustment as the tray is mounted onto the seat.
2. Depending on the type of seat the handle is flipped up or the finger screw is loosened on the arm rest rod.
3. Mount the rods from the tray into the tube beneath both arm rests.
4. Depending on the type of seat the tray is fastened by flipping down the handle or tightening the finger screw on the arm rest rod.
5. The bolts (C) under the tray can now be tightened firmly with the enclosed Allen key.

The tray can only be mounted on seats equipped with arm rests.

Any adjustments and addition of accessories must be placed and fitted correctly and checked before every use.

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