Manual till vikbar ramp

Art. Nr. IRP50, IRP90, IRP120, IRP150, IRP170, IRP200
IRC 75, IRC120, IRC150, IRC200

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Manual foldable ramp

Art. No. IRP50, IRP90, IRP120, IRP150, IRP170, IRP200
IRC 75, IRC120, IRC150, IRC200

- Thank you for choosing a product from FeAl!
- Check that no damage has occurred on the ramp during transport. The ramp should be easy to fold/unfold.
- Read the operating instructions and warnings carefully before using the ramp. Save instructions for future needs.
- On each ramp there is a label indicating the maximum load capacity for the product.
- The label also indicates the recommended level difference the ramps are suitable for.
Take it carefully and use the ramp with good judgment.
- Unfold the ramp and place the "lip" steadily resting on the upper plane. The lip is on the underside fitted with anti-slip tape to prevent the ramp from slipping on the ground. However, be careful when using so that no part of the wheelchair hooks stuck and pulling the ramp.
- Should a damage occur on the ramp, do not use it until it has been repaired by an expert. Spare parts are presented on a separate spare part list.
- The products has a 5 year warranty.

Maintenance

- The ramp must be kept clean from dirt and snow so that the grip in the surface is maintained. The ramp can be cleaned with plain detergent. Rinse clean with water.

Demolition and recycling

- All materials are 100% recyclable. Aluminum is left as metal scrap and the IRC ramps of carbon fiber are sorted as non-combustible waste.