


Etac Xact

Spareparts list english

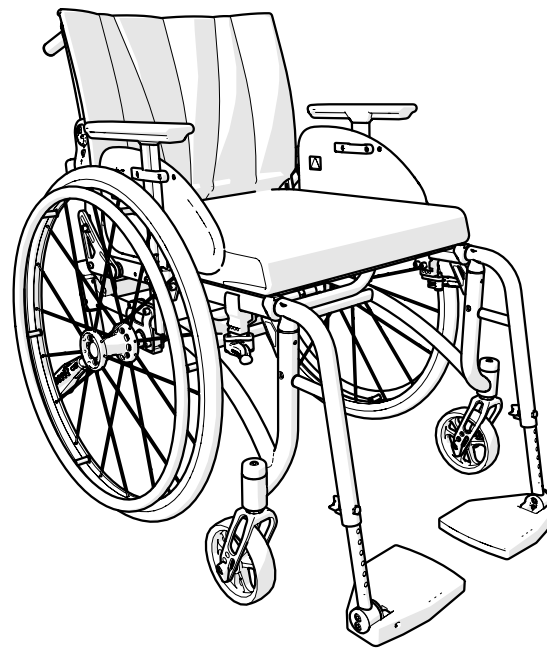


1a19461.4 2025-04-14



Xact

Index


Index	Page
Seat frame	3
Back support frame	4
Upholstery	5
Push handles	6
Leg supports	7
Foot plates	8
Rear wheels	9
Handrims	10
Tyres and inner tubes	11
Spoke guards	12
Brakes	13
Castors	14
Arm supports, mudguards	15
Anti-tips	16
Rear axle attachments	17
Attendant brake	18
One-arm drive	19
Widening kit	20
Crutch and cane holder	21
Attachment positioning belt	22

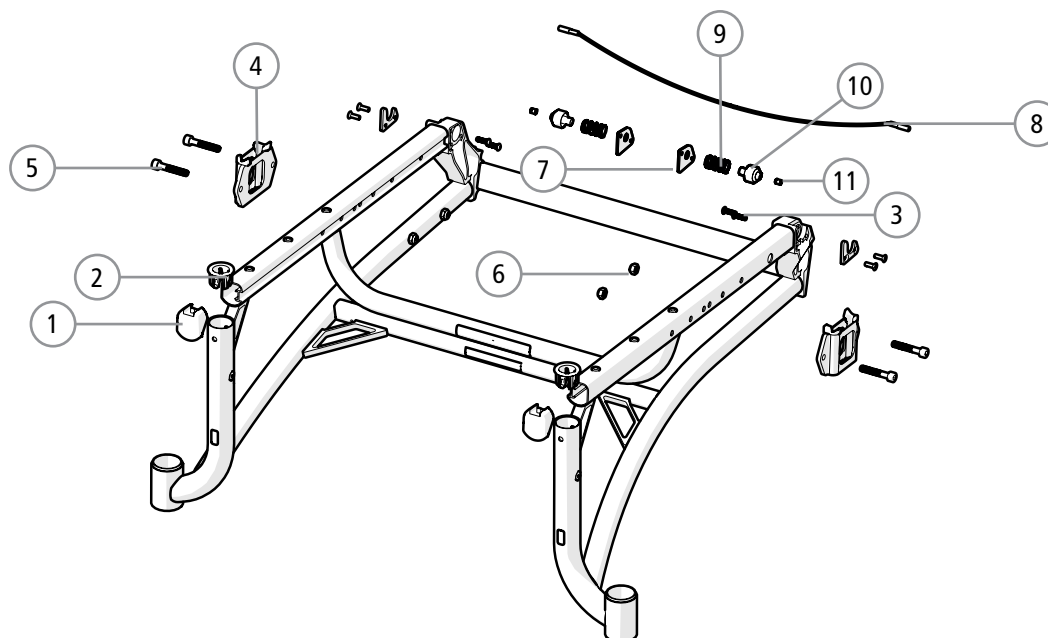

Pos.	Description	Art. no.
1	End cap seat tube	51281
2	Bushing leg support	51273
3	Screw M4x14 K6S 10.9 Fe Zn8 P4	11724
4	Arm support attachment H48 cpl	28859
5	Screw M6x35 MC6S black	11533
6	Locking nut M6 Zn8/P4 DIN 985-6	12035
7	Cover rear part inside	83929
8	Wire back folding 35 cm	83501
	Wire back folding 37,5 cm	83502
	Wire back folding 40 cm	83503
	Wire back folding 42,5 cm	83504
	Wire back folding 45 cm	83505
	Wire back folding 47,5 cm	83506
	Wire back folding 50 cm	83507
9	Spring D13,5 L24	83500


Pos.	Description	Art. no.
10	Lock back folding	83923
11	Stop screw M5x5 SK6SS A2	11530
8-11	Wire kit 35 cm	249630
	Wire kit 37,5 cm	249640
	Wire kit 40 cm	249660
	Wire kit 42,5 cm	249670
	Wire kit 45 cm	249680
	Wire kit 47,5 cm	249690
	Wire kit 50 cm	249700

Back support frame

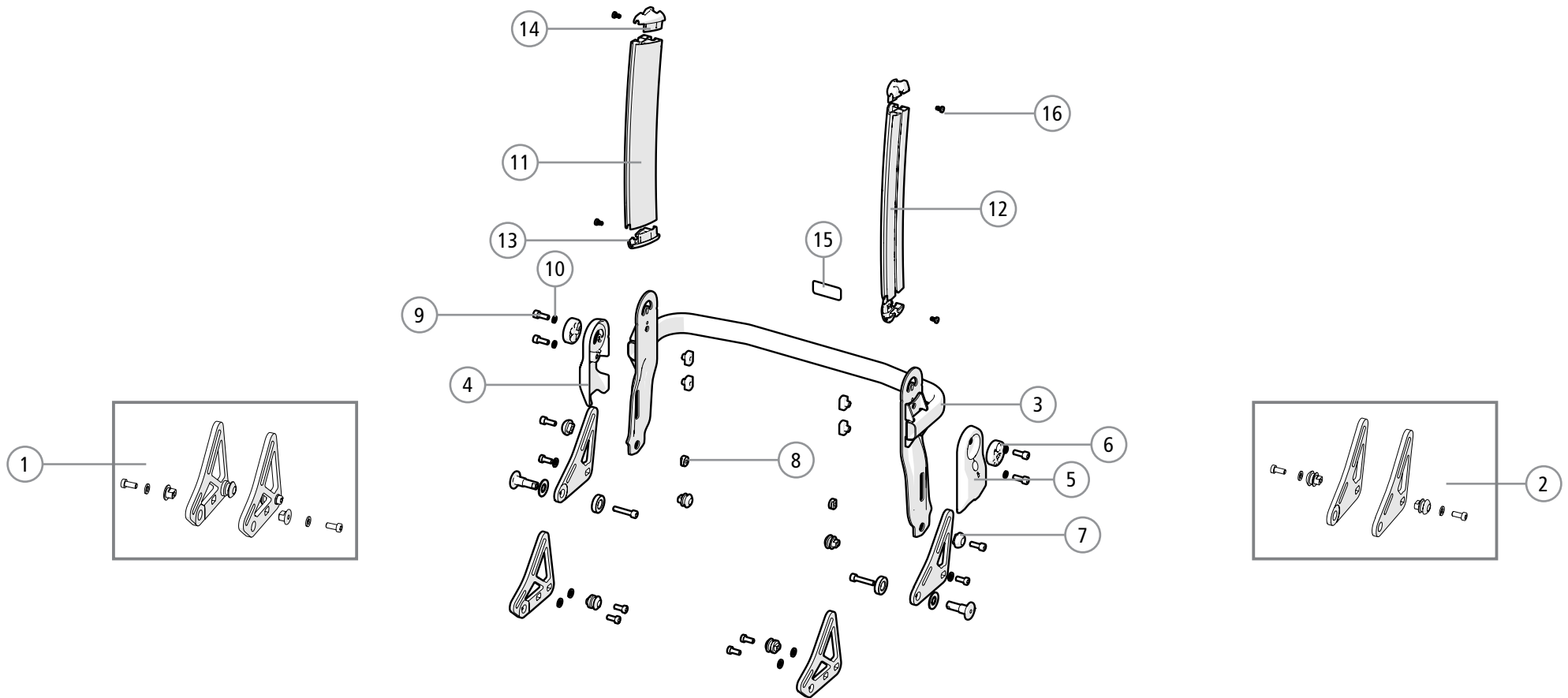


Pos.	Description	Art. no.
1	Seat depth kit short	24624
2	Seat depth kit std	24625
3	Back support frame 35 cm	24030-01
	Back support frame 37,5 cm	24031-01
	Back support frame 40 cm	24032-01
	Back support frame 42,5 cm	24033-01
	Back support frame 45 cm	24034-01
	Back support frame 47,5 cm	24035-01
	Back support frame 50 cm	24036-01
4	Cover back support ri	51287-1
5	Cover back support le	51287-2
6	Excenter back support	83291
7	Sleeve back angle	83918

Pos.	Description	Art. no.
8	Nut T-slot back	20028
9	Screw M6x16 MC6S 12.9	11253
10	Circlip M6 wavy FR DIN128A steel 2.1 zink plated	13113
11	Back rail Cr 3A mod 4 right	83013-1
12	Back rail Cr 3A mod 4 left	83013-2
13	Dust cover 1 with locking nut	84109
14	Dust cover 2 with locking nut	84110
15	Logo label Xact	72121
16	Screw M4x8 MRX sv. 4.8 pos	11441



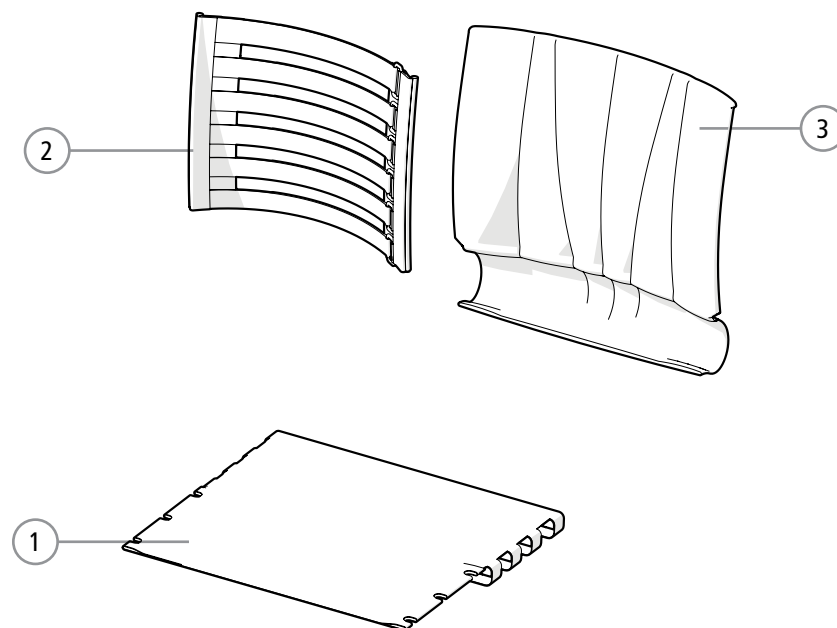


Pos.	Description	Art. no.
1	Seat upholstery 35 cm	63362-60
	Seat upholstery 37.5 cm	63363-60
	Seat upholstery 40 cm	63364-60
	Seat upholstery 42.5 cm	63365-60
	Seat upholstery 45 cm	63366-60
	Seat upholstery 47.5 cm	63367-60
	Seat upholstery 50 cm	63368-60
2	Back support upholstery 3A adj 35-40 cm	62953-60
	Back support upholstery 3A adj 42.5-50 cm	62954-60

Pos.	Description	Art. no.
3	Back support cover 2B/3A 35 cm airmesh	63204-68
	Back support cover 2B/3A 37.5 cm airmesh	63205-68
	Back support cover 2B/3A 40 cm airmesh	63206-68
	Back support cover 2B/3A 42.5 cm airmesh	63207-68
	Back support cover 2B/3A 45 cm airmesh	63208-68
	Back support cover 2B/3A 47.5 cm airmesh	63209-68
	Back support cover 2B/3A 50 cm airmesh	63210-68
	Back support cover 3A 35 cm dark grey	62941-60
	Back support cover 3A 37.5 cm dark grey	62942-60
	Back support cover Cross 3A 40 cm dark grey	62943-60
	Back support cover Cross 3A 42.5 cm dark grey	62944-60
	Back support cover Cross 3A 45 cm dark grey	62945-60
	Back support cover Cross 3A 47.5 cm dark grey	62946-60
	Back support cover Cross 3A 50 cm dark grey	62947-60


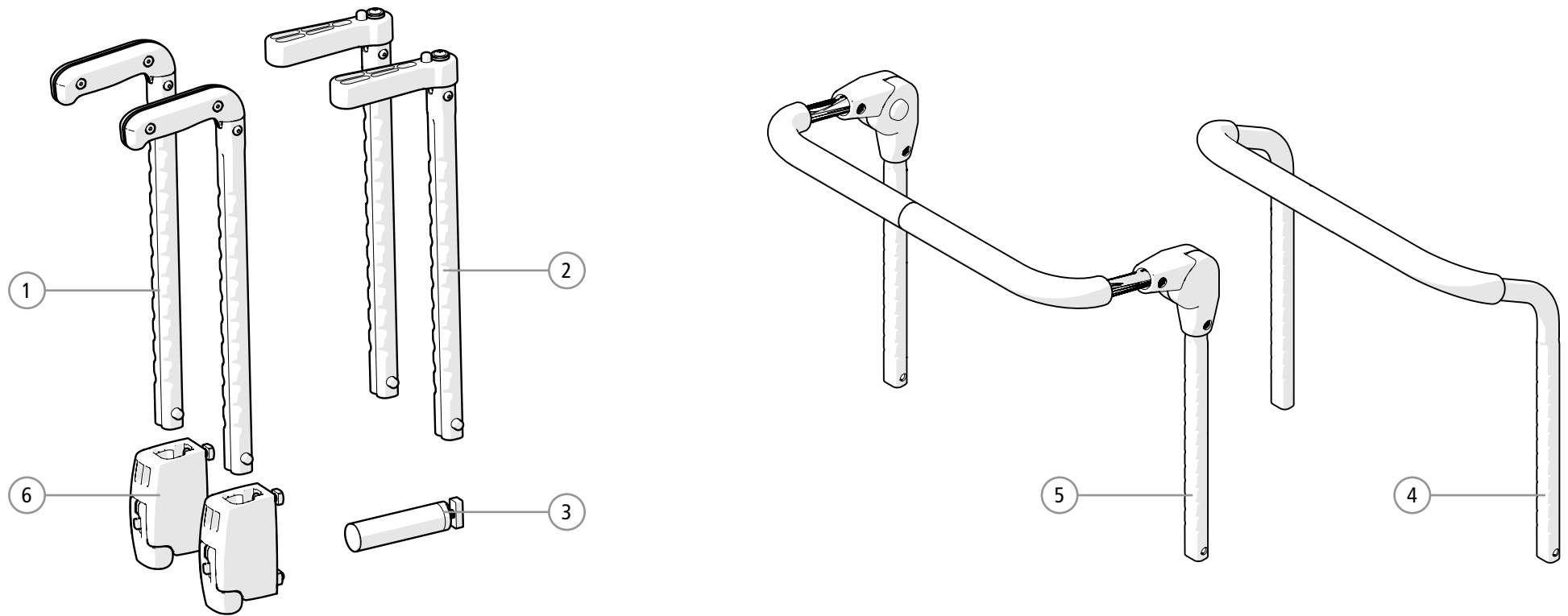


Pos.	Description	Art. no.
1	Push handle active height adj cpl	28873
2	Push handle swing-away cpl	24066
3	Push handle fixed 3A cpl	28875
4	Bow handle height adjustable 32,5 cm back 3A 35 cm cpl	27433-01
5	Bow handle angle adjustable 35 cm back 3A 37.5 cm cpl	27161
	Bow handle angle adjustable 37.5 cm back 3A 40 cm cpl	27162
	Bow handle angle adjustable 40 cm back 3A 42.5 cm cpl	27163
	Bow handle angle adjustable 42.5 cm back 3A 45 cm cpl	27164
	Bow handle angle adjustable 45 cm back 3A 47.5 cm cpl	27165
	Bow handle angle adjustable 47.5 cm back 3A 50 cm cpl	27166
6	Attachment push handle cpl	28576

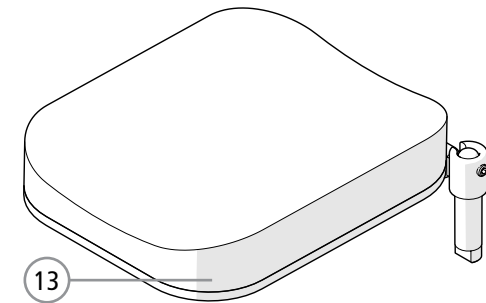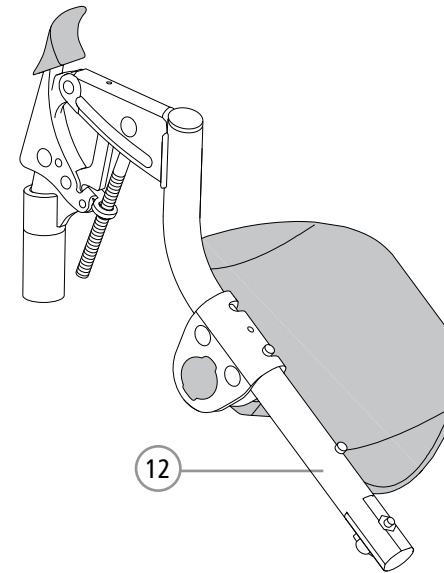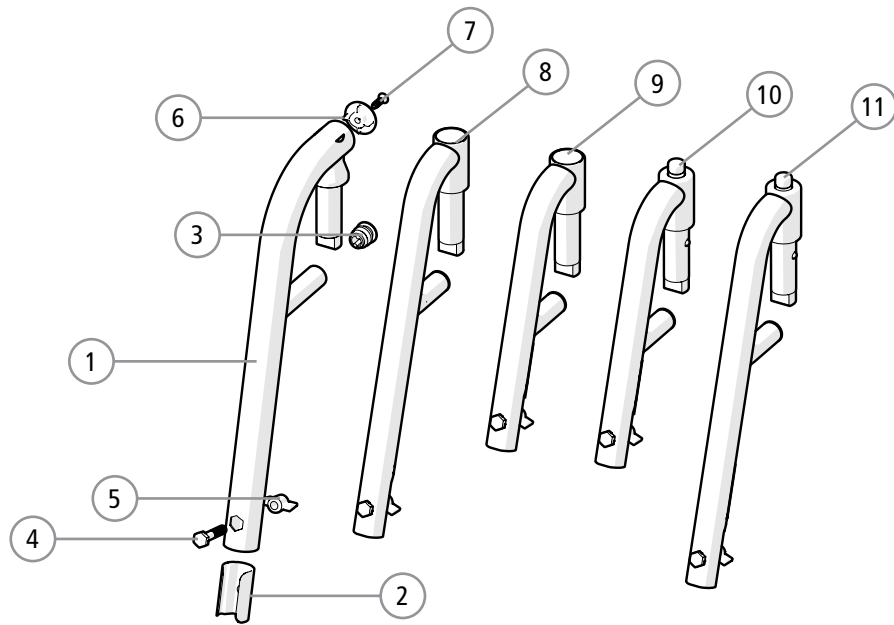


Pos.	Description	Art. no.
1-7	Leg support active	28888
2	Lock foot support Crissy active	51158
3	End cap IKPK	50240
4	Screw M6x30 M6S 8.8 ZnNi esloc	11555
5	Wing nut M6 PA 6.6 black	80847
6	End cap leg support active	51286
7	Screw B8x16 FXS black	11401
8	Leg support narrow angle	25570-01
9	Leg support narrow angle short	25571-01
10	Leg support narrow angle short lockable	25572-01
11	Leg support narrow angle lockable	25573-01
12	Leg support elevating compl with calf support	26100
13	Leg support amputee Cross	25516-60

Foot supports

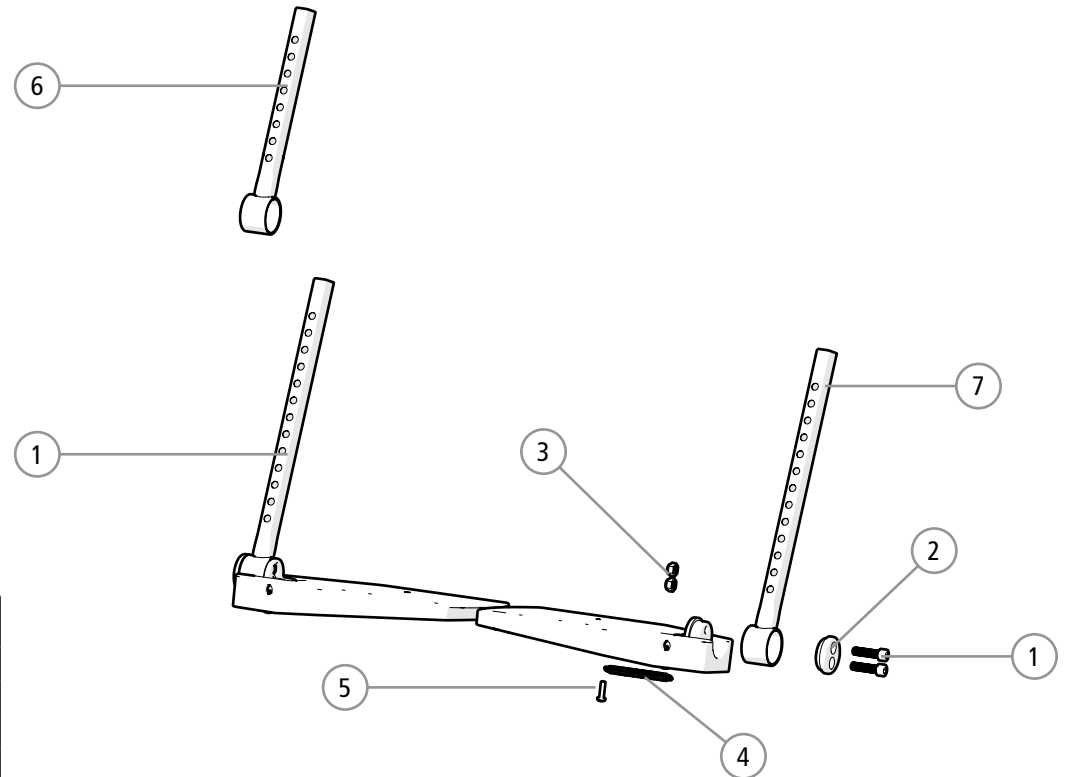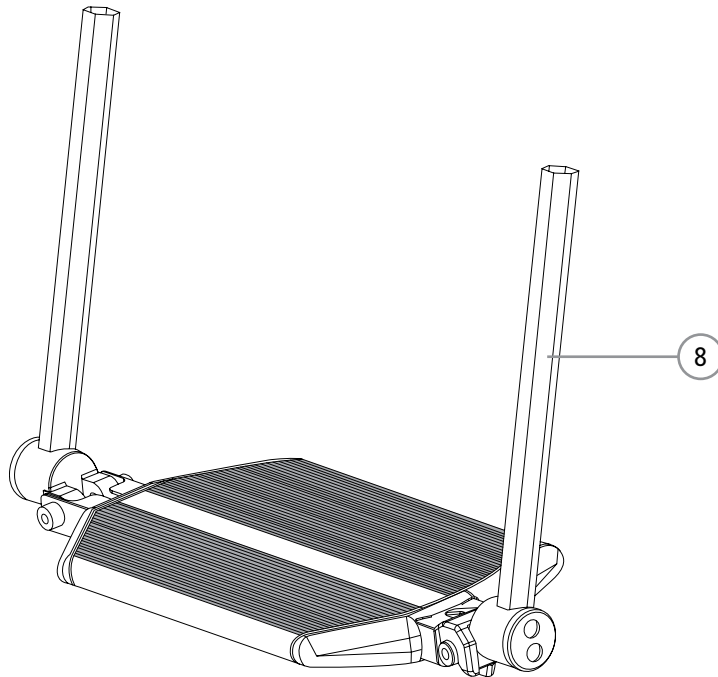


Pos.	Description	Art. no.
1	Screw M6x25 MC6S 8.8 zink plated	11033
2	Locking block foot support	20324
3	Locking nut M6 zink plated	12017
4	Spring foot support	80619
5	Screw K40x8 PT-KX Philips	11344
6	Attachment tube foot support short	25565
7	Attachment tube foot support std	25563
1-6	Foot support short 35 cm ri	25653-1
	Foot support short 35 cm le	25653-2
	Foot support short 37.5 cm ri	25654-1
	Foot support short 37.5 cm le	25654-2
	Foot support short 40 cm ri	25655-1
	Foot support short 40 cm le	25655-2
	Foot support short 42.5 cm ri	25656-1
	Foot support short 42.5 cm le	25656-2
	Foot support short 45 cm ri	25657-1
	Foot support short 45 cm le	25657-2
	Foot support short 47.5 cm ri	25738-1
	Foot support short 47.5 cm le	25738-2
	Foot support short 50 cm ri	25658-1
	Foot support short 50 cm le	25658-2

Pos.	Description	Art. no.
1-5, 7	Foot support std 35 cm ri	25641-1
	Foot support std 35 cm le	25641-2
	Foot support std 37.5 cm ri	25642-1
	Foot support std 37.5 cm le	25642-2
	Foot support std 40 cm ri	25643-1
	Foot support std 40 cm le	25643-2
	Foot support std 42.5 cm ri	25644-1
	Foot support std 42.5 cm le	25644-2
	Foot support std 45 cm ri	25645-1
	Foot support std 45 cm le	25645-2
	Foot support std 47.5 ri	25736-1
	Foot support std 47.5 le	25736-2
	Foot support std 50 cm ri	25646-1
	Foot support std 50 cm le	25646-2
8	Foot support solid cpl 35-37.5	25936
	Foot support solid cpl 40-50	25937

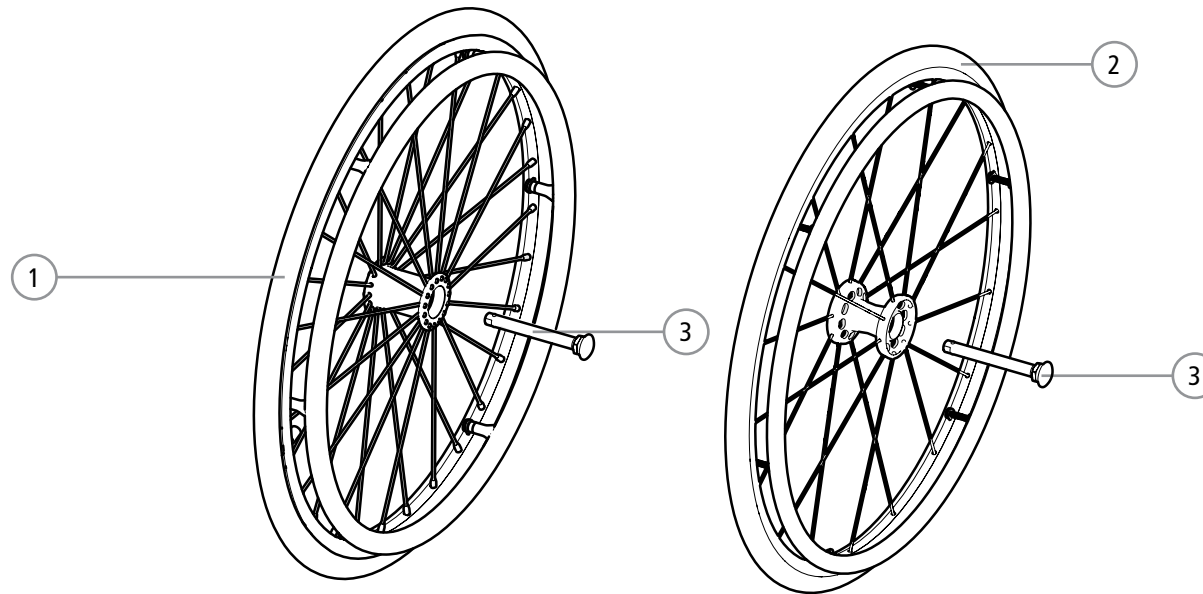


Pos.	Description	Art. no.
1	Rear wheel 20 C6 hp al Speedlite excl axle	83908
	Rear wheel 20 C6 hp ss Speedlite excl axle	83909
	Rear wheel 20 C6 hp plastic Speedlite excl axle	83884
	Rear wheel 20 C6 solid al excl axle	83885
	Rear wheel 20 C6 solid ss excl axle	83910
	Rear wheel 20 C6 solid plastic excl axle	83911
	Rear wheel 22 C6 hp al Speedlite excl axle	83880
	Rear wheel 22 C6 hp ss Speedlite excl axle	83906
	Rear wheel 22 C6 hp plastic Speedlite excl axle	83881
	Rear wheel 22 C6 solid al excl axle	83882
	Rear wheel 22 C6 solid ss excl axle	83907
	Rear wheel 22 C6 solid plastic excl axle	83883
	Rear wheel 24 C6 hp al Speedlite excl axle	83873
	Rear wheel 24 C6 hp ss Speedlite excl axle	83874
	Rear wheel 24 C6 hp plastic Speedlite excl axle	83875
	Rear wheel 24 C6 solid al excl axle	83876
	Rear wheel 24 C6 solid ss excl axle	83877
	Rear wheel 24 C6 solid plastic excl axle	83878


Pos.	Description	Art. no.
2	Rear wheel 22 Maxx hp hard al Speedlite excl axle	28956
	Rear wheel 22 Maxx hp plastic Speedlite excl axle	28957
	Rear wheel 22 Maxx hp hard al Trailblazer R excl axle	24052-1
	Rear wheel 22 Maxx hp hard al Trailblazer L excl axle	24052-2
	Rear wheel 22 Maxx hp plastic Trailblazer R excl axle	24053-1
	Rear wheel 22 Maxx hp plastic Trailblazer L excl axle	24053-2
	Rear wheel 24 Maxx hp hard al Speedlite excl axle	28950
	Rear wheel 24 Maxx hp plastic Speedlite excl axle	28951
	Rear wheel 24 Maxx hp hard al Trailblazer R excl axle	28952-1
	Rear wheel 24 Maxx hp hard al Trailblazer L excl axle	28952-2
	Rear wheel 24 Maxx hp plastic Trailblazer R excl axle	28953-1
	Rear wheel 24 Maxx hp plastic Trailblazer L excl axle	28953-2
	Rear wheel 25 Maxx hp hard al Trailblazer R excl axle	28964-1
	Rear wheel 25 Maxx hp hard al Trailblazer L excl axle	28964-2
	Rear wheel 25 Maxx hp plastic Trailblazer R excl axle	28965-1
	Rear wheel 25 Maxx hp plastic Trailblazer L excl axle	28965-2
3	QR-axle D12.7x99	83492

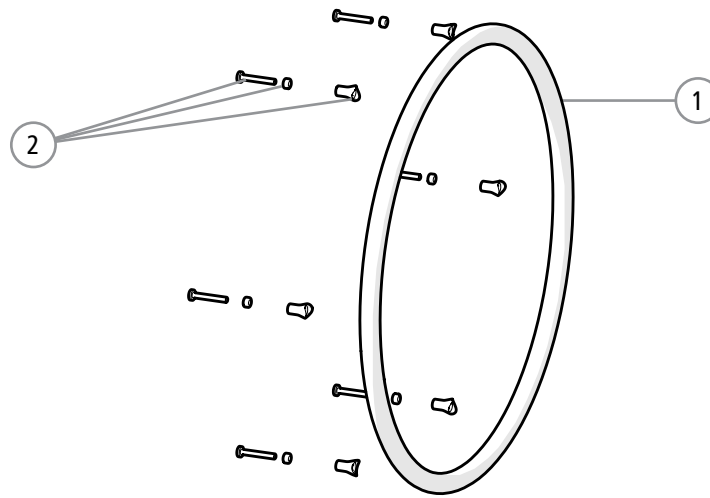

Pos.	Description	Art. no.
1	Handrim 20 al 6 rivet nut	83890
	Handrim 22 al 6 rivet nut	83888
	Handrim 24 al 6 rivet nut	83886
	Handrim 22 hard al 6 rivet nut	83336
	Handrim 24 hard al 6 rivet nut	83334
	Handrim 25 hard al 6 rivet nut	83340
	Handrim 20 ss 6 rivet nut	83891
	Handrim 22 ss 6 rivet nut	83889
	Handrim 24 ss 6 rivet nut	83887
	Handrim 20 plastic 6 rivet nut	83339
	Handrim 22 plastic 6 rivet nut	83338
	Handrim 24 plastic 6 rivet nut	83335
	Handrim 25 plastic 6 rivet nut	83341
2	Assembly kit handrim 6m	83892
	Assembly kit handrim 6m plas. Plastic handrim C6 Max	24081

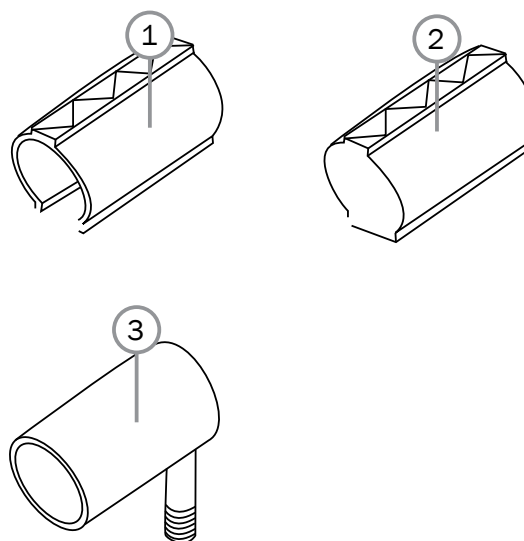


Pos.	Description	Art. no.
1	Tyre 20x1 (25-451) Speedlite black	83897
	Tyre 22x1 (25-501) Speedlite black	83898
	Tyre 24x1 (25-540) Speedlite black	83899
	Tyre 22x1 ((25-501) Trailblazer black	83350
	Tyre 24x1 (25-540) Trailblazer black	83351
	Tyre 25x1 (25-559) Trailblazer black	83900
2	Tyre 20x1 3/8 (37-451) solid black	83347
	Tyre 22x1 3/8 (37-501) solid black	83348
	Tyre 24x1 1/4 (32-540) solid black	83349
3	Inner tube 20x1 (25-451) high pressure	82199
	Inner tube 22x1 (25-501) high pressure	80136
	Inner tube 24x1 (25-540) high pressure	80560
	Inner tube 25x1 (25-559) high pressure	83904

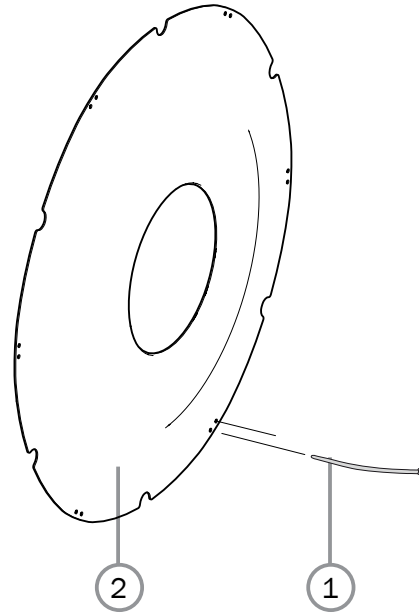

Pos.	Description	Art. no.
1	Assembly kit spoke guard, 6 pcs cable tie	26276
1-2	Spoke guard 20" cpl C6/Maxx	28934
	Spoke guard 22" cpl C6/Maxx	28935
	Spoke guard 24" cpl C6/Maxx	28936




Pos.	Description	Art. no.
1	Scissor brake ri cpl	28915-1
2	Scissor brake le cpl	28915-2
3	Brake knee lever red R cpl	28916-1
	Brake knee lever black R cpl	28917-1
4	Brake knee lever red L cpl	28916-2
	Brake knee lever black L cpl	28917-2
5	Plug for brake Cross angled handle red	51012
	Plug for brake Cross angled handle black	50366
6	Single hand brake ri	28918-1
	Single hand brake le	28918-2

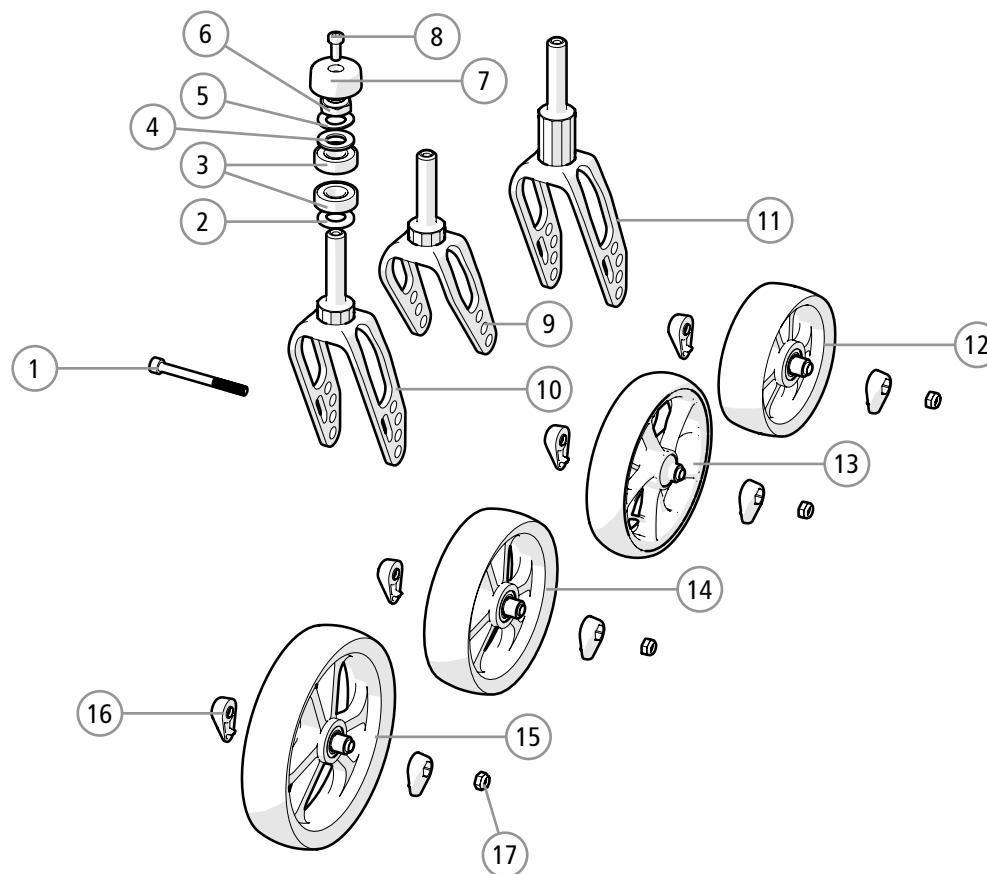


Pos.	Description	Art. no.
1	Screw M6 MC6S Zn8/P4 12.9	11408
2	Washer BRB 12.7 with chamfer	13032
3	Ball bearing 1/2" EE4 2RS	80224
4	Washer BRB 13,5x24x2 ZN8/P4	13031
5	pring washer D24xd13,5x1,2	13040
6	Locking nut axle front fork H=8	12037
7	Cover castor housing	51282
8	Screw M6x14 MC6LS SW5 Zn8/P4	11539
9	Front fork alu XS w axle	28923
10	Front fork alu S w axle	28924

Pos.	Description	Art. no.
11	Front fork alu S w axle +25	28925
12, 16-17	Castor 4 100x34 cpl	28919
13, 16-17	Castor 5 125x28 cpl	27299
14, 16-17	Castor 5 125x34 cpl	28921
15, 16-17	Castor 6 150x34 cpl	28922
16	Cover wheel axle	51284
	Cover wheel axle 50 pcs	28966
17	Locking nut M6 Zn8/P4 DIN 985-6	12035

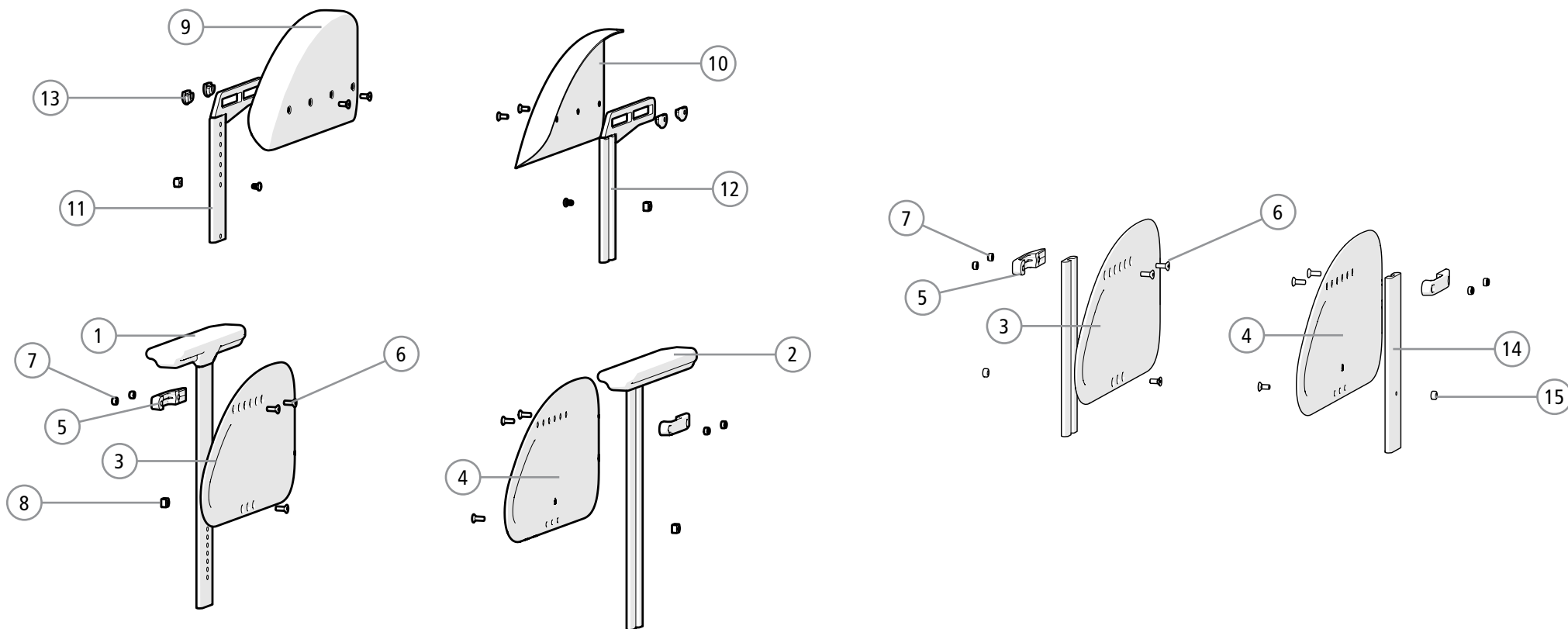


Pos.	Description	Art. no.
1, 3, 5-7	Arm supports active cpl ri	28855-1
2, 4, 5-7	Arm supports active cpl le	28855-2
3-7	Side guard active d adj cpl	28857
1	Arm support ri	
2	Arm support le	
3	Sideguard ri	
4	Sideguard le	
5	Attachment side guard	
6	Screw M5x14 MF6S Zn8/P esloc	
7	Locking nut stainless steel	
8	Stop bar arm support	
9	Mudguard S ri	51292-1
	Mudguard M ri	51293-1
	Mudguard L ri	51294-1
10	Mudguard S le	51292-2
	Mudguard M le	51293-2
	Mudguard L le	51294-2

Pos.	Description	Art. no.
11	Attachment mudguard 155 ri	24083-1
12	Attachment mudguard 155 le	24083-2
13	T-nut M5 offset	12079
6-9, 11, 13	Mudguard S profile ri 155 cpl	24091-1
	Mudguard M profile ri 155 cpl	24092-1
	Mudguard L profile ri 155 cpl	24093-1
6-8, 10, 12-13	Mudguard S profile le 155 cpl	24091-2
	Mudguard M profile le 155 cpl	24092-2
	Mudguard L profile le 155 cpl	24093-2
14	Vertical profile side guard	
15	Spacer handrim short	
3, 5-7, 14-15	Side guard active d adj R cpl	24592-1
4-7, 14-15	Side guard active d adj L cpl	24592-2

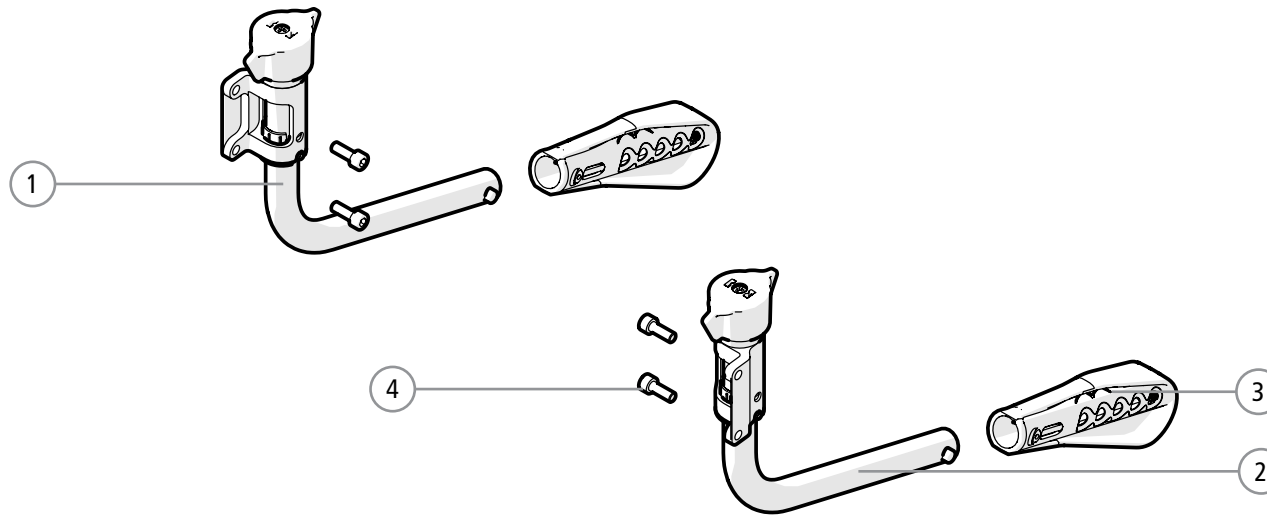


Pos.	Description	Art. no.
1, 3, 4	Anti-tip 1 ri cpl	24163-1
	Anti-tip 2 ri cpl	24164-1
	Anti-tip 3 ri cpl	24165-1
	Anti-tip 4 ri cpl	24166-1
2, 3, 4	Anti-tip 1 le cpl	24163-2
	Anti-tip 2 le cpl	24164-2
	Anti-tip 3 le cpl	24165-2
	Anti-tip 4 le cpl	24166-2
3	Anti-tip extension multi	51201
4	Screw M6x16 MC6S 12.9	11253

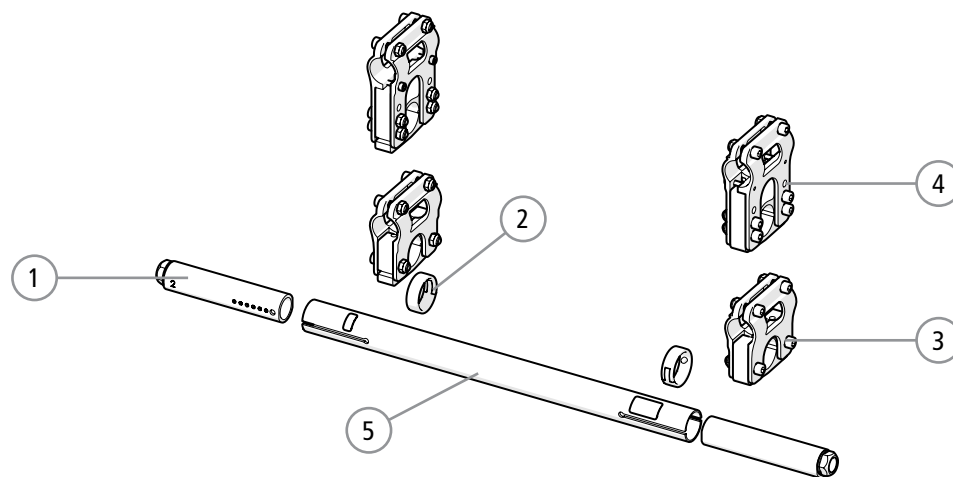


Pos.	Description	Art. no.
1	Camberplug 2° cpl	KI004356
2	Clip rear axle Ki Mobility	KI111808
3	Rear axle attachment std cpl	24004
4	Rear axle attachment high cpl	24019
5	Rear axle incl labels W 35	24523
	Rear axle incl labels W 37.5	24524
	Rear axle incl labels W 40	24525
	Rear axle incl labels W 42.5	24526
	Rear axle incl labels W 45	24527
	Rear axle incl labels W 47.5	24528
	Rear axle incl labels W 50	24529

Attendant brake

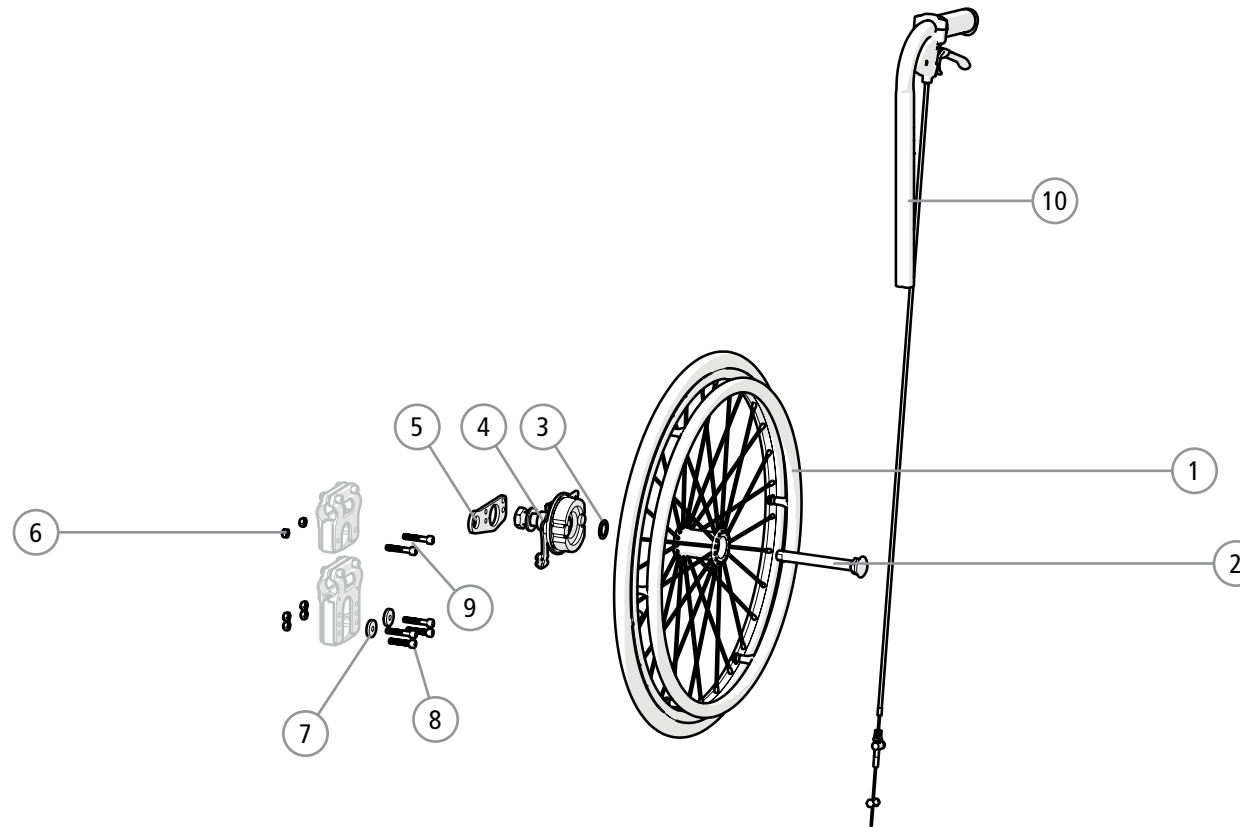


Pos.	Description	Art. no.
1	Rear wheel 20" solid al	81337
	Rear wheel 22" high pressure stainless st	81344
	Rear wheel 22" high pressure al	81342
	Rear wheel 22" solid plastic coated	81340
	Rear wheel 22" solid stainless st	81341
	Rear wheel 22" solid al	81339
	Rear wheel 24" high pressure plastic coated	28625
	Rear wheel 24" high pressure al	28626
	Rear wheel 24" solid plastic coated	81347
	Rear wheel 24" solid stainless st	81348
	Rear wheel 24" solid al	81346
2	QR-axle	81325
3	Schnorr washer	13084

Pos.	Description	Art. no.
4	Drum brake attendant brake kit ri	82822
	Drum brake attendant brake kit le	82823
5	Attachment attendant brake	83974
6	Locking nut M6	12035
7	Washer 6,4x22x4 zink plated	13062
8	Screw M6x35 MC6S	11533
9	Screw M6x30 MC6S	11398
10	Push handle hrb Cross 100 mm	27625-01
3-10	Attendant brake kit cpl	24001

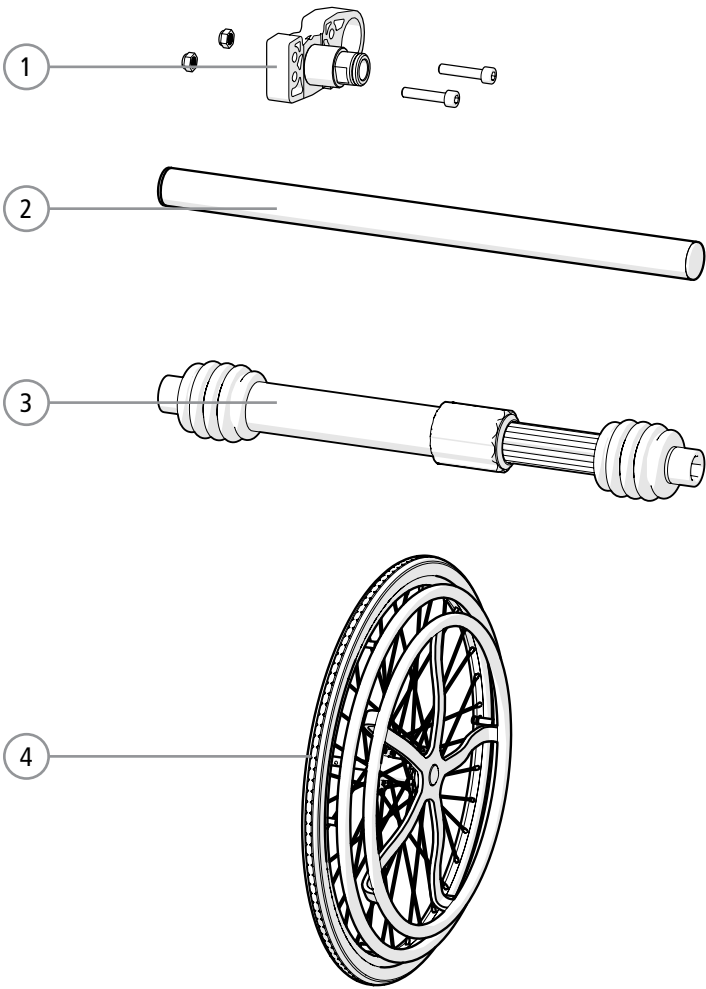


Pos.	Description	Art. no.
1	Assembly kit one arm drive std	24071
2	Cross bar one arm drive B35	24072
	Cross bar one arm drive B37,5	24073
	Cross bar one arm drive B40	24074
	Cross bar one arm drive B42,5	24075
	Cross bar one arm drive B45	24076
	Cross bar one arm drive B47,5	24077
	Cross bar one arm drive B50	24078


Pos.	Description	Art. no.
3	Telescopic shaft one arm drive 1	24171
	Telescopic shaft one arm drive 2	24172
	Telescopic shaft one arm drive 3	24173
	Telescopic shaft one arm drive 4	24174
	Telescopic shaft one arm drive 5	24175
4	Drive unit one arm drive 20	84101
	Drive unit one arm drive 22	84102
	Drive unit one arm drive 24	84103

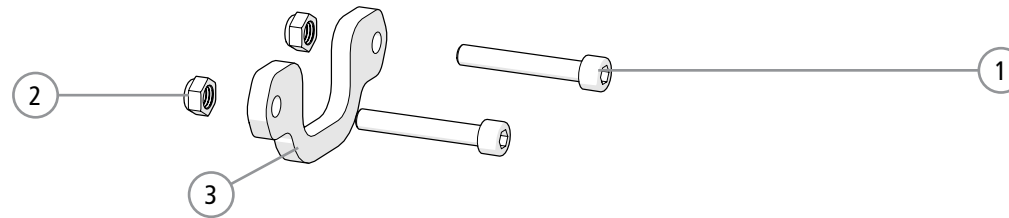

Pos.	Description	Art. no.
1-3	Widening kit cpl	24079
1	Screw M6x40 MC6S zink plated	
2	Locking nut M6	
3	Widening plate	

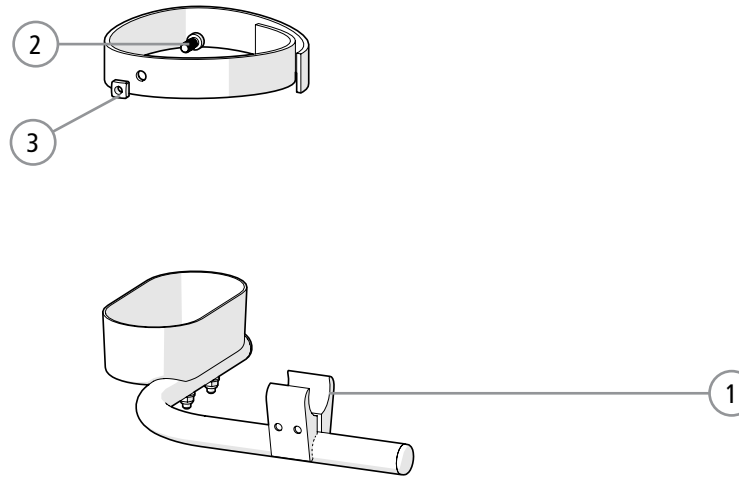

Pos.	Description	Art. no.
1-3	Crutch and cane holder cpl	28932
2	Screw MC6LS 6x14 SW=5 black	
3	Square nut high M6	

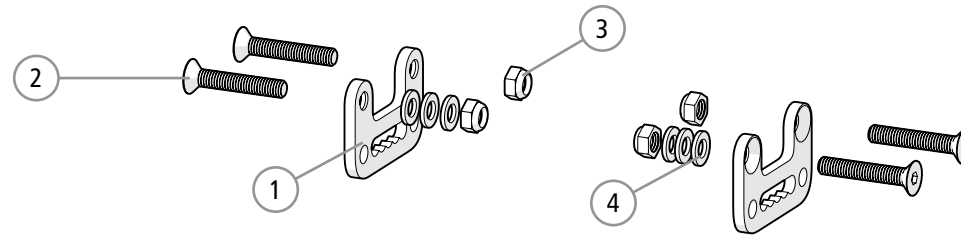

Pos.	Description	Art. no.
1-4	Attachment positioning belt	28933
1	Attachment positioning belt	
2	Screw M6x35 MF6S 10.9	
3	Locking nut M6	
4	Washer 6.4x12x1.6 Zn8/P4	