



# Elite

## Sparepart list

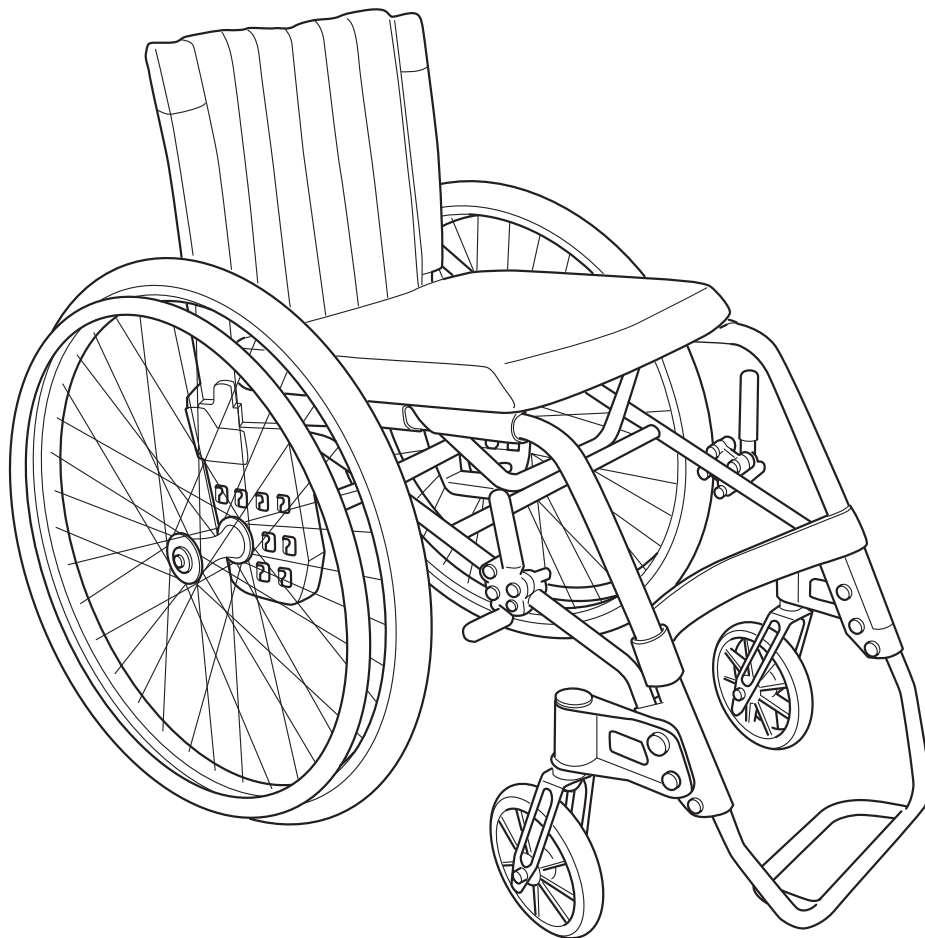

74070E 2012-02-24



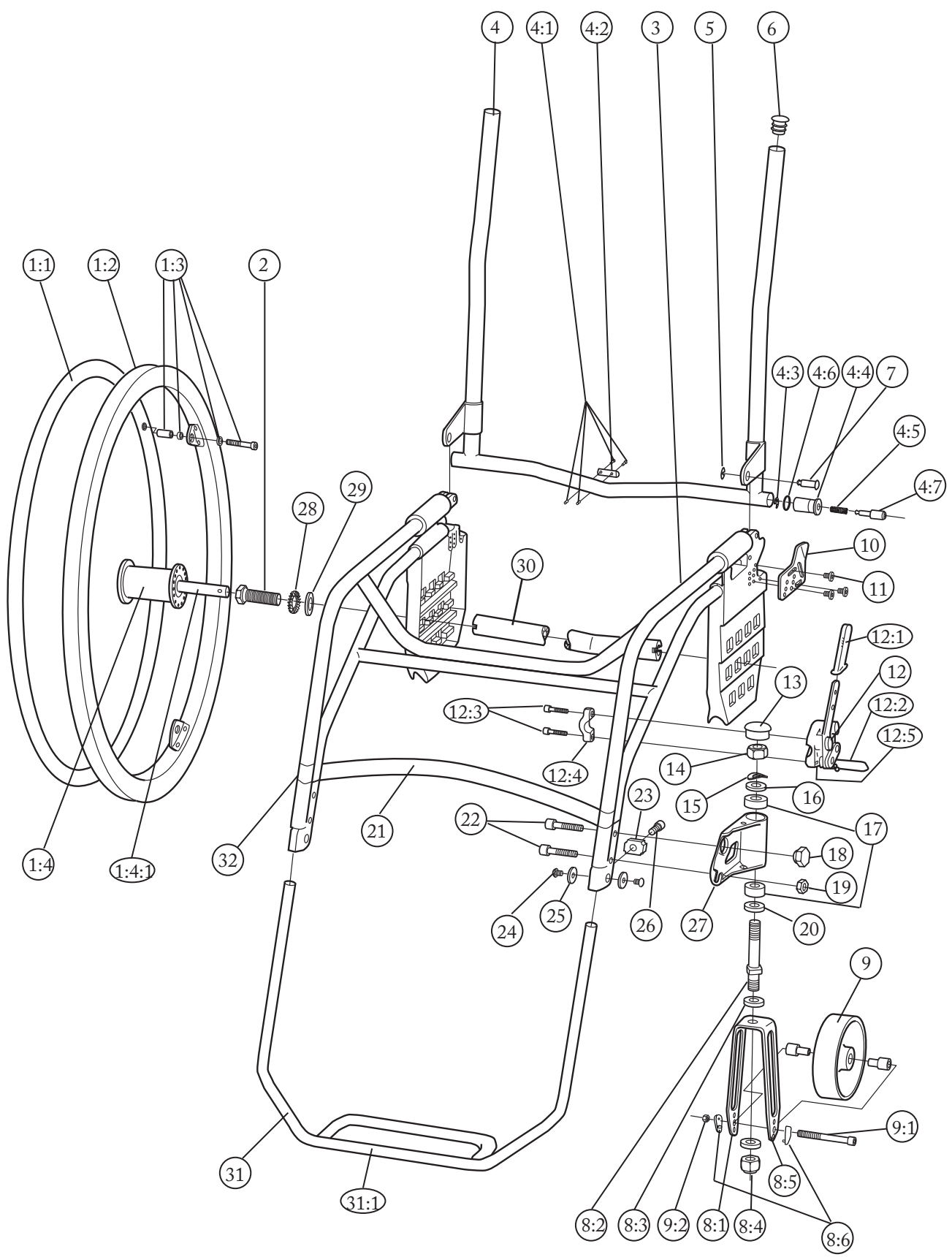


Pos	Description	Item no	Pos	Description	Item no
1	Rear wheel 24" high press titanium handrim ..	25093	9	Castor 125 x 30 solid .....	24522
	Rear wheel 24" high press stainless st handrim	25097		Castor 165 x 30 solid .....	25339
	Rear wheel 24" high press aluminium handrim	25182	9:1	Screw M6x70 MC6S black .....	11349
	Rear wheel 24" high press cellular rubber .....	25806	9:2	Locking nut M6 black .....	12035
	Rear wheel 24" high press plastic coated handr	25832	10	Backrest angle plate right .....	25237-1
	Tyre 24" pneumatic high pressure .....	80558		left.....	25237-2
	Innertube 24" pneumatic .....	80560	11	Screw M5x20 MC6LS NV4 fzb .....	11320
1:1	Handrim D540 wheel 24" titanium .....	24540	12	Brake höger .....	25450-1
	Handrim D540 wheel 24" stainless steele .....	24023		vänster .....	25450-2
	Handrim D540 wheel 24" aluminium .....	25181	12:1	Brake handle black .....	50188
	Handrim D540 wheel 24" cellular rubber .....	80800	12:2	Brake plunger L48 .....	20325
	Handrim D540 wheel 24" plastic coated .....	80894	12:3	Screw M 6x 22 MC6S 8.8 black chrome esloc...	11239
1:2	Rear wheel 24" high press without handrim ...	25023	12:4	Locking joint for brake .....	25449-01
1:3	Attachment kit handrim .....	24514	12:5	Screw M6x16 MC6S black .....	11253
1:4	Hub aluminium .....	80524	13	Dust cover 32x 1.5 black .....	50144
1:4:1	Axle aluminium hub .....	20349	14	Locking nut 1/2 UNF low .....	12037
2	Axle sleeve .....	20079	15	Washer 13x24x1.2 fzb .....	13040
3	Frame seat short 35 cm (14") .....	25215-*	16	Washer BRB 13.5 black .....	13031
	37.5 cm (15") .....	25216-*	17	Ball bearing 1/2 EE4 2RS .....	80244
	40 cm (16") .....	25217-*	18	Locking bolt attachment front fork .....	20341
	42.5 cm (17") .....	25218-*	19	Locking nut M8 black .....	12032
	45 cm (18") .....	25219-*	20	Washer BRB 13.5 black chrome plated .....	13032
	Frame seat long 35 cm (14") .....	25220-*	21	Calf strap 35/37.5 cm (14"/15")	62145
	37.5 cm (15") .....	25221-*		40/42.5 cm (16"/17")	62146
	40 cm (16") .....	25222-*		45 cm .....	62147
	42.5 cm (17") .....	25223-*	22	Screw M8x35 MC6S fzb .....	11276
	45 cm (18") .....	25224-*	23	Locking plate M8 footrest .....	20339
4	Frame backrest 35 cm (14") .....	24492-01	24	Screw M5x8 K6S black.....	11304
	37.5 cm (15") .....	24493-01	25	Washer 5,5x18x2 black .....	13056
	40 cm (16") .....	24494-01	26	Screw M8x20 MC6S 8.8 fzb .....	11222
	42.5 cm (17") .....	24495-01	27	Front fork housing left .....	24519-**
	45 cm (18") .....	24496-01		right .....	24518-**
	Backrest locking kit without wire .....	24574	28	Locking washer AZ 5/8 .....	13039
4:1	Stud M4x4 P6SS fzb .....	11267	29	Spacer rear axle .....	20370
4:2	Handle backrest lock .....	80373	30	Rear axle 2.5° 35 cm (14") .....	25232-*
4:3	Cir clip RSM 5 stainless .....	13037		37.5 cm (15") .....	25233-*
4:4	Sleeve backrest locking .....	20203		40 cm (16") .....	25234-*
4:5	Spring D9 d7 L15 .....	80340		42.5 cm (17") .....	25235-*
4:6	O-ring d13.1x1.6 nitril .....	80663		45 cm (18") .....	25236-*
4:7	Wire backrest folding 35 cm (14") .....	24637		Rear axle 0° 35 cm (14") .....	25247-01
	37.5 cm (15") .....	24638		37.5 cm (15") .....	25248-01
	40 cm (16") .....	24639		40 cm (16") .....	25249-01
	42.5 cm (17") .....	24640		42.5 cm (17") .....	25250-01
	45 cm (18") .....	24641		45 cm (18") .....	25251-01
5	Cir clip RSM 6 stainless .....	13036	31	Footrest 35 cm (14") .....	25278-***
6	Dust cover 19x 1.5 black .....	50010		37.5 cm (15") .....	25279-***
7	Leading bogie backrest D12 L25.5 .....	20205		40 cm (16") .....	25280-***
8	Front fork medium aluminium axle .....	24621-**		42.5 cm (17") .....	25281-***
8:1	Front fork medium without axle .....	24555-**		45 cm (18") .....	25282-***
8:2	Axle front fork aluminium .....	20185	31:1	Tape anti slide W25 mm .....	80478
8:3	Washer BRB 13,5 med fas black .....	13032	32	Velcro strap L70 mm .....	62138
8:4	Locking nut 1/2 UNF low .....	12037			
8:5	Washer BRB 13,5 black .....	13031			
8:6	Cover axle wheel .....				

\*) Colour code: 01=black, 02=red, 22=grey texture, 30=blue texture, 33=seegreen texture, 34=plum texture, 38=black texture, 37=blue chrome, 47=dull blast

\*\*\*) Colour code: 01=black, 35=black blast, 36=gold

\*\*\*\*) Colour code: 01=black, 35=black blast, 38=black texture

## Upholstery back and seat

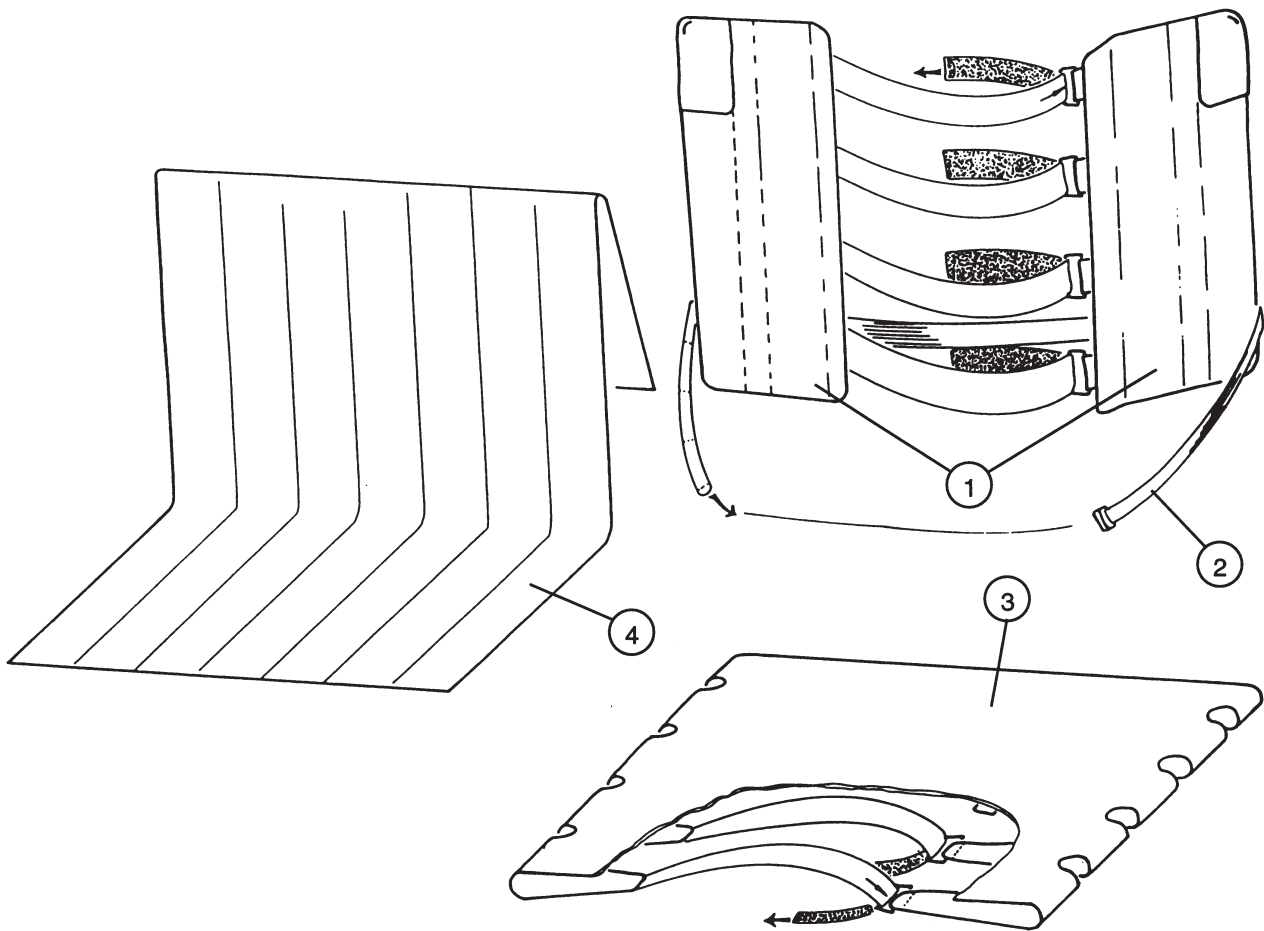

Pos	Description	Item no	
1	Upholstery back	35/37.5 cm (14"/15") .....	62120-60
		40/42.5 cm (16"/17") .....	62121-60
		45 cm (18") .....	62122-60
2	Elastic ribbon .....	62117	
3	Upholstery seat short	35/37.5 cm (14"/15") .....	62179-60
		40/42.5 cm (16"/17") .....	62276-60
		45 cm (18") .....	62277-60
	long	35/37.5 cm (14"/15") .....	62181-60
		40/42.5 cm (16"/17") .....	62278-60
		45 cm (18") .....	62279-60
4	Cover Upholstery	35 cm (14") .....	62135-60
		37.5 cm (15") .....	62387-60
		40 cm (16") .....	62136-60
		42.5 cm (17") .....	62388-60
		45 cm (18") .....	62137-60