


Act


Sparepart list

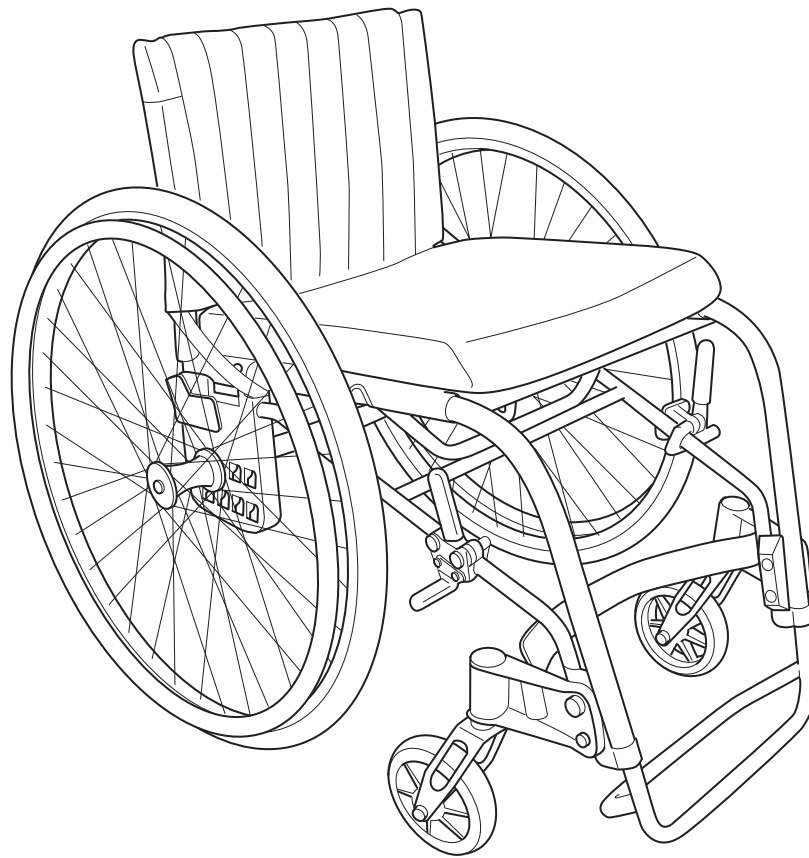

74079G 2012-02-24



Etac
Box 203, 334 24 Anderstorp
tel +46 371 58 73 00
fax +46 371 58 73 90
www.etac.com

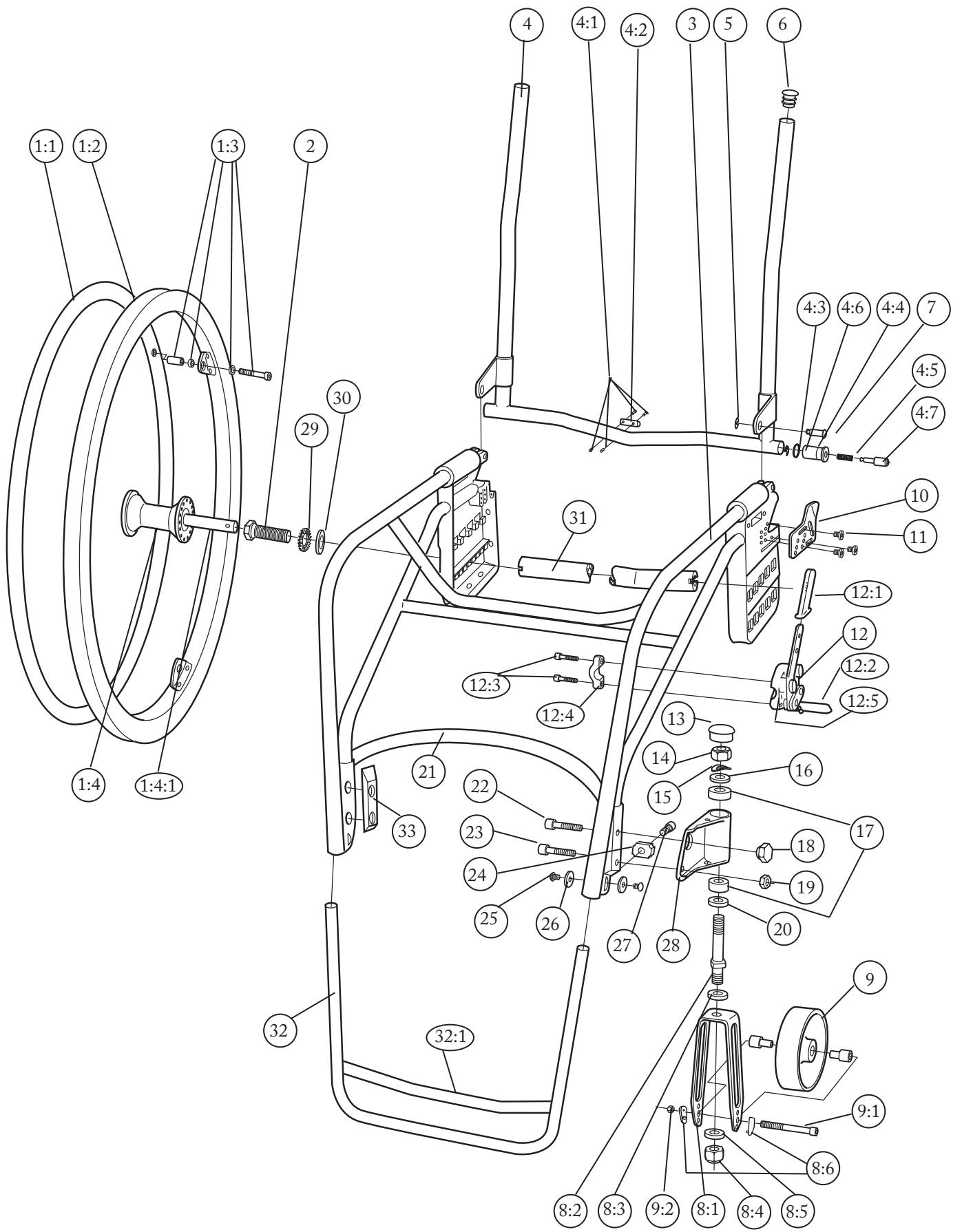

valid from serial number: 21 A 00100


Pos	Description	Item no	Pos	Description	Item no
1	Rear wheel 24" high press titanium handrim	25493	9	Castor 125 x 30 solid	24522
	Rear wheel 24" high press stainless steel handrim .	25494	9:1	Screw M 6x 70 MC6S 12.9 black chrome	11349
	Rear wheel 24" high press aluminium handrim	25505	9:2	Locking nut M 6 black chrome	12035
	Rear wheel 24" high press plastic coated handrim .	26155	10	Backrest angle plate right	25418-1
	Tyre 24" pneumatic high pressure	80558		left	25418-2
	Innertube 24" pneumatic	80560	11	Screw M 5x 20 MC6LS NV4	11320
1:1	Handrim D540 wheel 24 titanium	24540	12	Brake right	27277-1
	Handrim D540 wheel 24 stainless steel	24023		left	27277-2
	Handrim D540 wheel 24 aluminium	25181	12:1	Handle brake extension	50188
	Handrim D540 wheel 24 plastic coated	80894	12:2	Brake plunger L48	20325
1:2	Rear wheel 24" high press without handrim	25491	12:3	Screw M 6x 22 MC6S 8.8 black chrome esloc	11239
1:3	Attachment kit handrim	24514	12:4	Locking joint for brake	25449-01
1:4	Hub aluminium	80524	12:5	Skrøv M6x16 MC6S svkr	11253
1:4:1	Axle aluminium hub	20349	13	Dust cover 32x 1.5 black	50144
2	Axle sleeve	20079	14	Locking nut 1/2 UNF low	12037
3	Frame seat short 35 cm (14")	25396-*	15	Washer 13x24x1.2 fzb	13040
	37.5 cm (15")	25397-*	16	Washer BRB 13.5 black	13031
	40 cm (16")	25398-*	17	Ball bearing 1/2 EE4 2RS	80244
	42.5 cm (17")	25399-*	18	Locking bolt attachment front fork	20341
	45 cm (18")	25400-*	19	Locking nut M 8 black.....	12032
	long 35 cm (14")	25708-*	20	Washer BRB 13.5 black chrome plated	13032
	37.5 cm (15")	25401-*	21	Calf strap 35/37.5 cm (14"/15")... ..	62145
	40 cm (16")	25402-*		40/42.5 cm (16"/17")... ..	62146
	42.5 cm (17")	25403-*		45 cm (18")	62147
	45 cm (18")	25404-*	22	Screw M 8x 40 MC6S 8.8 black chrome	11386
4	Frame backrest 35 cm (14")	24492-*	23	Screw M 8x 45 MC6S 8.8 black chrome	11387
	37.5 cm (15")	24493-*	24	Locking plate M 8	20399
	40 cm (16")	24494-*	25	Screw M 5x 8 K6S black chrome	11304
	42.5 cm (17")	24495-*	26	Washer 5,5x18x2 black	13056
	45 cm (18")	24496-*	27	Screw M8x20 MC6S 8.8 fzb	11222
	Backrest locking kit without wire	24574	28	Front fork housing right	25414-**-1
4:1	Stud M4x4 P6SS fzb	11267		left	25414-**-2
4:2	Handle backrest lock	80373	29	Locking washer AZ 5/8	13039
4:3	Cir clip RSM 5 stainless	13037	30	Spacer rear axle	20370
4:4	Sleeve backrest locking	20203	31	Rear axle 2.5° 35cm (14")	25232-*
4:5	Spring D9 d7 L15	80340		37.5cm (15")	25233-*
4:6	O-ring Di=13.1x1.6 Nitril	80663		40 cm (16")	25234-*
4:7	Wire backrest folding 35 cm (14")	24637		42.5 cm (17")	25235-*
	37.5 cm (15")	24638		45 cm (18")	25236-*
	40 cm (16")	24639		Rear axle 0° 35 cm (14")	25247-01
	42.5 cm (17")	24640		37.5 cm (15")	25248-01
	45 cm (18")	24641		40 cm (16")	25249-01
5	Cir clip RSM 6 stainless	13036		42.5 cm (17")	25250-01
6	Dust cover 19x 1.5 black	50010		45 cm (18")	25251-01
7	Leading bogie backrest D12 L25.5	20205		Rear axle 6° 35 cm (14")	25419-01
8	Front fork short aluminium axle black blast	24620-**		37.5 cm (15")	25420-01
8:1	Front fork short without axle	24554-**		40 cm (16")	25421-01
8:2	Washer BRB 13.5 black chrome plated	13032		42.5 cm (17")	25422-01
8:4	Locking nut 1/2 UNF low	12037		45 cm (18")	25423-01
8:5	Washer BRB 13,5 black	13031			
8:6	Cover axle wheel	50880	32	Footrest 35 cm (14")	25424-***
				37.5 cm (15")	25425-***
				40 cm (16")	25426-***
				42.5 cm (17")	25427-***
				45 cm (18")	25428-***
			32:1	Tape anti slide W25 mm	80478
			33	Ankle pad Act right	50357
				left	50358

*) Colour code: 01=black, 02=red, 22=grey texture, 30=blue texture, 33=seegreen texture, 34=plum texture, 38=black texture, 37=blue chrome, 47=dull blast

**) Colour code: 01=black, 35=black blast, 36=gold

***) Colour code: 01=black, 35=black blast, 38=black texture

UPHOLSTERY BACK AND SEAT

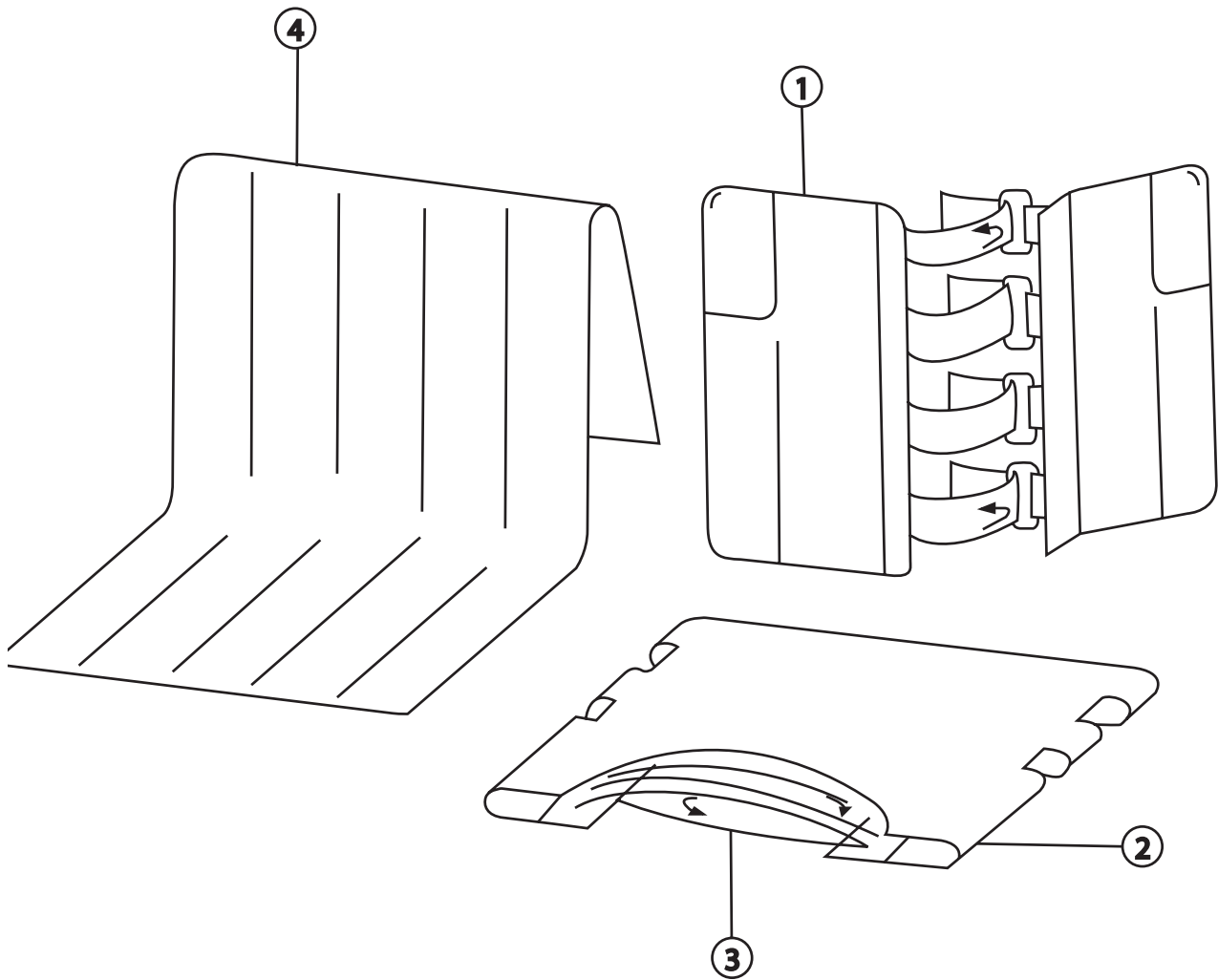

POS	DESCRIPTION		ITEM NO
1	Upholstery back	35/37.5 cm (14"/15") black	62120-60
		40/42.5 cm (16"/17")	62121-60
		45 cm (18")	62122-60
2	Elastic ribbon		62117
3	Upholstery seat short	35/37.5 cm (14"/15") black	62179-60
		40/42.5 cm (16"/17")	62276-60
		45 cm (18")	62277-60
	long	35/37.5 cm (14"/15")	62181-60
		40/42.5 cm (16"/17")	62278-60
		45 cm (18")	62279-60
4	Cover Upholstery	35 cm (14") dark grey	62135-60
		37.5 cm (15") dark grey	62387-60
		40 cm (16")	62136-60
		42.5 cm (17")	62388-60
		45 cm (18")	62137-60