


Etac Prio

Spareparts list english

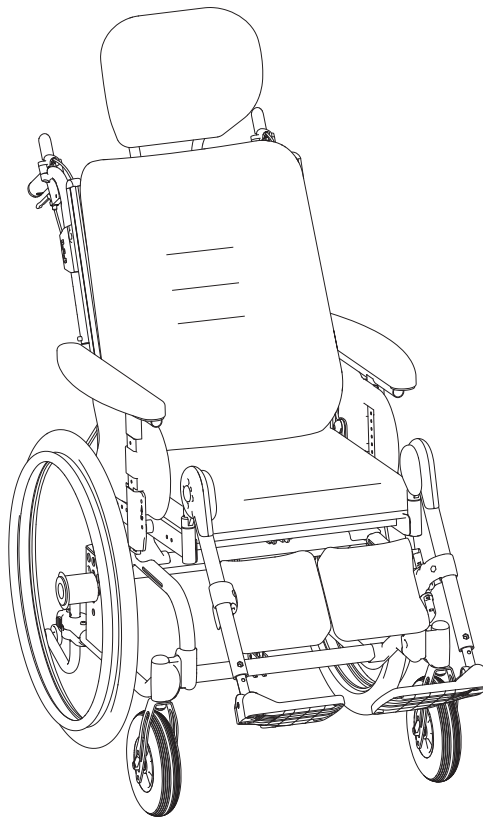

75221F 2017-03-14



Index


Index	Page
Frame	3
Seat frame	4
Gas pistons, comfort lock	5
Back support frame	6-9
Push handles	10
Bow handle	11
Bow handle angle adjustable	12
Leg support	13-14
Leg support fixed angle	15
Amputee leg support	16
Foot support	17-18
Foot support one piece	19
User brake	20
Rear wheel	21
Attendant brake	22
Tyre and innertube	23
Castor	24-25
Widening kit	26
Solid seat	27-28
Arm support	29
Arm support Hemi	30
Trunk support	31-32
Leg Abduction	33
Tray	34
Attachment harness	35
IV stand	36
Basket	37
Positioning belt	38
Anti tip	39
Spoke guard	40
Head support	41-44
Seat cushion	45-46
Seat cushion - Cover	47
Back support cushion	48
Back support upholstery	49-50

Explanation for list of points

- Applies to model

Prio equipped with 16" rear wheels	Prio	Prio AB	Prio 16"
Prio equipped with attendant brakes			
Prio equipped with handrim rear wheels			


Frame

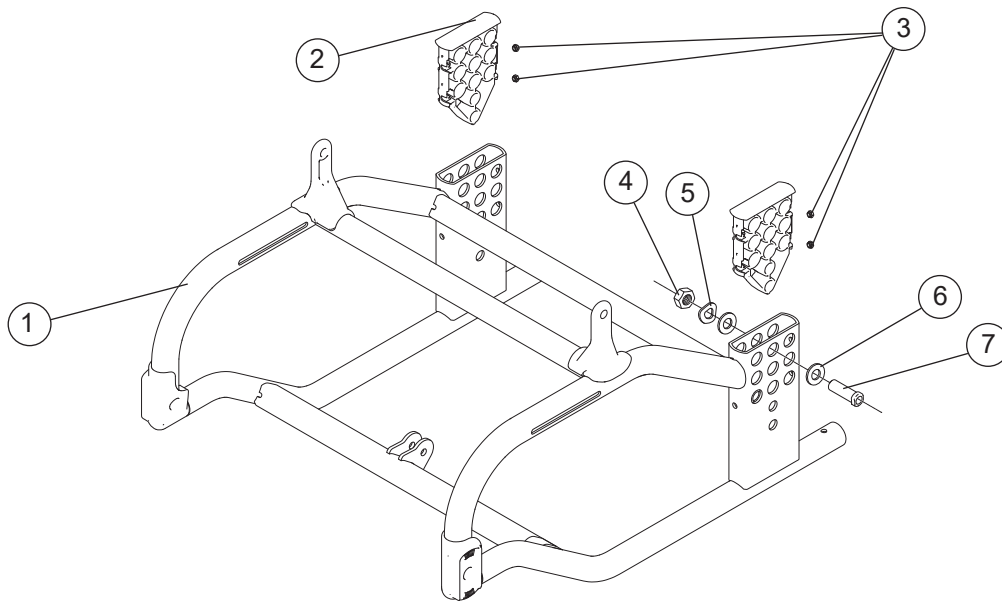

Pos.	Description	Item no.
1	Frame Prio	
2	Cover rear wheel attachm Prio	
3	Square nut high M6	
4	Nut M16x 1 NV 24 B8	12052
	Nut M12 Low DIN 439B	12076
5	Locking washer AZ 5/8	13039
6	BRB 17x30x3 Washer	13117
7	Axle sleeve M100 D12	20024


Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	
		●
●	●	
●	●	
●	●	

Seat frame

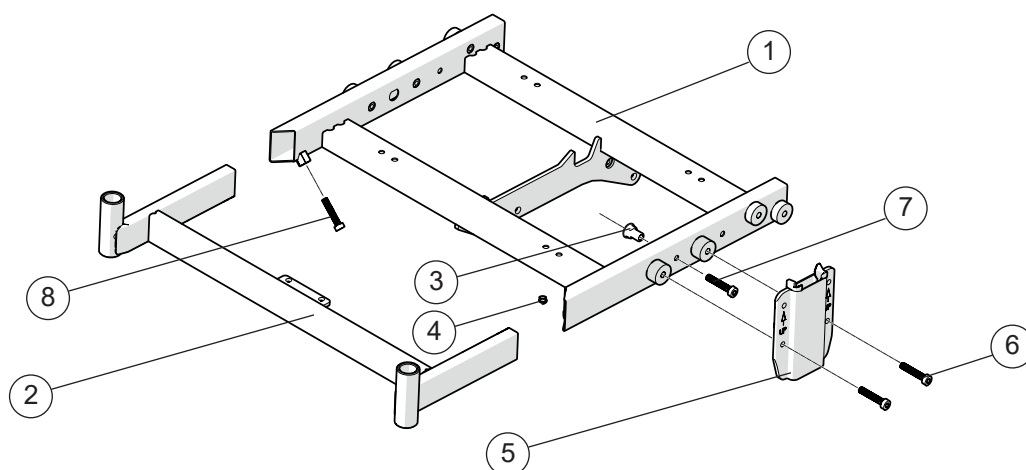

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Seat frame rear Prio		●	●	●
2	Seat frame front Prio		●	●	●
3	Joint nut seat frame Prio	12117	●	●	●
4	Wire attachment	50893	●	●	●
5	Attachment armrest	25535	●	●	●
6	Screw M 6x 16 MC6S 12.9	11253	●	●	●
7	Screw MC6LS 6x14 SW=5	11539	●	●	●
8	Screw M8x16 K6S A2	11014	●	●	●


Gas pistons, comfort lock

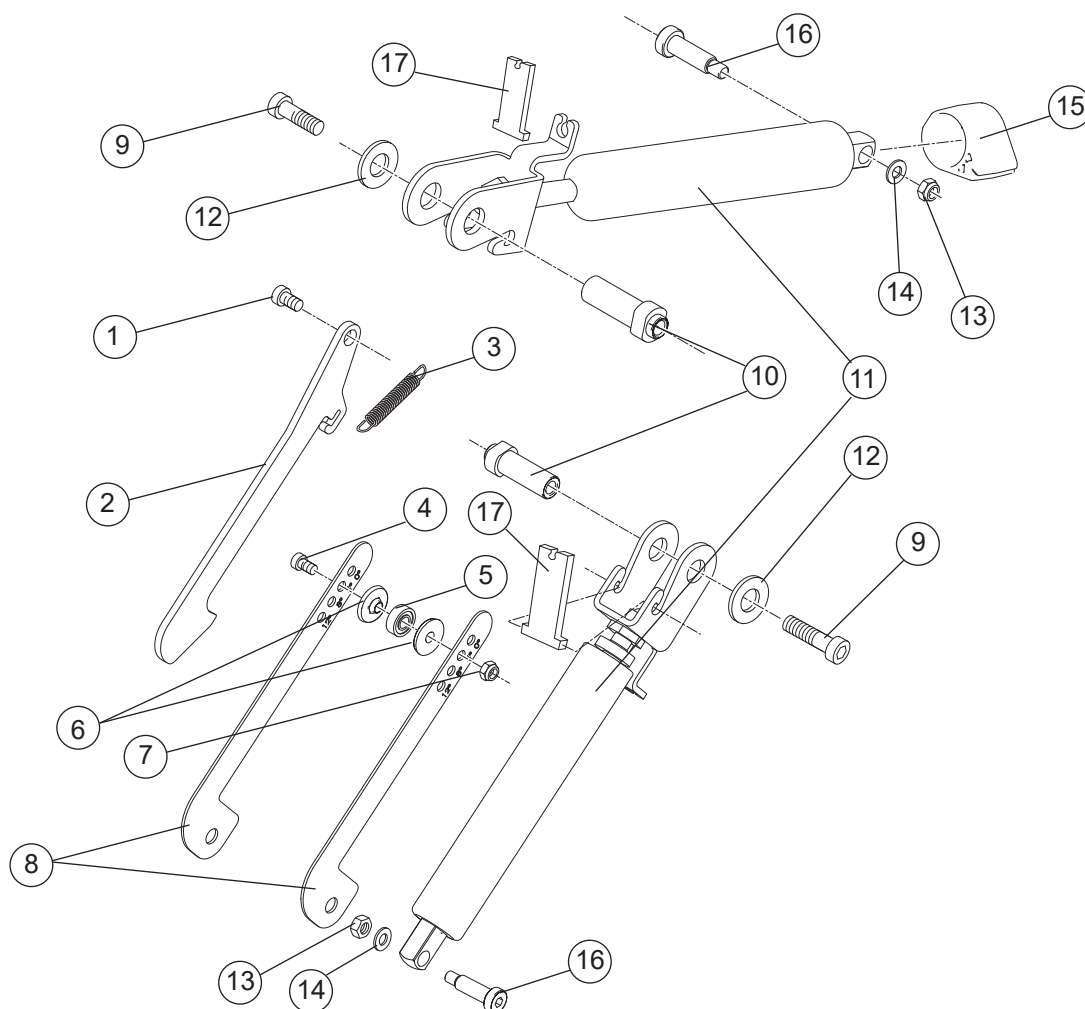

Pos.	Description	Item no.
1	Screw M 6x 16 MC6S 12.9	11253
2	Comfort lock	82667
3	Spring	82690
4	Screw M5x16	11424
5	Bearings Dy15 Di6 t5 (NOMO 696)	82660
6	Bearing bushing comfort lock K	20509
7	Locking nut M 5	12003
8	Adjustment plate comfort lock	82691
9	Screw M 8x 20 fzb	11529
10	Joint sleeve key socket gas pi	20511
11	Gas spring cpl	27682
12	Washer 25/10/1.5 black pa	50281
13	Locking nut M 6	12017
14	Washer BRB 6x3x10x1.5 fzb	13080
15	Pull mechanism active backrest plastic Prio	51042
16	Screw 12.9 8xM6x21.5 Prio	20513
17	Lock clapper	82697


Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Back support (from ser. no. 38B1)

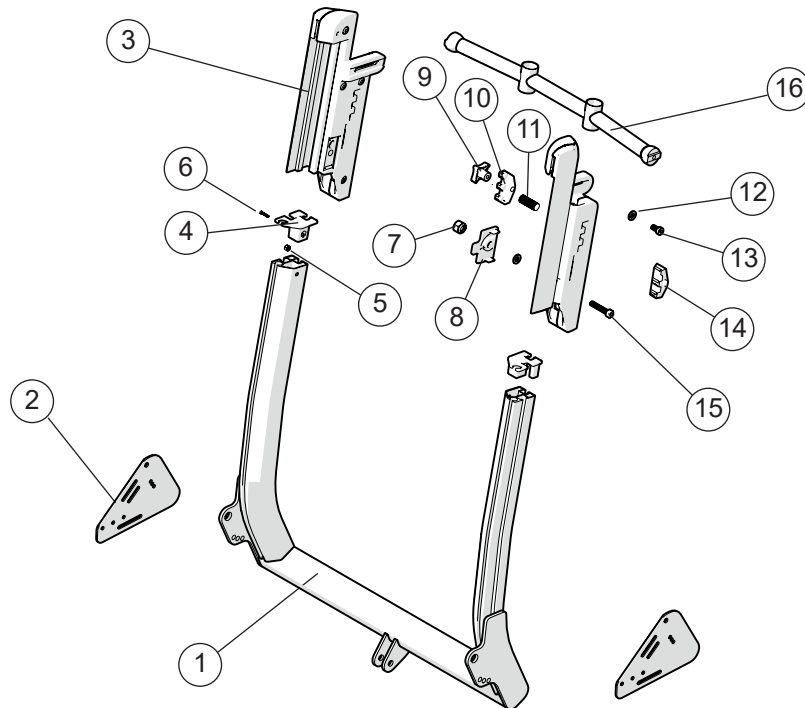

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1	Back support Prio		●	●	●
2	Back joint plate riveted right		●	●	●
	Back joint plate riveted left		●	●	●
3	Upper part right		●	●	●
	Upper part left		●	●	●
4	Dust cover backrest profile right	51104-1	●	●	●
	Dust cover backrest profile left	51104-2	●	●	●
5	Locking nut M 4 fzb	12026	●	●	●
6	Screw M 4x 12 MRX 4.8 pos	11637	●	●	●
7	Locking nut M8 DIN 985-6 black chrome	12032	●	●	●
8	Locking detail back support height Prio	82938	●	●	●
9	Knob M 5x32	82536	●	●	●
10	Locking plate push handle inner	50998	●	●	●
11	Spring 0.75x5.0x15 mm left wound	82934	●	●	●
12	Washer BRB M 6 black chrome	13030	●	●	●
13	Screw M6x18 MC6S 8.8 Fe/Zn8/P4	11539	●	●	●
14	Locking plate push handle outer	50999	●	●	●
15	Screw M8x35 MC6LS Fe/Zn8/P4	11677	●	●	●
16	Head support bracket		●	●	●

Back support (to ser. no. 38A9999)

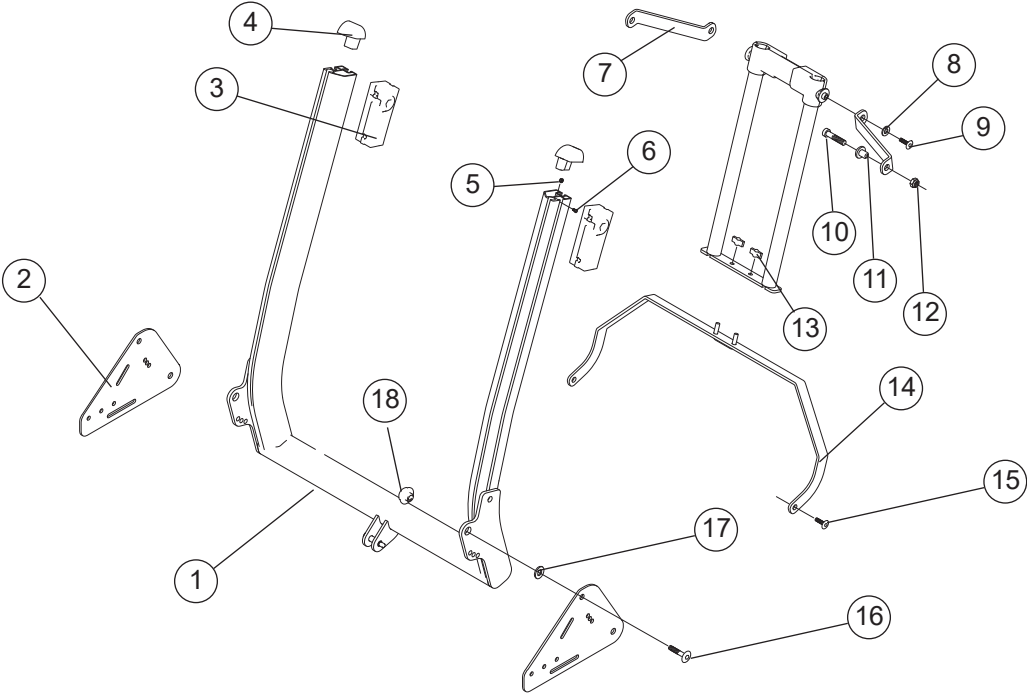


Pos.	Description	Item. no.			
1	Backrest welded 40 cm Prio	27674-01	●	●	●
	Backrest welded 45 cm Prio	27675-01	●	●	●
	Backrest welded 50 cm Prio	27676-01	●	●	●
	Backrest welded 55 cm Prio	27677-01	●	●	●
2	Back joint plate riveted right	27721-1	●	●	●
	Back joint plate riveted left	27721-2	●	●	●
3	Attachment push handle Prio right	27855-1	●	●	●
	Attachment push handle Prio left	27855-2	●	●	●
4	Dust cover backrest profile right	51044-1	●	●	●
	Dust cover backrest profile left	51044-2	●	●	●
5	Locking nut M 4 fzb	12026	●	●	●
6	Screw M 4x 12 MRX 4.8 pos	11637	●	●	●
7	Slewing bracket backrest kit 40	82719	●	●	●
	Slewing bracket backrest kit 45	82770	●	●	●
	Slewing bracket backrest kit 50	82771	●	●	●
	Slewing bracket backrest kit 55	82772	●	●	●
8	Washer 6.4x22x4 fzb	13062	●	●	●
9	Screw M 6x 14 K6S fzb	11358	●	●	●
10	Screw M 6x 40 K6S fzb	11594	●	●	●
11	Joint sleeve adapter mechanism	82703	●	●	●
12	Locking nut M 6 fzb	12017	●	●	●
13	Wing nut WN 314.1	80847	●	●	●

Back support (to ser. no. 38A9999)


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
14	Weld lower mechanism backrest 40	27738	●	●	●
	Weld lower mechanism backrest 45	27739	●	●	●
	Weld lower mechanism backrest 50	27740	●	●	●
	Weld lower mechanism backrest 55	27741	●	●	●
15	Screw M 6x 8 K6S black chrome	11410	●	●	●
16	Screw M 8x 20 fzb	11529	●	●	●
17	Washer 25/10/1.5 black pa	50281	●	●	●
18	Nut backrest angle Prio	82712	●	●	●

Back support 3D (from ser. no. 38B1)

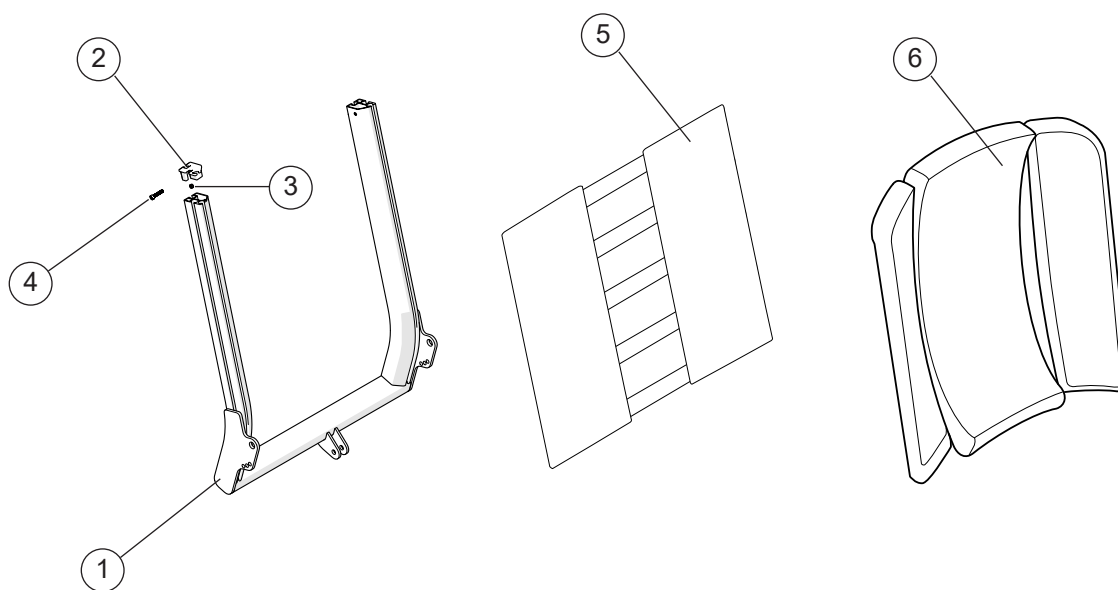


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-6	Back support 3D Prio 40	28026	●	●	●
	Back support 3D Prio 45	28027	●	●	●
	Back support 3D Prio 50	28028	●	●	●
	Back support 3D Prio 55	28029	●	●	●
1	Back support frame 3D Prio		●	●	●
2	Dust cover backrest profile right	51104-1	●	●	●
	Dust cover backrest profile left	51104-2	●	●	●
3	Locking nut M 4 fzb	12026	●	●	●
4	Screw M 4x 12 MRX 4.8 pos	11637	●	●	●
5	Back support upholstery 3D Prio 40	63148-60	●	●	●
	Back support upholstery 3D Prio 45	63148-60	●	●	●
	Back support upholstery 3D Prio 50	63148-60	●	●	●
	Back support upholstery 3D Prio 55	63148-60	●	●	●
6	Back support cushion 3D Prio 40 plush	28015-60	●	●	●
	Back support cushion 3D Prio 45 plush	28016-60	●	●	●
	Back support cushion 3D Prio 50 plush	28017-60	●	●	●
	Back support cushion 3D Prio 55 plush	28018-60	●	●	●

Push handles

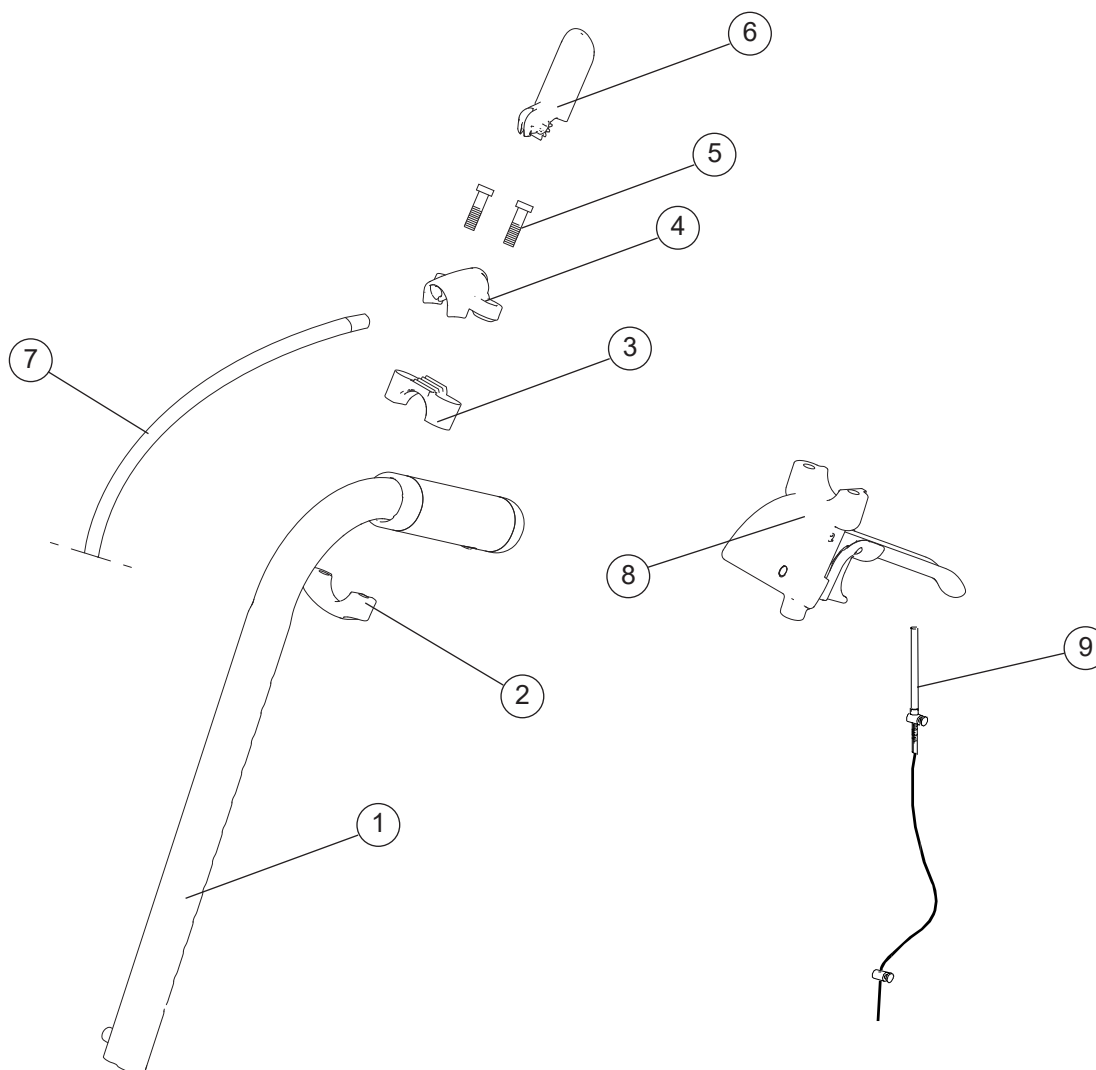

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-7	Push handle seat angle adj	27807-01	●	●	●
	Push handle back tilt controls	27806-01	●	●	●
1-9	Push handle seat tilt brake controls	27809-01	●	●	●
	Push handle back tilt brake controls	27808-01	●	●	●
1	Push handle		●	●	●
2	Lower attachment wire control		●	●	●
3	Attachment drum brake Prio		●	●	●
4	Upper attachment wire control		●	●	●
5	Screw DG50 x 30 WN		●	●	●
6	Control Prio		●	●	●
7	Wire back support angle		●	●	●
	Wire seat tilt		●	●	●
8	Brake handle Prio compl (see section "Drum brake")		●	●	●
9	Wire kit drum brake (see section "Drum brake")		●	●	●

Bow handle

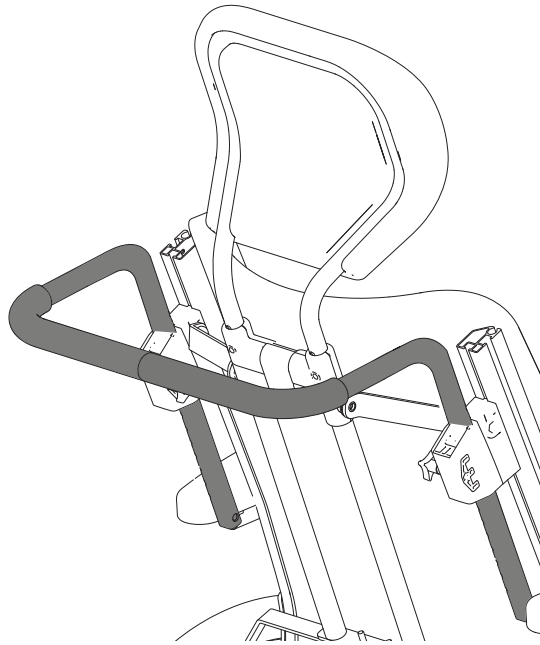


Pos.	Description	Item no.
1	Bow handle 40 cm height adj compl	25580-01
	Bow handle 45 cm height adj compl	25582-01
	Bow handle 50 cm height adj compl	26064-01
	Bow handle 55 cm height adj compl	26069-01

Prio

Prio AB


Prio 16"

Bow handle angle adjustable

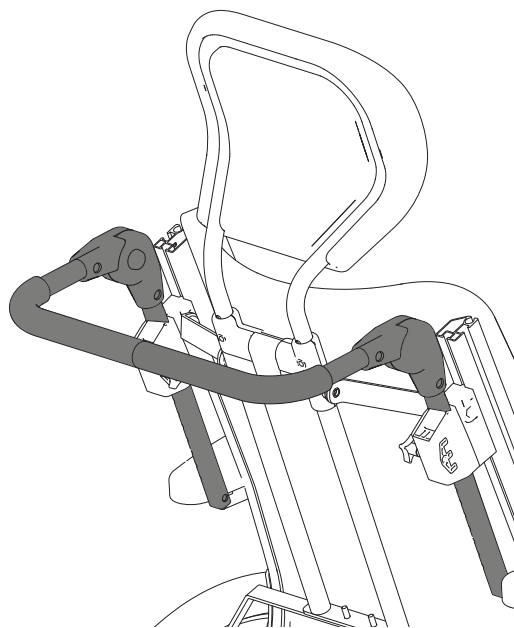

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Bow handle angle adj 40 cm	27163	●	●	●
	Bow handle angle adj 45 cm	27165	●	●	●
	Bow handle angle adj 50 cm	27167	●	●	●

Leg support elevating

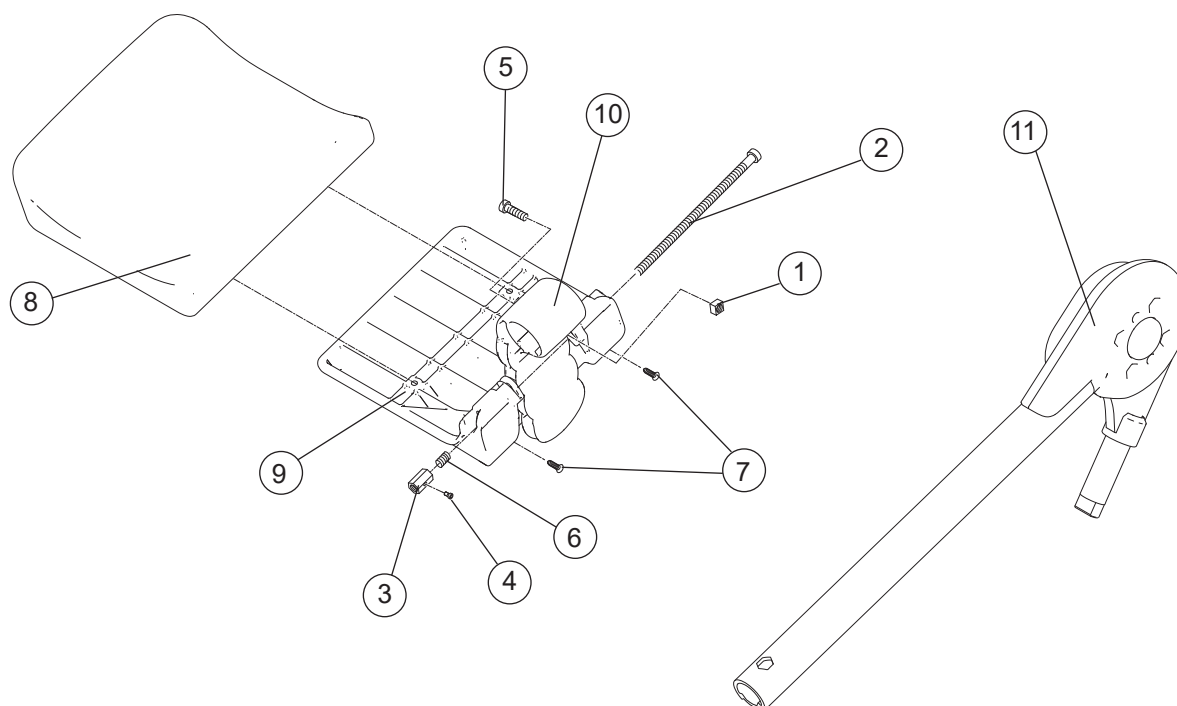


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Nut attachment calf support		●	●	●
2	Screw MC6S M6 x120 10.9		●	●	●
3	Nut M6 calf support		●	●	●
4	Screw M 4x 10 MC6S 8.8		●	●	●
5	Screw M 6x 20 MC6S 8.8		●	●	●
6	Spring		●	●	●
7	Screw B 8x19 RXS/A2 ph st 4.2	11064	●	●	●
8	(Pad calf support Prio 40 green/grey hygiene)		●	●	●
9	(Calf support)		●	●	●
10	(Attachment calf support)		●	●	●
7-10	Calf support cpl 40 cm green hygiene	27733-64	●	●	●
	Calf support cpl 40 cm grey hygiene	27733-65	●	●	●
	Calf support cpl 45 cm green hygiene	27734-64	●	●	●
	Calf support cpl 45 cm grey hygiene	27734-65	●	●	●
	Calf support cpl 50 cm green hygiene	27735-64	●	●	●
	Calf support cpl 50 cm grey hygiene	27735-65	●	●	●
	Calf support cpl 55 cm green hygiene	27736-64	●	●	●
	Calf support cpl 55 cm grey hygiene	27736-65	●	●	●

Leg support elevating


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
11	Leg support elevating right	27730-01-1	●	●	●
	Leg support elevating left	27730-01-2	●	●	●
1-11	Leg support elevating Prio 40 cm green hygiene complete right	27857-64-1	●	●	●
	Leg support elevating Prio 40 cm green hygiene complete left	27857-64-2	●	●	●
	Leg support elevating Prio 40 cm grey hygiene complete right	27857-65-1	●	●	●
	Leg support elevating Prio 40 cm grey hygiene complete left	27857-65-2	●	●	●
	Leg support elevating Prio 45 cm green hygiene complete right	27858-64-1	●	●	●
	Leg support elevating Prio 45 cm green hygiene complete left	27858-64-2	●	●	●
	Leg support elevating Prio 45 cm grey hygiene complete right	27858-65-1	●	●	●
	Leg support elevating Prio 45 cm grey hygiene complete left	27858-65-2	●	●	●
	Leg support elevating Prio 50 cm green hygiene complete right	27859-64-1	●	●	●
	Leg support elevating Prio 50 cm green hygiene complete left	27859-64-2	●	●	●
	Leg support elevating Prio 50 cm grey hygiene complete right	27859-65-1	●	●	●
	Leg support elevating Prio 50 cm grey hygiene complete left	27859-65-2	●	●	●
	Leg support elevating Prio 55 cm green hygiene complete right	27860-64-1	●	●	●
	Leg support elevating Prio 55 cm green hygiene complete left	27860-64-2	●	●	●
	Leg support elevating Prio 55 cm grey hygiene complete right	27860-65-1	●	●	●
	Leg support elevating Prio 55 cm grey hygiene complete left	27860-65-2	●	●	●

Leg support fixed

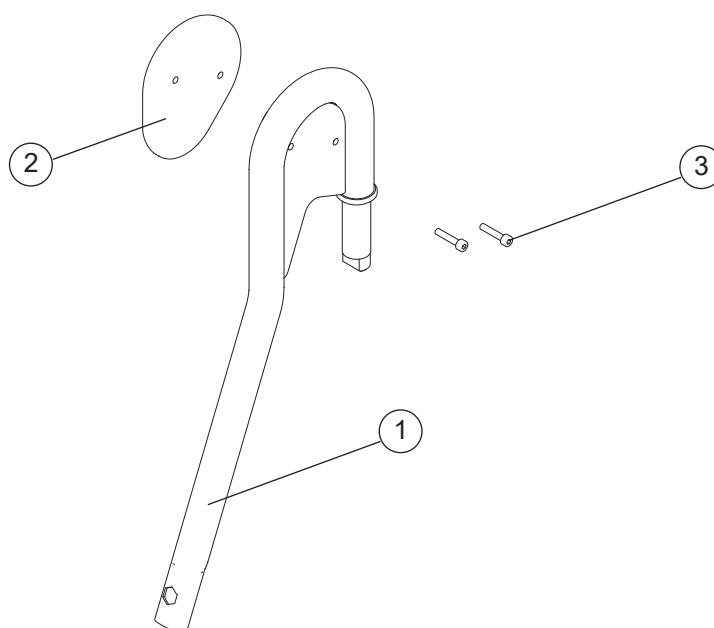


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Legrest fixed cpl right	27729-1	●	●	●
	Legrest fixed cpl left	27729-2	●	●	●
2	Knee pad pur cpl Prio	82666	●	●	●
3	Screw M 5x 20 MF6S	11662	●	●	●

Amputee leg support

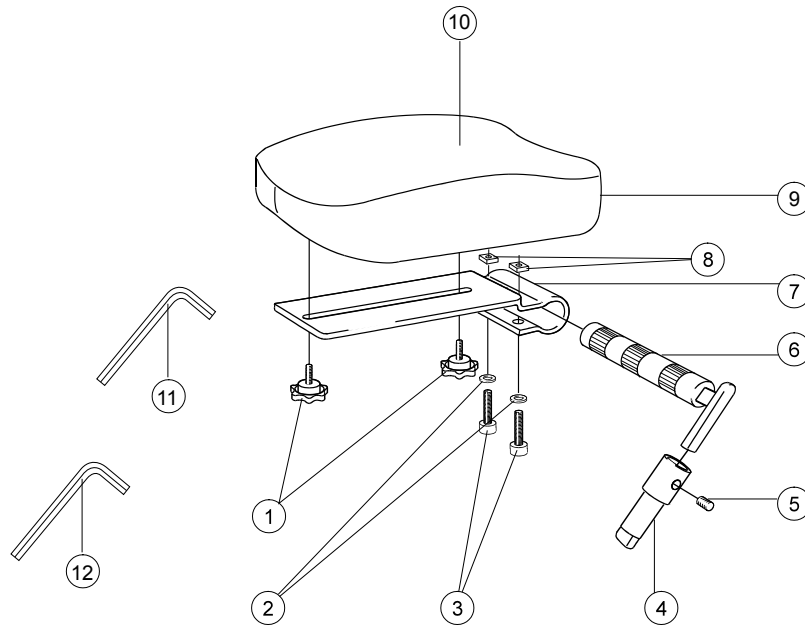


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Knob M 5x15 leg support amputee	80807	●	●	●
2	Washer BRB 8.4/A2 stainless	13011	●	●	●
3	Screw M 8x 35 MC6S	11276	●	●	●
4	Locking tube for leg support amputee	25518-01	●	●	●
5	Stop screw M 8x 8 black	11388	●	●	●
6	Axle leg support amputee compl	25519-01	●	●	●
7	Attachment calf support leg support amputee	80809	●	●	●
8	Square nut high M8	12056	●	●	●
9	Support pad leg support amputee dark grey	62335-60	●	●	●
	Support pad leg support Dartex	62335-63	●	●	●
	Support pad leg support amputee ri XL dark grey	62452-60-1	●	●	●
	Support pad leg support amputee le XL dark grey	62452-60-2	●	●	●
	Support pad leg support amputee ri XL dartex	62452-63-1	●	●	●
	Support pad leg support amputee le XL dartex	62452-63-2	●	●	●
10	Cover leg support amputee dark grey	62893-60	●	●	●
	Cover leg support amputee Dartex	62893-63	●	●	●
	Cover leg support amputee XL ri dark grey	62450-60	●	●	●
	Cover leg support amputee XL ri Dartex	62450-63	●	●	●
	Cover leg support amputee XL le dark grey	62761-60	●	●	●
	Cover leg support amputee XL le Dartex	62761-63	●	●	●
11	Allen key NV 4 DIN 911	80141	●	●	●
12	Allen key Nv 6 180 mm	82625	●	●	●
1-12	Amputee support compl Dark grey	25516-60	●	●	●
	Amputee support compl Dartex	25516-63	●	●	●
	Amputee support XL ri compl dark grey	26210-60-1	●	●	●
	Amputee support XL le compl dark grey	26210-60-2	●	●	●
	Amputee support XL right compl Dartex	26210-63-1	●	●	●
	Amputee support XL left compl Dartex	26210-63-2	●	●	●

Foot support

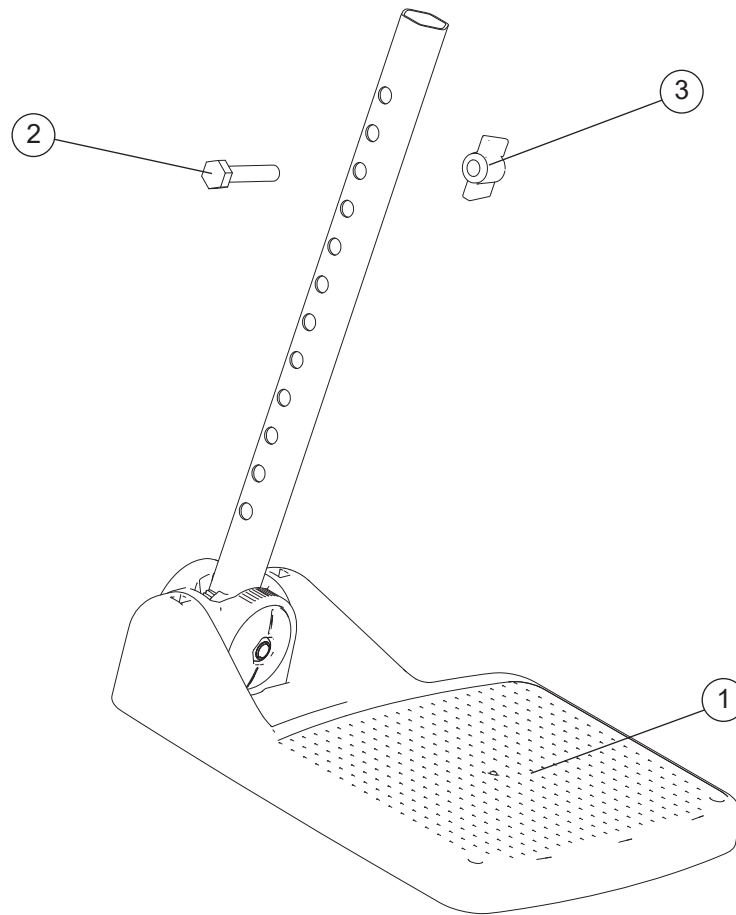


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-3	Foot support depth 15 cm 40 cm right long attachment tube	82354-1	●	●	●
	Foot support depth 15 cm 40 cm left long attachment tube	82354-2	●	●	●
	Foot support depth 15 cm 45 cm right long attachment tube	82356-1	●	●	●
	Foot support depth 15 cm 45 cm left long attachment tube	82356-2	●	●	●
	Foot support depth 15 cm 50 cm right long attachment tube	82357-1	●	●	●
	Foot support depth 15 cm 50 cm left long attachment tube	82357-2	●	●	●
	Foot support depth 15 cm 40 cm right medium attachment tube	82343-1	●	●	●
	Foot support depth 15 cm 40 cm left medium attachment tube	82343-2	●	●	●
	Foot support depth 15 cm 45 cm right medium attachment tube	82345-1	●	●	●
	Foot support depth 15 cm 45 cm left medium attachment tube	82345-2	●	●	●
	Foot support depth 15 cm 50 cm right medium attachment tube	82346-1	●	●	●
	Foot support depth 15 cm 50 cm left medium attachment tube	82346-2	●	●	●
2	Screw M6x 30 M6S	11555	●	●	●
3	Wing nut WN 314.1	80847	●	●	●

Foot support

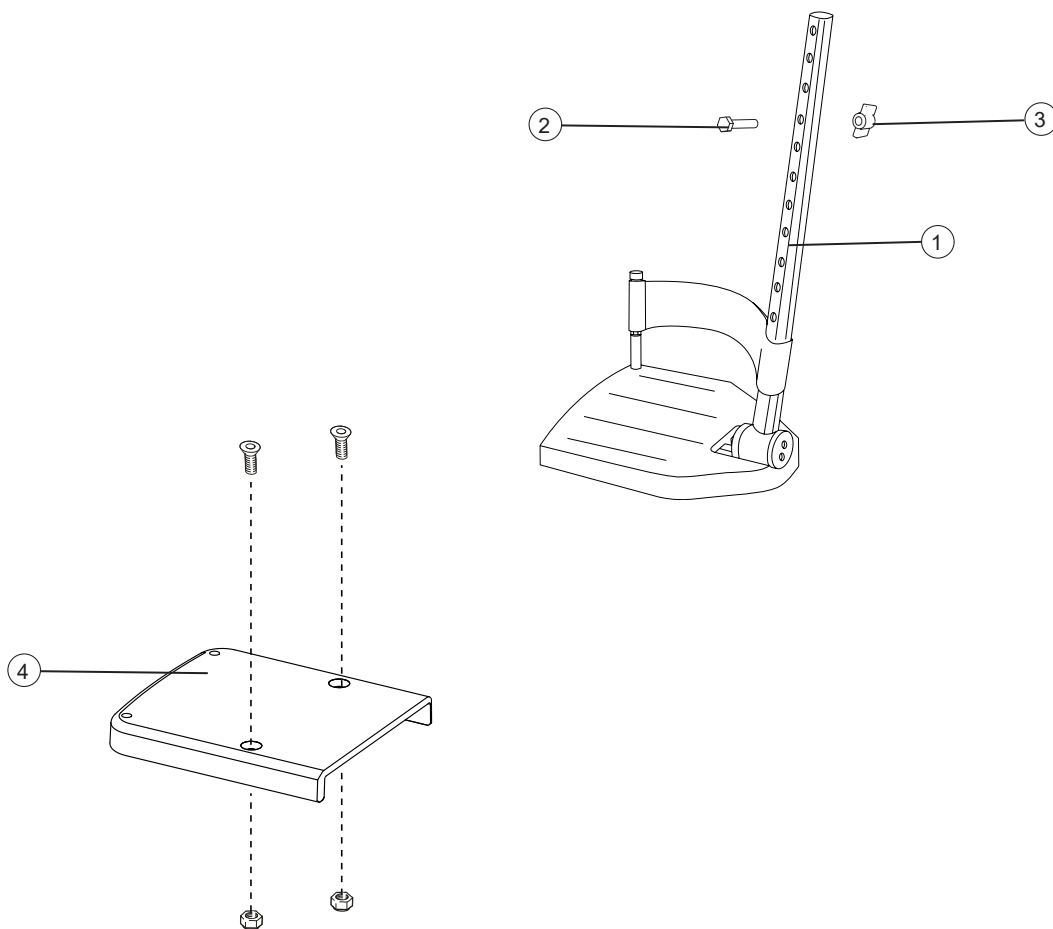


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-3	Foot support depth 13 cm 40 cm right long attachment tube	25650-1	●	●	●
	Foot support depth 13 cm 40 cm left long attachment tube	25650-2	●	●	●
	Foot support depth 13 cm 45 cm right long attachment tube	25737-1	●	●	●
	Foot support depth 13 cm 45 cm left long attachment tube	25737-2	●	●	●
	Foot support depth 13 cm 50 cm right long attachment tube	25652-1	●	●	●
	Foot support depth 13 cm 50 cm left long attachment tube	25652-2	●	●	●
	Foot support depth 13 cm 40 cm right medium attachment tube	25644-1	●	●	●
	Foot support depth 13 cm 40 cm left medium attachment tube	25644-2	●	●	●
	Foot support depth 13 cm 45 cm right medium attachment tube	25736-1	●	●	●
	Foot support depth 13 cm 45 cm left medium attachment tube	25736-2	●	●	●
	Foot support depth 13 cm 50 cm right medium attachment tube	25646-1	●	●	●
	Foot support depth 13 cm 50 cm left medium attachment tube	25646-2	●	●	●
2	Screw M6x 30 M6S	11555	●	●	●
3	Wing nut 26-M 6	80847	●	●	●
4	Foot support widened 52,5-60 cm compl	50885	●	●	●

Foot support one piece

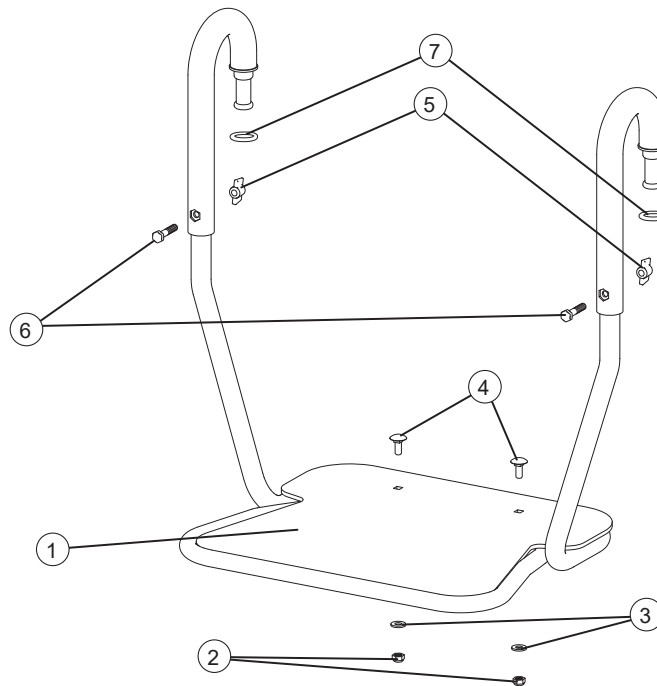


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-8	Foot support one piece=40	27905	●	●	●
	Foot support one piece=45	27906	●	●	●
	Foot support one piece=50	27907	●	●	●
	Foot support one piece=55	27908	●	●	●
2	Locking nut M 6 fzb	12017	●	●	●
3	Washer BRB 6x3x10x1.5	13080	●	●	●
4	Carriage bolt m6x16	11659	●	●	●
5	Wing nut WN 314.1	80847	●	●	●
6	Screw M 6x30 M6S 8.8	11555	●	●	●
7	O-ring 12x3 NBR70	82805	●	●	●
8	Damper adhesive 21x3	51064	●	●	●

User brake

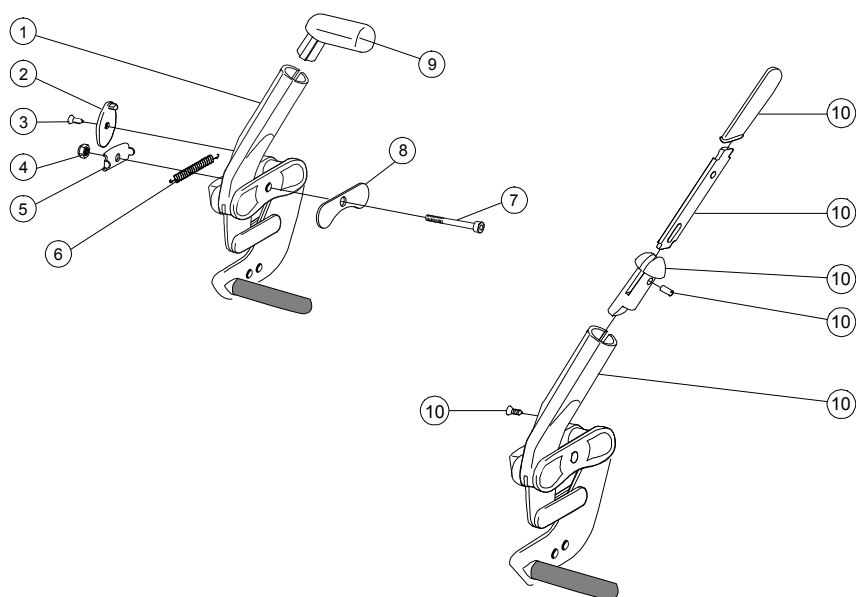

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-9	User brake angle red handle R Prio	27893-1	●	●	
	User brake angle red handle L Prio	27893-2	●	●	
	User brake angle black handle R Prio	27894-1	●	●	
	User brake angle black handle L Prio	27894-2	●	●	
2	Stop detail brake Cross mod new	50373	●	●	
3	Screw B 6 3.5x 9.5 FXS black	11384	●	●	
4	Locking nut M 6 zink plated	12017	●	●	
5	Washer inner brake mod 3	82564	●	●	
6	Spring for brake Cross	80796	●	●	
7	Screw M 6x 50 MC6S zink plated	11433	●	●	
8	Washer outer brake mod 3	82575	●	●	
9	Plug brake Cross red	51012	●	●	
	Plug brake Cross black	50395	●	●	
10	Brake lever extension Cross compl	25515	●	●	
1-8,10	User brake extension handle R Prio	27895-1	●	●	
	User brake extension handle L Prio	27895-2	●	●	


Rear wheel

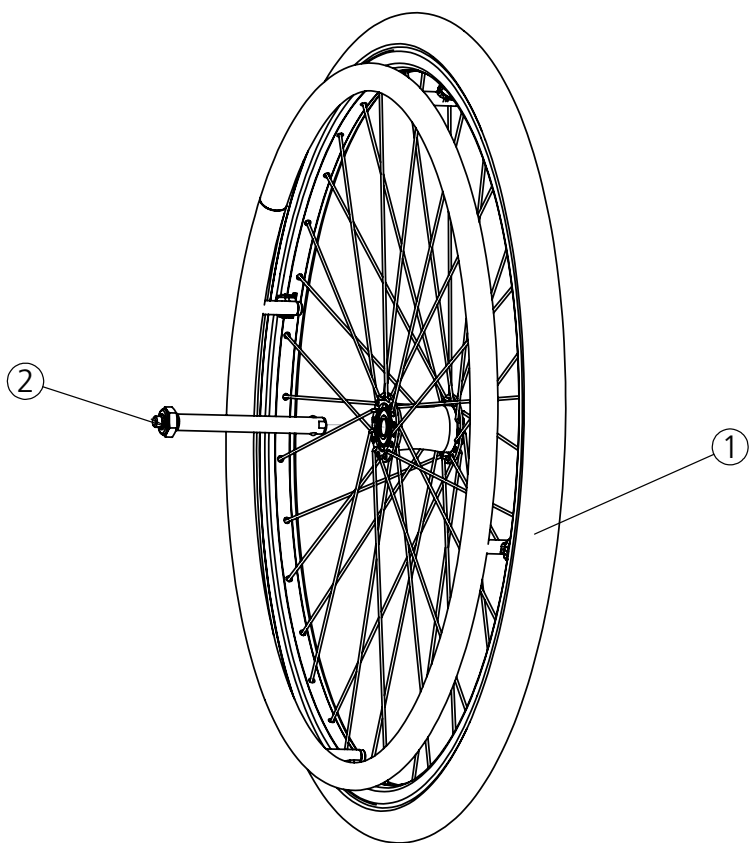

Pos.	Description	Item no.
1	Rear wheel 20 solid/alu M100	82486
	Rear wheel 22 solid/alu M100	82485
	Rear wheel 24 solid/alu M100	82484
2	Axle QR D12 M100 rear wheel	82483

Prio	Prio AB	Prio 16"


Attendant brake

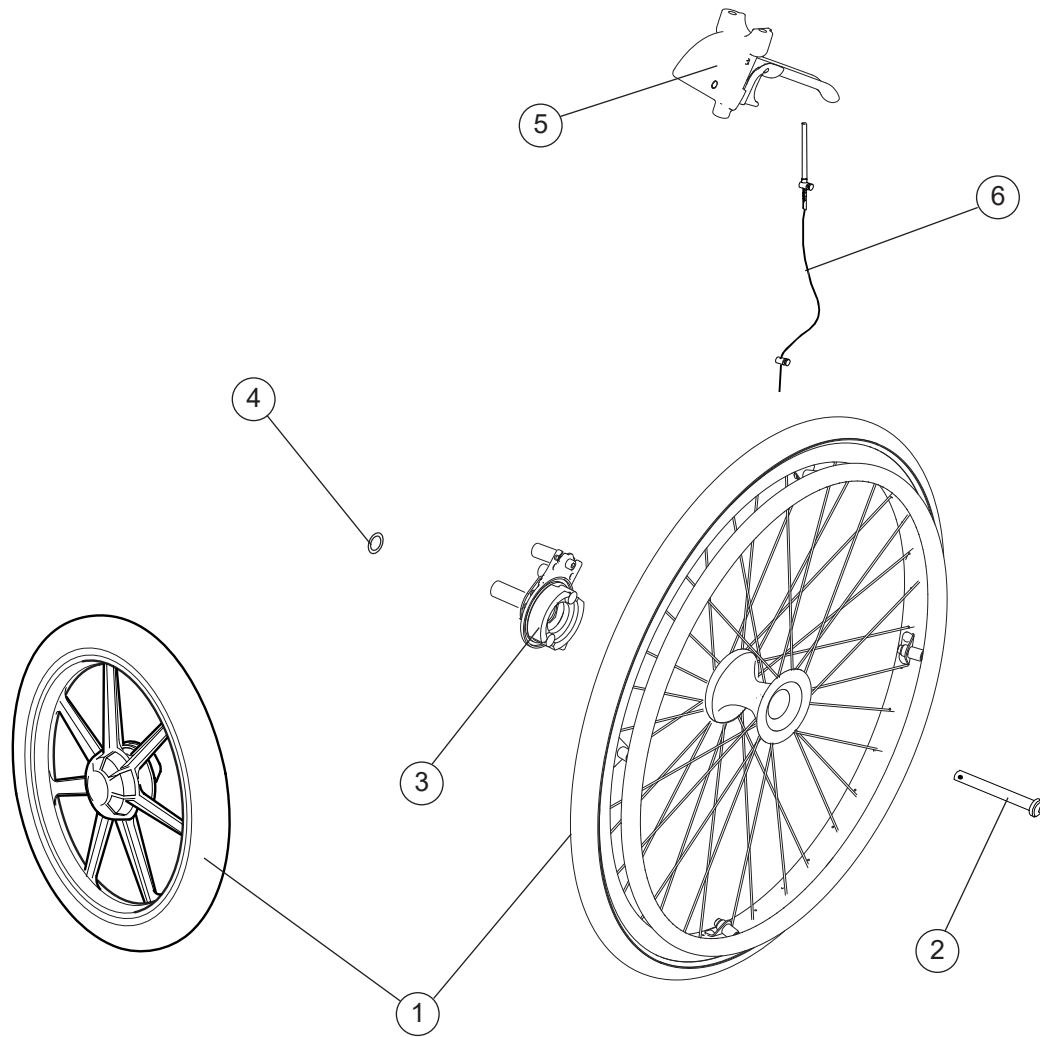

Pos.	Description	Item no.
1	Rear wheel 20 attendant brake	81337
	Rear wheel 22 attendant brake	81339
	Rear wheel 24 attendant brake	81346
	Wheel 16 solid Transit right	81173
	Wheel 16 solid Transit left	81174
2-4	Brake shoe kit attendant brake Prio right	82822
	Brake shoe kit attendant brake Prio left	82823
2	Qr axle attendant brake	81325
4	Schnorr washer 17x24x1.3	13084
5	Brake handle Prio compl	82824
6	Wire kit attendant brake	82856
	Push handles, see section "Push handles"	

Prio	Prio AB	Prio 16"
●	●	
●	●	
●	●	
		●
		●
●	●	
●	●	
●	●	●
●	●	●


Tyre and innertube

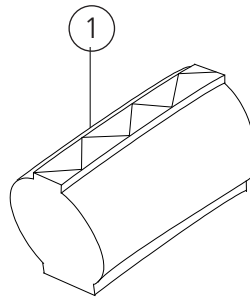

Pos.	Description	Item no.
1	Tyre 20" solid	80787
	Tyre 22" solid	81384
	Tyre 24" solid	80768

Prio	Prio AB	Prio 16"
●	●	
●	●	
●	●	

Castor

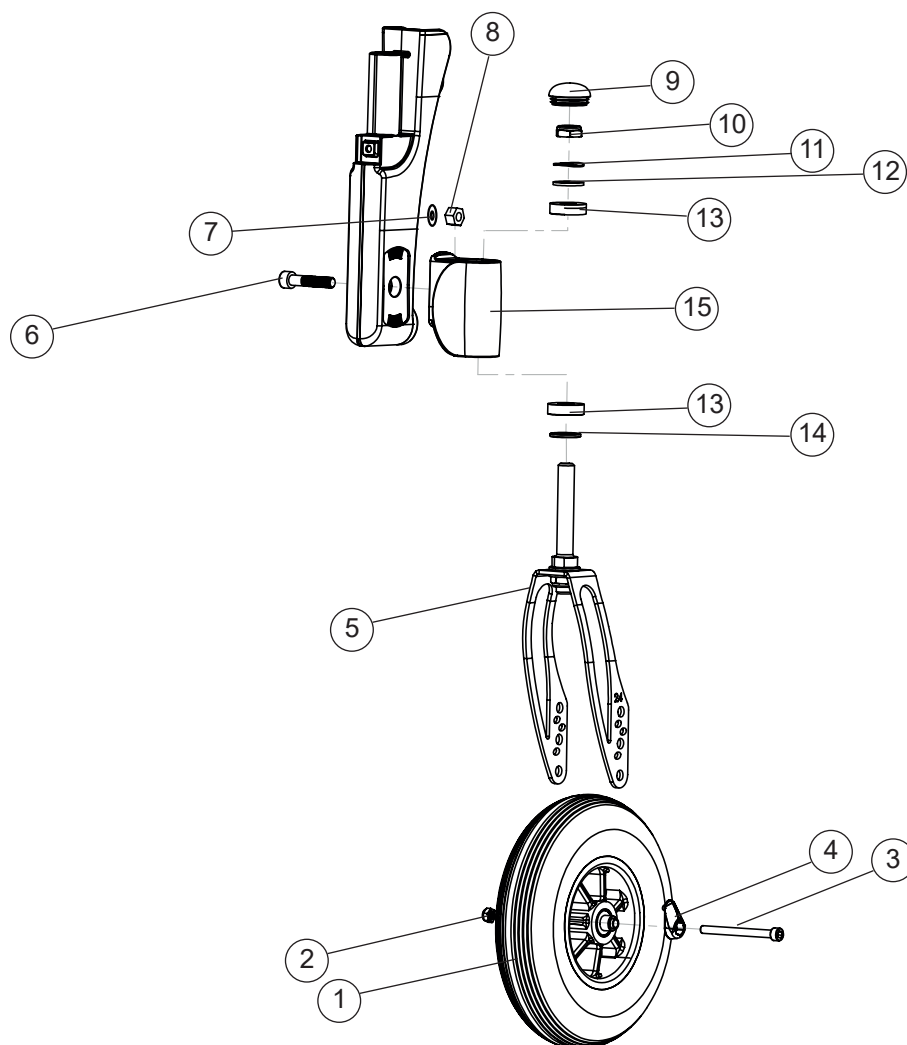


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-4	Castor 8 200x35 solid soft	28217	●	●	●
	Castor 8 200x50 solid soft	28218	●	●	●
	Castor 6 150x30 solid soft	28216	●	●	●
2	Locking nut M 6	12035	●	●	●
3	Screw M 6x 75 MC6S 12.9 black	11338	●	●	●
4	Cover axle wheel PA 6 grey	51049	●	●	●
	Cover axle wheel PA 6 (50 pcs) grey	27954	●	●	●
	Cover axle wheel PA 6 black	51135	●	●	●
5	Front fork M100 std grey	27126-42	●	●	●
	Front fork 6 grey	27125-01	●	●	●
	Front fork 8 black	28214-01	●	●	●
	Front fork 6 black	28215-01	●	●	●
6	Screw M 8x 1x40 MC6S	11305	●	●	●
7	Washer TBRUB 8.4x16x3 fzb	13087	●	●	●
8	Locking nut M 8x 1 DIN 985	12051	●	●	●
9	Plug frontfork housing black	51136	●	●	●
10	Locking nut 1/2 UNF low svkr	12037	●	●	●

Castor cont.

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
11	Washer 13x24x1.2 DIN 137B fzb	13040	●	●	●
12	Washer BRB 13.5 black chrome	13031	●	●	●
13	Ball bearing 1/2 EE4 2RS	80244	●	●	●
14	Washer BRB 13.5	13032	●	●	●
15	Front fork housing black	83029	●	●	●


Widening kit

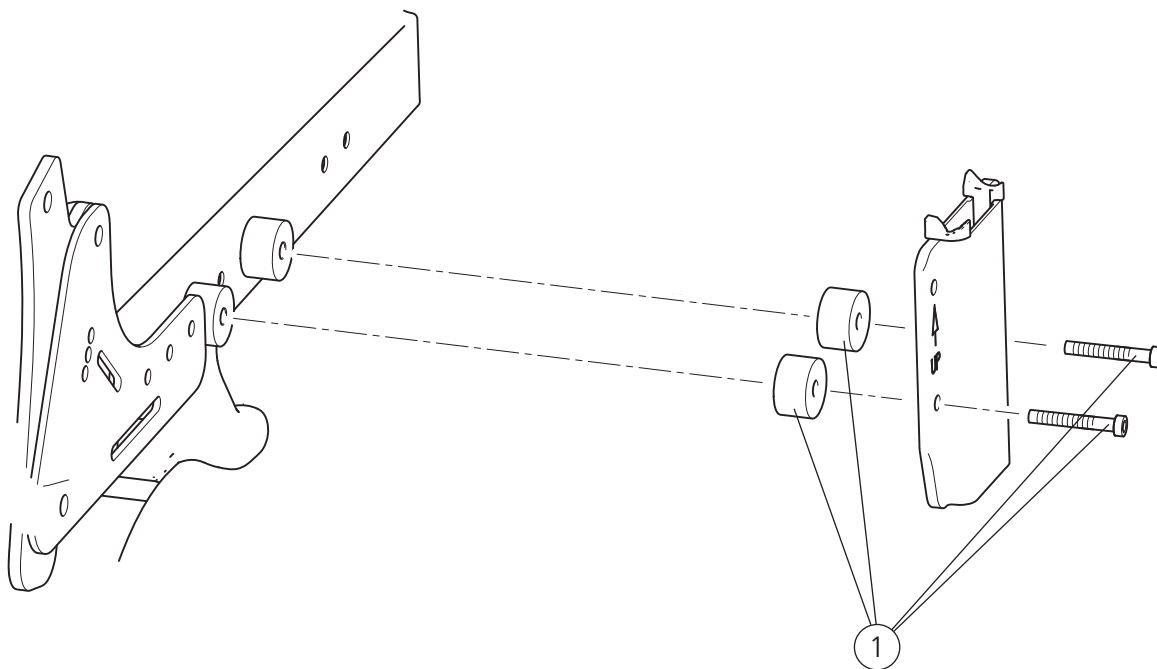

Pos.	Description	Item no.
1	Widening kit 25 mm	27892


Prio	Prio AB	Prio 16"
●	●	●

Solid seat (up to ser. no 38A9999)

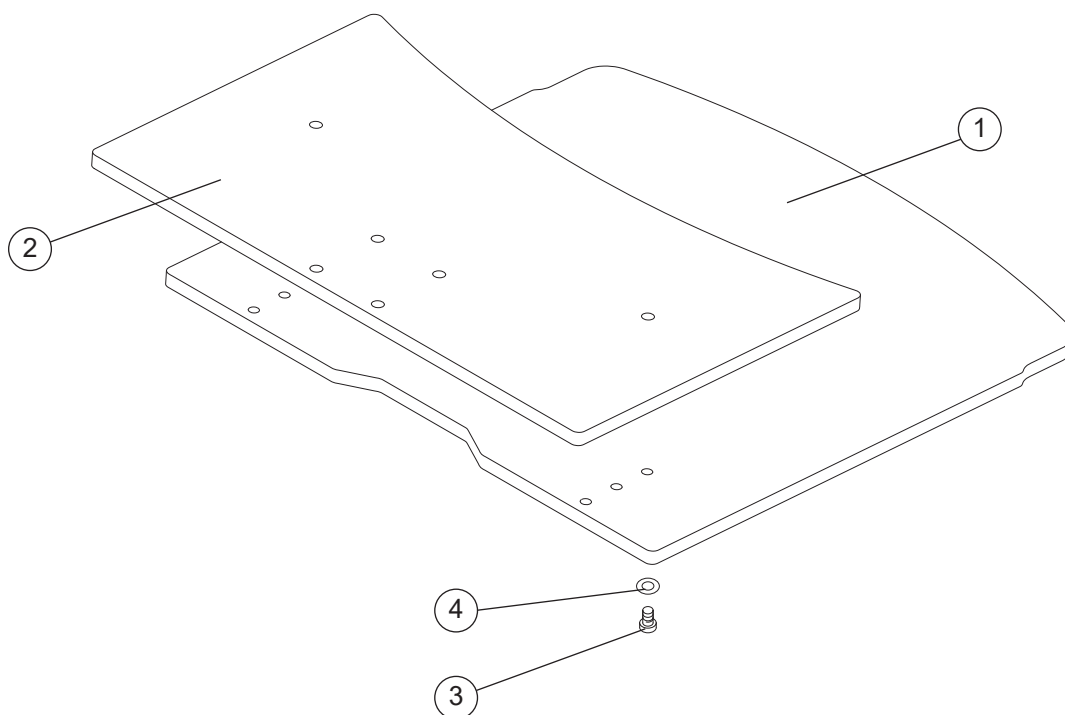


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Seat solid lower 40 cm Prio	82777	●	●	●
	Seat solid lower 45 cm Prio	82778	●	●	●
	Seat solid lower 50 cm Prio	82779	●	●	●
	Seat solid lower 55 cm Prio	82780	●	●	●
2	Seat solid upper cpl 40 short Prio	82781	●	●	●
	Seat solid upper cpl 45 short Prio	82782	●	●	●
	Seat solid upper cpl 50 short Prio	82783	●	●	●
	Seat solid upper cpl 55 short Prio	82784	●	●	●
	Seat solid upper cpl 40 long Prio	82785	●	●	●
	Seat solid upper cpl 45 long Prio	82786	●	●	●
	Seat solid upper cpl 50 long Prio	82787	●	●	●
	Seat solid upper cpl 55 long Prio	82788	●	●	●
3	Screw M 6x 40 MC65	11229	●	●	●
4	Washer BRB M 6	13030	●	●	●

Solid seat (from ser. no. 38B1)

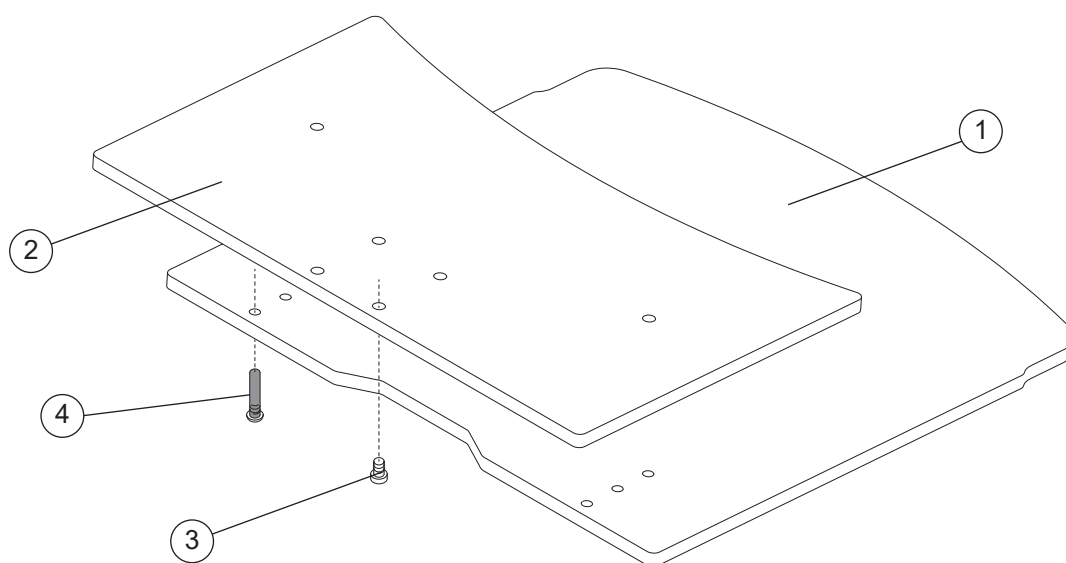

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Seat solid lower 40 cm Prio	82777	●	●	●
	Seat solid lower 45 cm Prio	82778	●	●	●
	Seat solid lower 50 cm Prio	82779	●	●	●
	Seat solid lower 55 cm Prio	82780	●	●	●
	Seat solid lower 40 cm Prio plastic	51121	●	●	●
	Seat solid lower 45 cm Prio plastic	51122	●	●	●
	Seat solid lower 50 cm Prio plastic	51123	●	●	●
	Seat solid lower 55 cm Prio plastic	51124	●	●	●
2	Seat solid upper cpl 40 short Prio	82952	●	●	●
	Seat solid upper cpl 45 short Prio	82953	●	●	●
	Seat solid upper cpl 50 short Prio	82954	●	●	●
	Seat solid upper cpl 55 short Prio	82955	●	●	●
	Seat solid upper cpl 40 long Prio plastic	51125	●	●	●
	Seat solid upper cpl 45 long Prio plastic	51126	●	●	●
	Seat solid upper cpl 50 long Prio plastic	51127	●	●	●
	Seat solid upper cpl 55 long Prio plastic	51128	●	●	●
3	Screw MC6LS M6 x 14	11539	●	●	●
4	Fixing pin	51114	●	●	●

Arm support

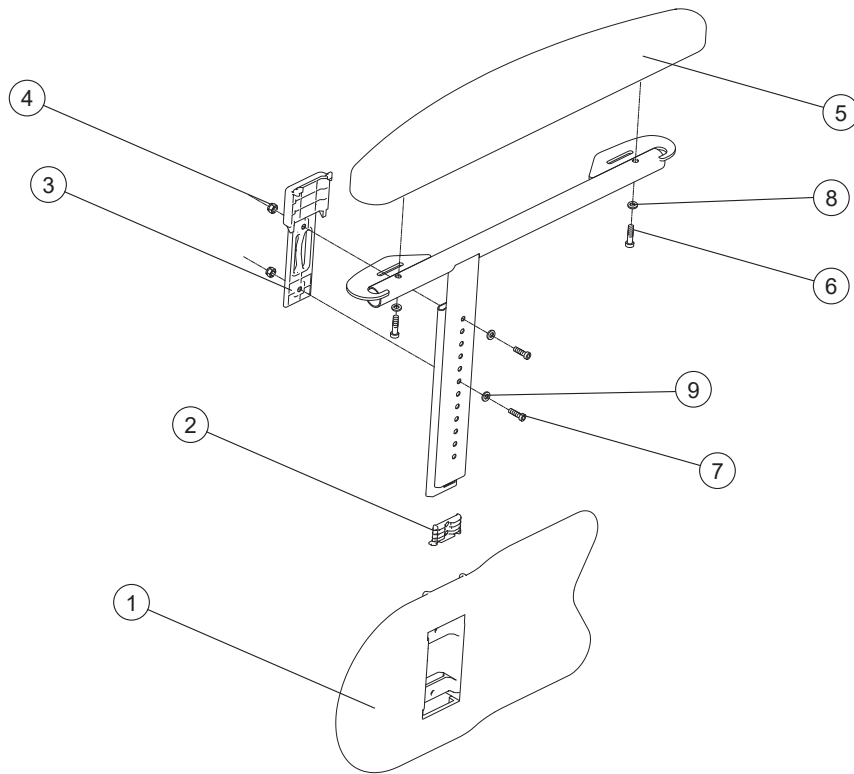

Pos.	Description	Item no.
1-8	Arm support Prio right	27714-1
	Arm support Prio left	27714-2
1	Side guard Prio	
2	Plug vertical profile Prio	
3	Cover/nut pocket side guard	
4	Square nut M5 DIN 557	
5	Armrest pad ri/le Prio	82664
6	Screw M 5x 14 MC6LS	11340
7	Screw K6S M5 x 18 10.9	11656
8	Washer BRB M 6	13030
9	Washer M5 anti loss	


Prio

Prio AB

Prio 16"


●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Arm support Hemi

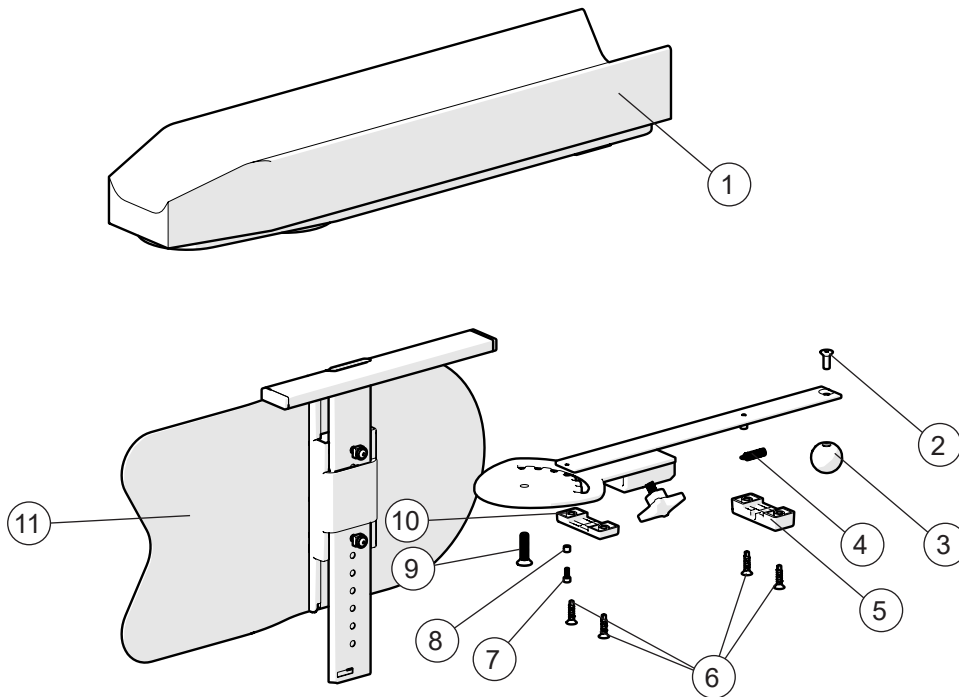

Pos.	Description	Item no.			
1-10	Arm support Prio right compl	28034-60-1	●	●	●
	Arm support Prio left compl	28034-60-2	●	●	●
2-10	Positioning lock Hemi right	28165-1	●	●	●
	Positioning lock Hemi left	28165-2	●	●	●
1	Arm support Prio right	28167-1	●	●	●
	Arm support Prio left	28167-2	●	●	●
2	Screw M 5x 14		●	●	●
3	Ball SD25 M5		●	●	●
4	Spring		●	●	●
5	Outer guide Hemi		●	●	●
6	Screw ST4.2		●	●	●
7	Spacer Hemi		●	●	●
8	Screw M3x8		●	●	●
9	Screw M6x25		●	●	●
10	Inner guide		●	●	●
11	Arm support bracket	28166-1	●	●	●
	Arm support bracket	28166-1	●	●	●


Trunk support

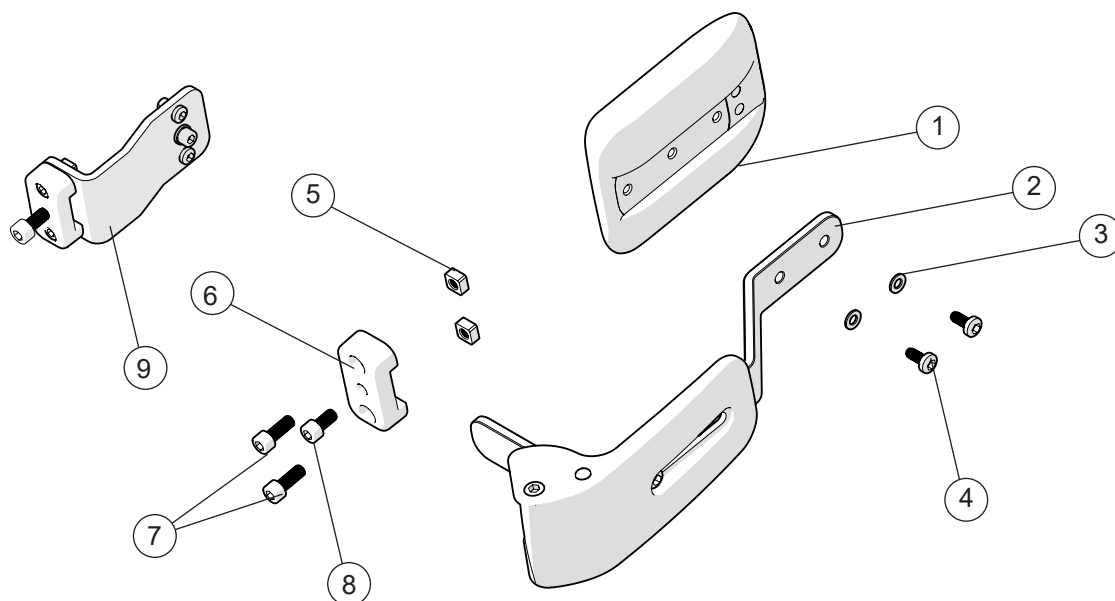

Pos.	Description	Item no.
1-8	Trunk support foldable right	28019-1
	Trunk support foldable left	28019-2
1-4, 9	Trunk support Prio incl attachment right (to ser no. 38A9999)	28141-1
	Trunk support Prio incl attachment left (to ser no. 38A9999)	28141-2
1	Trunk support pad	83005
2	Trunk support arm	
3	Washer 5.3x10x1 BRB DIN 125	13017
4	Screw M 5x 12 K6S 10.9	11208
5	Welding nut M 6 FK4M	12115
6	Attachment trunk support	82995
7	Screw M6x12 mc6s	11577
8	Screw M6x14 MF6S 10.9	11592
9	Trunk support attachment 3A compl	28146


Prio	PrioAB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Trunk support

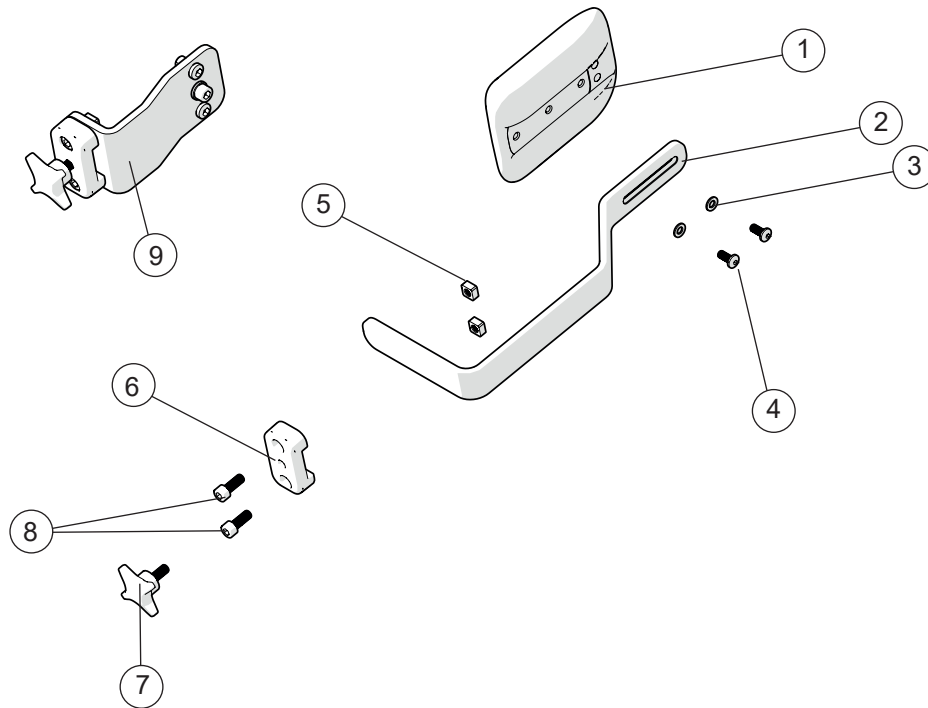


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-8	Trunk support Prio incl attachment	28020	●	●	●
1-4, 9	Trunk support Prio incl attachment (to ser no.38A9999)	28145	●	●	●
1	Trunk support pad	83005	●	●	●
2	Trunk support arm		●	●	●
3	Washer 5.3x10x1 BRB DIN 125	13017	●	●	●
4	Screw M 5x 12 K6S 10.9	11208	●	●	●
5	Welding nut M 6 FK4M	12115	●	●	●
6	Attachment trunk support	82995	●	●	●
7	Knob 40 M6x17	80297	●	●	●
8	Screw M6x12 mc6s	11577	●	●	●
9	Trunk support attachment 3A compl	28146	●	●	●

Leg abduction

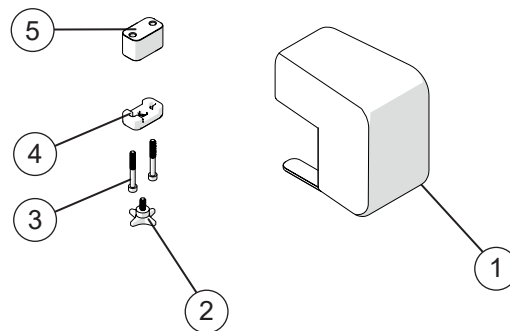


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-5	Leg abduction compl	28022	●	●	●
1	Leg abduction	28194	●	●	●
2	Knob 40 M6x17	80297	●	●	●
3	Screw	11037	●	●	●
4	Bracket	82995	●	●	●
5	Spacer	83042	●	●	●

Tray

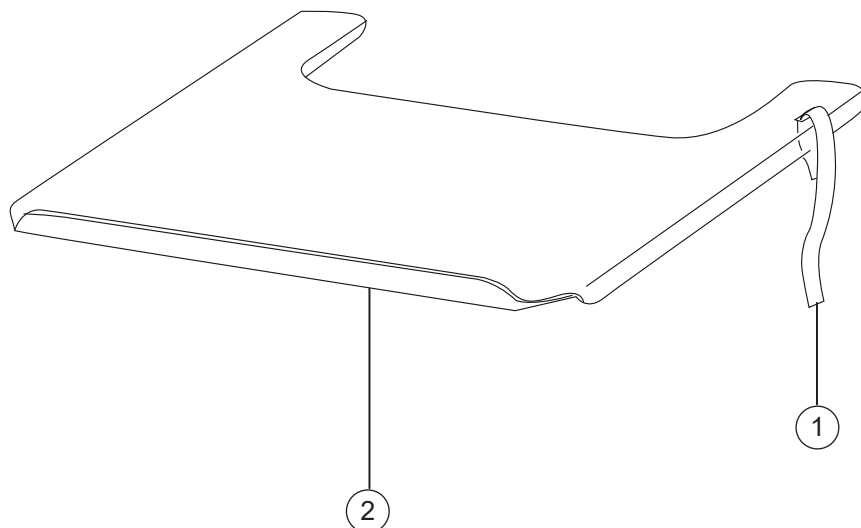


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Strap for tray 27.5-50 cm (11-20)	62103	●	●	●
1-2	Tray 57.5 cm compl for wheelchair 40 cm (19)	25731	●	●	●
	Tray 62.5 cm compl for wheelchair 45 cm (21)	26020	●	●	●
	Tray 67.5 cm compl for wheelchair 50 cm (23)	26022	●	●	●
	Tray 75,5 cm compl for wheelchair 55 cm (25)	27902	●	●	●

Attachment harness

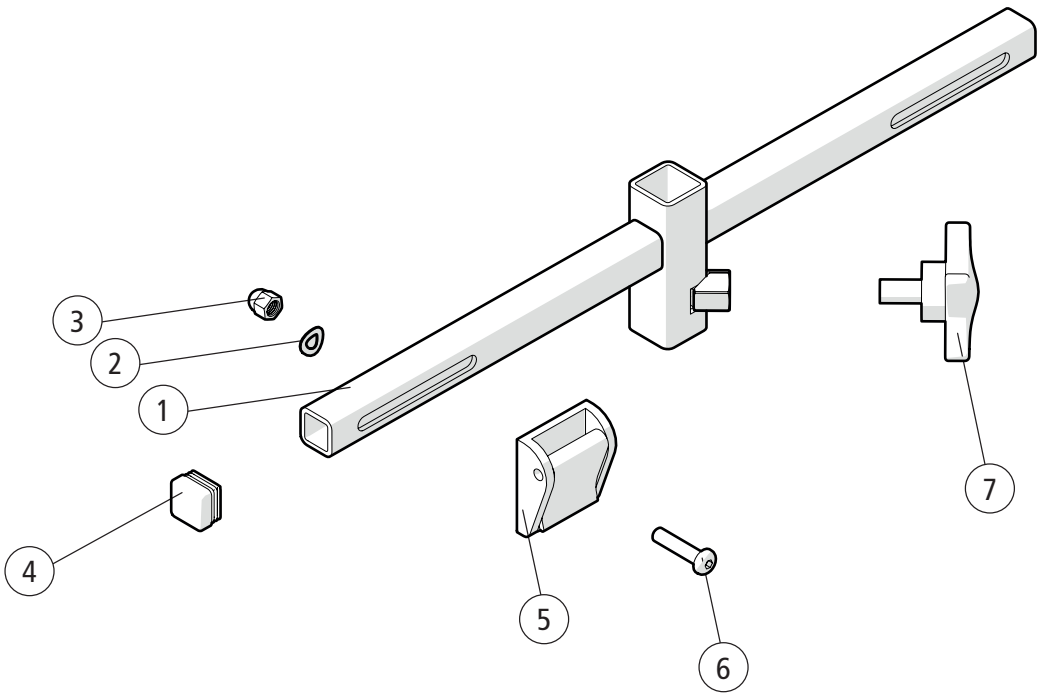

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-7	Attachment harness cpl	28186	●	●	●
1	Attachment harness		●	●	●
2	Washer D10x5.3x0.5	13017	●	●	●
3	Domed nut M5	12167	●	●	●
4	End plug	11539	●	●	●
5	Fix lock 25 mm	310445025	●	●	●
6	Screw M 5x 22 MK6S fzb	11075	●	●	●
7	Knob	400118	●	●	●

IV stand

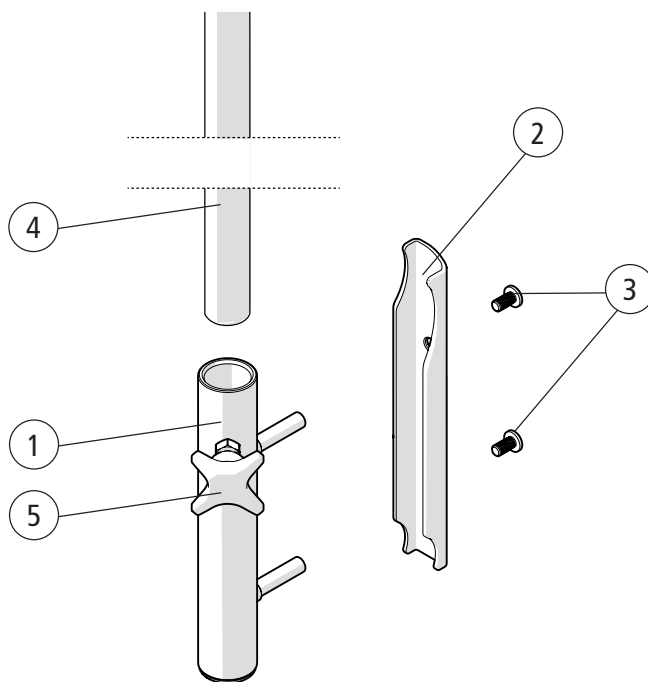


Pos.	Description	Item. no.
1-4	IV stand cpl	28160
1	IV stand welded Prio	
2	Bracket	
3	Screw M6x14 MC6LS SW5 black chrome	11539
4	Hook drip stand	83053
5	Knob	80297

Prio

Prio AB


Prio 16"

Basket

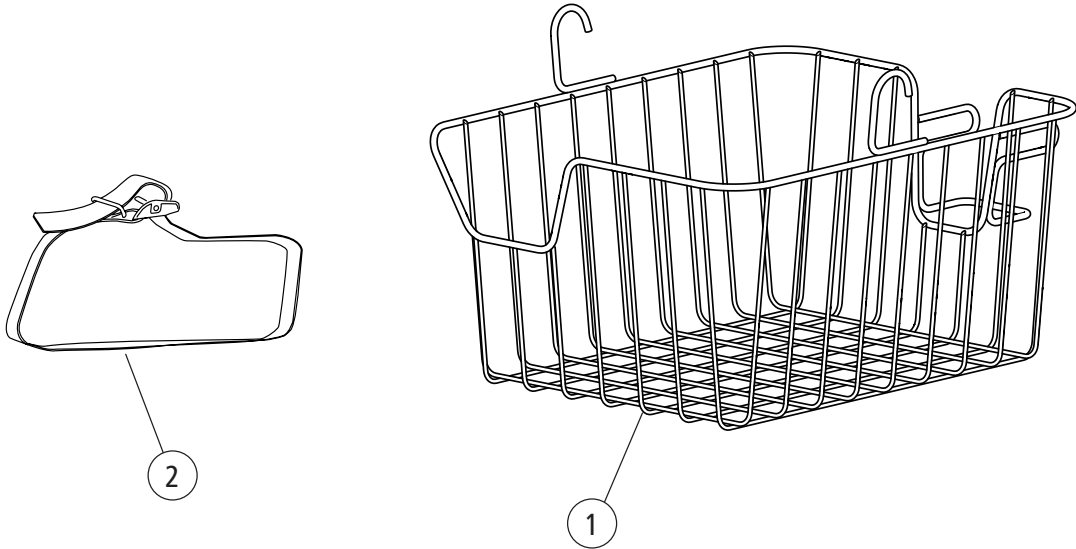

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-2	Basket compl Prio	28160	●	●	●
1	Basket		●	●	●
2	Strap	80459	●	●	●

Positioning belt

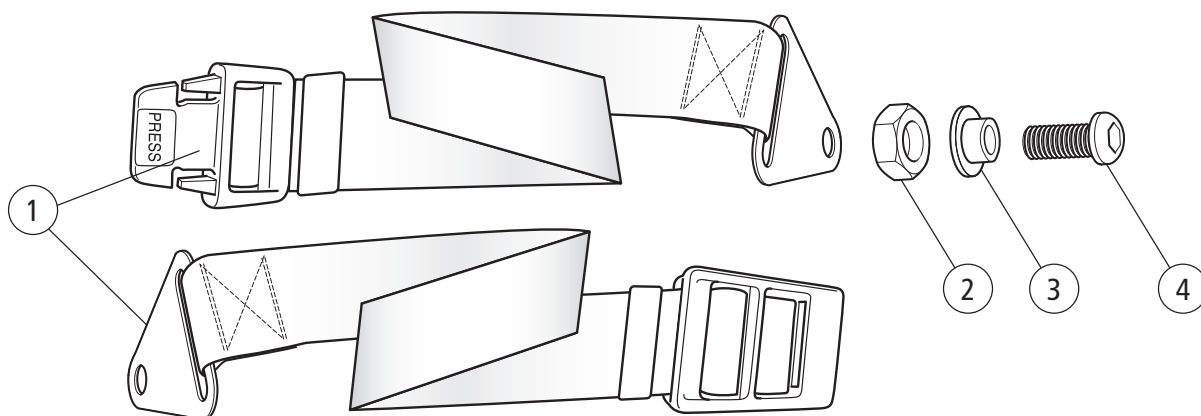


Pos.	Description	Item no.
1-4	Positioning belt	27904
2	Locking nut M 6	
3	Spacer positioning belt	
4	Screw M 6x 14 K6S	

Prio	Prio AB	Prio 16"

Anti tip


●	●	●
●	●	
●	●	●
●	●	●

Spoke guard

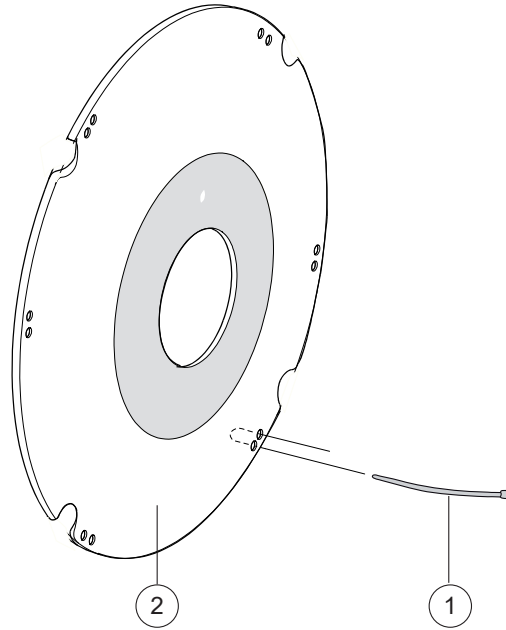

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Attachment kit spokeguard	26276	●	●	
1-2	Spoke guard 20 with logo Etac grey	25502-28	●	●	
	Spoke guard 20 with logo Etac yellow	25502-29	●	●	
	Spoke guard 22 with logo Etac grey	25503-28	●	●	
	Spoke guard 22 with logo Etac yellow	25503-29	●	●	
	Spoke guard 24 with logo Etac grey	25504-28	●	●	
	Spoke guard 24 with logo Etac yellow	25504-29	●	●	

Head support (up to ser. no. 38A9999)

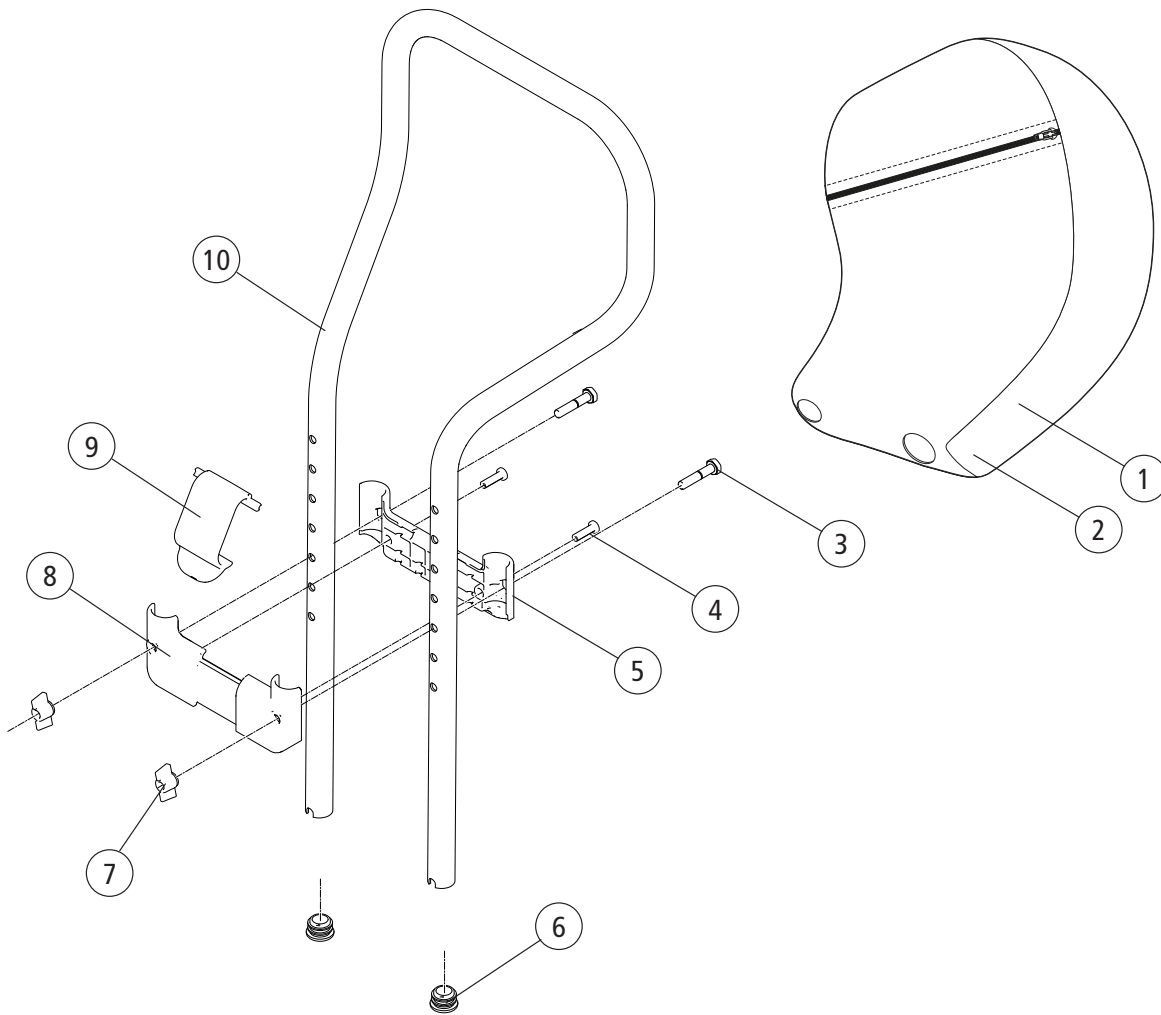


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Head support Prio wide STD Plush	27776-60	●	●	●
	Head support Prio wide STD Hygiene green	27776-64	●	●	●
	Head support Prio wide STD Hygiene grey	27776-65	●	●	●
2	Upholstery Head support Prio narrow STD Plush	63076-60	●	●	●
	Upholstery Head support Prio narrow STD Hygiene green	63076-64	●	●	●
	Upholstery Head support Prio narrow STD Hygiene grey	63076-65	●	●	●
	Upholstery Head support Prio wide STD Plush	63077-60	●	●	●
	Upholstery Head support Prio wide STD Hygiene green	63077-64	●	●	●
	Upholstery Head support Prio wide STD Hygiene grey	63077-65	●	●	●
	Upholstery Head support Prio Comfort Plush	63078-60	●	●	●
	Upholstery Head support Prio Comfort Hygiene green	63078-64	●	●	●
	Upholstery Head support Prio Comfort Hygiene grey	63078-65	●	●	●

Head support (up to ser. no 38A9999)


			Prio	Prio AB	Prio 16"
Pos.	Description	Item no.			
3	ScrewM 6x 30 M6S 8.8	11555	●	●	●
4	Thermoplastic screw panhead	11658	●	●	●
5	Plastic bridge head frame front	51037	●	●	●
6	Dust cover 19x 1.5 black	50010	●	●	●
7	Wing nut WN 314.1	80847	●	●	●
8	Plastic bridge head frame rear	51036	●	●	●
9	Plastic lock head frame Prio	51043	●	●	●
10	Head frame std Prio	82720	●	●	●
	Head frame std Prio +50 mm	82721	●	●	●
1-10	Head support wide active head support plush	27719-60	●	●	●
	Head support wide active head support hygiene green	27719-64	●	●	●
	Head support wide active head support hygiene grey	27719-65	●	●	●

Head support (from ser. no. 38B1)

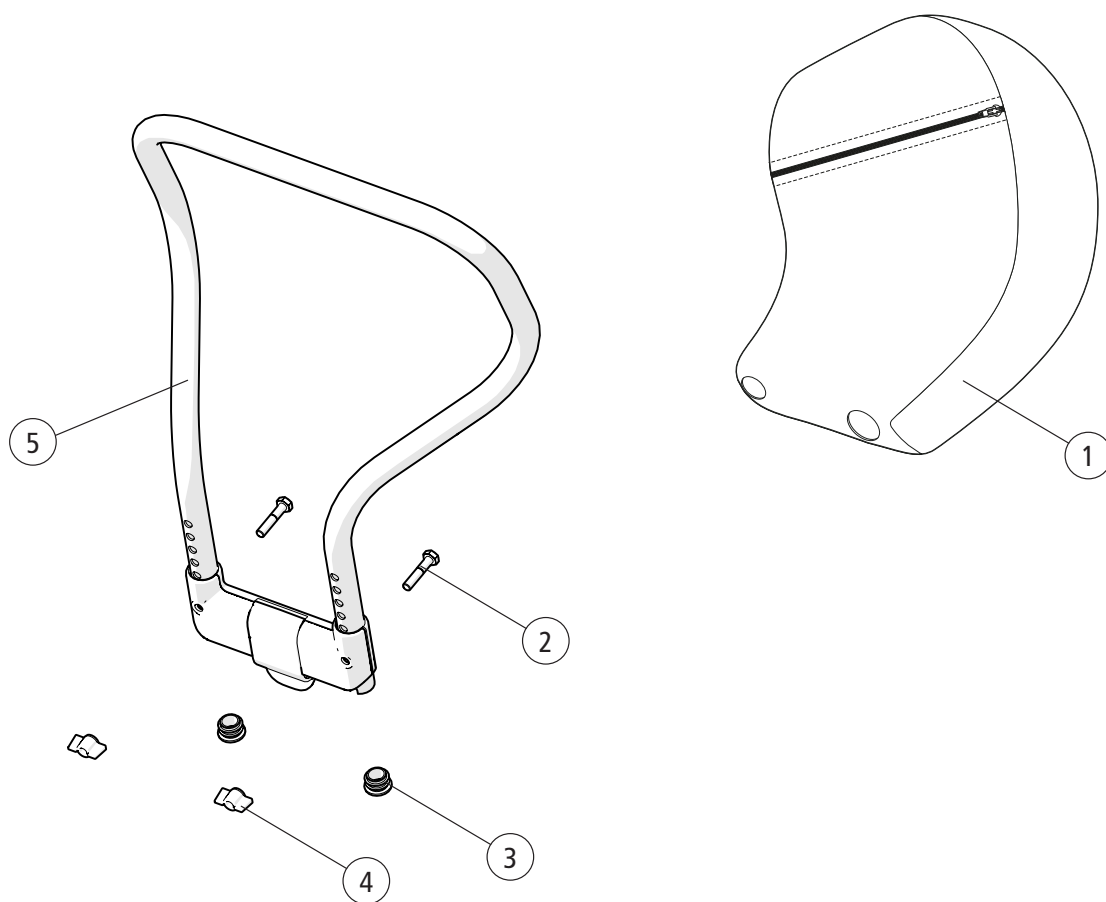


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-5	Head support wide std plush	28152-60	●	●	●
	Head support wide std hygiene green	28152-64	●	●	●
	Head support wide std hygiene grey	28152-65	●	●	●
1	Head support wide std plush	28192-60	●	●	●
	Head support wide std hygiene green	28192-64	●	●	●
	Head support wide std hygiene grey	28192-65	●	●	●
2	ScrewM 6x 30 M6S 8.8	11555	●	●	●
3	Dust cover 19x 1.5 black	50010	●	●	●
4	Wing nut WN 314.1	80847	●	●	●
5	Head support frame Prio		●	●	●

Head support

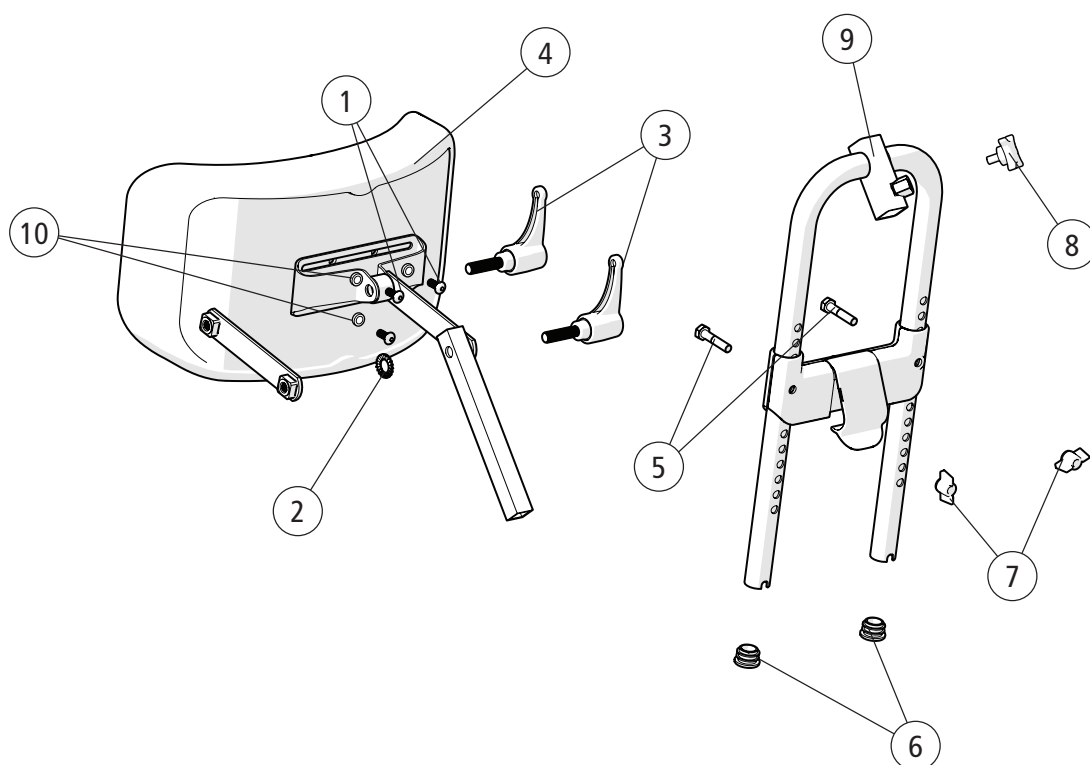


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Screw M 5x 16 MRX		●	●	●
2	Washer M10 IZ DIN 6798/A2		●	●	●
3	Handle M 8x30 65 mm black	80123	●	●	●
4	Head support cushion incl cover grey plush	63176-60	●	●	●
	Head support cushion incl cover grey hygiene	63176-65	●	●	●
1-4	Head support mod 1 Hygiene 26x14 cm incl tube with two joints excl attachment	28023-65	●	●	●
	Head support mod 1 dark grey 26x14 cm incl tube with two joints excl attachment	28023-60	●	●	●
5	Screw M6x 30 M6S 8.8	11555	●	●	●
6	Dust cover 19x 1.5 black	50010	●	●	●
7	Wing nut WN 314.1	80847	●	●	●
8	Knob M8x14	400118	●	●	●
9	Head support frame		●	●	●
5-9	Bracket head support Prio angled complete	28024	●	●	●
10	Joint bushing	50285	●	●	●
1-10	Head support compl grey plush	28193-60	●	●	●
	Head support compl grey hygiene	28193-65	●	●	●

Seat cushion

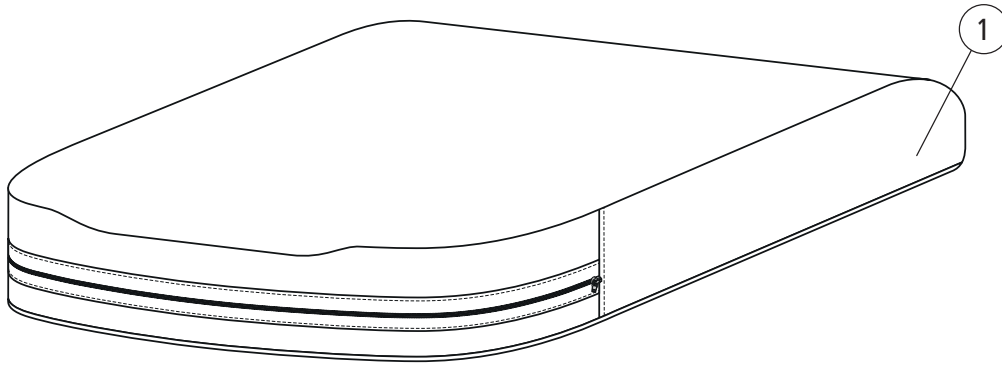


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Seat cushion Prio 40 cm short STD plush	27751-60	●	●	●
	Seat cushion Prio 40 cm short STD hygiene green	27751-64	●	●	●
	Seat cushion Prio 40 cm short STD hygiene grey	27751-65	●	●	●
	Seat cushion Prio 45 cm short STD plush	27752-60	●	●	●
	Seat cushion Prio 45 cm short STD hygiene green	27752-64	●	●	●
	Seat cushion Prio 45 cm short STD hygiene grey	27752-65	●	●	●
	Seat cushion Prio 50 cm short STD plush	27753-60	●	●	●
	Seat cushion Prio 50 cm short STD hygiene green	27753-64	●	●	●
	Seat cushion Prio 50 cm short STD hygiene grey	27753-65	●	●	●
	Seat cushion Prio 55 cm short STD plush	27754-60	●	●	●
	Seat cushion Prio 55 cm short STD hygiene green	27754-64	●	●	●
	Seat cushion Prio 55 cm short STD hygiene grey	27754-65	●	●	●
	Seat cushion Prio 40 cm short comfort plush	27755-60	●	●	●
	Seat cushion Prio 40 cm short comfort hygiene green	27755-64	●	●	●
	Seat cushion Prio 40 cm short comfort hygiene grey	27755-65	●	●	●
	Seat cushion Prio 45 cm short comfort plush	27756-60	●	●	●
	Seat cushion Prio 45 cm short comfort hygiene green	27756-64	●	●	●
	Seat cushion Prio 45 cm short comfort hygiene grey	27756-65	●	●	●
	Seat cushion Prio 50 cm short comfort plush	27757-60	●	●	●
	Seat cushion Prio 50 cm short comfort hygiene green	27757-64	●	●	●
	Seat cushion Prio 50 cm short comfort hygiene grey	27757-65	●	●	●
	Seat cushion Prio 55 cm short comfort plush	27758-60	●	●	●
	Seat cushion Prio 55 cm short comfort hygiene green	27758-64	●	●	●
	Seat cushion Prio 55 cm short comfort hygiene grey	27758-65	●	●	●

Seat cushion

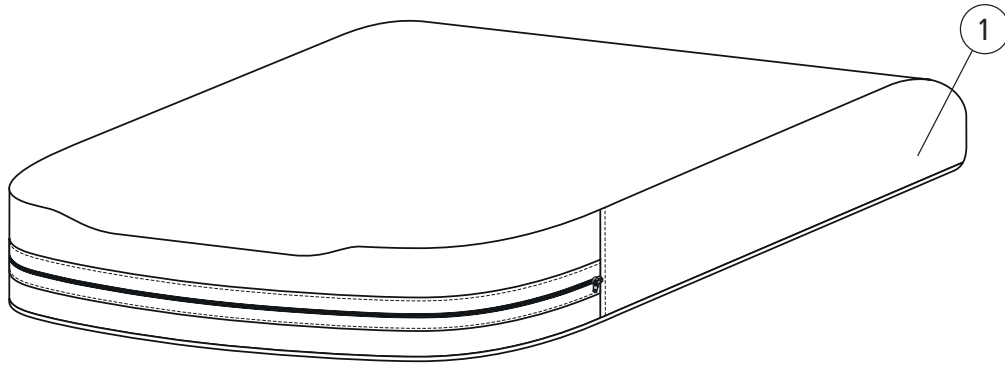


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
	Seat cushion Prio 40 cm long STD plush	27759-60	●	●	●
	Seat cushion Prio 40 cm long STD hygiene green	27759-64	●	●	●
	Seat cushion Prio 40 cm long STD hygiene grey	27759-65	●	●	●
	Seat cushion Prio 45 cm long STD plush	27760-60	●	●	●
	Seat cushion Prio 45 cm long STD hygiene green	27760-64	●	●	●
	Seat cushion Prio 45 cm long STD hygiene grey	27760-65	●	●	●
	Seat cushion Prio 50 cm long STD plush	27761-60	●	●	●
	Seat cushion Prio 50 cm long STD hygiene green	27761-64	●	●	●
	Seat cushion Prio 50 cm long STD hygiene grey	27761-65	●	●	●
	Seat cushion Prio 55 cm long STD plush	27762-60	●	●	●
	Seat cushion Prio 55 cm long STD hygiene green	27762-64	●	●	●
	Seat cushion Prio 55 cm long STD hygiene grey	27762-65	●	●	●
	Seat cushion Prio 40 cm long comfort plush	27763-60	●	●	●
	Seat cushion Prio 40 cm long comfort hygiene green	27763-64	●	●	●
	Seat cushion Prio 40 cm long comfort hygiene grey	27763-65	●	●	●
	Seat cushion Prio 45 cm long comfort plush	27764-60	●	●	●
	Seat cushion Prio 45 cm long comfort hygiene green	27764-64	●	●	●
	Seat cushion Prio 45 cm long comfort hygiene grey	27764-65	●	●	●
	Seat cushion Prio 50 cm long comfort plush	27765-60	●	●	●
	Seat cushion Prio 50 cm long comfort hygiene green	27765-64	●	●	●
	Seat cushion Prio 50 cm long comfort hygiene grey	27765-65	●	●	●
	Seat cushion Prio 55 cm long comfort plush	27766-60	●	●	●
	Seat cushion Prio 55 cm long comfort hygiene green	27766-64	●	●	●
	Seat cushion Prio 55 cm long comfort hygiene grey	27766-65	●	●	●

Seat cushion - Cover

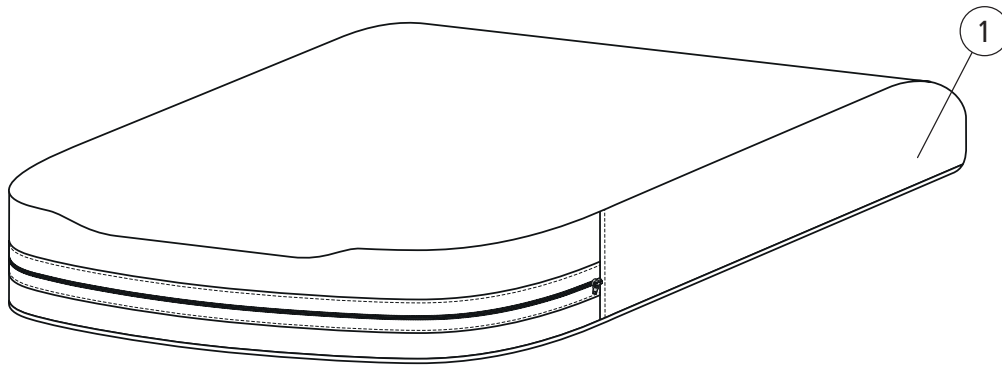


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Cover seat cushion 40 (16) short plysch Prio	63060-60	●	●	●
	Cover seat cushion 40 (16) short hygiene Prio	63060-64	●	●	●
	Cover seat cushion 40 (16) short hygiene grey Prio	63060-65	●	●	●
	Cover seat cushion 45 (18) short plysch Prio	63061-60	●	●	●
	Cover seat cushion 45 (18) short hygiene Prio	63061-64	●	●	●
	Cover seat cushion 45 (18) short hygiene grey Prio	63061-65	●	●	●
	Cover seat cushion 50 (20) short plysch Prio	63062-60	●	●	●
	Cover seat cushion 50 (20) short hygiene Prio	63062-64	●	●	●
	Cover seat cushion 50 (20) short hygiene grey Prio	63062-65	●	●	●
	Cover seat cushion 55 (22) short plysch Prio	63063-60	●	●	●
	Cover seat cushion 55 (22) short hygiene Prio	63063-64	●	●	●
	Cover seat cushion 55 (22) short hygiene grey Prio	63063-65	●	●	●
	Cover seat cushion 40 (16) long plysch Prio	63064-60	●	●	●
	Cover seat cushion 40 (16) long hygiene green Prio	63064-64	●	●	●
	Cover seat cushion 40 (16) long hygiene grey Prio	63064-65	●	●	●
	Cover seat cushion 45 (18) long plysch Prio	63065-60	●	●	●
	Cover seat cushion 45 (18) long hygiene green Prio	63065-64	●	●	●
	Cover seat cushion 45 (18) long hygiene grey Prio	63065-65	●	●	●
	Cover seat cushion 50 (20) long plysch Prio	63066-60	●	●	●
	Cover seat cushion 50 (20) long hygiene green Prio	63066-64	●	●	●
	Cover seat cushion 50 (20) long hygiene grey Prio	63066-65	●	●	●
	Cover seat cushion 55 (22) long plysch Prio	63067-60	●	●	●
	Cover seat cushion 55 (22) long hygiene green Prio	63067-64	●	●	●
	Cover seat cushion 55 (22) long hygiene grey Prio	63067-65	●	●	●

Back support cushion

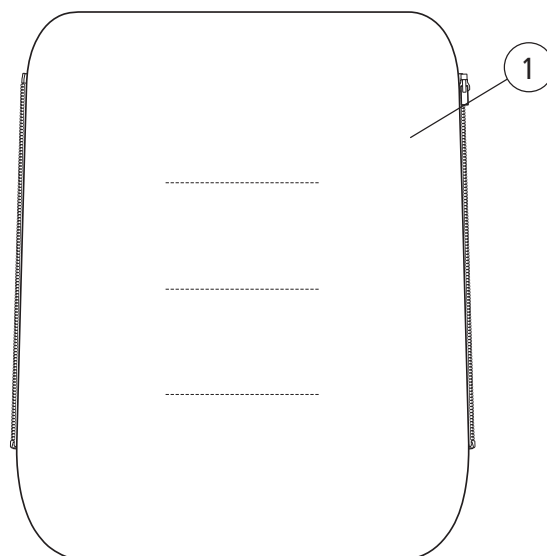

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
	Back support cushion 40 cm Comfort plush	28195-60	●	●	●
	Back support cushion 40 cm Comfort hygiene green	28195-64	●	●	●
	Back support cushion 40 cm Comfort hygiene grey	28195-65	●	●	●
	Back support cushion 45 cm Comfort plush	28196-60	●	●	●
	Back support cushion 45 cm Comfort hygiene green	28196-64	●	●	●
	Back support cushion 45 cm Comfort hygiene grey	28196-65	●	●	●
	Back support cushion 50 cm Comfort plush	28197-60	●	●	●
	Back support cushion 50 cm Comfort hygiene green	28197-64	●	●	●
	Back support cushion 50 cm Comfort hygiene grey	28197-65	●	●	●
	Back support cushion 55 cm Comfort plush	28198-60	●	●	●
	Back support cushion 55 cm Comfort hygiene green	28198-64	●	●	●
	Back support cushion 55 cm Comfort hygiene grey	28198-65	●	●	●
	Back support 40 cm with side support	28245-60	●	●	●
	Back support 45 cm with side support	28246-60	●	●	●
	Back support 50 cm with side support	28247-60	●	●	●
	Back support 55 cm with side support	28248-60	●	●	●

Back support upholstery comfort

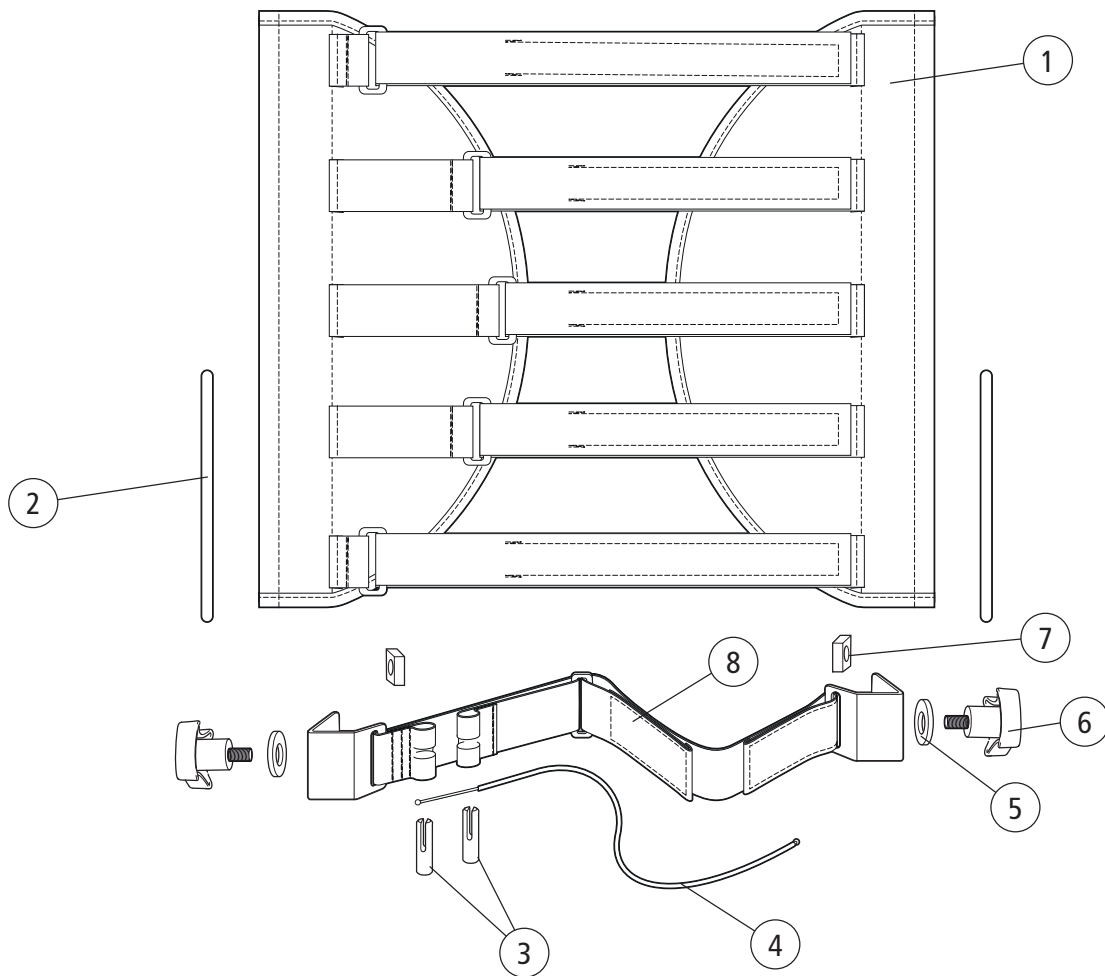


Pos.	Description	Item no.
1	Back support upholstery 40 cm	63072-60
	Back support upholstery 45 cm	63073-60
	Back support upholstery 50 cm	63074-60
	Back support upholstery 55 cm	63075-60
2	Tension rod kit Prio	28275
3	Wire attachment lumbar support	
4	Wire active lumbar support	82661
5	Washer 6,4x14x1,5	13062
6	Knob 40 M6x17 black	80297
7	Nut M 6 FK4M	12058
8	(Back support strap active lumbar support)	
3-8	Active lumbar support Prio 40 cm	28035
	Active lumbar support Prio 45 cm	28036
	Active lumbar support Prio 50 cm	28037
	Active lumbar support Prio 55 cm	28038

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Back support upholstery (up to ser. no 38A9999)

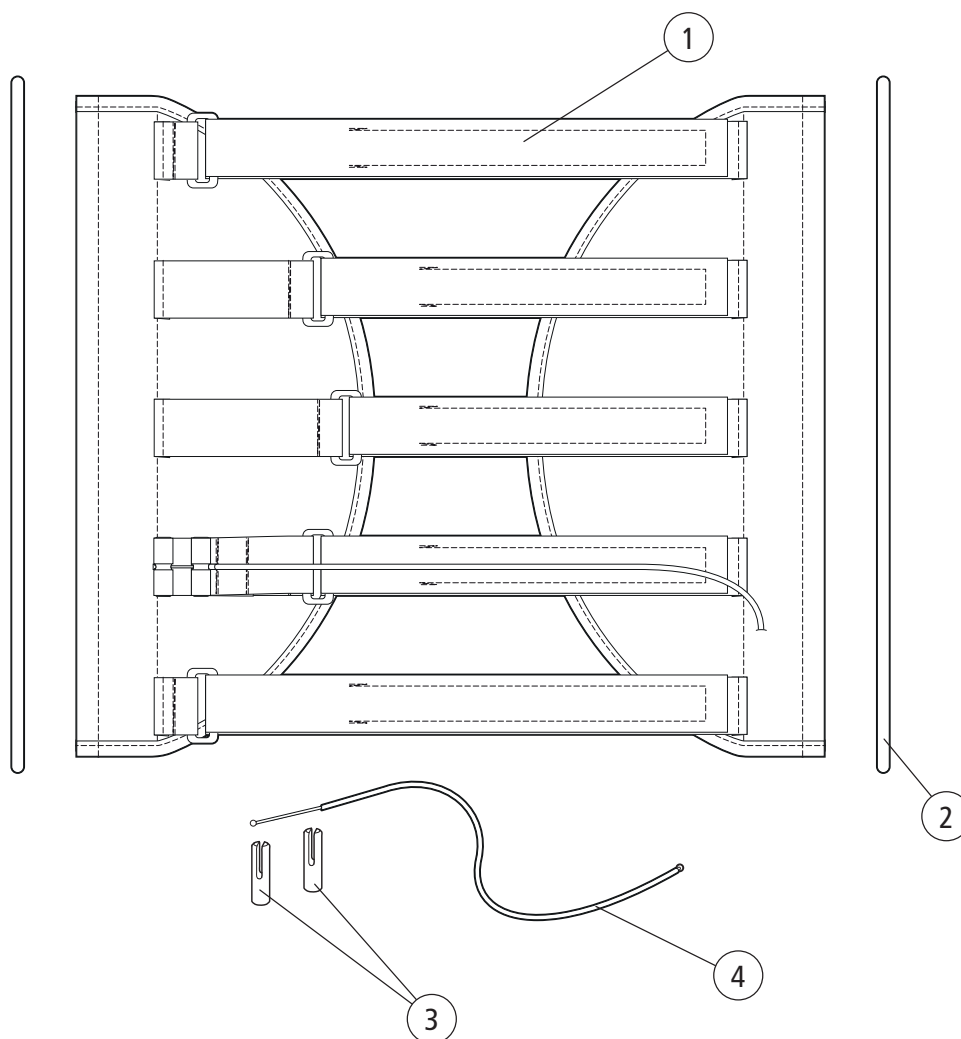

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Back support upholstery 40 cm	63154-60+28035	●	●	●
	Back support upholstery 45 cm	63155-60+28036	●	●	●
	Back support upholstery 50 cm	63156-60+28037	●	●	●
	Back support upholstery 55 cm	63157-60+28038	●	●	●
2	Tension rod Prio		●	●	●
3	Wire attachment lumbar support		●	●	●
4	Wire active lumbar support	82661	●	●	●